Infrared Thermometer

User Manual

Index

English	4
Nederlands	8
Deutsch	12
Français	16
Español	20
Italiano	24
Polski	28

1. PACKAGE CONTENTS

- 1. Infrared Thermometer
- 2. Manual

2. TECHNICAL SPECIFICATIONS

2.1 Product specifications

- Display accurate digits: 0.1°C (0.1°F)
- Storage temperature: -20°C -55°C
- Operating environment temperature: 16°C~35°C
- Relative humidity: 585%
- Power: DC 3V (2 AAA battery)
- Size: 160*100*40mm
- Weight: 100g

- Toggle key > Toggle button
- LED > LCD Display
- Backlight
- Memory key > Memory button

2.2 Measuring range

- Measures temperatures between: 32.0-42.9°C
- Range of measuring distance: 5-8 cm
- Automatically turns off after 20 seconds.

2.3 Measuring accuracy

32.0°C ~ 34.9°C	±0.3°C
35.0°C ~ 42.0°C	±0.2°C
42.1°C ~ 42.9°C	±0.3°C

3. INSTRUCTIONS FOR USE

This infrared thermometer is specialized in measuring the temperature of the human body by aiming at the forehead area. The temperature will be shown at the LCD display. Please note that this thermometer does not replace the doctor's diagnosis/advice.

3.1 Features

- Non-contact and high-precision body thermometer.
- The temperature can be displayed in °C or °F. Press the “°C/F” button for 8 seconds to switch.
- An alert value can be set. The default value is set at 38°.
- When measuring temperatures above the alert value, the thermometer will give a “buzzer” sound.
- The LCD display has backlight for clear readings in the dark. Press the “Backlight” button once to turn on the backlight. Press and hold the button for 8 seconds to turn the backlight off.
- Up to 32 measurements can be stored in the memory log. Press the up and down buttons to view stored measurements.
- The thermometer automatically saves measurements

3.2 Notes before use

Make sure to do a test measure before measuring the temperature of yourself or another person. Aim the thermometer at the area of the forehead and press the “measuring” button. A confirmation sound will be heard, indicating that the test has been completed. The measured temperature will be displayed on the LCD screen.

3.3 Explanation of the LCD display

Figure 2

Body temperature mode (BODY)
Measured temperature
Memory log
Temperature °C/F

4. OPERATING THE TEMPERATURE GUN

4.1 Measuring body temperature

Aim the thermometer at the area around the forehead above the eyebrows. Hold the thermometer at about 5-10 cm and press the "measure" button to measure the body temperature.

The measured temperature is shown on the LCD display and will automatically be stored in the memory log. Measures can be saved up to 32 times in the memory log.

4.2 Battery instructions

The battery cover is indicated by an arrow at the bottom of the thermometer handle. Push the cover lightly towards the direction the arrow is pointing to, to open the cover. You can now insert the batteries.

4.2 Notes of battery installation and replacement

Pay attention at how the battery needs to be placed according to the "+" and "-"-symbols.

When a new battery has been placed it can take up to 10 minutes for the thermometer to power on.

When the battery capacity is insufficient, a low battery symbol will appear on the display.

Please take out the battery when the product is not being used for a long time to prevent damage like leakage of battery liquids.

5. SAFETY INSTRUCTIONS

- Do not charge non-rechargeable batteries
- Do not expose the product to water
- Keep the product away from heating sources.

6. CLEANING

- Clean the product with a dry cloth.
Do not use any (chemical) liquids or oil!

7. PROBLEMS AND SOLUTIONS

7.1 LCD cannot display numerical values.

If the measured temperature is lower than 32°C or higher than 42.9°C, the LCD will display "LO" or "HI".

Reasons for 'LO' or 'HI' notification	Solution
The temperature value is affected by hair and sweat.	Make sure there are no obstacles when measuring the temperature.
The temperature is affected by airflow change.	Ensure stable air during temperature measurement.
The measuring distance is too far.	Make sure that the measuring distance is in the 5-10 cm range.

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