HORIGEN

USER MANUAL

Hands-Free Double Electric Breast Pump

Model: D6

Decent Go

★ **Caution:**

To avoid injury caused by misuse, please read the manual carefully before using the product, and use the product in strict conformity with instruction manual.

CONTENTS

1	Intended For Use	01
1.1	Indications For Use	01
1.2	Operating Life	01
2	Product Description	02
2.1	Features	02
2.2	Mode Description	02
2.3	Parts	03
2.4	Buttons Control	04
3	Safety Precautions	05
3.1	Warnings	05
3.2	Precautions	06
4	Getting Started	08
4.1	Cleaning And Sterilizing	08
4.2	Charging	11
4.3	Assembling The Breast Pump	12
4.4	Disassembling The Breast Pump	14
5	Using Your Breast Pump	15
5.1	Flange Sizing	15
5.2	Aligning Your Nipple	16
5.3	Placing Collection Cup within your bra	16
5.4	Starting Your Session	17
5.5	Handling And Storing Of Breast Milk	19
6	Maintenance	20
6.1	Battery Maintenance	20
6.2	Breast Pump Maintenance	20
7	Technical Specifications	21
8	Trouble Shooting	22

INTENDED FOR USE

1.1 Indications For Use

The "Decent GO Series" hands-free breast pump is a powered breast pump intended for lactating women to express and collect milk from their breasts.

The breast pump alleviate engorgement of the breast, maintain the ability of lactation, and provide mother's milk for future feedings when separation of mother and baby occurs.

The device is intended for a single user to be used in a home environment, and is capable of single and double pumping.

1.2 Operating Life

- The operating life of the Device is defined to be not less than 500 hours.
- The expected operating life of the replaceable washable components is 3-6 months. It also depends on how frequently the parts are used and washed.
- It's recommended to replace the washable components after long-term storage or before pumping for your next child.

PRODUCT DESCRIPTION

2.1 Features

- LED panel, clear and intuitive.
- 3 modes with 9 levels each mode.
- Auto-timer & Memory suction level.
- Safety Childlock function.
- Ultra quiet noise reduce design.
- Back housing belt clip design.
- Each cup holds up to 6 oz.
- 1400mAh large capacity battery.
- Freedom tubing direction.


2.2 Mode Description

3-Phase Expression

- **Stimulation Mode**
There are 9 levels in the Stimulation Mode, you can find the right massage levels by personal need.
- **Expression Mode**
There are 9 levels in the Expression Mode, you can find the right suction levels by personal need.
- **2 in 1 Mode**
There are 9 levels in 2-in-1 Mode, you can find the right suction levels by personal need.

PRODUCT DESCRIPTION

2.3 Parts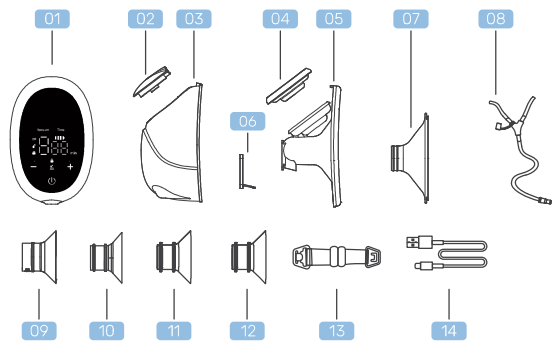


01	Pump Motor		1	09	21mm Flange Insert		2
02	Diaphragm Cover		2	10	19mm Flange Insert		2
03	180ml Collection Cup		2	11	17mm Flange Insert		2
04	Diaphragm		2	12	15mm Flange Insert		2
05	Flange		2	13	Bra Adjuster		2
06	Valve		2	14	Type-C Cable		1
07	25mm Cushion		2				
08	Tubing Kit		1				

*Spare part: Spare Silicone Valve *2

PRODUCT DESCRIPTION

2.4 Buttons Control

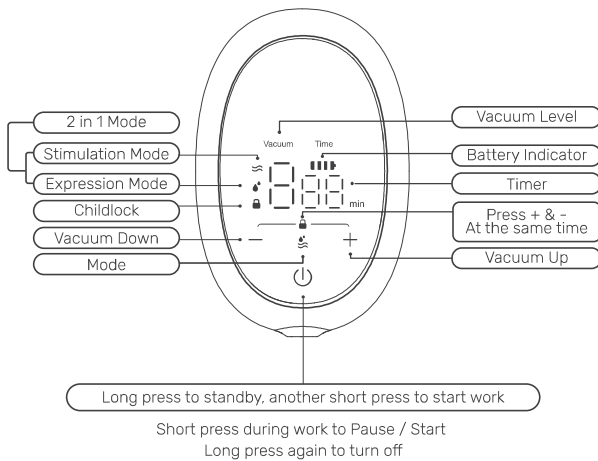TIP:

- This product has a built-in lithium-ion battery. When the battery indicator light shows only 1 bar, we recommend you that use a certified 5V 1A Power Adapter to charge the pump motor till it's fully charged.
- When the battery indicator light flashes during the working process, indicates that the pump motor should be charged as soon as possible.

SAFETY PRECAUTIONS

Please read the "Safety Precautions" carefully before using the product, use it in the correct way to avoid damage or danger caused by improper use.

3.1 Warnings

- This is a single-user product, use by more than one person may present a health risk and voids the warranty.
- Do not drop or immerse the Pump Motor in water or other liquids.
- Do not place or store the device where it can fall or be pulled into a tub or sink.
- Do not use the breast pump while bathing or sleeping or overly drowsy.
- Do not inhale or swallow small parts. Keep out of reach of children.
- Do not sanitize the kit in microwave oven.
- Do not store wet or damp parts, as mold may develop.
- Do not place the pump motor in the fridge or freezer.
- Do not use detergents containing alcohol, ammonia, benzene, or abrasives.
- Do not leave the device unattended when plugged into an electrical outlet.
- Do not use flammable materials while using or connecting the breast pump to a charger.
- Do not unplug power adapter/charger by pulling on the cord.
- Only use the manufactured parts with this breast pump, including the charging cable, do not substitute.
- It's recommended to use an adaptor compatible with both IEC60601-1 and your regional power source.

SAFETY PRECAUTIONS

- Never operate the device if it has a damaged cord or plug, or it has been dropped into water, or if any parts are broken or missing.
- Do not disassemble or modify the pump motor. When disassembled by unauthorized person, it voids warranty.
- Do not expose this breast pump or the build-in battery to direct sunlight, fire or excessive heat.
- The device and its components are not heat-resistant. Keep away from heat sources.

3.2 Precautions

- Always inspect all parts before use for damage or cleanliness.
- Sanitize and disinfect all washable components before the first and each use.
- After using the breast pump, remove and clean the parts that come into contact with your breast and breast milk ASAP to avoid bacterial growth and odor caused by breast milk residue.
- The Tubing Kit should not be soaked in water for cleaning. If accidentally drops into water, it must be completely dried before use.
- Always ensure all parts are completely dry before assembling.
- If tubing becomes moldy, discontinue use and replace tubing.
- Check whether the silicone flange and other parts are assembled in place before using the breast pump, to avoid the phenomenon of no suction or low suction in the process of use.
- Close supervision is necessary when the breast pump is used near infants, children or persons with disabilities.

SAFETY PRECAUTIONS

- It's recommended you consult with a licensed healthcare professional before using the breast pump if you have an active implanted device that would be affected by a magnetic field (e.g. pacemaker, implantable cardioverter defibrillator).
- Using a breast pump should not cause pain. Do not try to express with an intensity setting that is too high or uncomfortable. If excessive discomfort or pain is felt during pumping, turn the pump off and remove from your nipple immediately and discontinue the session.
- Contact your healthcare professional if you express minimal or no milk, or if pumping is painful.
- For assistance with correct flange sizing and comfort, pls see section 5.1 or talk to a lactation consultant or breastfeeding specialist.
- The breast pump should not be used while lying down, or engaging in strenuous exercise or activities with risk of harm
- When using this breast pump, frequency or level should be selected due to the actual situation of the individual. Started with lower level from beginning.

GETTING STARTED

4.1 Cleaning And Sterilizing

Cleaning The Pump Motor

- The Pump Motor contains electronics that should be kept dry.
- Never wash or sterilize the pump motor, wipe with a clean, damp soft cloth.
- Unplug and switch off the breast pump before cleaning the pump motor.
- Any other cleaning method will be classed as misuse under the warranty.


Do not get wet!

GETTING STARTED

Cleaning The Washable Parts

- For the first time and after each use, you must disassemble, wash and sterilize all the parts that come into contact with breast milk.
- Take care when cleaning the silicone cushion and diaphragm and valve, if it gets damaged, your breast pump will not function properly.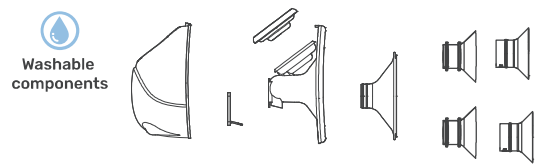


WARNINGS: Please take care of it during washing and sterilizing.

Clean washable components		Sterilize washable components
1. Rinse in cold, clear water (approx +20°C / +68°F).	+	Boil in water for not more than five minutes
2. Wash in warm water (approx +30°C / +86°F) using milk dish washing soap and wipe afterwards with a clean cloth.v		
3. Rinse with cold, clear water (approx +20°C / +68°F) for 10 to 15 seconds.		Suitable for steam or cold water sterilizing methods. We do not recommend to sterilize in steam sterilizer, microwave or dishwasher directly.
OR Place on the top shelf of the dishwasher		

GETTING STARTED

Cleaning The Tubing

- Normally, cleaning of the tubing is not necessary.
- If you find condensation or breast milk residue in the tubing or tubing connector, wash and dry it immediately or replace the tubing.
- Remove the tubing from the pump motor before cleaning the tubing.
- Rinse the tubing by pouring cool water into both short tubing ends until it flows out of the long tubing end.
- Wash the tubing in warm, soapy water. Rinse with clear water.
- Shake out any water droplets and hand to air dry.

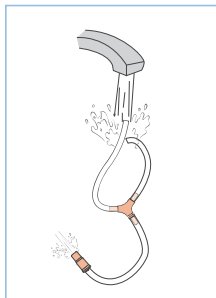WARNINGS:

1. Do not run the breast pump with wet tubing and make sure it is completely dry before using.
2. If the tubing becomes moldy, discontinue use and replace new.

GETTING STARTED

4.2 Charging

- When the battery is low in working process, the battery display will flash.
- When charging, the battery display will light up.
- Use the charging cable provided to connect the pump motor to a USB power source (5V 1A).
- The battery display shows all four bars when charging is complete.
- It takes approximately 2 hours to fully charge the battery, which provides approximately 2 hours usage time at standard settings.

TIP:



1. Charge immediately after finishing your pumping session if lower battery is indicated.
2. You can use the breast pump while charging the battery. However, when the pump is running it will charge in a slower mode.

GETTING STARTED

4.3 Assembling The Breast Pump

01 Assemble diaphragm and valve

Place the Diaphragm (04) into Flange (05), align the disc of Diaphragm (04) on the designated circular slot and ensure it is at the right place. Assembly Valve (06) with Flange (05), press around the edges to ensure it is compatible with the Flange (05).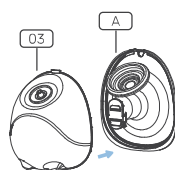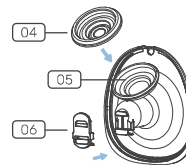


02 Assemble 180ml Collection Cup


First insert the pouring spout into the opening on the flange. Secondly push the Collection Cup (03) and Part (A) together tightly, ensuring the edge is fully sealed all the way around until you hear a snapping sound.

03 Assemble Silicone Cushion with Part B

Insert 25mm Standard Cushion (07) into Part (B), press tightly to ensure sealing correctly. Assemble the Diaphragm Cover (02) tightly with the Collection Cup (03).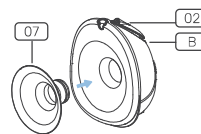

GETTING STARTED

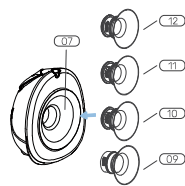
04 Assemble Flange Insert with standard Silicone Cushion

In case that the standard silicone cushion in 25mm (07) were too big, you may choose the 21mm (09) or 19mm (10) or 17mm (11) or 15mm (12) flange insert, to be placed in the center of Cushion (07).

05 Assemble Tubing Kit with Collection Cup


Connect the short ends of the Tubing (08) into the Diaphragm Cover (02) connection port. Make sure the tubing to be straight without any squeezing or bending.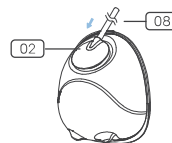

06 Assemble Collection Cup with Pump Motor


• Double Pumping

After both Correction Cup assemble with Tubing Kit, connect the long end of Tubing (with connector) to the Pump motor for double pumping.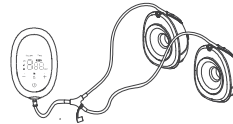

• Single Pumping

Block the unused tubing end on the Y-shaped tubing connector for single pumping.

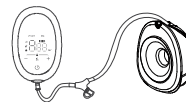

07 Clip the pump motor on your belt or pocket to pump on the go.

GETTING STARTED



4.4 Disassembling The Breast Pump

- 01 Remove the tubing from the diaphragm cover and pump motor.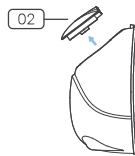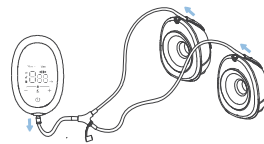

- 02 Remove the diaphragm cover from the front shell.

- 03 Open the collection cup(03) using the tabs at the bottom.

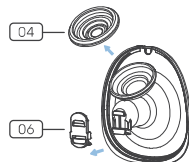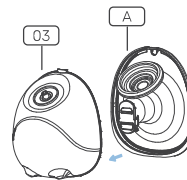- 04 Remove the valve(06) and diaphragm(04) from the flange.

USING YOUR BREAST PUMP

5.1 Flange Sizing

This hands-free breast pump comes with preassembled silicone cushion in 25mm and four additional flange inserts in size of 21mm/19mm/17mm/15mm, so you can find a flange that fits comfortable for you.

- Select the flange size by measuring the diameter of your nipple

Nipple Diameter	Recommended Flange Insert Size
11-13mm	15mm
13-15mm	17mm
15-17mm	19mm
17-20mm	21mm
21mm+above	25mm

Too Small


Correct size

Too Large

USING YOUR BREAST PUMP

5.2 Aligning Your Nipple

- Correct nipple alignment in the Breast Shield is key to comfort, expression and performance.
- Your nipple must be positioned correctly in the flange tunnel before pumping to avoid discomfort.
- Based on your personal preference, The tubing can be connected facing inwards (towards the breasts) or outwards (towards the arms). Ensure that the tubing is not pinched or bent during use and always placed outside your bra.
- The scale on the collection cup indicated you an approximate volume during pumping.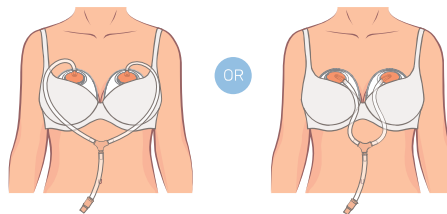


5.3 Placing Collection Cup within your bra

- This Hands-free Breast Pump is designed to be worn with your standard nursing bra and you can use it in either a vertical or slightly tilted position.
- Getting the right compression could impact milk volumes.

USING YOUR BREAST PUMP

- This Hands-free Breast Pump needs to be firmly pressed against your breast, there should be no gaps between the collection cup and your breast.
- Every bra has a different level of support, so make sure to get the compression right before each session. If you are using a particularly structured and inflexible bra, the included Bra Adjuster may be used to create more room.
- Once the buckle of each end is hung on the nursing bra, adjust the length of bra adjuster to make more comfortable.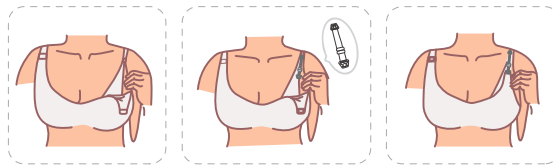

5.4 Starting Your Session

Using Guide

Long press "⏻" button for 1 second when the breast pump at off state to turn it on;

Long press "⏻" button for 1 second when the breast pump at working state to turn it off.

Short press during work to Pause / Start, Long press again to turn off.

USING YOUR BREAST PUMP

Stimulation Mode

Long press "⏻" button for 1 second, the breast pump turns on and enters the Stimulation Mode by default, "⌘" indicator light is on, and the suction level and frequency used in the last time are maintained.

- Press "—", "+" button can adjust the levels of Stimulation Mode, total 9 levels.

Expression Mode

Press "⌘" button when the breast pump is in Stimulation Mode, "⌘" indicator light will light up and switch to the Expression Mode.

- Press "—", "+" button can adjust the levels of Expression Mode, total 9 levels.

2 in 1 Mode

Press "⌘" button when the breast pump is in Expression Mode, "⌘" indicator light will be on and switch to the 2 in 1 mode.

- Press "—", "+" button can adjust the levels of 2 in 1 mode, total 9 levels.

Memory Function

This product comes with memory function, automatic memory of the last used parameters, and the level of all the Stimulation Mode, Expression Mode, 2 in 1 mode can be recorded.

- The level of the Stimulation Mode used last time will be automatically set when the breast pump is turned on.
- The level of the Expression Mode used last time will be automatically set when switching to suction mode.

USING YOUR BREAST PUMP


Child lock

To activate the child lock function, press the “—” and “+” at the same time for 3 seconds, the lock icon will display on the screen. Press again to deactivate.

5.5 Handling And Storing Of Breast Milk


- Carefully remove the collection cups from your bra.
- Handle with care and place on flat surface in an upright position to avoid spilled.
- Collected milk volume can be read by the volume scale markings on the shell.
- Detach the tubing from the collection cup.
- When pouring milk make sure the rounded outer shell to be facing down.
- Pour your expressed milk through the pouring spout on top into a bottle or a container of your choice for storage or feeding.
- When storing milk, always follow the guidance of your lactation consultant or health professional.

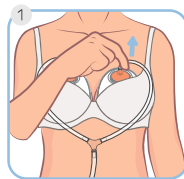Upright Removing

Carefully Take Out

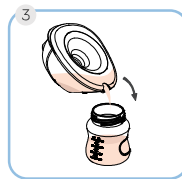Pouring into Bottle

MAINTENANCE

6.1 Battery Maintenance

- The breast pump cannot switch on in case of a completely discharged battery, allow the battery to be charged for at least 15 minutes before power on.
- Make sure that the voltage of the power adaptor(not included) is compatible with the power source.
- Only use the provided charging cable with this breast pump, do not substitute.
- Recharge the battery before it completely discharge.
- Fully charge the battery before storing your breast pump.
- Store your breast pump in a cool place to preserve the battery cycle.

6.2 Breast Pump Maintenance

- Switch the breast pump off when cleaning and maintaining.
When not in use for an extended amount of time, recharge the battery first before use.
- Do not store collection cups in assembled state over an extended period of time.
- Clean the Collection Cups and Pump Motor before storing your breast pump

TECHNICAL SPECIFICATIONS

Product	Hands-Free Double Electric Breast Pump
Dimension	75*118*49mm
Levels	9 Adjustable Levels
Modes	Stimulation / Expression / Mix
Max Suction	36 Kpa
Rated Input	AC 100-240V, 50/60Hz
Rated Output	DC 5V --- 1A, 5W
Display	LED Digital
Timer	30 minutes
Power Supply	DC 3.7V
Battery type /capacity	Li-ion Polymer Battery / 1400mAh
Charging Time	150min
Using Time	Approx. 180min
Charging Port	USB-Type C
Operation Conditions	0-40°C, 15-90%RH, 70-106kPa
Transportation & Storage Conditions	-20-60°C, 15-90%RH, 70-106kPa
Noise	≤50dB(A) (+/-2dB)

**Specification may change without notice by manufacturer for product enhancement.*

TROUBLE SHOOTING

1. No suction power

- a. Check that diaphragm, valve, cover and Tubing Kit are properly assembled;
- b. Inspect the parts for an breakage;
- c. Check whether the power is energized if you can not turn it on.

2. Less suction power

Check that diaphragm, valve, cover and Tubing Kit are properly assembled.

Suggestion: Please adjust suction power from low to high gradually according to the individual actual situation when using the breast pump, the most suitable suction power is that the milk comes out without feeling pain.

Stronger suction is not always better, it might harm the breast tissue, cause breast pain or other discomfort for the user. Therefore, the suction power that suitable for yourself is the best.

3. Milk comes out low

The speed of producing milk is different from person to person, will be affected by the structure of the mammary gland, the patency of the mammary gland, the degree of pleasure and other factors; sometimes the slowness of milking is not a problem with the suction of the breast pump, but may also be related to the user's current health or mood.

Using warm cloth to hot compress the breasts before pumping, and then massage the breast for 5 minutes in the Stimulation Mode, and then switch to the Expression Mode for sucking. You can also massage the breasts by hands to dredge the mammary glands.

TROUBLE SHOOTING

4. Failure to pump milk or breast pain

- a. Distending pain in breast, but no milk comes out after suction, it might causes by mammary gland block;
- b. Mammary gland obstruction will cause mastitis, should go to the hospital in time for examination and medical treatment;
- c. Feeling pain during suction may also be caused by milk mastitis, so timely emptying the breast is beneficial to preventing the occurrence of mastitis.

5. Breast pain during using

Because different people accept different degrees of suction, if you feel pain when using the breast pump, you can relieve it through the following methods:

- a. Adjust the position of the breast shield on the suction kit to ensure that the nipple is right with the mouth of the suction kit, so that the breast shield can massage breasts in a better way to relieve the pain of sucking;
- b. Lower suction level in Expression Mode;
- c. As the body gets used to the suction power and uses the breast pump more often, the pain will decrease gradually.

6. Still have milk can be squeezed by hand after sucking

Because the structure of the human mammary gland is reticulated, and the lactation mother is affected by the increase of prolactin, the mammary gland will keep secreting milk, the existing ways of sucking are difficult to completely suck it clean, so it is normal to use hand to squeeze and still have milk comes out after sucking.

Guangdong Horigen Mother & Baby Products Co.,Ltd.

Add: No. 18, Pingnan Industrial Zone, Mianbei Street, Chaoyang District,
515100 Shantou, Guangdong, PEOPLE'S REPUBLIC OF CHINA

Tel: +86 754 83613668

Website: www.horigen.cn

Fax: +86 754 83843338

Made in China

All pictures shown in the instructions are for illustration only. The actual product may vary due to local differences or product enhancement. HORIGEN reserves the right to substitute any component or accessory with a replacement of equivalent performance.

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