Cover^{BiologIQ} skin protectant

NL - Productinformatie

Het product

Cover, 60 g / 120 g Huidbeschermer

Bestanddelen

Dimethicon, Glycerine, Propyleenglycol, Polyethyleenglycol 400, Polyethyleenglycol 4000, Povidon, Cetostearylalcohol.

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Product description

Cover is a smooth, viscous, greaseless, fragrance-free, allergen-free skin protectant based on Dimethicone. Cover promotes skin repair by optimal protection against moisture-associated skin damage. Cover is also indicated for prevention of skin damage as a result of exposure to environmental factors such as shear forces. Cover adheres well to moist and dry skin and is easy to remove.

Intended purpose

The intended purpose of Cover is to form a mechanical barrier over the skin that protects against moisture-induced damage and shear forces. Cover can be used on intact and damaged skin.

Clinical benefits

- The product protects against moisture-induced skin damage, e.g. the peri-wound skin, skin around catheters, drains, tubes and nasal cannula sites.
- The product protects against aggressive chemicals from e.g. urine, fecal incontinence (including diaper rash), salivation, perspiration and exudation.
- The product protects against skin damage by exposure to friction forces.
- The product prevents dehydration of the skin.
- The product promotes natural skin repair by optimal protection of the skin.

Instruction for use

- Thoroughly cleanse the entire skin area with a suitable cleanser.
- Wash or cleanse hands.
- Unscrew the whole cap from the tube.
- Remove the seal from the tube opening.
- Replace the cap.
- Open the upper part of the flip top cap.
- Remove ointment from the tube and apply a layer of Cover by rubbing gently onto the skin.
- Repeat after each sanitary change or as often as required.

Indications

Cover is indicated for the care of moisture-associated damage of intact or injured skin as a result of frequent fecal incontinence, salivation, perspiration and exudation. It is also indicated for preventing and managing skin irritations like redness and chapping associated with pressure ulcers (up to stage 2) and intertrigo.

User and Use environment

- The product can be used by both lay persons and health care professionals and is suitable for all different patient groups.
- There are no specific user environments applicable.

Contra indications

- Do not use the product when the patient is known to be sensitive to one of the ingredients of the ointment (see Ingredients). Hypersensitivity to one of the ingredients of the ointment can cause redness, irritation and possibly light pain.

Side effects

- In the event of unexpected side effects after use, contact your doctor or the manufacturer. Side effects can cause inconvenience to the patient.
- In case of any serious incident this should be reported to the manufacturer and the competent authority.

Safety precautions

- Cover is for external use only.
- Avoid contact with eyes. If this happens, rinse thoroughly with water.
- Keep out of reach of children.
- Do not use if the tube is damaged as this can lead to contamination of the product with bacteria and to a lower performance of the product.
- Always replace the flip top cap on the tube after use. An open tube could lead to contamination with bacteria and to a lower performance of the product.
- The contents of the tube must be used within 3 months of opening. Longer use could lead to lower functionality and contamination of the product.
- Do not treat different patients with the same tube as this can lead to cross-contamination with bacteria and to a lower performance of the product.
- Do not use the product after the expiration date as this can lead to lower performance of the product and to inconvenience for the patient.

Storage conditions

For best results, store at room temperature. Avoid extreme heat. Do not freeze.

Disposal

Packaging materials and used products can be disposed of as normal waste.

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