
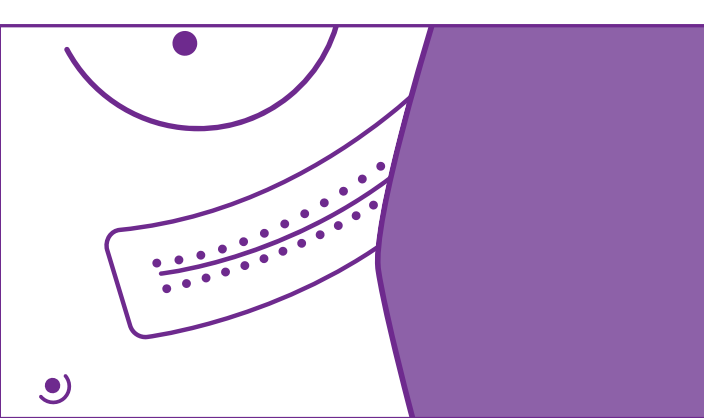
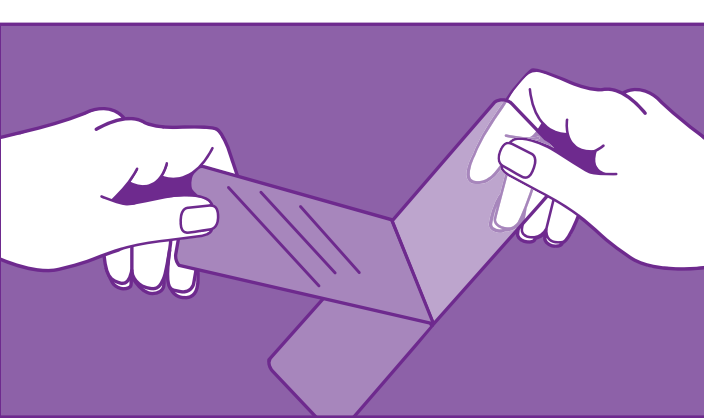
INSTRUCTIONS FOR USE

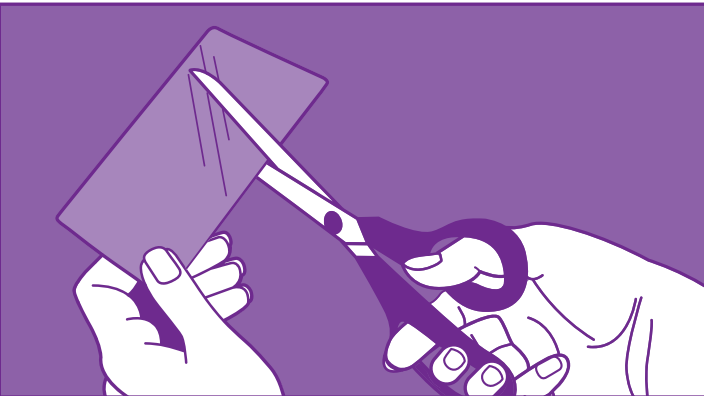ES

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EN

Nombre
BAPSCARCARE T silicone sheets.

Product description
-BAPSCARCARE T is a proven technology for the treatment of scars (hypertrophic and keloid) and for the prevention of abnormal scar development after surgery or injury.
-The BAPSCARCARE T is a self-adhesive, thin, transparent silicone sheet with high adhesive strength, which ensures that it remains securely in place on any area of the body requiring treatment. Suitable for use under clothing.

Composition
-Polyurethane film
-Medical grade silicones

Characteristics
-BAPSCARCARE T silicone sheets ensure that scars become flatter, more flexible and thinner and that complaints (persistent redness, 'pulling' sensations, itching, restriction of joint movement, discomfort and cosmetic concerns) are reduced.
-The silicone sheet creates a semi-occlusive (covering) layer on top of the skin, which results in increased hydration of the skin (epidermis). The increased hydration ensures that damaged, scarred skin recovers more quickly and functional problems (complaints) are reduced.

Indications/applications
BAPSCARCARE T silicone sheets are suitable for:
-both old and new scars
-all skin types
-all ages
-the treatment and prevention of abnormal scars (hypertrophic and keloid)
-the treatment of burn scars
-the treatment of surgical scars
-use under clothing and in compression therapy
the treatment of persistent functional complaints caused by a scar (persistent redness, 'pulling' sensations, itching, restriction of joint movement, discomfort and cosmetic concerns)

Instructions for use
-For an optimal result it is best to start using BAPSCARCARE T silicone therapy immediately after wound closure (after sutures have been removed and any scabs have disappeared).
-If necessary, the BAPSCARCARE T silicone sheet can be cut to size.
-Make sure that the silicone sheet covers the scar and a 2cm-wide area around it.
-To ensure good adhesion and durability of the silicone sheet, it is advisable to remove any hair around the scar location.
-Thoroughly clean and dry the skin before applying the silicone sheet on and around the scar.
-Remove the first part of the protective film of the silicone sheet and carefully apply the self-adhesive silicone sheet, starting at the edge of the scar.
-Remove the second part of the protective film and apply the next section of the silicone sheet to the scar. Depending on the size of the silicone sheet,

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NL

Benaming
BAPSCARCARE T siliconverband.

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