INSTRUCTIONS FOR USE

1	Name Scarban silicone sheets - Elastic and Light Product Description Scarban silicone sheets are a proven technology for treating scars (hypertrophic and keloid) and preventing abnormal scar development after surgery or an accident. The medical silicone sheet is self-adhesive (with the exception of the silicone sheets for fingers), washable and can be used for a long period of time. Scarban Silicone Cleaning Soap is a cleaning agent for silicone sheets. Composition - Textile layer: polyamide, elastane - Medical grade silicones Properties Scarban silicone sheets ensure that scars become flatter, smoother and thinner and prevent complications (like persistent redness, pulling of the scar, itching, restriction of joints movement, discomfort and cosmetic concerns). The silicone sheet provides a semi-occlusive (covering) layer on top of the skin, which results in increased hydration of the skin (epidermis). This increased hydration ensures that the damaged scar skin recovers more quickly and that functional objectives (complaints) are reduced. Scarban silicone sheets provide UV protection as standard (at least 93% UV blocked). Indications/applications Scarban silicone sheets are suitable for: - both old and new scars - all skin types - all ages - treatment and prevention of abnormal scars (hypertrophic and keloid) - treatment of burn scars - treatment of surgical scars - use under clothing and in compression therapy - treatment of persistent functional complaints caused by the scar (persistent redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic concerns)
2	
3	
4	
5	
6	
7	
8	

Instructions for use – Scarban silicone sheets <i>(For instructions on how to use the glove and finger, see below)</i>	Instructions for use – Scarban silicone sheets <i>(For instructions on how to use the glove and finger, see below)</i>
- For optimum results preferably start Scarban silicone therapy immediately after wound closure (after removal of sutures and the disappearance of any scabs).	- It is recommended that Scarban silicone therapy is build up gradually up to the maximum of 23 hours per day (see below). Scarban therapy is gradually to allow the skin to become accustomed to the silicone. For the best possible results, it is preferable that the silicone sheets are used for a minimum of 12 and a maximum of 23 hours per day.
- If necessary, Scarban silicone sheets can be cut to the correct size (fig. 1).	- If necessary, Scarban silicone sheets can be cut to the correct size (fig. 1).
- Ensure that the silicone sheet extends two centimetres beyond the edges of the scar.	- Ensure that the silicone sheet extends two centimetres beyond the edges of the scar.
- Ensure good adhesion and durability of the silicone sheet by removing any hair around the scar site.	- Ensure good adhesion and durability of the silicone sheet by removing any hair around the scar site.
- Thoroughly clean and dry the hand before applying the silicone sheet to the scar and the area around the scar.	- Thoroughly clean and dry the hand before applying the silicone sheet to the scar and the area around the scar.
- Remove the protective film from the back of the silicone sheet (fig. 3) and apply Scarban to the scar (fig. 4).	- Remove the protective film from the back of the silicone sheet (fig. 3) and apply Scarban to the scar (fig. 4).
- If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage (fig. 5).	- If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage (fig. 5).
- After removal of the silicone sheet, the skin may be treated with a hydrating cream.	- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

Instructions for use – Scarban silicone sheet: Glove	Instructions for use – Scarban silicone sheet: Glove
- The Scarban silicone sheet should be used for scarring on the palm of the hand.	- The Scarban silicone sheet should be used for scarring on the palm of the hand.
- To put on the glove, first fold the cuff outwards. This prevents the textile cuff from tearing from the glove when pulling it on and helps the glove to easily onto the hand.	- To put on the glove, first fold the cuff outwards. This prevents the textile cuff from tearing from the glove when pulling it on and helps the glove to easily onto the hand.
- If necessary, the hand (and especially the fingers) can be rubbed with a neutral, water-based lotion/cream in order to allow the glove to slide on easily. This will ensure that the hand slides easily into the glove.	- If necessary, the hand (and especially the fingers) can be rubbed with a neutral, water-based lotion/cream in order to allow the glove to slide on easily. This will ensure that the hand slides easily into the glove.
- Pull the finger sections of the glove over the fingers, twisting movements.	- Pull the finger sections of the glove over the fingers, twisting movements.
- After donning, turn back the cuff of the glove.	- After donning, turn back the cuff of the glove.

Instructions for use – Scarban silicone sheet: Finger	Instructions for use – Scarban silicone sheet: Finger
- The Scarban silicone sheet finger is used for scarring on the fingers.	- The Scarban silicone sheet finger is used for scarring on the fingers.
- First determine the length of the silicone sheet that is needed on the side, lay the finger on the sheet. There are several colored markings on the sheet. If the sheet is too long, the silicone sheet can be cut at the bottom edge up to the colored mark at a 90° angle.	- First determine the length of the silicone sheet that is needed on the side, lay the finger on the sheet. There are several colored markings on the sheet. If the sheet is too long, the silicone sheet can be cut at the bottom edge up to the colored mark at a 90° angle.
- First, the sheet with the silicone layer should be put on the finger.	- First, the sheet with the silicone layer should be put on the finger.
- If necessary, the finger can be rubbed with a neutral, water-based lotion/cream to help it slide easily into the sheet.	- If necessary, the finger can be rubbed with a neutral, water-based lotion/cream to help it slide easily into the sheet.

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EN

1	Name Scarban silicone sheets - Elastic and Light Product Description Scarban silicone sheets are a proven technology on littenks (hypertrophic and keloid) to be handled in or on a surgery of onvegal operation or an accident. The medical silicone sheet is self-adhesive (with the exception of the silicone sheets for fingers), washable and can be used for a long period of time. Scarban Silicone Cleaning Soap is a cleaning agent for silicone sheets. Composition - Textile layer: polyamide, elastane - Medical grade silicones Properties Scarban silicone sheets ensure that scars become flatter, smoother and thinner and prevent complications (like persistent redness, pulling of the scar, itching, restriction of joints movement, discomfort and cosmetic concerns). The silicone sheet provides a semi-occlusive (covering) layer on top of the skin, which results in increased hydration of the skin (epidermis). This increased hydration ensures that the damaged scar skin recovers more quickly and that functional objectives (complaints) are reduced. Scarban silicone sheets provide UV protection as standard (at least 93% UV blocked). Indications/applications Scarban silicone sheets are suitable for: - both old and new scars - all skin types - all ages - treatment and prevention of abnormal scars (hypertrophic and keloid) - treatment of burn scars - treatment of surgical scars - use under clothing and in compression therapy - treatment of persistent functional complaints caused by the scar (persistent redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic concerns)
2	
3	
4	
5	
6	
7	
8	

Instructions for use – Scarban silicone sheets <i>(For instructions on how to use the glove and finger, see below)</i>	Instructions for use – Scarban silicone sheets <i>(For instructions on how to use the glove and finger, see below)</i>
- For optimum results preferably start Scarban silicone therapy immediately after wound closure (after removal of sutures and the disappearance of any scabs).	- It is recommended that Scarban silicone therapy is build up gradually up to the maximum of 23 hours per day (see below). Scarban therapy is gradually to allow the skin to become accustomed to the silicone. For the best possible results, it is preferable that the silicone sheets are used for a minimum of 12 and a maximum of 23 hours per day.
- If necessary, Scarban silicone sheets can be cut to the correct size (fig. 1).	- If necessary, Scarban silicone sheets can be cut to the correct size (fig. 1).
- Ensure that the silicone sheet extends two centimetres beyond the edges of the scar.	- Ensure that the silicone sheet extends two centimetres beyond the edges of the scar.
- Ensure good adhesion and durability of the silicone sheet by removing any hair around the scar site.	- Ensure good adhesion and durability of the silicone sheet by removing any hair around the scar site.
- Thoroughly clean and dry the hand before applying the silicone sheet to the scar and the area around the scar.	- Thoroughly clean and dry the hand before applying the silicone sheet to the scar and the area around the scar.
- Remove the protective film from the back of the silicone sheet (fig. 3) and apply Scarban to the scar (fig. 4).	- Remove the protective film from the back of the silicone sheet (fig. 3) and apply Scarban to the scar (fig. 4).
- If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage (fig. 5).	- If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage (fig. 5).
- After removal of the silicone sheet, the skin may be treated with a hydrating cream.	- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

Instructions for use – Scarban silicone sheet: Glove	Instructions for use – Scarban silicone sheet: Glove
- The Scarban silicone sheet should be used for scarring on the palm of the hand.	- The Scarban silicone sheet should be used for scarring on the palm of the hand.
- To put on the glove, first fold the cuff outwards. This prevents the textile cuff from tearing from the glove when pulling it on and helps the glove to easily onto the hand.	- To put on the glove, first fold the cuff outwards. This prevents the textile cuff from tearing from the glove when pulling it on and helps the glove to easily onto the hand.
- If necessary, the hand (and especially the fingers) can be rubbed with a neutral, water-based lotion/cream in order to allow the glove to slide on easily. This will ensure that the hand slides easily into the glove.	- If necessary, the hand (and especially the fingers) can be rubbed with a neutral, water-based lotion/cream in order to allow the glove to slide on easily. This will ensure that the hand slides easily into the glove.
- Pull the finger sections of the glove over the fingers, twisting movements.	- Pull the finger sections of the glove over the fingers, twisting movements.
- After donning, turn back the cuff of the glove.	- After donning, turn back the cuff of the glove.

Instructions for use – Scarban silicone sheet: Finger	Instructions for use – Scarban silicone sheet: Finger
- The Scarban silicone sheet finger is used for scarring on the fingers.	- The Scarban silicone sheet finger is used for scarring on the fingers.
- First determine the length of the silicone sheet that is needed on the side, lay the finger on the sheet. There are several colored markings on the sheet. If the sheet is too long, the silicone sheet can be cut at the bottom edge up to the colored mark at a 90° angle.	- First determine the length of the silicone sheet that is needed on the side, lay the finger on the sheet. There are several colored markings on the sheet. If the sheet is too long, the silicone sheet can be cut at the bottom edge up to the colored mark at a 90° angle.
- First, the sheet with the silicone layer should be put on the finger.	- First, the sheet with the silicone layer should be put on the finger.
- If necessary, the finger can be rubbed with a neutral, water-based lotion/cream to help it slide easily into the sheet.	- If necessary, the finger can be rubbed with a neutral, water-based lotion/cream to help it slide easily into the sheet.

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