Liberty Puck Handleiding

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Liberty Puck Manual

Prefer to watch the video? Use the QR code on the packaging.

This vibrating element provides the most effective way to wake up people who are known to sleep very deep, are afraid of loud noises, want to be woken up discreetly or are hard of hearing. In addition, the vibrating element is very easy to use as it is wireless.

Charging before use

The vibrating element has a rechargeable battery that needs to be charged before use. To do this, insert the supplied adapter into the opening of the vibrating element. As it charges, the light will pulse orange. As soon as the battery is full, the light will turn a constant green.

Turning the vibrating element on/off

There is an on/off switch on the sliding button of the vibrating element. The vibrating element is turned on when the sliding button is pushed to the right (O). When the battery is (partially) charged, the light will be on.

Tip! Switch the vibrating element off during the day when it's not in use. This will prolong the battery life.

Preparing for use

The vibrating element must be connected to both the orange transmitter and the receiver of the Liberty bedwetting alarm before use. **First, make sure both the orange transmitter and the vibrating element are fully charged.** Then follow the next steps:

1. Press the pairing button (P) at the bottom of the Liberty bedwetting alarm. The LED ring on top of the transmitter will flash a blue light.
2. Press the pairing button (P) at the bottom of the vibrating element. The light on the side of the vibrating element will flash blue.
3. Take the orange transmitter and with a sharp point (e.g., the end of a paperclip) briefly press the small sunken button between the 2 pushbuttons.

The transmitter, receiver and vibrating element are now connected. The flashing of the LED ring on the receiver and the light on the side of the vibrating element will stop and become constant.

Please note!

- > Only continue with step 3 when both the LED ring on the receiver and the light on the side of the vibrating element flash blue.
- > Was the vibrating element purchased at a later date to be connected to your Liberty? In that case, all 3 steps have to be followed as well!

Testing if the pairing worked

It is advisable to test whether the 3 steps described above have been successfully completed. Connect the two push buttons on the orange transmitter with a metal object (e.g., a paperclip or scissors). If a metal bridge is thus formed between the two buttons - and the receiver is switched on - the alarm should go off and the vibrating element should vibrate heavily. Both alarms can be switched off simultaneously by pressing the drop on top of the receiver.

What does the colour of the light mean?

The light on the side of the vibrating element may vary in colour. This is what the colours mean:

- Constant green: Sufficiently charged and ready for use.
- Constant orange: Charge the battery to prevent it from running out at night.
- Pulsating orange: The battery is charging.
- Flashing blue: Pairing mode. The vibrating element can be paired with the receiver and the transmitter.
- Flashing red: There is no contact between the transmitter and receiver.

Important information

- Fully charging an empty battery should take approximately 3 hours.
- The standby time of the battery is approximately 100 hours, but this number of hours will decrease rapidly with use. We advise to charge the battery every day.
- Are you only using the vibrating element and not the sounding alarm? Then, in case of an alarm, switch off the vibrating element with the on/off button to stop the vibration.

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