HIP'SAFE

AIRBAG FOR SENIORS

USER GUIDE / MANUEL D'UTILISATION

Airbag certifié / Certified airbag

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Helite

www.helite.com
hipsafe@helite.com

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USER GUIDE

THANK YOU
FOR YOUR
PURCHASE

To get the most out of your
Hip'Safe airbag, read
this manual carefully.

General Information	19
First Use	20
Safety Precautions	22
Technical Description	23
Reactivating the airbag system	26
Precaution of use.....	29
Maintenance	30
Delivery & After sales service	31
Disclaimer	32

General information

The Hip'Safe package contains: a protection cover, a cartridge, an electronic system and a USB charger.

A Hip'Safe

B USB Cable

C Cartridge

D Belt

E Protection cover

A Hip'Safe: the complete belt protects against hip fractures in case of fall.

D Belt: picture of the belt without the cover. Should not be used alone.

B USB Cable: to recharge the battery of the belt.

E Protection cover: to protect the system. Light, comfortable and washable.

C Cartridge: inflates both airbags in case of fall. It is for single use only and should be replaced after each fall.

FOR WHOM?

Hip'Safe is designed for people aged above 65 years old with mobility troubles and a high risk of falling. The device is adapted for a daily use at home or in a nursing home.

First use

1- Test the belt on your hips

The belt is ergonomic and fits exactly over the hips. It must neither be too tight nor too loose.

Too tight.

Too loose.

Good position.

2- Adjust the size

If the belt does not fit exactly over the hips:

Remove the protection cover entirely.

The adjusting system is on the back of the belt.

Adjust the belt to your size by placing the strap in the good position to increase or reduce the size.

Pull the strap to adjust.

Replace the cover starting from the center.

3. Buckling the belt will turn the system ON

SYSTEM ON- OFF

HOW TO PLACE THE BELT CORRECTLY?

Pull the belt over your hips and close it to activate the system.

Check if the belt is in the right position. (Picto "Up" and the figure are upward)

Turn and center the belt. (Buckle on your back, arrow on your front)

WRONG POSITIONS

The belt is not turned.

The belt is on the wrong side (the arrow must point upwards).

The belt is not centered.

HOW TO KNOW IF HIP'SAFE IS ON THE RIGHT SIDE?

CORRECT

- The symbols "UP" and the figure point upwards.
- A single "beep" when the system is ready to function.

NOT CORRECT

- The symbols "UP" and the figure are backwards.
- A 10 seconds continuous beep.
- Red LED is on.

4. Remove the belt and turn OFF the system

Open the buckle and remove the belt.

That's it! The system turns OFF automatically when you remove the belt.

Never close the belt when you are not wearing it!
(Risk of unexpected triggering)

For further information, watch the tutorial and visit our website: www.helite.com.

Safety Precautions:

Here are some safety instructions for your security:

- Once used, the CO₂ cartridge must be replaced with a new Hip'Safe cartridge of the same capacity (50 cc or 60 cc) (on sale on our website www.helite.com or at our dealers.)
- The belt should be stored carefully when not used. Keep it away from high temperatures (over 45°C) and humidity.
- Failure to follow the instructions could lead to malfunction or reduce the effectiveness of the airbag system.
- Do not wear Hip'Safe under pants.
- Do not wear Hip'Safe directly on the skin.
- Make sure that Hip'Safe is on the right side before using it.
- Do not remove any part of the airbag system or of the cartridge.
- Never close the belt when you are not wearing it (to avoid unexpected triggering)
- Never close the belt when you are charging it (to avoid unexpected triggering)
- Modifications or misuse of the airbag system could decrease dangerously the effectiveness of your system.
- Do not wash the belt, only the cover is washable in a washing machine or by hand.
- The belt is not suitable for sport practice
- Do not use under the shower or in a bath.
- Keep out of reach of children
- The belt is not suitable for people with dementia symptoms or neuro-degenerative diseases (Alzheimer, Parkinson)


Technical Description

THE CONCEPT

Hip'Safe is a wearable airbag that protects the hips in case of fall. When a fall is detected both airbags deploy automatically (in 0,08 seconds) above the hips to ensure an optimal protection before ground impact.

Daily use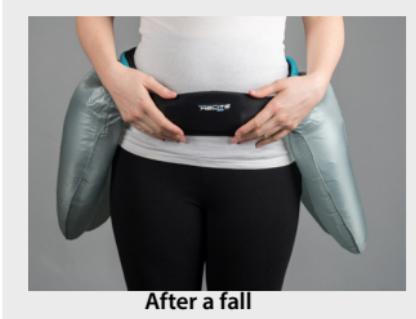

After a fall

ALGORITHM AND ELECTRONIC SYSTEM

The belt includes an electronic system (situated inside the belt behind the CO₂ cartridge and not accessible for the user). It uses a complex algorithm and powerful sensors which analyze more than 1000 times per second people's motion in context and can determine if the person is walking or falling down.

The fall is detected in 200 ms, the airbags inflate in 80ms. The person is fully protected in 280 ms.

AIRBAG PROTECTION


The airbags are made and tested in our workshop in Dijon. They are made of ultra strong nylon which absorbs and resists the shock. When the air cushions are totally inflated, the maximum pressure is about 200 mbar. Then the pressure decreases progressively thanks to a decompression leak.

CO2 CARTRIDGE

The belt is sold with a new and usable CO2 cartridge ready to work. To install a new one, you must screw it directly in the black cylinder.

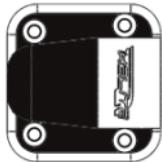Black cylinder

Helite CO2 cartridge

The cartridge is for single use only and must be replaced after each inflation. We advise to store 1 or 2 cartridges and keep them available in the event of fall. It is possible to order CO2 replacement cartridges on our website www.helite.com or at one of our dealers.

CAUTION

- The CO2 cartridges are for single use only and must be replaced after each inflation.
- The CO2 cartridge must be a Helite branded cartridge (GG 50cc / GG 60cc). Do not use any other cartridge to replace the used one.
- The cartridge must be entirely screwed. Never connect the cartridge before screwing it entirely in the black cylinder.
- The cartridge should be stored and used in temperatures below 45°C. Do not leave full cartridges in the car in high temperatures for example.
- Never let the cartridge fall down.
- If rust or other corrosive marks appear on the outer surface of the cartridges, change them immediately and discard the cartridge.
- Never cut the cartridge, do not throw into fire.
- Keep out of children's reach.

TABLE OF SIZE

The belt must fit perfectly over your hips to ensure an optimal protection. Please verify that it is not too tight or too loose. The belt is available in five different sizes (T1, T2, T3, T4 and T5). It is also possible to adjust the belt thanks to the strap under the belt. To find your size, you need to measure your waist (with clothes on) and look at the table below:

	Product size / Taille du produit/ Produktgröße				
	T1	T2	T3	T4	T5
 Waist line Tour de taille Taillenumfang (in centimeters and inches)	82-91 cm (32-36")	90-103 cm (36-40")	101-112 cm (40-44")	110-124 cm (44-49")	122-134 cm (49-53")

BATTERY

Hip'Safe has 1 week autonomy. The system is 100% automatic and consumes little energy : it switches off automatically when the belt is removed or when no moves are detected. You can recharge your belt like any electronical device with any USB cable.

Never close the buckle when the belt is charging.

The belt is unloaded: Plug the charger into the buckle.

The belt is charged and ready for use again.

Low battery:


If a continuous and quick «beep» and a red LED appear when you close the belt that means the battery is low. Battery left: 1 day

Full battery:

When the belt is fully charged, the LED turns green. The charging time is about 3 hours.

PROTECTION COVER

A protection cover is included with Hip'Safe. Other covers with different colors and sizes are available at our online shop: www.helite.com or via our dealers. The cover can be washed and removed easily: following the pressure buttons to take it off.

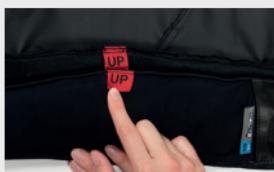Never use the belt without the cover.

Always check that the cover is in the right position:

- The arrow symbols "UP" and the figure must point upwards.
- The "UP" symbols in red and yellow inside the cover must match.

Reactivating the airbag system:

The belt is reusable after each fall as long as the fabric is not damaged. To re-use Hip'Safe you need to fold and store both airbags and change the CO2 cartridge. Here are the steps to follow:

1. Open the belt

Remove the cover entirely.

Open the pocket.

2. Remove the empty cartridge

Disconnect the cartridge.

Unscrew the cartridge.

Remove the cartridge.

3. Deflate the airbags

Compress the airbags with hands to remove the air.

4. Fold and store the airbags

Roll the airbag with your two hands and fold from top to bottom.

IMPORTANT

To roll the airbag: The blue side must be rolled inside and the white side outside.

RIGHT FOLDING
the white part is outside.

WRONG FOLDING
the blue part is outside.

5. Close the airbag pockets

Starting from the corner and following the edge.

Repeat the operation
on the other side.

6. Replace the cartridge

Take a new cartridge.

Pull the wire through the
black cylinder.

Screw the new cartridge
into the black cylinder.

Connect the cartridge.

IMPORTANT

The cartridge must be entirely screwed.

Never connect the cartridge before screwing it entirely in the black cylinder.

YES

NO

7. Close the belt

Close the pocket.

Replace and close the cover.
Starting from the middle and following the pressure buttons.

CAUTION:

A wrong reconditioning could damage the system and the effectiveness of the airbag belt. If you have any doubts about Hip'Safe reconditioning don't hesitate to contact us via email hipsafe@helite.com or via phone +33 03 80 35 48 26 or contact your dealer. You can also watch our tutorials: "How to re-use my Hip'Safe after a fall down ?" on our website: www.helite.com.

Safety Precautions:

CAUTION BEFORE USING THE AIRBAG BELT

If one of these elements does not work, please check your settings for you may not be well protected.

CHECK LIST BEFORE USING THE DEVICE

- ✓ The belt is well adjusted over your hips.
- ✓ The belt is turned, centered and on the right side symbols "Up" and figure upwards and "^" centered.
- ✓ The buckle is closed (mode ON).
- ✓ The belt is charged (green LED on).
- ✓ You hear a short beep when you close the buckle.
- ✓ You see a green LED light.

 The system will be activated 30 seconds after closing the buckle
(time to initiate the different sensors).

CAUTION AFTER AIRBAG DEPLOYMENT

Be aware of these following points once the airbags are deployed:

- The gas will escape gradually through the decompression hole. The sensation of pressure will remain for 5-10 seconds. The airbag will take back its normal shape.
- If any external damage to the airbag garment is caused during deployment, take it to an authorised dealer for servicing. Please note that: in the event of serious damage to the airbag garment, it may not be possible to repair it.

CHECK LIST BEFORE REUSE

- ✓ The CO2 cartridge has been replaced by a new Helite CO2 cartridge.
- ✓ Both airbags have been carefully folded and stored in their pockets.
- ✓ The cover is upward and correctly closed with the pressure buttons.

Maintenance:

GAS CARTRIDGE:

The cartridge comes with a 10 years warranty. The CO2 cartridges have a long life span but we do recommend to check them annually. The check consists in weighing your cartridge on a scale and making sure that the weight is approximately (+/- 3g) equivalent to the weight written on your cartridge.

Example :

a cartridge of 50 CC weighs 182 g : its weight should be between 179g and 185g.
a cartridge of 60 CC weighs 212 g : its weight should be between 209g and 215g.

WASHING

The cover can be washed in the washing machine or by hand washing.

Protection cover

Belt

CAUTION

Do NOT wash the belt (hand washing and washing machine).
It is only possible to use a humid cloth to clean the device.
Washing the belt could seriously damage the electronic system.

STORAGE

Follow the guidelines below to enjoy long and reliable use of your airbag garment:

- Avoid high temperatures (direct sunlight, if left in a vehicle, near a heat source or flame).
- Do not expose to the rain when not wearing it.
- When stored, do not expose to steam or excessive humidity.
- When stored do not cover with anything heavy.
- Do not leave within reach of rodents or other animals.
- Keep out of children's reach.
- Do not close the belt when stored.

RECYCLING

We recycle the empty CO2 cartridge.

Put them aside and send them to the following address:

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21121 Fontaine Lès Dijon

or bring it back to one of our dealers.

Delivery & After sales services

If you have any questions regarding the Hip'Safe airbag, please contact us via email: hipsafe@helite.com or by phone +33 03 80 35 48 26.

DELIVERY

The delivery is made by a private transport service within 48 hours after the order. The belt is stored in its own package. For transport we recommend using cardboard packaging. The device may be controlled by different legislations according to the country. Please gather information in advance when you are travelling with the device.

REPLACEMENT AND WARRANTY

The belt comes with a **2 years warranty**. It is extendible for 2 more years if you register your belt online. If you notice any of the problems described below, consult a dealer or contact our after sales service: hipsafe@helite.com:

- If the airbag garment or the interior materials (fabrics, sheaths) are torn, worn out or the internal bag is visible.
- If any breaks, cracks or corrosion appear on the trigger mechanism or on the gas cartridge.
- In the event of deterioration or significant fading in the materials.

HIP'SAFE MALFUNCTION CASE

What should I do in case of Hip'Safe malfunction ?

ISSUE 1: The belt makes a repeated and a non-compliant sound when it is turned ON

NORMAL CASE: The belt should make a quick beep sound when it is turned ON

POSSIBLE CAUSES	ACTIONS
The battery is low.	Recharge the belt with the USB cable.
The CO2 cartridge is already used.	Replace the CO2 cartridge.
The CO2 cartridge is not plugged in correctly.	Unplug and replug the cartridge.
The belt is on the wrong side.	Check if belt is on the right side: look at the symbols "UP" and the figure on the cover.

If none of these actions help, please contact your dealer or our after sales service: hipsafe@helite.com or call +33 03 80 35 48 26.

ISSUE 2: There is no beep sound when I turn on my Hip'Safe

NORMAL CASE: The belt should make a quick beep sound when it is turned on

POSSIBLE CAUSES	ACTION
The belt is out of battery.	Recharge Hip'Safe with the USB cable included in the belt.

If this action does not help, please contact our after sales service: hipsafe@helite.com or call +33 03 80 35 48 26. and send us back the belt.

Disclaimer :

- This product is "Certified CE (compliance with EU Regulation 2016/425) by ALIENOR CERTIFICATION, Zone du Sanital, 21 Rue Albert Einstein CHATELLERAULT-France-organisme notifié n° 2754"
- **Please note, no body protector can prevent death or serious injuries in certain accidents.**
- Helite will be held in no way responsible for any bodily injury or death, and/or property damage that may occur whilst using any Helite airbag garment.
- Helite accepts no responsibility or liability for any improper installation and usage of Hip'Safe.
- All Helite airbag garments are rigorously tested before being sold. Every airbag garment is inflated with double pressure as normal, however we cannot guarantee the airbag garment will inflate every time.
- Helite cannot be held responsible should the airbag garment not inflate
- The belt does not cover 100% of risks during a fall.
- The belt does not detect 100% of falls. In case of slow fall, the system may not detect it.
- Hip'Safe does not protect parts outside the airbags (The coccyx for example).
- Any alterations, wrong use or wrong storage of the device may cause a malfunction.
- This PPE does not contain any allergenic substances. It does not cause any irritation or discomfort to the user.

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