The Gecko Nasal Pad may help:

- reduce skin breakdown and facial sores
- improve overall mask comfort
- reduce mask leak.

RESMED


RESMED

Small

Solutions made easy

Recommended replacement frequency is every 30 days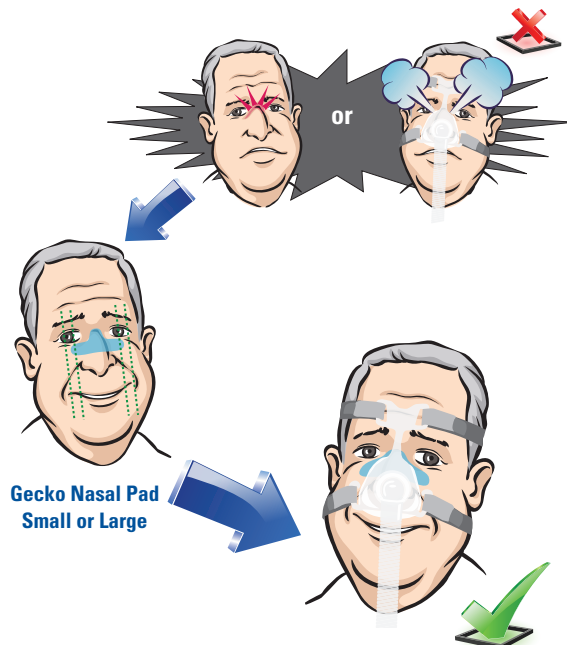

To use:

1. Before each use, wash the Gecko Nasal Pad in warm water using mild soap, then rinse.
2. Place the Gecko Nasal Pad on bridge of nose as shown.
3. Fit your mask as usual.

Intended use

Skin pressure protector device intended for single patient re-use only.

Recommended cleaning

Wash in warm water using a mild soap. Rinse well with drinking quality water and allow to air-dry away from direct sunlight.

Material

Polymer gel composed of USP grade mineral oil. Non-toxic, fully washable, hypoallergenic. Does not contain natural rubber latex; silicone free.

CAUTION

- Do not clean with shampoo
- Do not place in microwave
- Do not place in dishwasher
- Keep product away from open flames
- Keep product away from direct sunlight
- Do not ingest or place in mouth

Suitable for nasal and full face masks

LOT
Batch Code

REF **61912**
Catalogue Number

Temperature limitation
60°C / 140°F
-20°C / -4°F

Humidity limitation
95%


Consult accompanying document

MANUFACTURER
ResMed Ltd
1 Elizabeth Macarthur Drive,
Bella Vista NSW 2153 Australia.
Tel: +61 (2) 8884 1000
Email: reception@resmed.com.au

DISTRIBUTED BY
ResMed Corp
9001 Spectrum Center Boulevard,
San Diego, CA 92123 USA.
ResMed (UK) Ltd
[EC REP] 96 Milton Park, Abingdon, Oxfordshire
OX14 4RY UK.

See www.resmed.com for other ResMed locations worldwide. ResMed products are the subject of granted and pending patents and design registrations. © 2011 ResMed Ltd. R619-308/2 2011-06

Comfort is a key factor in successful therapy

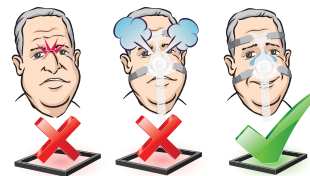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