

LÖWENSTEIN
medical

JULIA




More Room for Dreams.

 With people in mind

Feel free to Dream

JULIA is a small nasal mask that is particularly suitable for side sleepers and glasses wearers. It adapts to many different face shapes and does not require a forehead support. If you want to relax during your evening ritual, you can wear the mask and your glasses at the same time.

Thanks to the slim design and the precisely fitting mask cushion, JULIA sits almost weightlessly on the face, follows every movement and therefore simply proves...that less is more. More freedom and more room for dreams.

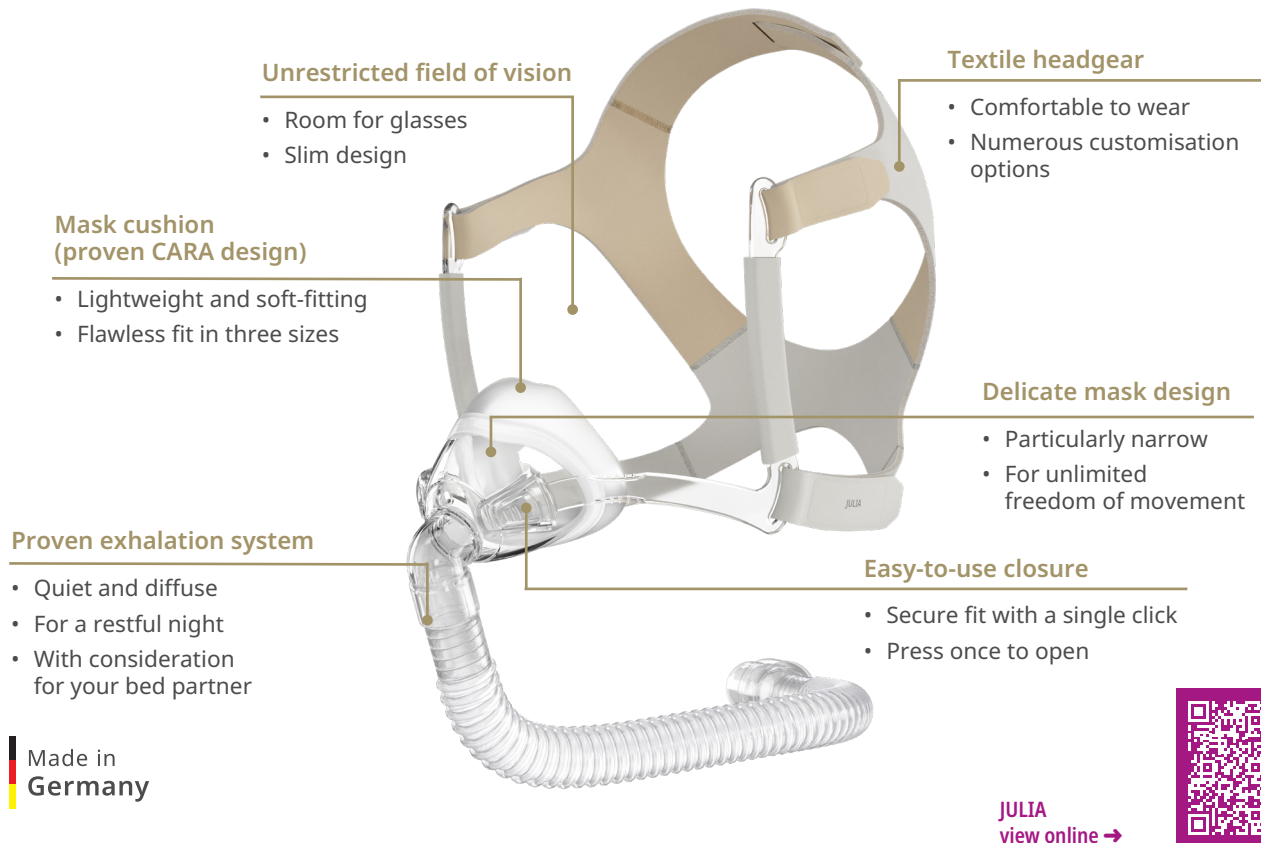

Unrestricted field of vision

- Room for glasses
- Slim design

Mask cushion (proven CARA design)

- Lightweight and soft-fitting
- Flawless fit in three sizes

Textile headgear

- Comfortable to wear
- Numerous customisation options

Delicate mask design

- Particularly narrow
- For unlimited freedom of movement

Proven exhalation system

- Quiet and diffuse
- For a restful night
- With consideration for your bed partner

Easy-to-use closure

- Secure fit with a single click
- Press once to open

Made in
Germany

JULIA
view online →



Flawless fit, slim design

With JULIA, therapy feels completely natural. The slim design offers freedom during the evening routine and while sleeping. The excellent fit has been adopted from the tried and tested CARA model. The mask cushion moulds to your individual facial contours and ensures lasting comfort - regardless of the shape and size of your face.

Unrestricted field of vision for glasses wearers and side sleepers

A new feature of JULIA is the unrestricted field of vision without a forehead support. Reading, watching TV or looking at your mobile phone in the evening is now possible with both a mask *and* glasses at the same time. Side sleepers lie particularly comfortably without a forehead support. Nothing feels tight or shifts in this position.

Comfortable textile headgear

The JULIA headgear ensures that the mask fits securely and, thanks to the soft, skin-friendly materials, feels comfortable against the skin.

Slim mask tube for unlimited freedom of movement

The filigree-like mask design includes a matching mask tube. It is significantly narrower than the usual device tubes and very flexible. It adapts to movement during the night so that sleep is not disturbed.

Easy-to-use closure

The mask closure is easy to use. The clip can be easily inserted into the mask body and closes with an audible click. In the morning, the closure is released by pressing the ribbed side area with a finger.

The sustainable choice

Resource-efficient manufacturing processes

We have achieved a sustainable improvement in the mask headgear, significantly reducing water, energy, and chemical use. We can now integrate fibre production and dyeing, eliminating the need for an additional dyeing process.


With renewable resources

The packaging of JULIA is made from 80% renewable sugarcane.

Easy to recycle

The JULIA mask components are designed to be almost entirely recyclable as single-material parts. We design without material mixes to ensure straightforward recycling at the end of the mask's life cycle.

This results in the following savings¹:

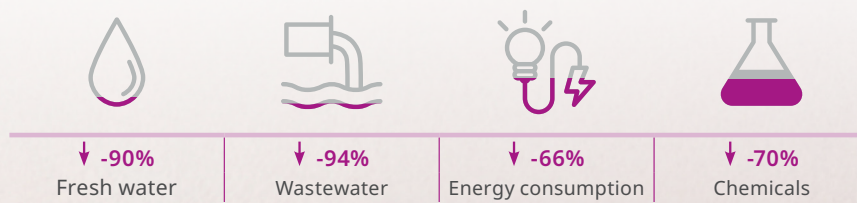


¹ Values determined by the manufacturer

Spare parts



**Headgear JULIA
incl. headgear clips**
standard: LMT 26856
large: LMT 26949

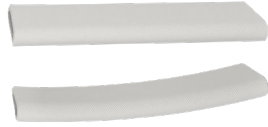

Protective cover 2 units
LMT 17830



Mask cushion JULIA
Size S: LMT 26636
Size M: LMT 26637
Size L: LMT 26638



Mask hose with connections
LMT 26713

Further information on our therapy solutions, accessories, and mask systems can be found at loewensteinmedical.com

JULIA Technical specifications

Product class as per MDR (EU) 2017/745	II a	Therapeutic pressure	4 hPa - 30 hPa
Dimensions (W x H x D)		Temperature range	
• size S	53 x 61 x 75 mm	• Operation	+5 °C to +40 °C
• size M	58 x 60 x 76 mm	• Transport and storage	-20 °C to +70 °C
• size L	62 x 64 x 77 mm	Flow resistance	
Weight		• at 50 l/min	0.6 hPa
• Size S	77 g	• at 100 l/min	2.3 hPa
• Size M	78 g	Declared two-number noise emission value per ISO 4871:	
• Size L	80 g	• Sound pressure level (SPL)	18 dB(A)
Dead space volume		• Sound power level (Lw).	26 dB(A)
• size S	68 ml	• Uncertainty factor	3 dB(A)
• size M	78 ml	Service life (total)	5 years
• size L	92 ml	Service life (in use)	up to 12 months ¹
Conical tubing connector compliant with EN ISO 5356-1	Ø 22 mm (male)	Applied standards	EN ISO 17510: 2020

¹ The service life (in use) depends on the cleaning and the cleaning agent used, the daily wearing time, the therapy pressure and the individual sweat secretion. The use of the second mask cushion (optionally included) does not extend the service life.

Device name	Size	Article no.
JULIA	S	LMT 26710
JULIA	M	LMT 26720
JULIA	L	LMT 26730
JULIA incl. 2nd Mask cushion	S	LMT 26710-1000*
JULIA incl. 2nd Mask cushion	M	LMT 26720-1000*
JULIA incl. 2nd Mask cushion	L	LMT 26730-1000*

* 2nd mask cushion of the same size included in the scope of delivery

CE 0197

Made in
Germany



Printed on
100% recycled paper




Löwenstein Medical Technology
Kronsaalweg 40
22525 Hamburg, Germany
T: +49 40 54702-0
F: +49 40 54702-461
info@loewensteinmedical.de
loewensteinmedical.com

Sales + Service
Löwenstein Medical
Arzbacher Straße 80
56130 Bad Ems, Germany
T. +49 2603 9600-0
F. +49 2603 9600-50
info@loewensteinmedical.com
loewensteinmedical.com

 With people in mind

