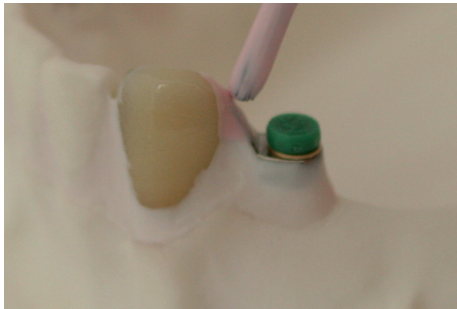
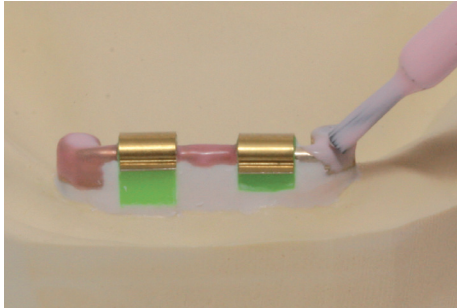
PRECI-SEP

6501 15 ML

AXIAL

ATTACHMENTS

CEKA AXIAL
PRECI-CLIX AXIAL
PRECI-CLIX RADICULAR
PRECI-BALL

EXTRACORONAL ATTACHMENTS

CEKA EXTRACORONAL
PRECI-VERTIX
PRECI-CLIX EXTRACORONAL
PRECI-SAGIX
PRECI-S2

PROFILE ATTACHMENTS

PRECI-BAR
PRECI-HORIX
PRECI-CLIP

DIGITAL LIBRARIES

SPECIALTIES

PRECI-PROFILE
PRECI-POST
CEKA SOL
CEKA SITE
CEKA BOND
3C-BOND
CEKA Multi - CEKA Multi O
PERMA-RET
PRECI-SEP
PLASTICWAX
EXPANDO
MEASURING GAUGE

CEKA PRECI-LINE

SPECIALTIES

PRECI-SEP

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CEKA PRECI-LINE

WWW.CKPL.EU

SIDE EFFECTS, WARNING AND PRECAUTIONS

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat products containing titanium.
Do not use products containing nickel in case of nickel allergy.
The accessories RE H79 and H 35 must be used outside the mouth.

CE0197

Prospectus no 294.M1-ed.G
© Copyright ALPHADENT NV 02/2019
BTW BE 0414.789.321 RPR Kortrijk

ALPHADENT NV
Mannebeekstraat 33,
8790 Waregem, Belgium
T +32 56 629 531
E info@ckpl.eu

WWW.CKPL.EU

PRECI-SEP

PRECI-SEP is a latex-based separator, spacer and release agent for multiple dental applications:

- use as a spacer for precision attachments
- use as a spacer for composite and acrylic provisional restorations
- use as a spacer for customized abutments when modelling with self-cure or light-cure acrylic
- protects against sandblasting and acids
- use as a spacer with vacuum formed trays
- release agent for crowns when pouring a pick-up impression

Apply with a brush and let dry. To speed up drying, use compressed air. The pink colour will become translucent. Use multiple layers when more thickness is desired. Peels off clean.

Use a few drops of distilled water if PRECI-SEP thickens.

CAUTION: contains natural rubber and may cause allergic reactions.

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