VERMEIREN

Eco-Light

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: C, 2015-08

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2015

EN Contents

1. Technical details	2
2. Components	2
3. Explanation of symbols	2
4. Check after delivery	2
5. Intended use	3
6. Fold and unfold the rollator (Fig. 1)	3
7. Assembly	3
8. Pushing handles (Fig. 2)	3
9. Brake system (Fig. 3)	4
10. Basket	4
11. Safety instructions	4
12. Care	5
13. Disinfection	5
14. Repairs / Service / Inspection	5
15. Shipping and storage	5
16. Warranty	5
17. Disposal	5
18. Declaration of conformity	6

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4-wheels rollator Eco-Light

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren		
Address	Vermeirenplein 1/15, B-2920 Kalmthout		
Type	Rollator		
Model	Eco-Light		
Width (unfolded)	600 mm		
Width (folded)	600 mm		
Length (unfolded)	760 mm (with basket) 670 mm (without basket)		
Length (folded)	345 mm (handgrips mounted lowest height) 270 mm (handgrips dismantled)		
Maximum height	950 mm		
Minimum height	820 mm		
Height (folded)	980 mm		
Distance between handgrips	480 mm		
Seat height	550 mm		
Seat width	410 mm		
Seat depth	340 mm		
Weight	7,00 kg	with basket	7,60 kg
Wheels	8 x 1 ¼ (Ø 200 mm)		
Brakes	Multifunctional brake		
Max. inclination forwards	15°		
Max. inclination backwards	7°		
Max. inclination sideways	5°		
Maximum user weight	max. 120 kg		
Maximum basket weight	max. 5 kg		
Turning width	1400 mm		
Storage temperature	+5°C - +41°C		
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / °.			

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the rollator, read the instruction manual carefully: it will familiarise you with this product.


You will also be given hints about the care and service of your rollator. Please remember that observance of our hints would keep your rollator in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

2. Components

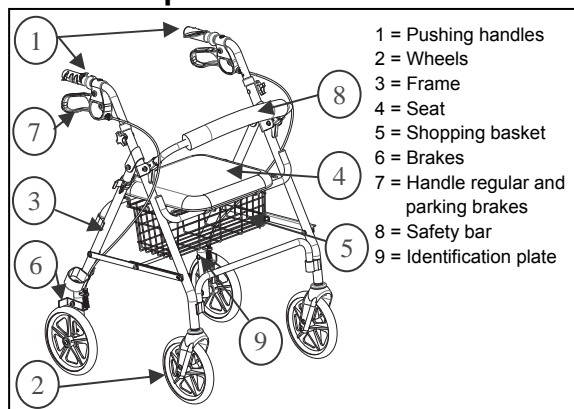3. Explanation of symbols

Maximum mass

CE conformity

Type designation

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- rollator frame with seat, 4 wheels (mounted)
- height adjustable pushing handles with brakes
- shopping basket
- safety bar
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

⚠ WARNING: Risk of falling - Do not use the rollator if you have weak arm function or poor equilibrium.

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide a better stability, support and safety during walking.

However, it cannot be used by persons with a visible weakened hand and/or arm function and also not by persons with a balance problem.

The rollator can be used indoor and outdoor. The rollator should only be used on surfaces where all four wheels are touching the ground. Not to be used on rocky terrain or steep hills (see technical details: max. inclination) so the rollator can become unstable.

The product is not meant for transport of people and/or goods. The seat can only be used when the parking brake is locked.

6. Fold and unfold the rollator (Fig. 1)

⚠ WARNING: Risk of clamping - Keep fingers, buckles and clothes away from the folding mechanism.


1. Folding rollator

- Lift the seat upwards.
- Remove the shopping basket.
- Pull the attachment plates ① a little bit upwards. **Do not place your fingers between the folding mechanism.**
- Push the rear tubes ② forwards against the front tubes to fold the rollator further.
- Fold up the safety bar ③ by pulling these upwards.

2. Unfolding rollator

- Pull the rear tubes ② backwards to unfold the rollator.
- Push the attachment plates ① on both sides downwards until these lock in position.
- Lift the seat upwards, if necessary.
- Place the shopping basket on the horizontal tubes where the seat rest on.
- Turn the seat downwards.
- Unfold the safety bar ③ by pulling these downwards.
- **Adjust the pushing handles to the desired height (see further).**

Fig. 1 – Folding / Unfolding rollator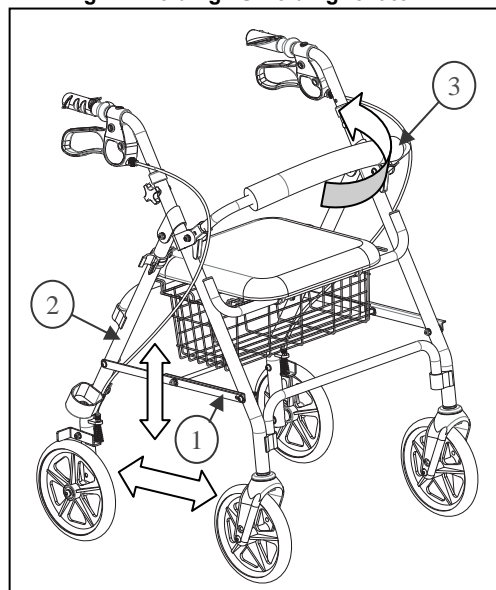

7. Assembly


- Slide the pushing handles in the front frame.
- Unfold the rollator.
- **Adjust the pushing handles to the desired height (see further).**

8. Pushing handles (Fig. 2)

⚠ CAUTION: Risk of falling - Secure the handles firmly before using the rollator.

⚠ CAUTION: Risk of falling - Do not exceed the maximum height adjustment of the pushing handles, your rollator will not be stable anymore.

Fig. 2 – Adjustment pushing handle

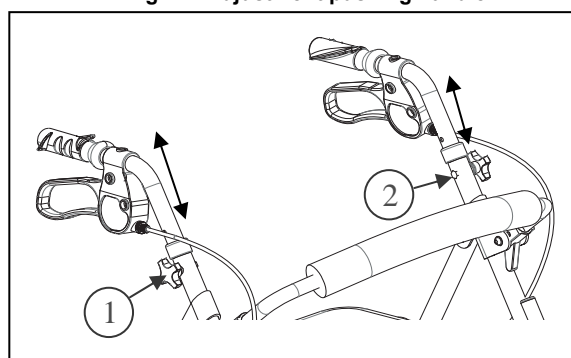The pushing handles can be adjusted in height by following steps:

- Loosen the star button ①.
- Remove the star button ① completely.
- Remove the bolt ② out the tube.
- Adjust the pushing handle to the desired height (7 different positions: steps from 25 mm). When standing straight, the handles should be as high as your wrists. Leave your arms hanging loose.
- Place the bolt ② back in the hole of the tube.
- Place the star button ① on the bolt ②.
- Retighten the star button ① firmly by hand.
- Adjust the other pushing handle with the same instructions.
- Check that the two pushing handles are adjusted with the same height.

9. Brake system (Fig. 3)

⚠ WARNING: Good operation of the brakes is influenced by wear and contamination of the wheels (water, oil, mud, ...) - Check the condition of the wheels before each use.


⚠ WARNING: The brakes can wear - Check the operation of the brakes before each use.

The rollator has been equipped with a multifunctional brake system. The brake handles underneath the pushing handles are regular brakes as well as parking brakes.

For the usage of a short brake (regular brakes), pull the brake handle upwards (toward the pushing handle).

For the usage of a long brake (parking brake), push the brake handle downwards till you hear a locking sound of the brake. The wheels (use both parking brakes left and right simultaneously) are now blocked until you pull the parking brakes upwards again.

Fig. 3 – Pushing handle / brake

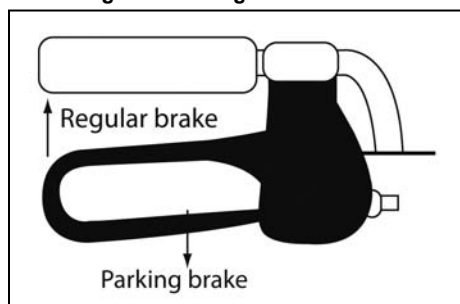10. Basket

⚠ WARNING: Risk of falling, tipping over - Do not overload the shopping basket.

The shopping basket is meant to be mounted on below the seat. Mount the shopping basket with the four clips to the two horizontal tubes were the seat rest on.

The maximum carrying weight from the shopping basket is 5 kg.

11. Safety instructions

- Ensure there are no stones or any other objects blocking the regular usage of the wheels of the rollator.
- Ensure the brakes or the rollator are always in parking-mode before using your rollator. When you are standing comfortable and stable, you may unlock the parking brakes.
- Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- Before every usage, check that the star buttons for your pushing handles are firmly locked.
- Verify if the safety against unwanted folding/unfolding is functioning properly.
- Only proceed moving with your rollator when you are standing comfortably and when both your hands are resting on the pushing handles.
- Do not hold the pushing handles with wet hands. If you do so, you can lose your grip and lose your balance.
- When you detect that your body is getting tired, lock the parking brakes of your rollator and have a seat on the bench.
- **The maximum users weight for your rollator is 120 kg** and can only be used on a flat underground.
 - Do not use your rollator on a staircase or bumps higher than 40 mm.
- When using the rollator outdoors, find yourself flat and stable areas to use your rollator. Avoid curbs and/or deep holes. Find a place where the curbs are bevelled.
- Do not proceed with your rollator into traffic. Always stay on the pavement.
- Avoid bumpy or sandy roads. Otherwise you might lose control over the stability of your rollator.
- Avoid roads with a very steep inclination.

- Do not use your rollator to transport people or items.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

12. Care

It is recommended to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish or the woodwork, consult your specialist dealer. Regular care would ensure that your rollator is preserved in a perfectly functional condition.

13. Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and wood surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: www.rki.de). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

14. Repairs / Service / Inspection

Lifetime of the rollator is influenced by its use, storage, regular maintenance, servicing and cleaning.

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the rollator, it should be serviced according to the following criteria:

- Completeness
- Folding mechanism
- Check the castors (stability, tracking, fixing screws)
- Frames (deformation, stability, connections)
- Pushing handle (safe locking from height adjustment system, good support from the handles)
- Brakes
- Seat, safety bar and shopping bag on wear or damage

- Cleaning
- **Damaged varnish (remove possible rust and repaint)**

Every 8 weeks:

- Lubricating (moving parts shall be greased)

Every 6 months or for every new user, it should be serviced according to the following criteria:

- General review
- Disinfection

15. Shipping and storage

The shipping and storage of the rollator shall be according following instructions:

- Store in a dry place (between +5°C and +41°C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- The rollator must be stored without being subjected to strains. (Do not put too heavy parts on the rollator, not clamping between something, ...)

16. Warranty

Excerpt from the "General Business Conditions".

(...)

5. The terms of the guarantee may differ from country to country. Consult your specialist dealer for the guarantee period for warranty claims.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

17. Disposal

When disposing of the rollator, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

18. Declaration of conformity

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V

Address :

Vermeirenplein 1/15

2920 Kalmthout

Belgium

declares under his sole responsibility that the CE marked devices :

Productgroup: Rollators

Productgroup (GMDN): Walker, foldable (GMDN 37951)

FAMHP (FAGG) registration

number: BE/CA01/11/2-02965-26-CLI

Brand: Vermeiren

Type: 251, 253, 275, 275BIS, 266, 286B, 286D, 286E, Streeter, 201 //
Delta, 4-light, ECO-Light, Quadri-Light

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,

***and is manufactured in full conformity with the European instructions below -
including the latest modifications - and with the national law, that organizes
this directions :***

Medical devices directive MDD 93/42/EEC: 2007

and is in conformity with the relevant European harmonized standards:

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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N.V. VERMEIREN N.V.
Vermeirenplein 1/15
B-2920 Kalmarhout
BELGIUM

SERVICE

The rollator was serviced:
Le rollator roulant contrôlé:
De rollator is gecontroleerd:

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Notes

A series of horizontal dotted lines for taking notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Vermeiren Italia

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.reatime.it
e-mail: info@reatime.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Czech Republic

Vermeiren ČR S.R.O.

Nadrazni 132
702 00 Ostrava 1
Tel: +420 596 133 923
Fax: +420 596 133 277
website: www.vermeiren.cz
e-mail: info@vermeiren.cz

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

Vermeiren Austria GmbH

Schärdinger Strasse 4
A-4061 Pasching
Tel: +43(0)732 37 13 66
Fax: +43(0)732 37 13 69
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Hühnerhubelstraße 59
CH-3123 Belp
Tel: +41(0)31 818 40 95
Fax: +41(0)31 818 40 98
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5
Sant Gregori Parc Industrial Edifici A
17150 Sant Gregori (Girona)
Tel: +34 972 42 84 33
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es