# VERMEIREN

## Jazz S50

USER MANUAL  
MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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*Bedienungshandbuch*

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*Manual de instrucciones*

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This page is intentionally left blank

## Contents

<b>Contents.....</b>	<b>1</b>
<b>Preface.....</b>	<b>2</b>
<b>1 Your product.....</b>	<b>3</b>
1.1 Accessories.....	3
<b>2 Before use.....</b>	<b>4</b>
2.1 Intended Use.....	4
2.2 Explanation of symbols.....	5
2.3 General safety instructions.....	5
<b>3 Using your wheelchair.....</b>	<b>6</b>
3.1 Carrying the wheelchair.....	6
3.2 Mounting/Taking off the rear wheels (if available).....	6
3.3 Folding/Unfolding the wheelchair.....	6
3.4 Mounting or removing of the footrests.....	7
3.5 Operating the brakes.....	7
3.6 Mounting or removing of armrests.....	8
3.7 Transfer in and out the wheelchair.....	9
3.8 Correct position in the wheelchair.....	9
3.9 Riding the wheelchair.....	9
3.10 Moving on slopes.....	9
3.11 Negotiating steps or curbs.....	10
3.12 Transport in the car.....	11
3.13 Use of the wheelchair as seat in a motor vehicle.....	12
<b>4 Installation and adjustment.....</b>	<b>14</b>
4.1 Tools.....	14
4.2 Manner of delivery.....	14
4.3 Adjusting of the footrests.....	14
4.4 Adjust the brakes.....	15
4.5 Pneumatic tyre changing.....	15
<b>5 Maintenance.....</b>	<b>17</b>
5.1 Points of maintenance.....	17
5.2 Maintenance instructions.....	17
5.3 Troubleshooting.....	18
5.4 Expected lifespan.....	18
5.5 End of use.....	18
<b>6 Technical specifications.....</b>	<b>19</b>

## Preface

Congratulations! You are now owner of a Vermeiren wheelchair!

This wheelchair is made by qualified and committed personnel. It is designed and produced according to high quality standards, guarded by Vermeiren.

Please read this manual carefully; it will help you to get familiar with the operation, capabilities and limitations of your wheelchair. If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

### Important note

To ensure your safety, and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

The warranty of this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

### Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

User manual

For user and specialist dealer

Installation instructions

For specialist dealer


Service manual for wheelchairs

For specialist dealer

EC declaration of conformity

# 1 Your product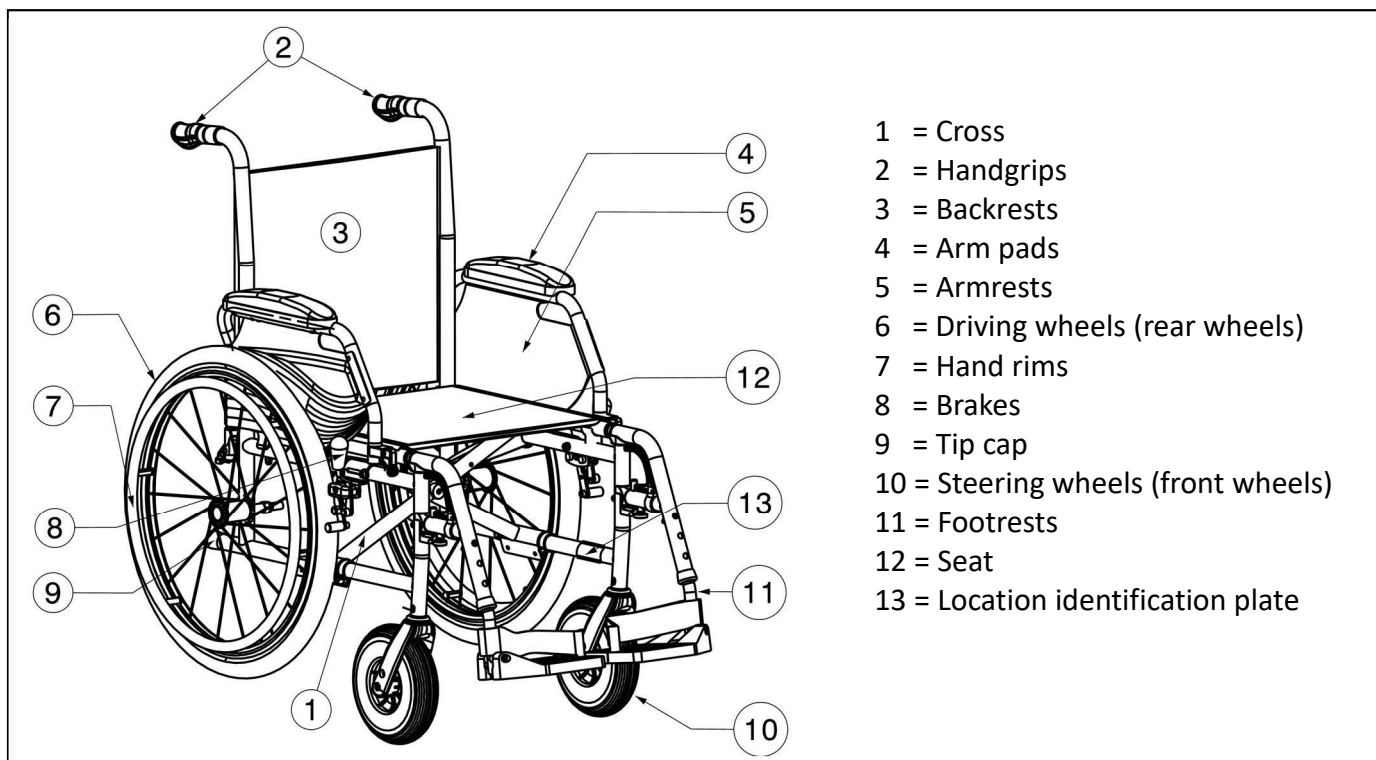

- 1 = Cross
- 2 = Handgrips
- 3 = Backrests
- 4 = Arm pads
- 5 = Armrests
- 6 = Driving wheels (rear wheels)
- 7 = Hand rims
- 8 = Brakes
- 9 = Tip cap
- 10 = Steering wheels (front wheels)
- 11 = Footrests
- 12 = Seat
- 13 = Location identification plate

## 1.1 Accessories

The following accessories are available for the Jazz S50:

- Travel wheels (B86)
- Armrests (B05)

Contact your specialist dealer about other accessories. He will gladly advise you.

## 2 Before use

### 2.1 Intended Use

- The wheelchair is intended for people with walking difficulties or no walking abilities.
- The wheelchair is designed to transport 1 person with a maximum weight of 130 kg.
- The wheelchair is suited for indoor and outdoor use.
- The user can propel the wheelchair by himself or have the wheelchair pushed by an attendant.
- The different types of fittings and accessories, and the modular construction allow full use by persons disabled by:
  - paralysis
  - loss of limbs (leg amputation)
  - limb defects or deformations
  - stiff or damaged joints
  - heart insufficiencies and poor blood circulation
  - balance disturbances
  - cachexia (decrease in muscle)
  - and also for aged persons.
- Your wheelchair should only be used on surfaces where all four wheels are touching the ground and where there is sufficient contact to propel the wheels equally.
- You should practice for use on uneven surfaces (cobblestones, etc.), slopes, curves and to get past obstacles (curbs, etc.).
- The wheelchair should not be used as a ladder, nor is it a transport for heavy or hot objects.
- When used on mats, carpeted floors or loose floor coverings, the floor covering can get damaged.
- Use only Vermeiren approved accessories.
- The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.
- Compliance with the user and maintenance instructions are an essential part of the guarantee conditions.
- Visually impaired people can contact the dealer for the instructions for use.

The wheelchair complies to the requirements set up in:

ISO 7176-8: Requirements and test methods for static, impact and fatigue strengths.

ISO 7176-16: Resistance to ignition of upholstered parts

## 2.2 Explanation of symbols

Maximum mass

Indoor and outdoor use

Maximum Safe Slope

CE conformity

Type designation

## 2.3 General safety instructions

- To prevent injury and/or damage to your wheelchair, make sure that no objects and/or body parts are caught in the spokes of the driving wheels.
- The parking brakes should be applied before getting into and out of the wheelchair.
- When getting into and out of the wheelchair, do not stand on the footplates. These should be folded up beforehand, or the footrests should be swung completely outwards out of the way.
- Investigate the effects of shifting the centre of gravity on the behaviour of the wheelchair, for example on up or down gradients, on laterally sloping ground, or when overcoming obstacles. Obtain support from an attendant.
- If you want to pick up something (lying in front of, on the side, or to the rear of the wheelchair), you should not lean too far out to avoid tipping over.
- When moving through doors, arches, etc. ensure that there is enough room at the sides so that you do not get your hands or arms caught or crushed and that there is no damage to the wheelchair.
- Avoid uncontrolled rolling against obstacles (steps, curbs, doorframes, etc) or dropping down from ledges. The manufacturer cannot assume liability for damage caused by overloading, collision or other improper use.
- Stairs may only be negotiated when aided by another person. If furnishings like drive-on, ramps or lifts are available, use them.
- When moving on the public highway, you are subject to the rules of the road.
- When driving your wheelchair, you should not be under the influence of alcohol as in the case of driving other vehicles. This also applies to indoor driving.
- When travelling outdoors, adapt your driving to weather and traffic conditions.
- To be better visible when driving in the dark, wear the brightest possible clothing or clothes with reflectors, and check that the reflectors mounted on the sides and rear of the wheelchair are clearly visible.
- Be careful when using possible causes of fire such as cigarettes since they may set the seat and back covers alight.
- Never exceed the maximum load of 130 kg.

## 3 Using your wheelchair

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**


The wheelchair is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the wheelchair are given in §4.

### 3.1 Carrying the wheelchair

The best way to carry the wheelchair is to make use of the wheels and roll the wheelchair. If this is not possible (e.g. when the rear wheels are taken off for transportation in a car), firmly grasp the frame on the front and the handgrips. Do not use the foot- or armrests or the wheels to grasp the wheelchair.

### 3.2 Mounting/Taking off the rear wheels (if available)


1. Take the rear wheel and push on button (1).
2. Keep the button pushed in and mount the rear wheels axle until it stops.
3. Release the button (1).
4. Check that the wheel is secured.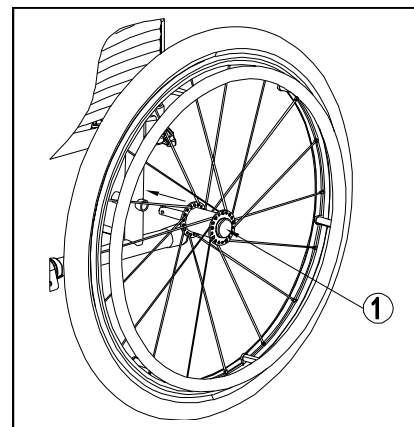

To facilitate the transport of the wheelchair, the rear wheels can be taken off:

1. Make sure the brakes standing in the off position.
2. Take the wheelchair to the side frame where you want to remove the wheel.
3. Press the button (1) in the centre of the wheel hub.
4. Pull the wheel away from the frame.

### 3.3 Folding/Unfolding the wheelchair

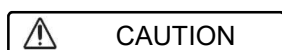CAUTION

**Chance of pinching – Do not place fingers between the components of the wheelchair.**

*To fold:*

1. Fold or remove the footplates (see § 3.4).
2. Take the seat on the front side and backside and pull it up.
3. Push on the handgrips to fold up the wheelchair further.

*To unfold:*

1. Position yourself behind the wheelchair.
2. Use the handgrips to open the wheelchair as much as possible.
3. Position yourself at the front of the wheelchair.
4. Push both seating tubes down until they are fixed in their position.


### 3.4 Mounting or removing of the footrests

CAUTION

Risk of injury

- Check that the footrests are securely locked in place before use.
- Keep objects away from the footrests swing range while mounting or removing the footrests.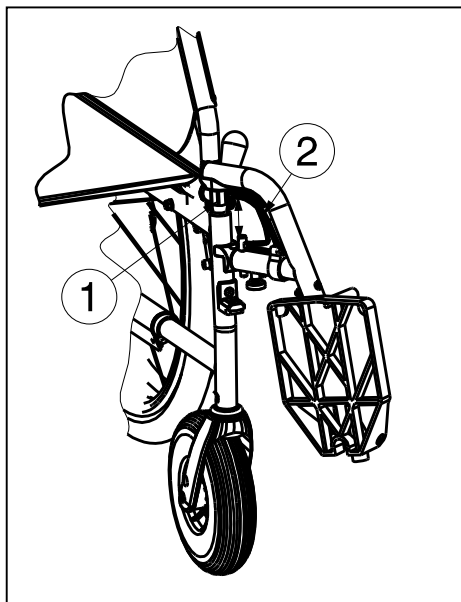

The mounting of the footrests is done as follows:

1. Hold the footrest sideways at the outside of the wheelchair frame and mount the tube hood (1) into the frame.
2. Swing the footrest inwards until it clicks in position.
3. Swing the footplate downwards.

To take off the footrests:

1. Pull lever (2).
2. Swing the footrest to the outside of the wheelchair until it comes loose from the guidance.
3. Pull the footrest from tube hood (1).

### 3.5 Operating the brakes


WARNING

Risk of injury

- The brakes are not used to slow down the wheelchair during movements – Use the brake only to prevent the wheelchair from unintended movements.
- Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...) – Check the condition of the tires before each use.
- The brakes are adjustable and can wear – Check the operation of the brakes before each use.
- Make sure the wheelchair is on a flat horizontal surface before releasing the brakes. Never release both brakes simultaneously.

To apply the brakes:

1. Push the brake levers (1) forward until you feel a distinctive click.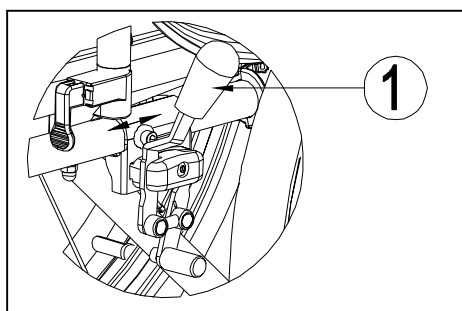

To release the brakes:

1. Release one brake by pulling the lever (1) backwards.
2. Hold the hand rim of the released wheel with your hand.
3. Release the second brake by pulling the lever backwards.


### 3.6 Mounting or removing of armrests

**WARNING**
**Risk of injury**

- Check that the armrests are securely locked in place before use.
- Keep fingers and clothes away from the bottom side of the armrest.


The armrests of the wheelchair can be mounted by following instructions.

1. Mount the rear tube of the armrest in tube hood (1) (Fig A).
2. Make sure the armrest is mounted very well.
3. Fold the armrest forward.
4. Pull the lever (2) on tube hood (3) upwards (Fig B).
5. Click the front tube of the armrest (4) in tube hood (3) (Fig B).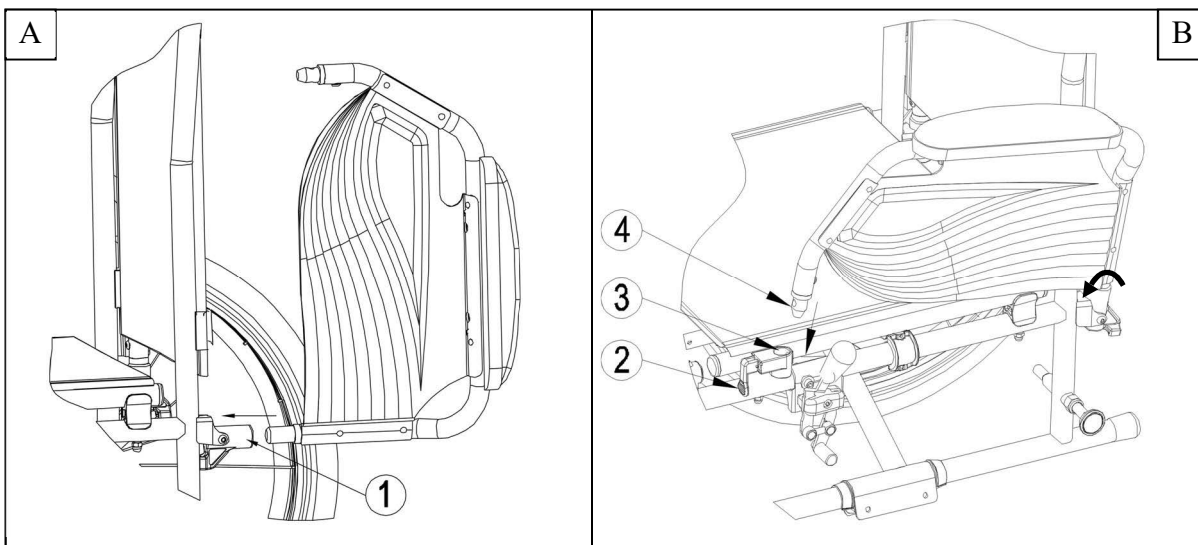

To open and remove the armrest:

1. Pull lever (2) and pull the front of the armrest (1) upwards (Fig C).
2. Fold the armrest backwards.
3. To remove the armrest, pull the rear of the armrest from tube hood (3) (Fig D).

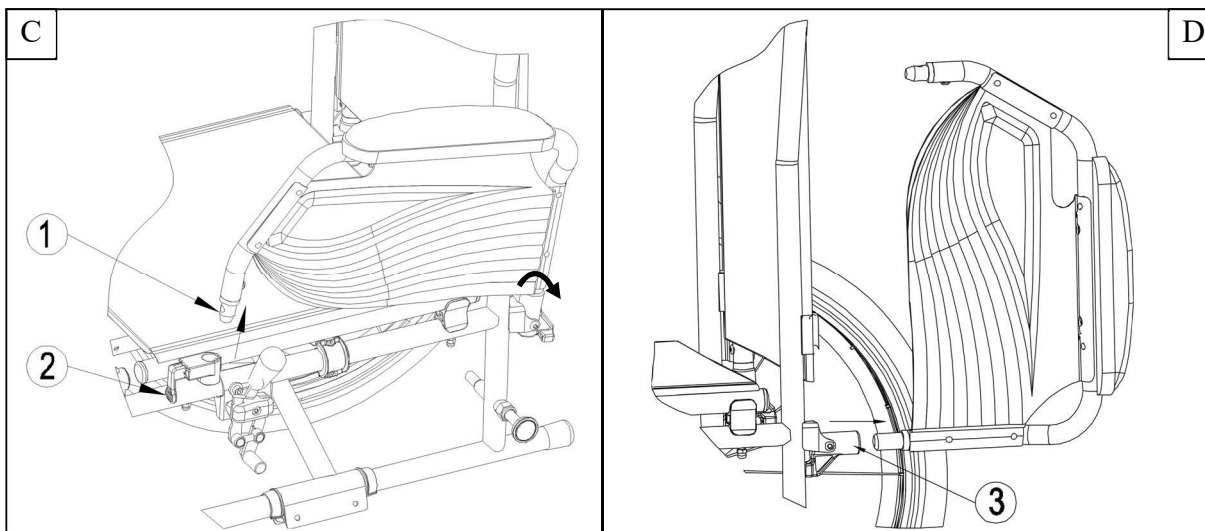
### 3.7 Transfer in and out the wheelchair

WARNING

**Risk of injury and/or damage**

- **In case you cannot perform the transfer in a safe manner, ask someone to assist you.**
- **Do not stand on the foot plates.**

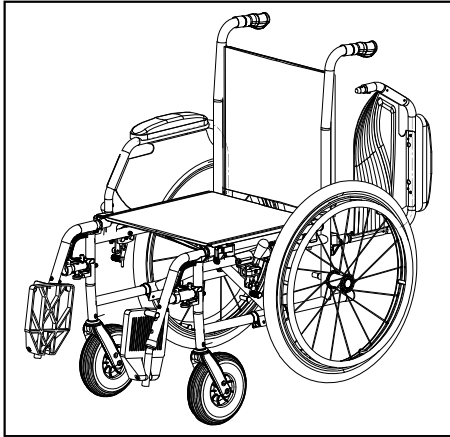1. Position the wheelchair as close as possible to the chair, couch or bed to/from you wish to transfer.
2. Check both brakes from the wheelchair are in the on position.
3. Fold the footplates upwards to prevent standing on them.
4. If the transfer is on the side of the wheelchair, fold the armrest on that side upwards (see § 3.6).
5. Transfer to/from the wheelchair.

### 3.8 Correct position in the wheelchair

Some recommendations for a comfortable use of the wheelchair:

- Position your backside as close as possible to the backrest.
- Make sure your upper legs are horizontal – If needed adjust the length of the footrests (see § 4.3).

### 3.9 Riding the wheelchair

WARNING

**Risk of injury**

- **Prevent your fingers from being caught by the wheels spokes.**
  - **Be careful passing through restricted passages (e.g. doors).**
  - **Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.**
1. Release the brakes.
  2. Take both hand rims at their highest position.
  3. Lean forward and push the hand rims forward until straight arms.
  4. Swing your arms loosely back to the hand rims topside and repeat the movement.

### 3.10 Moving on slopes

WARNING

**Risk of injury**

- **Moving on slopes as slow as possible.**
- **Consider the capacities of your attendant – If your attendant does not have enough force to control the wheelchair, put on the brakes.**
- **Lean forward to move your centre of gravity forward. This improves stability.**

1. If available on the wheelchair, wear the safety belt.
2. Do not attempt moving on too high slopes. The maximum slope angles (upwards and downwards) are mentioned in §6.
3. Ask an attendant to help you moving on the slope.
4. Lean forward to move your centre of gravity forward.

### 3.11 Negotiating steps or curbs

#### 3.11.1 Getting down steps or curbs

**Risk of tipping over – If you do not have enough experience with your wheelchair, ask assistance of an attendant.**

Riding down low curbs can be done moving forwards. Make sure that the footrests do not touch the ground. A practiced user can negotiate small steps or curbs by himself:

1. Bring balance on the rear wheels to reduce the pressure on the front wheels.
2. Negotiate the curbs.

Higher curbs can be taken forward with an attendant:

1. Ask the attendant to tip the wheelchair slightly backwards.
2. Get past the curbs while moving on the rear wheels.
3. Put the wheelchair back on the four wheels.

An experienced user can negotiate higher curbs by himself. This is best done backwards.

1. Turn the wheelchair to have the rear wheels facing the curb.
2. Lean forward to move your centre of gravity forward.
3. Move the wheelchair close to the curbs.
4. Use the hand rim to roll-off wheelchair from the curb in a controlled manner.

#### 3.11.2 Moving up steps or curbs

**Risk of tipping over – If you do not have enough experience with your wheelchair, ask assistance of an attendant.**

Moving up steps or curbs with attendant as follows:

1. Prevent the footrests from touching the curb.
2. Ask the attendant to tip the wheelchair backwards, just enough to move the front wheels over the curb.
3. Lean backwards to move your centre of gravity above the rear wheel.
4. Place the front wheels on the curb.
5. Roll rear wheels of the wheelchair over the curb.

Higher curbs are negotiated backwards:

1. Turn the wheelchair to have the rear wheels facing the curb.
2. Lean backwards and move your centre of gravity above the rear wheels.
3. Ask the attendant to pull the wheelchair on the curb.
4. Take back the normal position in the wheelchair.

An experienced user can negotiate curbs by himself:

1. Drive until the curbs.
2. Ensure that the footrest do not touch the curbs.
3. Lean backwards so you are balancing on the rear wheels.
4. Role the front wheels balancing over the curbs.
5. Bend forwards for more stability.
6. Role the rear wheels over the curbs.

### 3.11.3 Taking stairs

 **WARNING** Risk of tipping over – Taking of stairs shall always with 2 attendants.

Taking of stairs while you staying in the wheelchair shall be according to following rules:

1. Remove the footrests.
2. One attendant tip the wheelchair slightly backwards.
3. The second attendant take the front of the frame.
4. Stay calm, avoid sudden movements and keep your arms inside the wheelchair.
5. Take the steps on the rear wheels of the wheelchair.
6. Mounting the footrests back after taking the stair.

### 3.12 Transport in the car

 **WARNING** Risk of injury

- See that the wheelchair is attached properly to avoid injury from the passengers during collision or sudden braking.
- NEVER use the same seatbelt to attach the wheelchair and passenger.

To transport the wheelchair in the car use following steps:

1. Remove footrests and accessories.
2. Store footrests and accessories safely.
3. If possible, fold the wheelchair and remove the wheels.
4. Place the wheelchair in the luggage place.
5. If the wheelchair and the passenger compartment is NOT separated, attach the frame of the wheelchair securely to the vehicle. You can use the available safety belts in the vehicle.

### 3.13 Use of the wheelchair as seat in a motor vehicle

 WARNING

- **The wheelchair has passed the crash of ISO 7176-19: 2008 and, as such, has been designed and tested for use only as forward-facing seat in a motor vehicle.**
- **The wheelchair's pelvic belt alone is not suited as an occupant restraint belt.**

The wheelchair is tested using the four-point strap-tie system and a 3-point occupant-restraint system. Whenever feasible, use the seat of the vehicle and store the wheelchair in the cargo area.

Steps to secure the wheelchair in a motor vehicle:

1. Check that the vehicle is equipped with a suitable wheelchair tie down and occupant-restraint system, conform ISO 10542.
2. Check that the components of the wheelchair tie down and occupant restraint system are not frayed, contaminated, damaged or broken.
3. If equipped with an adjustable seat and/or back tilt, make sure that the wheelchair user is sitting as upright as possible. If the user's condition prevents this, a risk assessment should be done to evaluate the user's safety during transit.
4. Remove all mounted accessories such as trays and respiratory equipment, and secure them in a safe place.
5. Position the wheelchair facing forward in the travelling direction, centrally between the tie-down rails mounted in the floor of the vehicle.
6. Mount the front securement straps according to the instructions of the strap-system manufacturer at the indicated place. (figure 1)  
This place is marked on the wheelchair with a symbol. (figure 2)
7. Roll back the wheelchair until the front straps are tight.
8. Apply the wheelchair brake.
9. Mount the back securement straps according to the instructions of the strap-system manufacturer at the indicated place. (figure 1)
10. This place is marked on the wheelchair with a symbol. (figure 2)

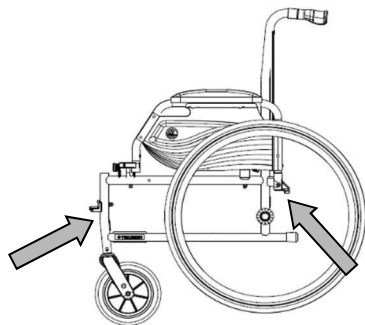
Figure 1

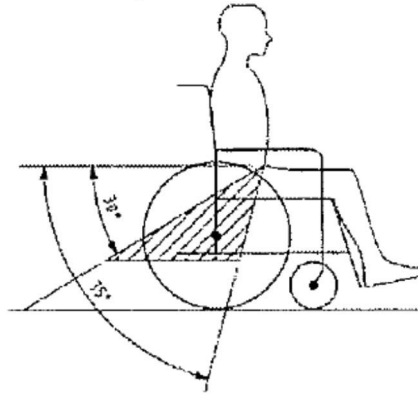
Figure 2


### Steps to secure the wheelchair user:

1. Remove both arm rests.
2. If present, attach the wheelchair's pelvic belt.
3. Attach the occupant restraint belts according to the instructions of the strap-system manufacturer.
4. Wear the pelvic belt low across the front of the pelvis, so that the angle of the pelvic belt is within the preferred zone of 30° to 75° to the horizontal, similar to that shown below.

5. A steeper (greater) angle within the preferred zone is desirable.
6. Adjust the belt tightly according to the instructions of the strap-system manufacturer, consistent with the user's comfort.
7. Ensure that the restraint belt connects in a straight line to the anchor point in the vehicle and that no bends in the belt are visible, for instance at the axle of the rear wheel.
8. Install the arm rests, if desired. make sure that belts are not twisted or held away from the body by wheelchair components such as arm rests or wheels.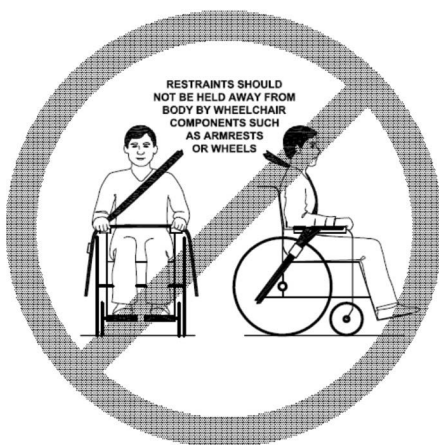

## 4 Installation and adjustment

The instructions in this chapter are for the specialist dealer.

The Vermeiren Jazz S50 has been designed as wheelchair with a minimum of adjustments. There is no need for extra stock of spare parts.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility.

- Use only the settings described in this manual.
- Variation of allowed adjustments can still change the stability of your wheelchair (tilt back or sideways).

### 4.1 Tools

To set up the wheelchair the following tools are needed.

- Wrench set n° 7 to n° 22
- Allen key set n° 3 to n° 8
- Screwdriver Phillips head

### 4.2 Manner of delivery

The Vermeiren Jazz S50 shall be delivered with:

- 1 frame with armrests, rear and front wheels (seat height of 500 mm)
- 1 pair footrests
- Tools
- Manual
- Accessories (optional)
- Tyre pump


### 4.3 Adjusting of the footrests

**Risk of damage – Avoid that the footrests make any contact with the ground. Keep a minimum distance from 60 mm above the ground.**

Adjust the length of the footrests as follow:

1. Remove the screw (1).
2. Adjust the length of the footrest to a comfortable length.
3. Tighten the screw (1) properly.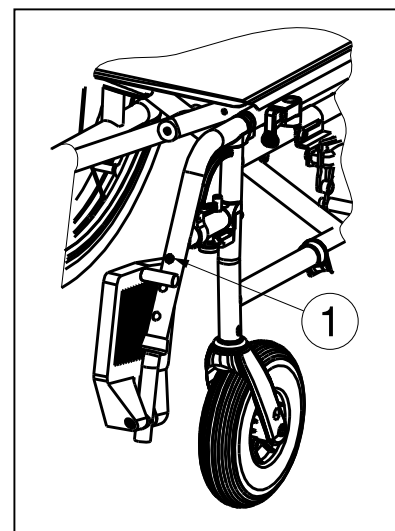


## 4.4 Adjust the brakes

 **WARNING** Risk of injury – Brakes may only be adjusted by your specialist dealer.

Adjust the brakes according to the following rules:

1. Install the wheels according to § 3.2.
2. Disconnect the brakes by pulling lever (1) backwards (*Fig A*).
3. Loosen bolt (2) so the brake mechanism can slide over the frame (3) (*Fig B*).
4. Pull the brake mechanism over the frame (3) to the desired position (*Fig B*). The adjustment of the brakes shall be in the range of the marked dashes (4) on the brake mechanism.
5. Retighten the bolt (2) (*Fig B*).
6. Check working of the brakes.
7. If necessary repeat the above steps until the brakes are adjusted well.

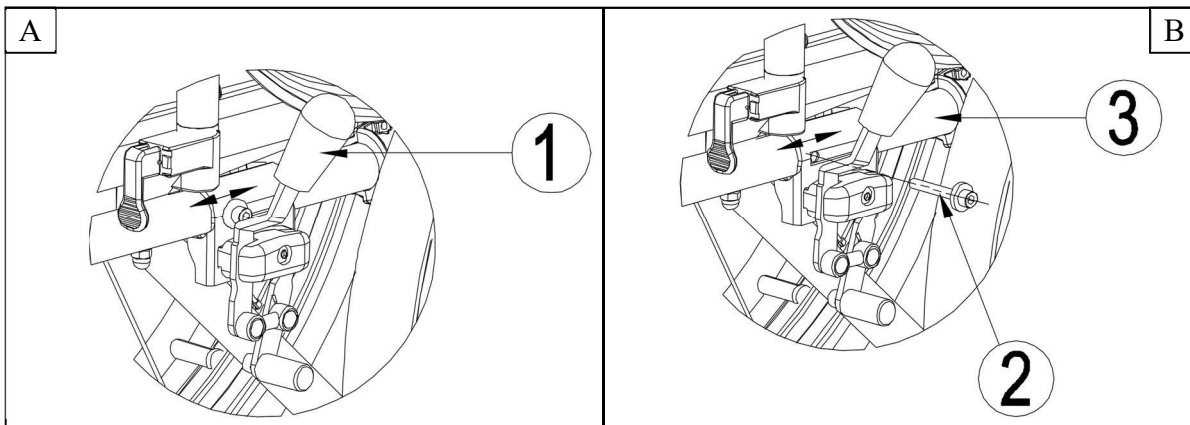## 4.5 Pneumatic tyre changing

 **CAUTION** Risk of injury and/or damage

- If handled improperly, the rim might be damaged. Rather let an expert perform these activities.
  - When mounting a tyre, make sure no objects or body parts are pinched between the tyre and the rim.
  - When inflating the tyres, always check that the pressure is correct. The correct pressure is given on the tyre walls.
  - Use only inflating equipment which complies with regulations and indicates the pressure in bar, or use the supplied air pump. We do not accept any liability for damage caused by inflating equipment not supplied by the manufacturer.
- i**
- Always make sure that the wheels are sufficiently inflated, since it could affect the driving behaviour.
  - We do not accept any liability for wheels not supplied by the manufacturer.
  - Only an expert can guarantee correct assembly. Work not done by your specialist dealer, would void the warranty of claims.

If you want to change the tyres or inner tubes, you should note the following:

1. Let all the air out of the inner tube.
2. Insert a tyre lever between the tyre and the rim.
3. Slowly and gently push the tyre lever downwards. This will pull the tyre over the edge of the rim.
4. Then move the lever along the rim, the tyre will jump out.
5. Now carefully remove the tyre from the rim and then remove the tube.

Before replacing the tyre, check the rim bed and the inside wall of the tyre for foreign matter and clean these properly if necessary. Check the condition of the rim bed, especially around the position of the air valve.

Please use only genuine original replacement parts. We do not accept any liability for damage caused by non-genuine parts. Kindly contact your specialist dealer.

Place the rim belt in position over the air valve before inserting it into the rim. Then the rim belt can be pulled over easily. Check that all spoke heads are covered (a rim belt is not required in the case of a plastic rim).

Push the tyre over the edge of the rim, starting behind the air valve. Inflate the tube slightly until it is round, and place it inside the tyre.

If the tube fits snugly inside the tyre without any folds (in the case of folds: let out some air), then the upper side of the tyre can be pressed lightly onto the rim with both hands, starting at the air valve.

Check all around on both sides that the tube is not pinched between the rim and the edge of the tyre. Lightly push the air valve inwards and pull it out again to make sure that the tyre is positioned properly in the region of the air valve.

To ensure that the wheel inflated correctly, admit only so much air initially that the tyre can still be easily pushed inwards by using your thumbs. If the check-lines are equidistant from the edge of the rim on both sides of the tyre, then the tyre is centred properly. If not - let out the air and position the tyre afresh.

Now the tyre can be inflated to its full operating pressure (note the maximum!!) and the valve cap should be replaced.

## 5 Maintenance

### 5.1 Points of maintenance

Risk of injuries and damage

Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.

**i** The service frequency depends on the frequency and intensity of use. Contact your dealer to agree to a common timetable for inspection/maintenance/repair.

#### *Before each use*

Inspect the following points:

- All parts: Present and undamaged or unworn.
- All parts: Clean, see § 5.2.2.
- Wheels, backrest, seat, calf rests, arm rests, footplate: Well secured.
- Condition of wheels/tyres, see § 5.2.1;
- Condition of frame parts: No deformation, instability, weakness or loose connections
- Seat, backrest, arm pads, calf pads and head rest (if applied): No excessive wear (like dented spots, damage or tears).
- Brakes: Undamaged and functioning

Contact your specialist dealer for possible repairs or part replacements.

#### *Yearly or more often*

Have your wheelchair inspected and serviced by your specialist dealer, at least once a year, or more often. The minimum maintenance frequency depends on use and should therefore be commonly agreed on with your specialist dealer.

### 5.2 Maintenance instructions

#### 5.2.1 Wheels and tyres

**i** Proper working of the brakes depends on the state of the tyres, and can change due to wear and contamination (water, oil, mud, ...).

Keep your wheels free of wires, hair, sand and fibres.

Check the profile of the tyres. If the tread depth is less than 1 mm, the tyres need to be replaced. Contact your specialist dealer for this matter.

Inflate each pneumatic tyre to the correct pressure (see pressure indication on the tyres).

### 5.2.2 Cleaning

CAUTION

Risk of damage by moisture

- Never use a hose or high-pressure cleaner to clean the wheelchair.

Wipe all rigid parts of the wheelchair with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

The upholstery can be cleaned with lukewarm water and a mild soap. Do not use abrasive cleaning agents to clean.

### 5.2.3 Disinfection

CAUTION

Risk of damage

- Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

## 5.3 Troubleshooting

Even if you use your wheelchair properly, a technical problem may occur. If this is the case, contact your local specialist dealer.

WARNING

Risk of injuries and damage to the wheelchair

- NEVER attempt to repair the wheelchair yourself.

The following symptoms may indicate a serious problem. Therefore always contact your specialist dealer if you spot any of the following deviations:

- Strange sound;
- Uneven tread wear on one of the tyres;
- Jerky movements;
- Wheelchair deflects to one side;
- Damaged or broken wheel assemblies.

## 5.4 Expected lifespan

The average lifespan of your wheelchair is 5 years. Depending on the frequency of use, driving circumstances and maintenance, the lifespan of your wheelchair will increase or decrease.

## 5.5 End of use

At end of life, you need to dispose your wheelchair according to the local environmental legislation. The best way to do so, is to disassemble the wheelchair to facilitate the transport of recyclable parts.

## 6 Technical specifications

The technical details below are only valid for this wheelchair, at standard settings and optimal ambient conditions. Take these details into account during use.

The values are no longer applicable if your wheelchair has been modified, damaged or suffers from severe wear.

*Table 1: Technical specifications*

<b>Make</b>	<b>Vermeiren</b>
<b>Address</b>	<b>Vermeirenplein 1/15, B-2920 Kalmthout</b>
<b>Type</b>	<b>Manual wheelchair</b>
<b>Model</b>	<b>Jazz S50</b>
<b>Maximum occupant mass</b>	<b>130 kg</b>

Description	Min.			Max.		
Overall length with footrest	1120 mm					
Effective seat width	390 mm	420 mm	440 mm	460 mm	480 mm	500 mm
Overall width (depends on the seat width)	565 mm	595 mm	615 mm	635 mm	655 mm	675 mm
Folded length without footrest	825 mm					
Folded width	290 mm					
Folded height	935 mm					
Total mass	16,60 kg					
Mass of heaviest part	8,05 kg					
Masses of parts that can be dismantled or removed	<b>Footrests: 1,80 kg; Armrests: 1,80 kg; Rear wheels: 4,95 kg</b>					
Static stability downhill	15°					
Static stability uphill	10°					
Static stability sideways	15°					
Obstacle climbing	60 mm					
Seat gap	30 mm					
Effective seat depth	450 mm					
Seat surface height at front edge	500 mm					
Seat angle	5°					
Backrest height	420 mm					
Backrest angle	0°					
Distance between footrest and seat	370 mm			440 mm		
Angle between seat and footrest	7°					
Distance between armrest and seat	220 mm					

Front location of armrest structure	310 mm	
Hand rim diameter	535 mm	
Horizontal location of axle (deflection)	10 mm	
Minimum turning diameter	1820 mm	
Diameter Rear wheels	24" (optional 12")	
Tyre pressure, rear (driving) wheels	Max. 3.5 bar	
Diameter Steering wheels	200 mm	
Tyre pressure, front (steering) wheels	Max. 2.5 bar	
Storage and use temperature	+ 5 °C	+ 41 °C
Storage and use humidity	30%	70%
<b>We reserve the right to introduce technical changes. Measurement tolerance <math>\pm 15</math> mm / 1,5 kg / 1,5°</b>		

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:
By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:
By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:
By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:
By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:	By (stamp):           Kind of work: I / S / R / D Date:

Vermeiren GROUP  
Vermeirenplein 1 / 15  
B-2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)

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### **All rights reserved, including translation.**