D-Lite

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Parts of Transport wheelchair

- | | |
|------------------------------|---------------------|
| 1 Push handle | 7 Quick Release pin |
| 2 Back canvas | 8 Parking brake |
| 3 Handgrips hight adjustment | 9 Castor wheel |
| 4 Half-fold mechanism | 10 Leg rest |
| 5 Armrest | 11 Anti-tipper |
| 6 Rear wheel | |

Introduction

The transport-wheelchair D-Lite been designed with elegance and quality in mind. It is suitable for occasional or frequent use, and can be used outdoors and indoors.

It's aluminium frame is lightweight and durable. The transport-wheelchair is designed for a single user of up to 125 kg (19 stone).

The transport-wheelchair comes with 12,5" or 24" wheels which allow either the user or an attendant to propel the transport-wheelchair.

All D-Lite transport-wheelchairs are manufactured to the highest quality and are CE compliant.

Drive Medical is an DIN EN ISO 13485: 2012 certified medical device manufacturer.

Indikation

Inability to walk or severely restricted walking ability as part of the basic need to move around in your own home and to leave your home to get some fresh air on a short walk or to reach the places usually in the surrounding area of your home where everyday business can be carried out.

Provision of transportable wheelchairs is advised if disabled persons need to put their wheelchair e.g. in a car themselves unaided or if the accompanying persons of disabled persons are not in the position to transport a standard wheelchair, if applicable including with removable wheels, due to its high weight.

The transportable wheelchair exclusively serves people who are not able to walk and people who have problems with walking with their individual requirements when travelling themselves and travelling with others for everyday use, both inside and outside on even and uneven ground.

The use of the transportable wheelchair is not suitable for:

- Disorders of perception
- Severe loss of balance
- Loss of limbs to both arms
- Joint contractions/joint damage to both arms
- Inability to sit down

Important Safety Information

Before Using the transport-wheelchair read the safety notices below:

- Use slow speeds on gradients. Do not exceed the maximum gradient stated.
- The transport-wheelchair is only suitable for single occupancy.
- Keep your feet on the footplates when moving. Do not stand on the footplates.

- Do not use escalators.
- Do not reverse down a gradient
- Maintain proper balance at all times. Users should not move their centre of gravity out of the seating area.
- Do not reach for items further than your arm will extend.
- Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc.
- We recommend you consult your healthcare professional for advice about transferring to and from the transport-wheelchair. The parking brakes should always be applied when transferring.
- The transport-wheelchair brakes and parking brakes are not suitable for slowing the transport-wheelchair down.

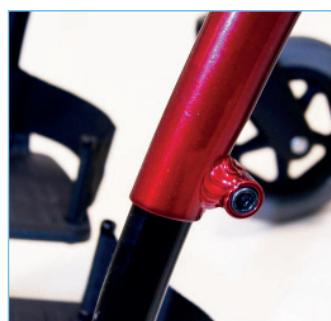
Adjustment of Armrest

The angle and front height of the armrests can be adjusted to suit each user. To adjust the angle, press the button shown in the photograph whilst pulling up or down on the front of the armrest. When the desired angle is reached, release the button to lock.

Adjustment of Legrest

The length of the leg rests can be adjusted. To adjust the length of the leg rest, first loosen the allen bolt on the legrest using the allen key provided. Then slide the footplate in and out until the desired position is reached. Finally retighten the allen bolt to secure.

Folding back mechanism

The transport-wheelchair has a half fold back mechanism which has been designed so that the transport-wheelchair will fit in to small spaces by reducing the wheelchair height.

To fold the back, push in the black lever using your thumb, whilst pulling the push handle down. Then repeat for the other side.

Warning:

Do not pull on the black lever as this could result in damage and/or injury.

Handgrips height adjustment

The D-Lite is equipped with a sliding handle for stepless height adjustment. Loosen the T-handle knob for adjusting the needed height. After you have reached the needed height, please tighten the T-handle knob again.

Disassembly and folding

The leg rests, rear wheels and cushions are easily removable without tools to facilitate a light carry weight and minimize storage space required.

Assembly of the Legrest

The legrests can swing in / out and are removable for convenience.

Pull the button (arrowed) on to unlock the legrest and then swing the legrest in or out, as required.

To remove the legrest, swing the legrest out approximately 45° and pull up on the leg rest.

To reinstall the leg rest, insert the leg rest at 45° then swing the legrest inwards until it locks in position.

Warning:

Always ensure the legrest is properly reinstalled and locked in position before use.

Assembly/Disassembly of the 24"-rear wheels

The 24"-rear wheels on the transport-wheelchair can be easily removed for transportation or storage.

With one hand holding up the transport-wheelchair frame, the other hand should grip the centre of the wheel, push the centre pin inwards and in the same movement pull the wheel hub away from the wheel.

To put the wheels back the procedure should be reversed.

The 12,5"-wheels are removable only with tools.

Warning:

Check that the wheel is securely fastened before use by pulling the wheel and ensuring there is no sideways movement.

Folding the transport-wheelchair

The transport-wheelchair has been designed with an easy fold mechanism. To fold the chair, ensure that the legrests are removed or the footplates flipped up. Remove the seat and back cushions and remove the rear wheels if required. Pull up in the centre of the front and rear of the seat canvas and the transport-wheelchair will naturally fold up.

How to Use

Either the transport-wheelchair user or an attendant can manually propel the wheelchair.

If the user is propelling the wheelchair, they should use the handrims attached to the rear wheels. The user can slow the wheelchair by applying pressure to the handrim using their hands.

The transport-wheelchair should not be propelled by the user in environments where the user cannot stop the wheelchair promptly and comfortably.

If an attendant is propelling the transport-wheelchair, they should push using the push handles.

Brakes

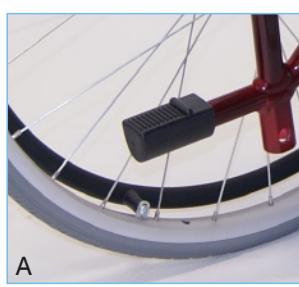
The transport-wheelchair comes with right and left parking brakes for both variants (12,5" and 24" wheels).

To apply the brakes, push down on the brake handle (shown by blue arrow).

To unlock the brakes, pull up on the brake handle (shown by white arrow).

Brake for attendant only 12.5 "version

Using the tip assist (See picture A)

Use the stepper tube to raise the front castors (when mounting a kerb for example). To use, push down on the stepper tube with a foot.

Warning:

Do not raise the front castors by pushing down on the push handles as this could result in damage to the transport-wheelchair.

To mount a kerb.

Approach the kerb head on. Then the attendant uses the stepper tube to raise the front castors, and lowers

the front castors on the raised kerb. Finally the attendant should push the transport-wheelchair forward, lifting it up slightly to mount the kerb if required.

To go down a kerb.

Line up the front castors with the edge of the kerb. The attendant uses the stepper tube to raise the front castors and tip the user slightly back. Keeping the castors raised, slowly lower the transport-wheelchair down the kerb.

Warning:

Do not Lift the front wheels while pushing down the push handles on, as this may cause damage to the transport-wheelchair.

Use in a vehicle

The transport-wheelchair user should transfer to a normal vehicle seat and the transport-wheelchair should be safely stored away. The transport-wheelchair has not been designed to be used as a seat in a motor vehicle, and failure to follow the above instructions could result in injury.

Tyre pressure and inflation

The D-Lite transport-wheelchair comes with solid tyres on both front and rear wheels, and hence there is no need to monitor tyre pressure.

Care and maintenance

Before each use of the transport-wheelchair, the brakes should be checked.

The transport-wheelchair should be stored in a dry environment, away from direct sunlight. When in storage the transport-wheelchair can be folded up.

The transport-wheelchair should be kept clean and dust free. This can be done with a duster or damp cloth.

The transport-wheelchair comes with a toolkit to perform simple maintenance tasks and most common adjustments.

The user should routinely check the following items. It is recommended that a Drive Medical dealer services the transport-wheelchair annually, where these items should be repaired, replaced, adjusted and/or lubricated if required.

- Tyre wear
- Castors
- Seat upholstery
- Legrest locking mechanism
- Arm pads
- Arm rest mechanisms
- Rear wheel quick release pin
- Anti-tip bar adjustment
- Half folding back mechanism
- Back posts
- Transport-wheelchair (folds up)
- Wheel bearings
- Brakes
- Back upholstery

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