SERVICEHANDLEIDING TROPHY

© 2011 Handicare

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LED code	Trip code	Error type	Explanation
1	2c00	Low battery error	Low battery voltage
1	2c01	Low battery error	Low battery voltage
2	3B01	Motor fault 1	Motor disconnected
3	1400	Bridge fault 1	Motor wiring fault
4	7000	Startup with push selected	Parking brake off
4	7001	Push activated in drive mode	Parking brake off
6	1e03	Inhibit activated	Inhibit active
7	0810	Tiller fault 1	Throttle trip
7	0814	Tiller reference error	Throttle trip
7	0815	Tiller lo reference error	Throttle trip
7	0816	Tiller high reference iso error	Throttle trip
7	0817	Tiller lo reference error	Throttle trip
7	0300	Speed Limit Potentiometer wiper open	Throttle trip
7	1d02	Front end spec change trip	Throttle potentiometer trip
7	2f01	Tiller displaced error	Throttle displaced at power up
8	0002	Eeprom calibration checksum error 2	Possible controller trip
8	0003	Eeprom program checksum error 2	Possible controller trip
8	0100	Running checksum 1	Possible controller trip
8	0203	Eeprom calibration checksum error 1	Possible controller trip
8	0204	Eeprom program checksum error 1	Possible controller trip
8	1501	Brake fault 2	Possible controller trip
8	1705	Relay fault 1	Possible controller trip
8	1802	Watchdog voltage ref error	Possible controller trip
8	1805	Salty finger ref error	Possible controller trip
8	1b20	Both clamps measuring current	Possible controller trip
8	2102	Progres counter error	Possible controller trip
8	2d01	Front end spec change trip	Possible controller trip
8	3100	Battery bridge difference error 1	Possible controller trip
8	3102	Temperature settings incorrect	Possible controller trip
8	3210	No internal timers	Possible controller trip
8	3211	Internal timer out of range	Possible controller trip
8	3212	Atod readings not finished before next read	Possible controller trip
8	3213	Unknown mux channel	Possible controller trip
8	3600	Hardware error	Possible controller trip
8	3601	Test trip latch 1	Possible controller trip
8	3602	Test trip latch 2	Possible controller trip
8	3603	Test trip latch 3	Possible controller trip
8	3608	Test trip latch 8	Possible controller trip
8	3609	Test trip latch 9	Possible controller trip
8	360a	Test trip latch 10	Possible controller trip
8	360b	Test trip latch 11	Possible controller trip
8	360c	Test trip latch 12	Possible controller trip
8	360d	Test trip latch 13	Possible controller trip
8	360e	Test trip latch 14	Possible controller trip
8	4401	Generic ctrl error	Possible controller trip
9	1500	Brake fault 1	Solenoid brake fault
9	1502	Brake over current error	Solenoid brake fault
10	1600	High battery error	High battery voltage

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5.2 Controller settings PG Drives EGIS

Trophy 3 en 4 wiel standaard

Trophy Egis 120 15kmh

Speed Settings	Fast	Slow
Forward Acceleration (0,1 Seconds)	25	25
Forward Deceleration (0,1 Seconds)	13	13
Reverse Acceleration (0,1 Seconds)	40	40
Reverse Deceleration (0,1 Seconds)	25	25
Forward Speed (%)	100	100
Reverse Speed (%)	50	50

Operation Settings

Sleep timer (minutes)	0
Throttle Invert	On

Battery Settings

Low battery Flash Inhibit	On
TruCharge Cable Resistance (milli-Ohms)	40
TruCharge Calibration	97
Max Charge Amps (0,1 Amps)	15
Min Charge Amps (0,1 Amps)	0

Inhibit Settings

Inhibit Mode	1
--------------	---

General Settings

Soft Stop	On
Output Voltage (Volts)	24
Pulsed Reverse Alarm	Off
Brake Time (0,01 Seconds)	60
Freewheel Voltage	100
Freewheel Time-out (0,01 Seconds)	20

Motor Settings

Current Limit Max (Amps)	110
Current Foldback Time (Seconds)	15
Current Foldback Level (%)	50
Hold Factor (%)	160
Bridge Hold Time (centi-Secs)	80
Motor Compensation (milli Ohms)	45

Factory Settings	Max	Min
Forward Acceleration (0,1 Seconds)	100	1
Forward Deceleration (0,1 Seconds)	100	1
Reverse Acceleration (0,1 Seconds)	100	1
Reverse Deceleration (0,1 Seconds)	100	1
Forward Speed (%)	100	0
Reverse Speed (%)	100	0

Advanced Settings

Profile Top	40
Profile Bottom	80

Throttle Settings

Wig-Wag Throttle	On
Throttle Deadband (%)	15
Throttle Gain (%)	130
Iso Test	On

Options Settings

Options Register	17
------------------	----

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Speed Settings

	Fast	Slow
Forward Acceleration (0,1 Seconds)	25	25
Forward Deceleration (0,1 Seconds)	13	13
Reverse Acceleration (0,1 Seconds)	40	40
Reverse Deceleration (0,1 Seconds)	25	25
Forward Speed (%)	100	100
Reverse Speed (%)	40	40

Operation Settings

Sleep timer (minutes)	0
Throttle Invert	Off

Battery Settings

Low battery Flash Inhibit	On
TruCharge Cable Resistance (milli-Ohms)	40
TruCharge Calibration	97
Max Charge Amps (0,1 Amps)	15
Min Charge Amps (0,1 Amps)	0

Inhibit Settings

Inhibit Mode	1
--------------	---

General Settings

Soft Stop	On
Output Voltage (Volts)	24
Pulsed Reverse Alarm	Off
Brake Time (0,01 Seconds)	60
Freewheel Voltage	100
Freewheel Time-out (0,01 Seconds)	20

Motor Settings

Current Limit Max (Amps)	110
Current Foldback Time (Seconds)	15
Current Foldback Level (%)	50
Hold Factor (%)	156
Bridge Hold Time (centi-Secs)	80
Motor Compensation (milli Ohms)	45

Factory Settings

	Max	Min
Forward Acceleration (0,1 Seconds)	100	1
Forward Deceleration (0,1 Seconds)	100	1
Reverse Acceleration (0,1 Seconds)	100	1
Reverse Deceleration (0,1 Seconds)	100	1
Forward Speed (%)	100	0
Reverse Speed (%)	100	0

Advanced Settings

Profile Top	40
Profile Bottom	80

Throttle Settings

Wig-Wag Throttle	Off
Throttle Deadband (%)	15
Throttle Gain (%)	120
Iso Test	On

Options Settings

Options Register	17
------------------	----

5.3 Controller settings PG Drives S-Drive 120A

Trophy S-Drive 120 15kmh

Speed Settings

	Fast	Slow
Forward Acceleration (0,1 Seconds)	35	35
Forward Deceleration (0,1 Seconds)	13	13
Reverse Acceleration (0,1 Seconds)	40	40
Reverse Deceleration (0,1 Seconds)	25	25
Max Forward Speed (%)	100	100
Min Forward Speed (%)	10	10
Max Reverse Speed (%)	40	40
Min Reverse Speed (%)	5	5
Speed Limit Pot Enabled	On	

Operation Settings

Sleep timer (minutes)	0
Throttle Invert	Yes

Battery Settings

Low battery Flash Level (Bars)	0
Cable Resistance (milli-Ohms)	50
Calibration Factor	100
Low Battery Alarm	Off

Inhibit Settings

Inhibit 1: Mode	1
Inhibit 1: Operation	Latching
Inhibit 1: Speed (%)	0
Inhibit 2: Mode	1
Inhibit 2: Operation	Latching
Inhibit 2: Speed (%)	0
Inhibit 3: Mode	1
Inhibit 3: Operation	Latching
Inhibit 3: Speed (%)	0
Aux Output: Mode	0

General Settings

Soft Stop	On
Brake Time (milli-seconds)	1500
Output Voltage (Volts)	24
Status Output Type	0
Diagnostic Flash Sequence	2
Reverse Alarm	Off
Pulsed Reverse Alarm	Off
Diagnostic Alarm	Off
Brake Disconnected Alarm	Off
Brake Fault Detect	On
Brake Light	On
Freewheel Speed Limit	35
Freewheel Time Out (0,01 Seconds)	20
Freewheel Enable	Disable
Inhibit 2 Horn Input Enable	Off

(0 = Trucharge, 1 = LED, 2 = Analogue)

Motor Settings

Current Limit Max (Amps)	110
Boost Drive Current (Amps)	120
Boost Drive Time (Seconds)	10
Current Foldback Threshold (Amps)	110
Curerent Foldback Time (Seconds)	15
Current Foldback Level (%)	50
Motor Cooling Time (Seconds)	120
Current Foldback Temperature (Deg C)	80
Current Limit Min (Amps)	18
Motor Compensation (milli Ohms)	20
Anti Rollback Level	12
Pull-away Delay (centi-Seconds)	0
Slope Factor	1275
Anti Rollback Velocity	0

Factory Settings

	Max	Min
Acceleration (0,1 Seconds)	100	1
Deceleration (0,1 Seconds)	100	1
Forward Speed (%)	100	0
Reverse Speed (%)	100	0

Throttle Settings

Throttle Type	1
Throttle Deadband (%)	15
Throttle Gain (%)	140
Throttle Operated at Power-up	2
Throttle Reference Test	On
Iso Test Resistor	Off

(0 = Drive, 1 = Inhibit, 2 = Trip)

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Trophy S-Drive 120 15kmh voetgas

Speed Settings	Fast	Slow
Forward Acceleration (0,1 Seconds)	35	35
Forward Deceleration (0,1 Seconds)	13	13
Reverse Acceleration (0,1 Seconds)	40	40
Reverse Deceleration (0,1 Seconds)	25	25
Max Forward Speed (%)	100	100
Min Forward Speed (%)	10	10
Max Reverse Speed (%)	40	40
Min Reverse Speed (%)	5	5
Speed Limit Pot Enabled	On	

Operation Settings

Sleep timer (minutes)	0
Throttle Invert	No

Battery Settings

Low battery Flash Level (Bars)	0
Cable Resistance (milli-Ohms)	50
Calibration Factor	100
Low Battery Alarm	Off

Inhibit Settings

Inhibit 1: Mode	1
Inhibit 1: Operation	Latching
Inhibit 1: Speed (%)	0
Inhibit 2: Mode	1
Inhibit 2: Operation	Latching
Inhibit 2: Speed (%)	0
Inhibit 3: Mode	1
Inhibit 3: Operation	Latching
Inhibit 3: Speed (%)	0
Aux Output: Mode	0

General Settings

Soft Stop	On
Brake Time (milli-seconds)	1500
Output Voltage (Volts)	24
Status Output Type	0
Diagnostic Flash Sequence	2
Reverse Alarm	Off
Pulsed Reverse Alarm	Off
Diagnostic Alarm	Off
Brake Disconnected Alarm	Off
Brake Fault Detect	On
Brake Light	On
Freewheel Speed Limit	35
Freewheel Time Out (0,01 Seconds)	20
Freewheel Enable	Disable
Inhibit 2 Horn Input Enable	Off

(0 = Trucharge, 1 = LED, 2 = Analogue)

Motor Settings

Current Limit Max (Amps)	110
Boost Drive Current (Amps)	120
Boost Drive Time (Seconds)	10
Current Foldback Threshold (Amps)	110
Curerent Foldback Time (Seconds)	15
Current Foldback Level (%)	50
Motor Cooling Time (Seconds)	120
Current Foldback Temperature (Deg C)	80
Current Limit Min (Amps)	18
Motor Compensation (milli Ohms)	20
Anti Rollback Level	12
Pull-away Delay (centi-Seconds)	0
Slope Factor	1275
Anti Rollback Velocity	0

Factory Settings

	Max	Min
Acceleration (0,1 Seconds)	100	1
Deceleration (0,1 Seconds)	100	1
Forward Speed (%)	100	0
Reverse Speed (%)	100	0

Throttle Settings

Throttle Type	0
Throttle Deadband (%)	15
Throttle Gain (%)	180
Throttle Operated at Power-up	2
Throttle Reference Test	On
Iso Test Resistor	Off

(0 = Drive, 1 = Inhibit, 2 = Trip)

5.4 Controller settings PG Drives S-Drive 140A

Trophy S-Drive 140 15kmh

Speed Settings

	Fast	Slow
Forward Acceleration (0,1 Seconds)	35	35
Forward Deceleration (0,1 Seconds)	13	13
Reverse Acceleration (0,1 Seconds)	40	40
Reverse Deceleration (0,1 Seconds)	25	25
Max Forward Speed (%)	100	100
Min Forward Speed (%)	10	10
Max Reverse Speed (%)	40	40
Min Reverse Speed (%)	5	5
Speed Limit Pot Enabled	On	

Operation Settings

Sleep timer (minutes)	0
Throttle Invert	Yes

Battery Settings

Low battery Flash Level (Bars)	0
Cable Resistance (milli-Ohms)	50
Calibration Factor	100
Low Battery Alarm	Off

Inhibit Settings

Inhibit 1: Mode	1
Inhibit 1: Operation	Latching
Inhibit 1: Speed (%)	0
Inhibit 2: Mode	1
Inhibit 2: Operation	Latching
Inhibit 2: Speed (%)	0
Inhibit 3: Mode	1
Inhibit 3: Operation	Latching
Inhibit 3: Speed (%)	0
Aux Output: Mode	0

General Settings

Soft Stop	Off
Brake Time (milli-seconds)	1400
Output Voltage (Volts)	24
Status Output Type	0
Diagnostic Flash Sequence	2
Reverse Alarm	Off
Pulsed Reverse Alarm	Off
Diagnostic Alarm	Off
Brake Disconnected Alarm	Off
Brake Fault Detect	On
Brake Light	On
Freewheel Speed Limit	35
Freewheel Time Out (0,01 Seconds)	20
Freewheel Enable	Disable
Inhibit 2 Horn Input Enable	Off

(0 = Trucharge, 1 = LED, 2 = Analogue)

Motor Settings

Current Limit Max (Amps)	130
Boost Drive Current (Amps)	140
Boost Drive Time (Seconds)	10
Current Foldback Threshold (Amps)	80
Current Foldback Time (Seconds)	70
Current Foldback Level (%)	59
Motor Cooling Time (Seconds)	660
Current Foldback Temperature (Deg C)	30
Current Limit Min (Amps)	10
Motor Compensation (milli Ohms)	25
Anti Rollback Level	12
Pull-away Delay (centi-Seconds)	0
Slope Factor	1275
Anti Rollback Velocity	0

Factory Settings

	Max	Min
Acceleration (0,1 Seconds)	45	30
Deceleration (0,1 Seconds)	30	10
Forward Speed (%)	100	0
Reverse Speed (%)	50	0

Throttle Settings

Throttle Type	1
Throttle Deadband (%)	15
Throttle Gain (%)	140
Throttle Operated at Power-up	2
Throttle Reference Test	On
Iso Test Resistor	Off

(0 = Drive, 1 = Inhibit, 2 = Trip)

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Trophy S-Drive 140 15kmh voetgas

Speed Settings

	Fast	Slow
Forward Acceleration (0,1 Seconds)	35	35
Forward Deceleration (0,1 Seconds)	13	13
Reverse Acceleration (0,1 Seconds)	40	40
Reverse Deceleration (0,1 Seconds)	25	25
Max Forward Speed (%)	100	100
Min Forward Speed (%)	10	10
Max Reverse Speed (%)	40	40
Min Reverse Speed (%)	5	5
Speed Limit Pot Enabled	On	

Operation Settings

Sleep timer (minutes)	0
Throttle Invert	Yes

Battery Settings

Low battery Flash Level (Bars)	0
Cable Resistance (milli-Ohms)	50
Calibration Factor	100
Low Battery Alarm	Off

Inhibit Settings

Inhibit 1: Mode	1
Inhibit 1: Operation	Latching
Inhibit 1: Speed (%)	0
Inhibit 2: Mode	1
Inhibit 2: Operation	Latching
Inhibit 2: Speed (%)	0
Inhibit 3: Mode	1
Inhibit 3: Operation	Latching
Inhibit 3: Speed (%)	0
Aux Output: Mode	0

General Settings

Soft Stop	Off
Brake Time (milli-seconds)	1400
Output Voltage (Volts)	24
Status Output Type	0
Diagnostic Flash Sequence	2
Reverse Alarm	Off
Pulsed Reverse Alarm	Off
Diagnostic Alarm	Off
Brake Disconnected Alarm	Off
Brake Fault Detect	On
Brake Light	On
Freewheel Speed Limit	35
Freewheel Time Out (0,01 Seconds)	20
Freewheel Enable	Disable
Inhibit 2 Horn Input Enable	Off

(0 = Trucharge, 1 = LED, 2 = Analogue)

Motor Settings

Current Limit Max (Amps)	130
Boost Drive Current (Amps)	140
Boost Drive Time (Seconds)	10
Current Foldback Threshold (Amps)	80
Curerent Foldback Time (Seconds)	70
Current Foldback Level (%)	59
Motor Cooling Time (Seconds)	660
Current Foldback Temperature (Deg C)	30
Current Limit Min (Amps)	10
Motor Compensation (milli Ohms)	25
Anti Rollback Level	12
Pull-away Delay (centi-Seconds)	0
Slope Factor	1275
Anti Rollback Velocity	0

Factory Settings

	Max	Min
Acceleration (0,1 Seconds)	45	30
Deceleration (0,1 Seconds)	30	10
Forward Speed (%)	100	0
Reverse Speed (%)	50	0

Throttle Settings

Throttle Type	0
Throttle Deadband (%)	15
Throttle Gain (%)	180
Throttle Operated at Power-up	2
Throttle Reference Test	On
Iso Test Resistor	Off

(0 = Drive, 1 = Inhibit, 2 = Trip)

Controller settings PG Drives S-Drive 140A Alpine

Trophy S-Drive 140 Alpine

Speed Settings

	Fast	Slow
Forward Acceleration (0,1 Seconds)	35	35
Forward Deceleration (0,1 Seconds)	13	13
Reverse Acceleration (0,1 Seconds)	40	40
Reverse Deceleration (0,1 Seconds)	25	25
Max Forward Speed (%)	100	100
Min Forward Speed (%)	10	10
Max Reverse Speed (%)	40	40
Min Reverse Speed (%)	5	5
Speed Limit Pot Enabled	On	

Operation Settings

Sleep timer (minutes)	0
Throttle Invert	Yes

Battery Settings

Low battery Flash Level (Bars)	0
Cable Resistance (milli-Ohms)	50
Calibration Factor	100
Low Battery Alarm	Off

Inhibit Settings

Inhibit 1: Mode	1
Inhibit 1: Operation	Latching
Inhibit 1: Speed (%)	0
Inhibit 2: Mode	1
Inhibit 2: Operation	Latching
Inhibit 2: Speed (%)	0
Inhibit 3: Mode	1
Inhibit 3: Operation	Latching
Inhibit 3: Speed (%)	0
Aux Output: Mode	0

General Settings

Soft Stop	Off
Brake Time (milli-seconds)	1000
Output Voltage (Volts)	24
Status Output Type	0
Diagnostic Flash Sequence	2
Reverse Alarm	Off
Pulsed Reverse Alarm	Off
Diagnostic Alarm	Off
Brake Disconnected Alarm	Off
Brake Fault Detect	On
Brake Light	On
Freewheel Speed Limit	35
Freewheel Time Out (0,01 Seconds)	20
Freewheel Enable	Disable
Inhibit 2 Horn Input Enable	Off

(0 = Trucharge, 1 = LED, 2 = Analogue)

Motor Settings

Current Limit Max (Amps)	130
Boost Drive Current (Amps)	130
Boost Drive Time (Seconds)	10
Current Foldback Threshold (Amps)	80
Curerent Foldback Time (Seconds)	70
Current Foldback Level (%)	59
Motor Cooling Time (Seconds)	660
Current Foldback Temperature (Deg C)	30
Current Limit Min (Amps)	10
Motor Compensation (milli Ohms)	35
Anti Rollback Level	12
Pull-away Delay (centi-Seconds)	0
Slope Factor	500
Anti Rollback Velocity	0

Factory Settings

	Max	Min
Acceleration (0,1 Seconds)	40	10
Deceleration (0,1 Seconds)	30	13
Forward Speed (%)	100	0
Reverse Speed (%)	50	0

Throttle Settings

Throttle Type	1
Throttle Deadband (%)	15
Throttle Gain (%)	140
Throttle Operated at Power-up	2
Throttle Reference Test	On
Iso Test Resistor	Off

(0 = Drive, 1 = Inhibit, 2 = Trip)

Trophy met voetgas S-Drive 140A Alpine

Trophy S-Drive 140 Alpine voetgas

Speed Settings

	Fast	Slow
Forward Acceleration (0,1 Seconds)	35	35
Forward Deceleration (0,1 Seconds)	13	13
Reverse Acceleration (0,1 Seconds)	40	40
Reverse Deceleration (0,1 Seconds)	25	25
Max Forward Speed (%)	100	100
Min Forward Speed (%)	10	10
Max Reverse Speed (%)	40	40
Min Reverse Speed (%)	5	5
Speed Limit Pot Enabled	On	

Operation Settings

Sleep timer (minutes)	0
Throttle Invert	Yes

Battery Settings

Low battery Flash Level (Bars)	0
Cable Resistance (milli-Ohms)	50
Calibration Factor	100
Low Battery Alarm	Off

Inhibit Settings

Inhibit 1: Mode	1
Inhibit 1: Operation	Latching
Inhibit 1: Speed (%)	0
Inhibit 2: Mode	1
Inhibit 2: Operation	Latching
Inhibit 2: Speed (%)	0
Inhibit 3: Mode	1
Inhibit 3: Operation	Latching
Inhibit 3: Speed (%)	0
Aux Output: Mode	0

General Settings

Soft Stop	Off
Brake Time (milli-seconds)	1000
Output Voltage (Volts)	24
Status Output Type	0
Diagnostic Flash Sequence	2
Reverse Alarm	Off
Pulsed Reverse Alarm	Off
Diagnostic Alarm	Off
Brake Disconnected Alarm	Off
Brake Fault Detect	On
Brake Light	On
Freewheel Speed Limit	35
Freewheel Time Out (0,01 Seconds)	20
Freewheel Enable	Disable
Inhibit 2 Horn Input Enable	Off

(0 = Trucharge, 1 = LED, 2 = Analogue)

Motor Settings

Current Limit Max (Amps)	130
Boost Drive Current (Amps)	130
Boost Drive Time (Seconds)	10
Current Foldback Threshold (Amps)	80
Current Foldback Time (Seconds)	70
Current Foldback Level (%)	59
Motor Cooling Time (Seconds)	660
Current Foldback Temperature (Deg C)	30
Current Limit Min (Amps)	10
Motor Compensation (milli Ohms)	35
Anti Rollback Level	12
Pull-away Delay (centi-Seconds)	0
Slope Factor	500
Anti Rollback Velocity	0

Factory Settings

	Max	Min
Acceleration (0,1 Seconds)	40	10
Deceleration (0,1 Seconds)	30	13
Forward Speed (%)	100	0
Reverse Speed (%)	50	0

Throttle Settings

Throttle Type	0
Throttle Deadband (%)	15
Throttle Gain (%)	180
Throttle Operated at Power-up	2
Throttle Reference Test	On
Iso Test Resistor	Off

(0 = Drive, 1 = Inhibit, 2 = Trip)

5.5 Controller settings PG drives S-Drive 140 met CT transaxle

Trophy S-Drive 140 15kmh CT

Speed Settings	Fast	Slow
Forward Acceleration (0,1 Seconds)	35	35
Forward Deceleration (0,1 Seconds)	13	13
Reverse Acceleration (0,1 Seconds)	40	40
Reverse Deceleration (0,1 Seconds)	25	25
Max Forward Speed (%)	100	100
Min Forward Speed (%)	10	10
Max Reverse Speed (%)	40	40
Min Reverse Speed (%)	5	5
Speed Limit Pot Enabled	On	

Operation Settings

Sleep timer (minutes)	0
Throttle Invert	Yes

Battery Settings

Low battery Flash Level (Bars)	0
Cable Resistance (milli-Ohms)	50
Calibration Factor	100
Low Battery Alarm	Off

Inhibit Settings

Inhibit 1: Mode	1
Inhibit 1: Operation	Latching
Inhibit 1: Speed (%)	0
Inhibit 2: Mode	1
Inhibit 2: Operation	Latching
Inhibit 2: Speed (%)	0
Inhibit 3: Mode	1
Inhibit 3: Operation	Latching
Inhibit 3: Speed (%)	0
Aux Output: Mode	0

General Settings

Soft Stop	Off
Brake Time (milli-seconds)	1400
Output Voltage (Volts)	24
Status Output Type	0
Diagnostic Flash Sequence	2
Reverse Alarm	Off
Pulsed Reverse Alarm	Off
Diagnostic Alarm	Off
Brake Disconnected Alarm	Off
Brake Fault Detect	On
Brake Light	On
Freewheel Speed Limit	35
Freewheel Time Out (0,01 Seconds)	20
Freewheel Enable	Disable
Inhibit 2 Horn Input Enable	Off

(0 = Trucharge, 1 = LED, 2 = Analogue)

Motor Settings

Current Limit Max (Amps)	130
Boost Drive Current (Amps)	140
Boost Drive Time (Seconds)	10
Current Foldback Threshold (Amps)	80
Curerent Foldback Time (Seconds)	70
Current Foldback Level (%)	59
Motor Cooling Time (Seconds)	660
Current Foldback Temperature (Deg C)	30
Current Limit Min (Amps)	10
Motor Compensation (milli Ohms)	25
Anti Rollback Level	12
Pull-away Delay (centi-Seconds)	10
Slope Factor	1175
Anti Rollback Velocity	0

Factory Settings

	Max	Min
Acceleration (0,1 Seconds)	45	1
Deceleration (0,1 Seconds)	30	1
Forward Speed (%)	100	0
Reverse Speed (%)	50	0

Throttle Settings

Throttle Type	1
Throttle Deadband (%)	15
Throttle Gain (%)	140
Throttle Operated at Power-up	2
Throttle Reference Test	On
Iso Test Resistor	Off

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Operation Settings

Sleep timer (minutes)	0
Throttle Invert	Yes

Battery Settings

Low battery Flash Level (Bars)	0
Cable Resistance (milli-Ohms)	50
Calibration Factor	100
Low Battery Alarm	Off

Inhibit Settings

Inhibit 1: Mode	1
Inhibit 1: Operation	Latching
Inhibit 1: Speed (%)	0
Inhibit 2: Mode	1
Inhibit 2: Operation	Latching
Inhibit 2: Speed (%)	0
Inhibit 3: Mode	1
Inhibit 3: Operation	Latching
Inhibit 3: Speed (%)	0
Aux Output: Mode	0

General Settings

Soft Stop	Off
Brake Time (milli-seconds)	1400
Output Voltage (Volts)	24
Status Output Type	0
Diagnostic Flash Sequence	2
Reverse Alarm	Off
Pulsed Reverse Alarm	Off
Diagnostic Alarm	Off
Brake Disconnected Alarm	Off
Brake Fault Detect	On
Brake Light	On
Freewheel Speed Limit	35
Freewheel Time Out (0,01 Seconds)	20
Freewheel Enable	Disable
Inhibit 2 Horn Input Enable	Off

(0 = Trucharge, 1 = LED, 2 = Analogue)

Motor Settings

Current Limit Max (Amps)	130
Boost Drive Current (Amps)	140
Boost Drive Time (Seconds)	10
Current Foldback Threshold (Amps)	80
Current Foldback Time (Seconds)	70
Current Foldback Level (%)	59
Motor Cooling Time (Seconds)	660
Current Foldback Temperature (Deg C)	30
Current Limit Min (Amps)	10
Motor Compensation (milli Ohms)	25
Anti Rollback Level	12
Pull-away Delay (centi-Seconds)	10
Slope Factor	1175
Anti Rollback Velocity	0

Factory Settings


	Max	Min
Acceleration (0,1 Seconds)	100	1
Deceleration (0,1 Seconds)	100	1
Forward Speed (%)	100	0
Reverse Speed (%)	100	0

Throttle Settings

Throttle Type	1
Throttle Deadband (%)	15
Throttle Gain (%)	180
Throttle Operated at Power-up	2
Throttle Reference Test	On
Iso Test Resistor	Off


(0 = Drive, 1 = Inhibit, 2 = Trip)

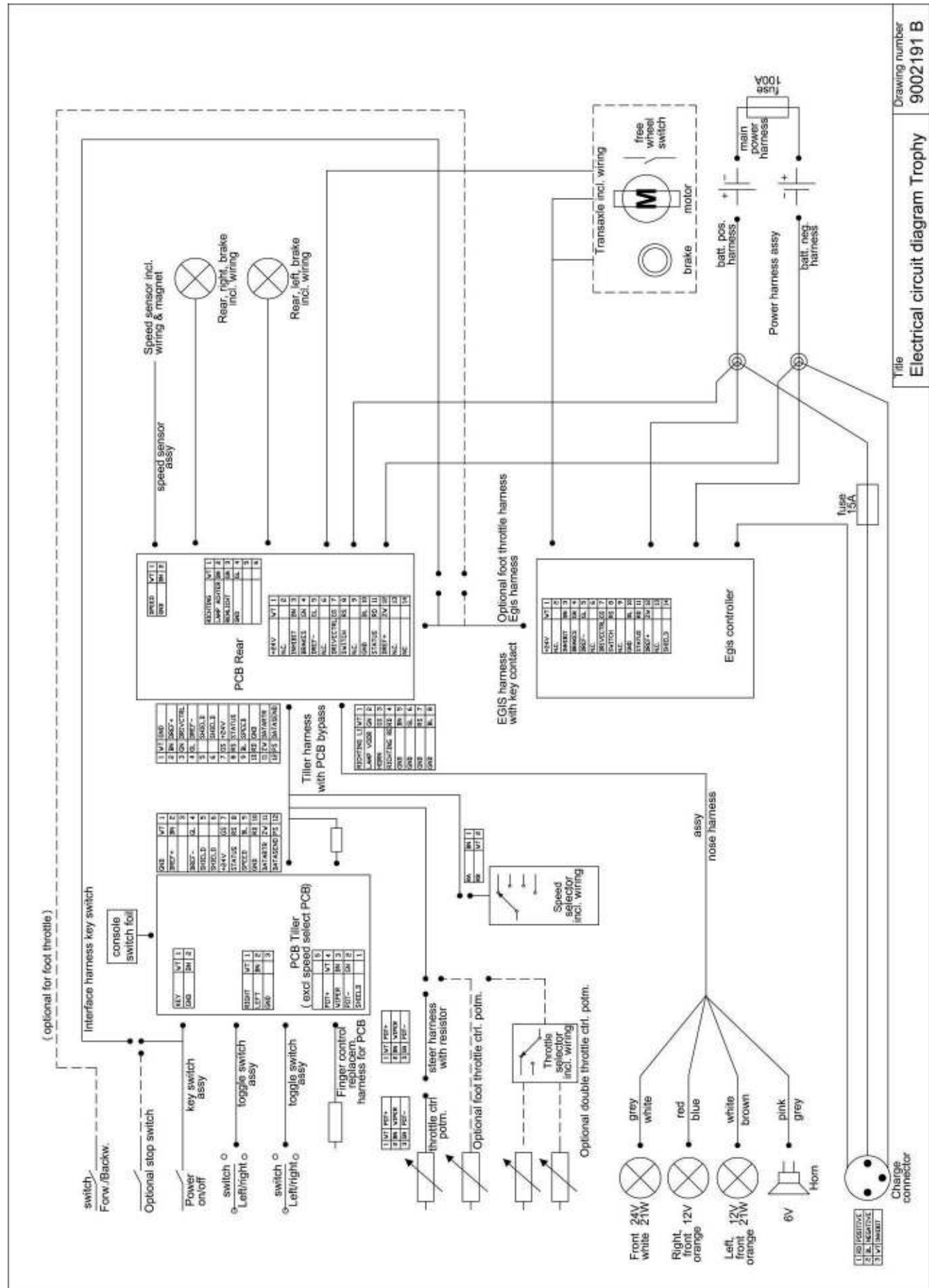5.6 Elektrisch schema Trophy Egis

Electrical circuit diagram Trophy
S-drive

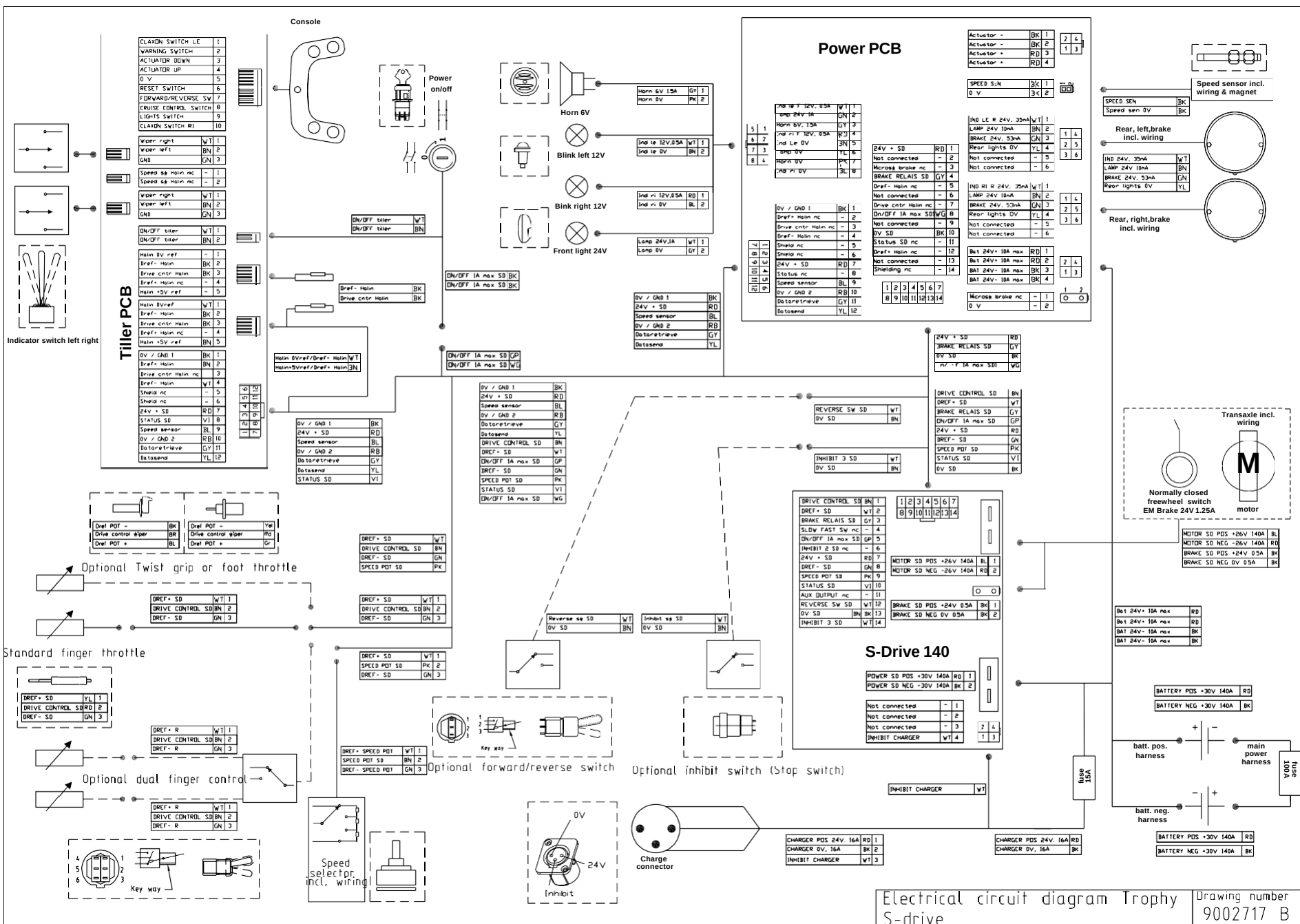Drawing number	9002717 B
----------------	-----------

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