



dry-glasses uv[®]

Smart drying cap for AI- and hearing glasses.

INSTRUCTION FOR USE

EN	INSTRUCTION FOR USE	2 - 3
DE	GEBRAUCHSANWEISUNG	4 - 5
FR	MODE D'EMPLOI	6 - 7
ES	MANUAL DE INSTRUCCIONES	8 - 9
PT	MANUAL DE INSTRUÇÕES	10 - 11
IT	MANUALE DI ISTRUZIONI	12 - 13
NL	GEBRUIKSAANWIJZING	14 - 15
GR	ΕΓΧΕΙΡΙΔΙΟ ΟΔΗΓΙΩΝ	16 - 17
TR	KULLANIM KILAVUZU	18 - 19
DK	BRUGERVEJLEDNING	20 - 21
SE	BRUKSANVISNING	22 - 23
SRB	UPUTSTVO ZA UPOTREBU	24 - 25
PL	INSTRUKCJA OBSŁUGI	26 - 27
KR	사용 지침	28 - 29
CN	用法說明	30 - 31
JP	取扱説明書	32 - 33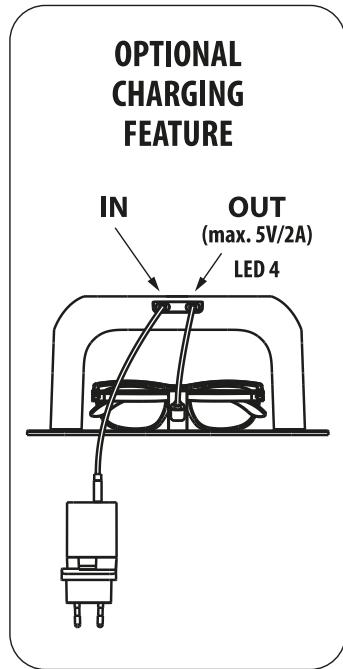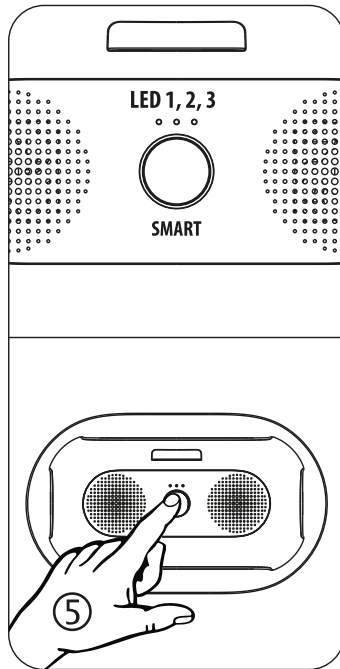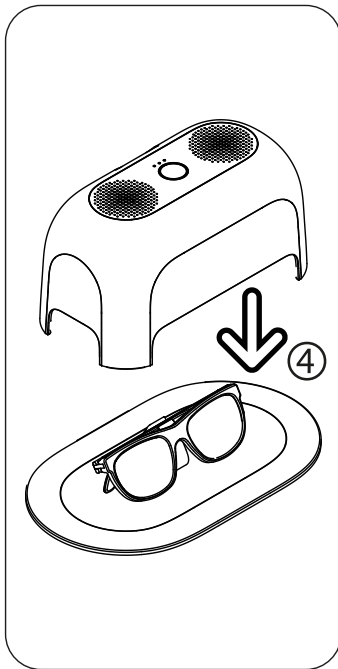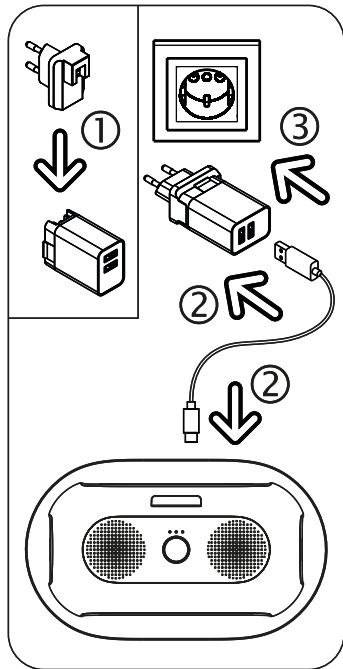

EN INSTRUCTION FOR USE

SMART mode (90, 120, or 180 minutes)

When you press this button, the drying time is automatically set to 90, 120, or 180 minutes based on the current dry room temperature and humidity measured by the built-in sensor.

Operation

1. Insert your country-specific adapter into the slot of the supplied USB universal power supply (5V/2.5A).
2. Plug the USB-A side of the longer USB cable into the USB-A socket of the power supply and the USB-C side of the cable into the USB-C socket (IN) on the back of your dry-glasses uv®.
3. Then plug the USB power supply into a power outlet.
4. Place the dry-glasses uv® over the charging station with the rechargeable AI/ hearing glasses. If the AI/ hearing glasses charging station has a lid, please open it before drying.
5. Press the SMART-button and the care program (drying + UV-C) will start automatically. ❗ LED 1, 2, or 3 will light up.
 - Drying by 2 fans with room air is controlled by a microprocessor and runs for 90, 120, or 180 minutes depending on the dry room temperature and humidity.
 - During drying, ultraviolet LED light (275 nm) eliminates 99.99% of germs and bacteria at the start and end of the drying cycle.
 - After drying and sanitizing, the dry-glasses uv® automatically switches to standby mode. ❗ LED 1, 2, or 3 switch off.

Drying and hygiene are complete, charging of the AI/ hearing glasses in the charging station continues and can remain under the dry-glasses uv® after the charging station is switched off.

Optional charging function

The dry-glasses uv® has a second USB-C output (OUT) on the back. This can be used to charge the AI/ hearing glasses or charging station. The shorter USB-C cable supplied is well suited for this purpose. Please note that the output power is limited to 5V/2A. LED 4 lights up green when the power is less than or equal to 5V/2A and

LED 4 switches off when the connected device requires more than 5V/2A of power. In this case, please use a separate power supply to charge your AI/ hearing glasses or charging station.

Maintenance and operation

We recommend using moist wipes to clean your dry-glasses uv[®]. Please also read the care and user instructions of your AI/ hearing glasses and charging station. The dry-glasses uv[®] is very energy efficient; only disconnect the power supply from the socket if you are going to be away for a longer period of time. You can interrupt the drying process by pressing and holding the SMART-button for longer than 3 seconds or by lifting and tilting the device.

! Warning notices

Avoid contact with water. Do not use the dry-glasses uv[®] near sinks, etc. In the event of a malfunction, unplug the power cord and take the dry-glasses uv[®] to your dealer. Do not disassemble the dry-glasses uv[®] and only use the USB power supply unit supplied. Children must be supervised to ensure that they do not use the dry-glasses uv[®] as a toy. The dry-glasses uv[®] is not intended for use by persons with limited physical, sensory, or mental abilities or lack of experience and/or knowledge, unless they are supervised by a person responsible for their safety or have been instructed by that person on how to use the dry-glasses uv[®].

CAUTION: Do not look into the UV-C light, as this may damage your eyes and skin. The dry-glasses uv[®] is equipped with a 3G motion sensor. If the dry-glasses uv[®] is moved, turned, or tilted during use, the 3G motion sensor will switch off the device! In idle mode, you must then restart the device. Please only use with UV-C cleanable products, never with humans or animals!

Disposal

Do not dispose of electrical appliances in unsorted municipal waste, but at separate collection points. If electrical appliances are disposed of in landfills, hazardous substances can enter the groundwater and the food chain and affect your health and well-being.

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flow-med GmbH
Otoplastic AG