
HUMANTECHNIK

lisa
signolux
sound**monitor**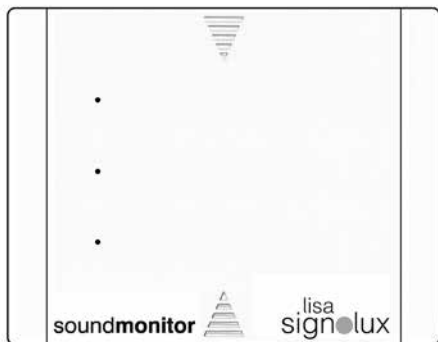

lisa signolux soundmonitor

DE

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Seite 4

EN

User manual

Page 16

FR

Mode d'emploi

Page 28

NL

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Pagina 40

FI

Käyttöohje

Sivu 52

DA

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Side 64

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Figure 1: View of top / side / front


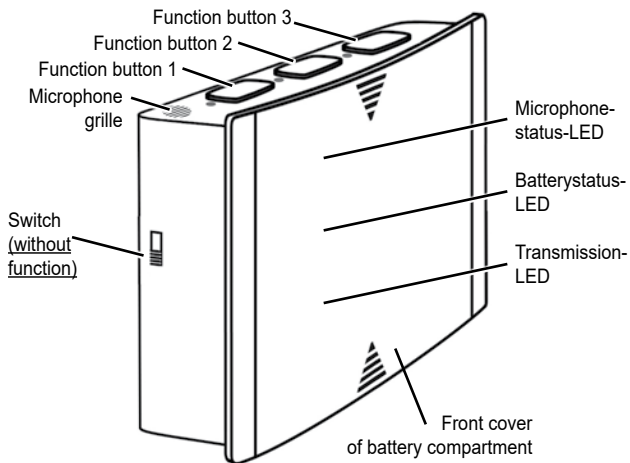
Figure 2: View of underside

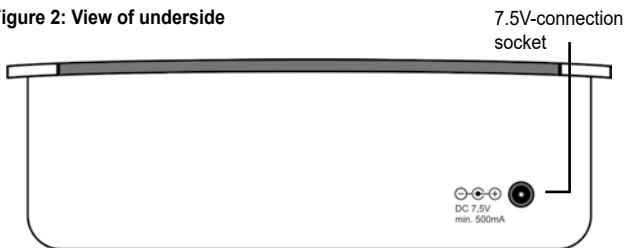The *Lisa-Signolux* »soundmonitor« helps people with hearing problems to cope with everyday life. It enables the perception of acoustic events in the home through visible signals.

The »soundmonitor« is part of a medical system and is compatible with all devices of the *Lisa-Signolux* alerting system.

Package contents

Please check if the following parts are included in the package:

- »soundmonitor«
- 3 x NiMH rechargeable battery, type AA, 1,2V
- 7,5 V power supply unit
- Wall mount + 2 screws and dowels
- User manual
- Warranty card

If the packaging or the packaging seal (round transparent sticker) is damaged, check the condition of the parts. If parts are missing or damaged, please contact your specialist dealer or the manufacturer directly.

Immediate safety information

- Please read these instructions carefully and completely before setting up and using the appliance. Be sure to observe the other warnings and safety instructions on page 23 and page 24.
- Keep this user manual in a safe place so that it is accessible to other users at all times. Every main user must have access to it.
- Keep *Lisa-Signolux* devices out of the reach of children! This packaging contains small parts - there is a risk of choking and suffocation!
- The flashes of light emitted by the receiver devices can trigger epileptic seizures in people with photosensitive epilepsy (PSE).

Function principle

The »soundmonitor« has been designed to learn sounds and to recognise them if they occur again. Up to three different sounds can be memorized.

This makes it possible to bind special acoustic signals into the *Lisa-Signolux* alerting system. The »soundmonitor« scans the environmental noise around it for those sounds it has been taught to react to.


If a memorised sound has been recognised, the »soundmonitor« will send out a radio signal to all *Lisa-Signolux*-receivers within range.

The receivers indicate the event signal with light-, vibration- or an acoustic alert (this depends upon the type of receiver).


Please note that the sound recognition only works with exactly repeatable sounds.

Inserting / replacing the rechargeable batteries (included and already inserted on delivery)


Remove the battery compartment lid: Push the front plastic cover upwards. A certain resistance, caused by the latching, has to be overcome (Figure 3).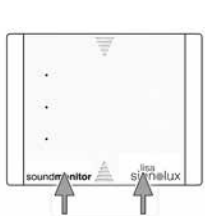

(Fig. 3)

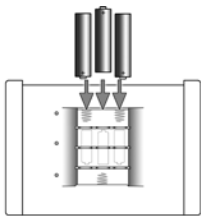(Fig. 4)

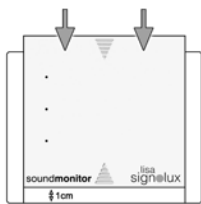(Fig. 5)

The plastic cover can now be taken off, the battery compartment has become accessible. Observe the correct polarity when inserting / replacing the batteries (Figure 4).

Re-attach the plastic cover: Lay the cover onto the housing with approx 1 cm distance to the lower edge (Figure 5). Push the cover downwards until it clicks back into place. The front cover is now attached again.

Indicators and controls

3 indicator LEDs are located under the dark transparent front cover:

Microphone status indicator

This LED lights up whenever a random sound triggers the activation of the microphone analyser.

Battery status indicator

A red battery status LED indicates that the rechargeable batteries are almost depleted and must be recharged. During recharging, this LED will show an orange light. A permanent green light shows that the batteries are charged and the »soundmonitor« is active.

Transmission indicator

The transmission indicator lights up briefly when a radio signal is transmitted to the *Lisa-Signolux* receiver devices.

Function buttons

The transmission and the triggered event indication on the receiver can be tested by pressing one of these buttons.

The function buttons also serve to teach the »soundmonitor« the sounds it will react to.

Setup

Choosing the best position

The »soundmonitor« scans and analyses the noises in its vicinity for the patterns of the sounds it has been taught. In order to assist this function, the »soundmonitor« should be placed as close as possible to the sound source it is supposed to identify (for example placed on a wall underneath the door bell).

Power supply

When using for the first time: Open the battery compartment and remove the battery stopper. Connect the 7,5 V power supply with the according socket on the underside of the »soundmonitor«. Plug the power supply unit into a regular 230V mains electricity outlet.

Blackout-Backup

Please note that this device has been designed to be primarily powered over the power supply unit. The rechargeable batteries serve only as an emergency backup during power cuts. The standby provided by the batteries covers approx. 16 hours.

The batteries are charged when the »soundmonitor« is connected to the mains power supply. The batteries have a charging time of approx. 12 hours.

Teaching the desired sounds

To teach a new sound, press and hold one of the function buttons, located on the upper face of the device, for 2 to 3 seconds until the adjacent LED of the button starts to blink.

Trigger the desired sound on the sound source. The sound should have a length of at least 1,5 seconds.

During the recording, the microphone status indicator must be lit constantly!

The new sound has been taught. The other two function buttons can also be programmed as described above.

Important notice:

When teaching a sound, please avoid any other environmental noise.

The »soundmonitor« should remain in the position in which the according sound was recorded, in order to assist a secure recognition of the sound when it occurs again.

Please note:

This device can only recognise repeatable and unaltering electronic sounds. Natural and mechanical sound sources or voice cannot be identified.

Automatic pairing

Lisa-Signolux devices can be used together immediately and without prior pairing when delivered.

By default, the »soundmonitor« will be indicated by receiver devices with the general alarm icon .

Important notice 1:

If the transmitter is to be displayed with an event icon of your own choice, this can be done as described in the chapter Pairing and assigning to event icons.

However, this causes the according receiver to lose the automatic pairing function and every transmitter previously indicated on the receiver must then be individually paired.

Important notice 2:

Should there be repeated false alerts when using the preset transmitter indication, this may hint at a neighbouring *Lisa-Signolux* installation causing interference. In this case, the problem can be solved by individually pairing all transmitter and receiver units within the setup (see Pairing and assigning to event icons).

Linking transmitter alerts with event icons

The »soundmonitor« can be individually paired with every *Lisa-Signolux*-receiver, which is later supposed to react to it. By doing so, the indication type is also defined.

To register the »soundmonitor« on a receiver, press and hold the according receiver's function key. The receiver's event indicator icons light up in sequence.

Release the function key as soon as the desired icon is lit up (for example the fire icon). Now press the function button of the »soundmonitor«.

Wait for approx. 20 seconds and then test the setup by pressing any of the function buttons of the »soundmonitor«. The receiver should now react, showing the according icon. This procedure needs to be repeated for every receiver unit.


Please note:

Every recognised sound is indicated by the same event icon on the receivers.

Mounting the »soundmonitor«

The »soundmonitor« can be mounted using the supplied wall mount. To use it, fix the wall mount onto a wall (or rafter, cupboard etc.)

When mounting onto a brick- or stone wall, you may need a power drill and stone bit to do so (observe the power drill manufacturer's safety instructions!). Use the supplied dowels in this case. Once fixed, hang the »soundmonitor« onto the wall mount bracket.

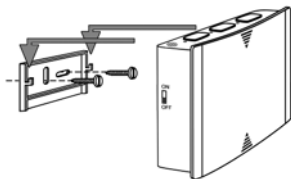Environmental conditions

Units of the *Lisa-Signolux* system should only be operated in a dry environment within the guideline values for temperature and humidity specified in these operating instructions. Humidity and moisture can lead to failure of the units.

Maintenance and cleaning

The unit is maintenance-free. If it becomes dirty, it can be cleaned with a soft, damp cloth. If necessary, use only commercially available alcohol-based disinfectants.

Never use cleaning agents or disinfectants that contain acetone or hydrogen peroxide or that are bleaching, corrosive or abrasive.

No liquid or moisture may penetrate the openings of the unit.

Compatible *Lisa-Signolux* receiver devices

This transmitter is compatible with all receivers of the *Lisa-Signolux* product range:

- Receiver S (A-2616-W)
- Receiver T (A-2617-0)
- Portable vibrating receiver (A-2619-0)
- Receiver alarm clock (A-2634-0 / W)
- Flashing lamp (A-2644-W)
- Flashing lamp with Type F (CEE 7/3) socket (A-2645-0)
- Receiver T Connect (A-2676-0)

Warnings and safety instructions

This chapter lists further important information for safe handling and other conditions for operation.

Safety instructions for rechargeable batteries and power supply

- Charge the batteries fully at room temperature before the first use.
 - Only use the specified battery type NiMH AA (HR6)! There is a risk of explosion if the wrong type of battery is used.
 - Batteries must not be exposed to high or low extreme temperatures and low air pressure during use, storage or transport at high altitude.
 - Disposing of batteries in fires, ovens or by mechanical crushing and cutting may cause fires or explosions. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.
 - Batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.
 - Use only the original power supply with the receiver.
 - Remove the batteries from the unit if it will be out of use for a longer period of time (> 3 months).
 - If the batteries are not sufficiently charged, event messages from the system may be missed!
- Lay the cable of the power supply unit in such a way as to avoid tripping hazards and possible damage to the cable.
- Rechargeable batteries and regular batteries are toxic and must not be swallowed! Keep them out of the reach of children and pets. In case of emergency, seek appropriate medical attention immediately.

Information on product safety

- Please observe these instructions to avoid damage to the unit and possible invalidation of the warranty.
- The device is designed for use in living areas. The unit must not be exposed to direct sunlight for long periods of time and must also be protected from excessive heat, moisture or strong mechanical shocks.
- Only place the unit on flat surfaces. Do not drop the unit. Dropping the unit on a hard surface can damage or destroy it.
- Do not disassemble the device or the power supply unit; there is a risk of electric shock. Opening and modifying the unit will immediately void the warranty. Only use accessories from Humantechnik.

- This unit is not suitable for outdoor use. Do not allow the unit to become damp or wet.
- Only use units of the *Lisa-Signolux* system where electronic devices are permitted. When using in aircraft, it is essential to obtain the prior consent of the flight personnel.
- Strong interference signals from third-party devices can impair or completely prevent the function. Do not place potentially interfering devices such as screens and mobile phones closer than 30 cm to *Lisa-Signolux* devices and, if necessary, their cabling, as important event messages such as fire warnings may be prevented.

For your safety: To ensure the function of the system, a function test must be carried out after installation.

If smoke detectors are integrated into the system, it is a legal requirement to check the detectors and their alerting regularly!

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products.

Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material.

You will receive further information on the recycling of this product from your local community, your local disposal company or your local dealer.

Disposal of batteries

The included rechargeable batteries are recyclable. Please dispose of the batteries by using a dedicated battery container or return them to the specialist shop. Please dispose only discharged batteries to ensure environmental protection.

Service, warranty and warranty conditions

The unit has a high level of operational safety. If faults should occur despite proper operation, please contact your specialist dealer or contact Humantechnik GmbH directly.

The warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Likewise, damage due to natural forces, such as fire and flooding, is not covered by the warranty.

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the serial number in any event.


Humantechnik's UK authorised representative:
Sarabec Ltd.
15 High Force Road
MIDDLESBROUGH TS2 1RH, United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments.

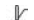UK declaration of conformity available from: Sarabec Ltd.

Markings

-  This is the labelling of Humantechnik GmbH for a medical product according to EU 2017/745.
-  This is the marking for the product identification number to identify the medical device.
-  This is the marking for the serial number to identify the medical device. This is indicated on the device and the packaging.
-  This is the identification for the manufacturer's item number.
-  This is the marking for the manufacturer of the medical device.
-  This symbol indicates the production date of the medical device.
-  This is the marking of the country of production of the medical device. This is shown on the device and the packaging.
-  This symbol indicates distributors for the medical device.
-  This symbol indicates that the user should read these operating instructions attentively.
-  This sign indicates warnings that are important for the user as well as important information on use and operational safety.
-  Temperature range for transport and storage: -5°C to 40°C
Temperature range for operation: 10°C to 30°C , i optimum ambient temperature: 25°C
-  Humidity during transport and storage: $< 90\%$ (non-condensing)
Operating humidity: 15% to 90% (non-condensing)
-  Atmospheric pressure range during transport, storage and operation: 700 hpa to 1060 hpa
-  This mark indicates that this appliance does not require sterilisation.
-  The CE mark on the device confirms conformity with the European requirements of the Medical Device Regulation (2017 / 745 / EU Medical Device Directive (MDR)).
- 2011 / 65 / EC + 2015 / 863 / EC RoHS-Directive
 - 2012 / 19 / EC WEEE-Directive
 - 2014 / 53 / EC RED-Directive

Complete declarations of conformity are available on the Internet at www.humantechnik.com in the service area.

All serious incidents must be reported to the manufacturer and the according local authority!

Specifications

 Working range:	10 to 30°C, optimum ambient temperature: 25°C
 Operation:	< 90%, non-condensing (transport, storage and operation)
Protection class:	IP 20
Electrical safety:	Protection class 2
Power supply:	DC 7,5V / 1500mA
Power backup:	by 3 rechargeable batteries 1,2V (Type HR6)
Rech. battery standby:	approx. 16 hours
RF-frequency:	868,425 MHz
Max. transmitting power:	<10 mW
Transmission range:	up to 200 m in open spaces, up to 80 m within buildings
Microphone status indicator:	1 red LED
Battery status indicator:	1 red/green LED
Transmission indicator:	1 red LED
Recording indicators:	3 red LEDs
Height x width x depth:	95 mm x 122 mm x 39 mm
Weight:	320 g (including batteries)
Expected lifetime of the unit:	approx. 10 years

Technical specifications may be subject to change without prior notice.

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