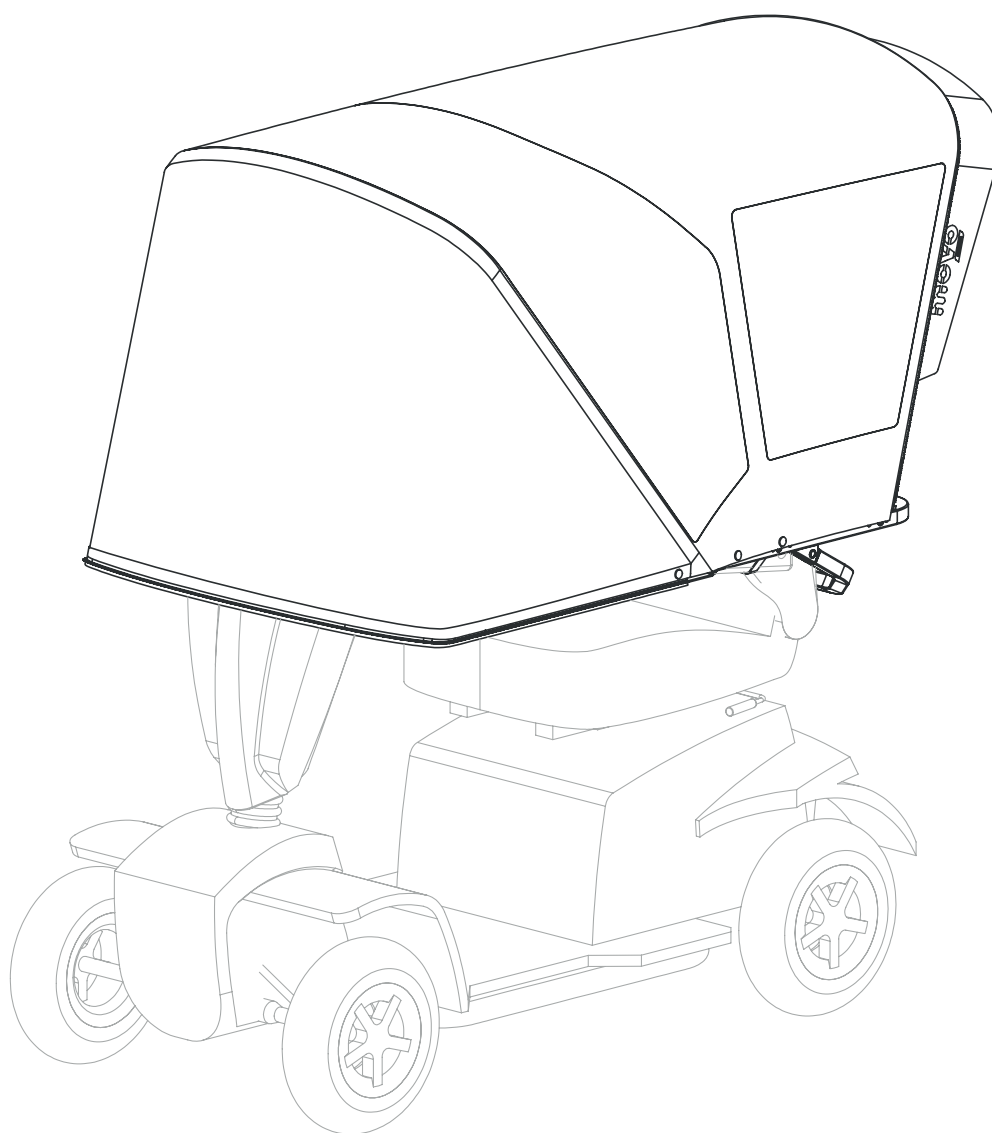
Gebruikershandleiding
User manual
Manuel d'utilisation

mevo

evolution

MV-100

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MOVO WARRANTY CONDITIONS AND LIMITATIONS :

Movo Evolution Canopy MV-100

1-year LIMITED WARRANTY: Please read and understand the instructions before attempting to install the new Movo Evolution canopy.

A warranty period of 1 year applies to your new Movo Canopy. This warranty does not apply to damages caused by strong winds or hail, improper installation, misuse, or neglect, use of abrasive or chemical cleaners, overexposure to UV rays, accidents, lack of maintenance, unsuitable environment such as chlorine, alterations or abuse. Although the material used is very resistant, it can tear during installation if not done correctly and Movo Evolution will not assume any responsibility for such installation. In addition, wear and fading resulting from normal use are excluded from this warranty. Warranties apply only within the EU and this warranty voids and replaces any other warranty stipulated by law or communicated orally, except as warranted in writing by Skyline Mobility.

UNAUTHORIZED USAGE

- Do not use in high winds, extreme rain, hail and other unusual weather conditions.
- Do not use on an electric vehicle with a maximum speed exceeding 25 km/h.
- Do not use on any type of vehicle other than four or three wheel electric scooters.
- Do not use in temperatures below freezing, below zero degrees Celsius.
- Do not fold and unfold repeatedly at an ambient temperature of less than 7 degrees Celsius, as the canopy will become more rigid.
- This canopy is not a substitute for a protective storage cover.
- For some scooters, the mirrors may need to be "folded in" to maintain optimal steering control, as the canopy is designed to be no wider than the average door opening.
- Never use the product if it causes blind spots while driving. If necessary, readjust the canopy and seat position to get a full view.
- Do not use if canopy obscures vehicle's lights or reflectors. The user is responsible for the visibility of his vehicle.
- Do not fold the canopy up or down with one hand. Always use two hands and lift both sides at the same time.

Skyline Mobility is not responsible for any necessary adjustments to secure the canopy under the seat. It is the responsibility of the purchaser and retailer that the insertion tube under the seat ensures a proper fitting and sturdy mounting of the Movo Evolution. Skyline Mobility is relieved of all responsibilities in this regard.

FABRIC MAINTENANCE

Do not use corrosive, gritty or abrasive cleaning agents.

To keep the fabric in good condition and avoid the need for a deep, vigorous cleaning, the user should gently clean the fabric monthly with clean water to prevent dirt from settling deep into the fabric.

Use a soft brush and lukewarm water with a suitable fabric cleaner, air dry only.

Never apply heat to the fabric. Always allow the fabric to dry completely before folding and storing.

Always test the cleaner on an inconspicuous area of the fabric before using it.

Mica can react to certain substances. Always test the cleaner on an inconspicuous area first.

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

Ajust the seat to an adequate driving position.

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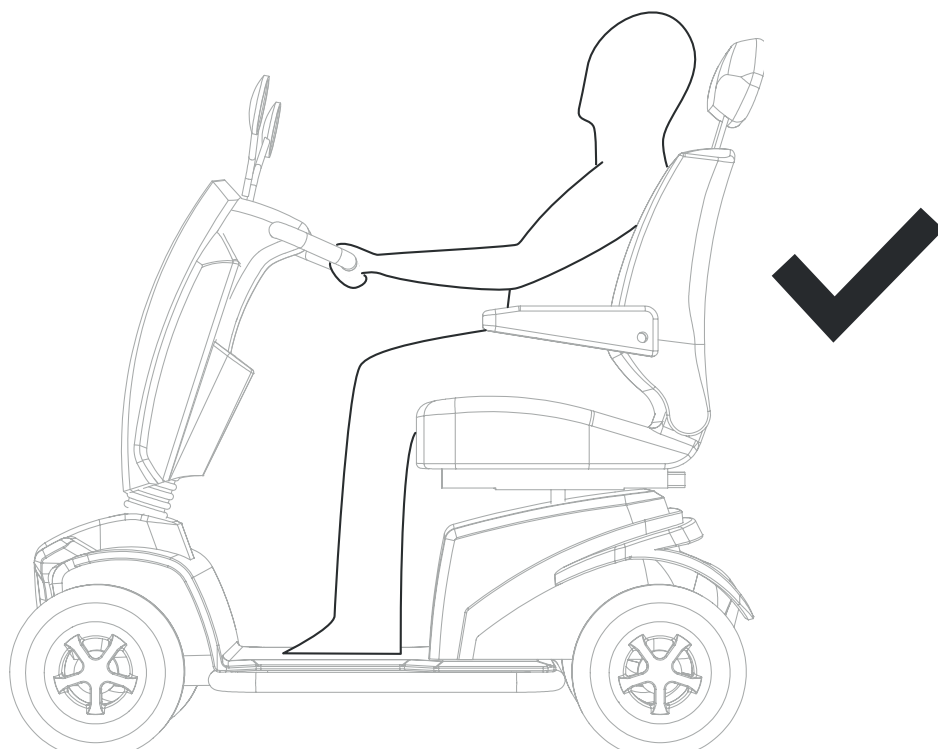
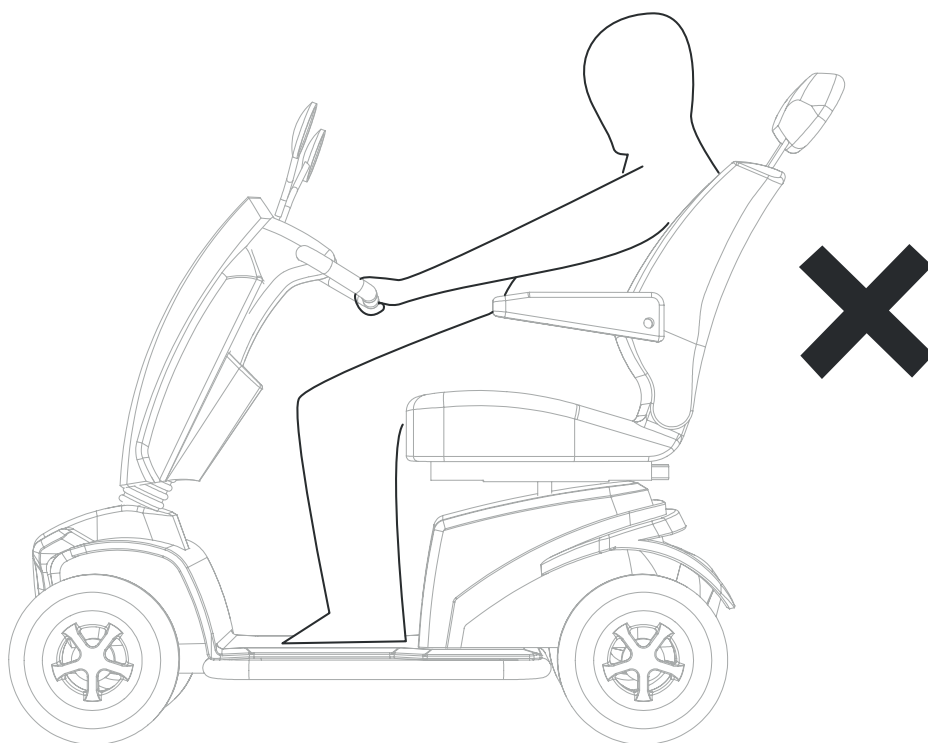
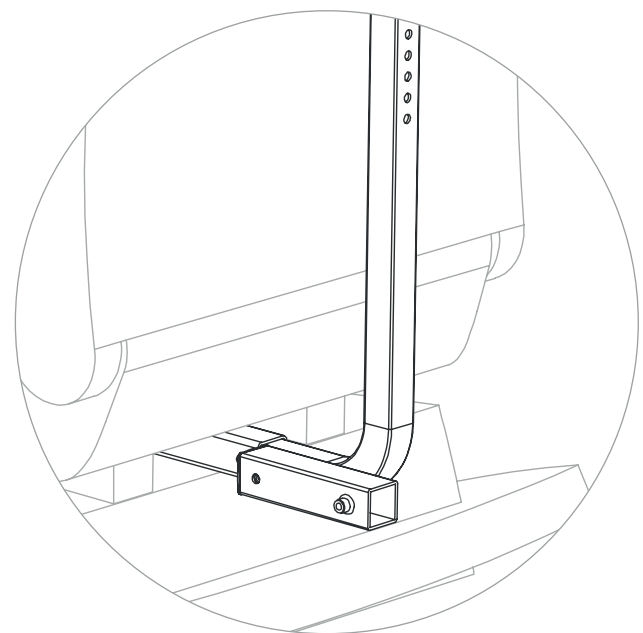
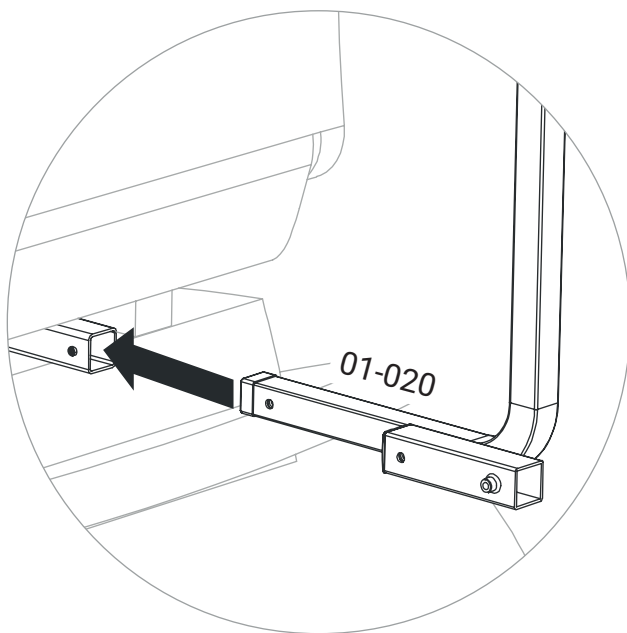
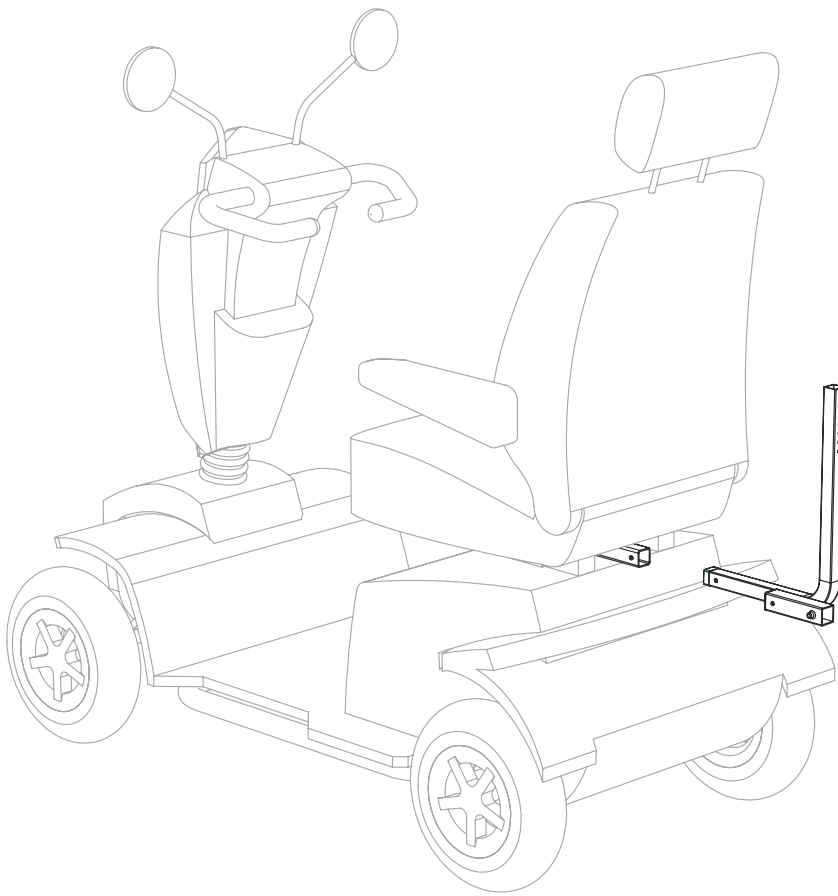
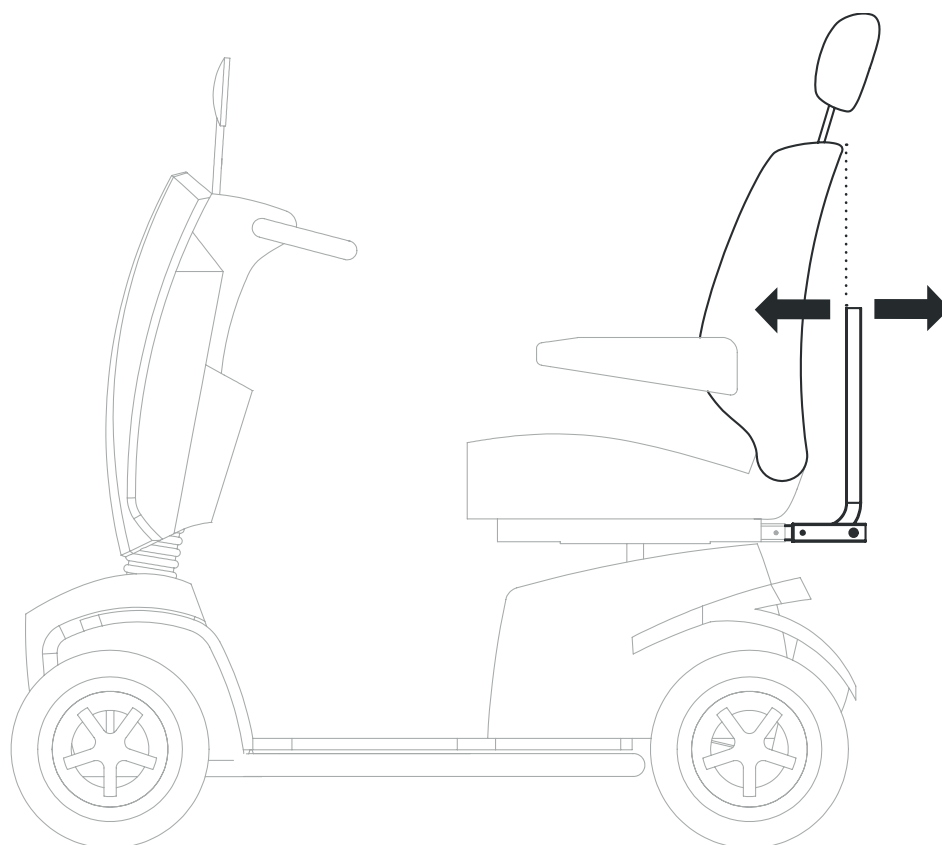
If the hitch isn't deep enough, see page 13-14.

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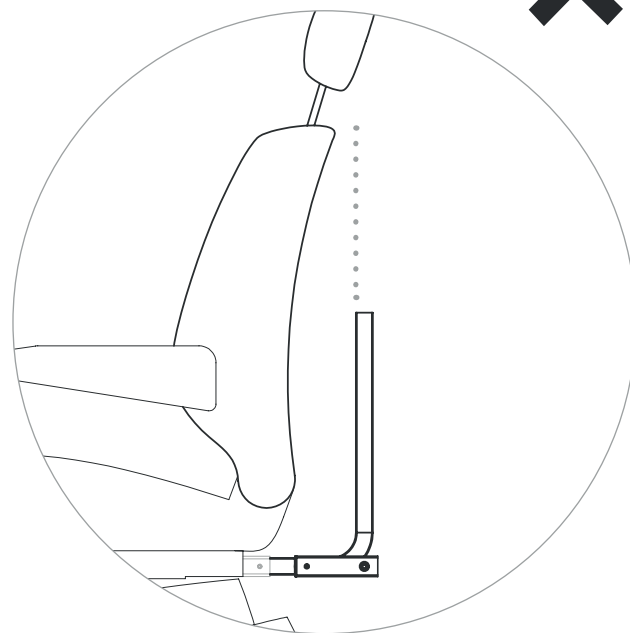
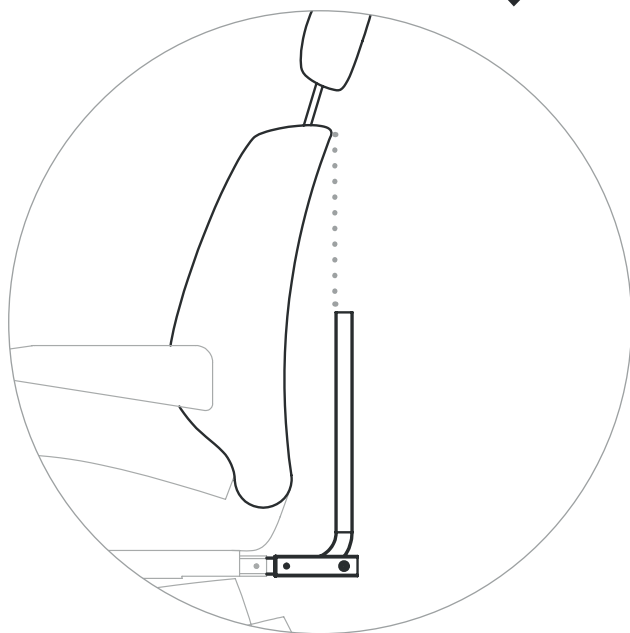
Secure 01-020 using the fastener supplied with the scooter insertion tube.

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

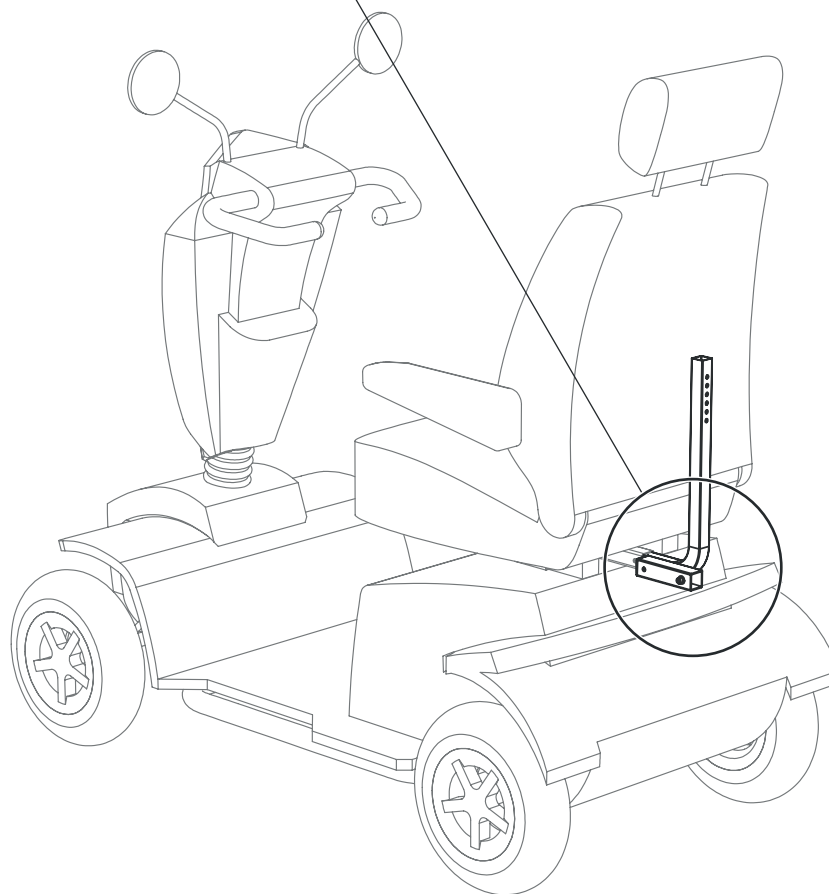
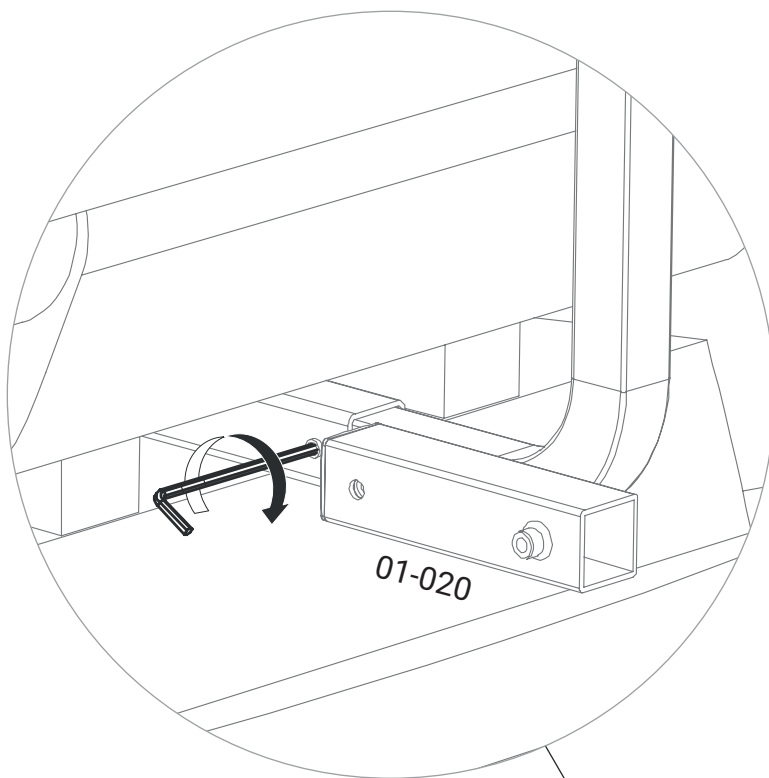
If the hitch isn't deep enough to allow the alignment of 01-020 to the top of the back rest, use a hack saw and shorten it by 5 cm. Repeat this process if 01-020 is still too long. The product will work fine without the plastic cover.

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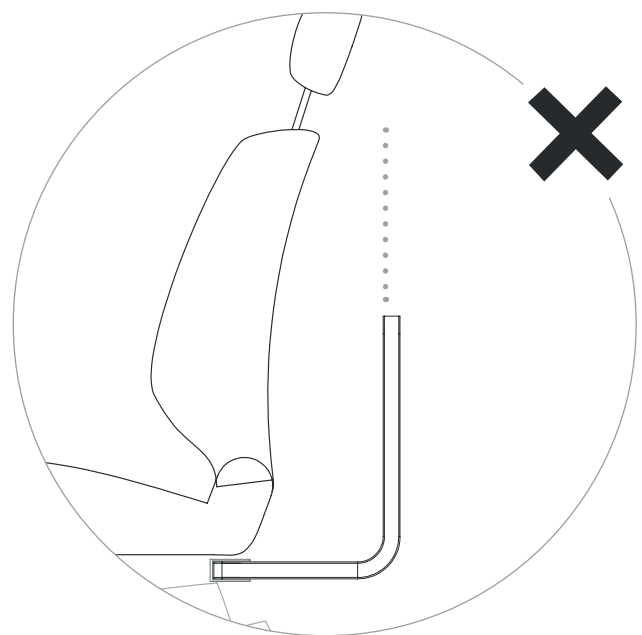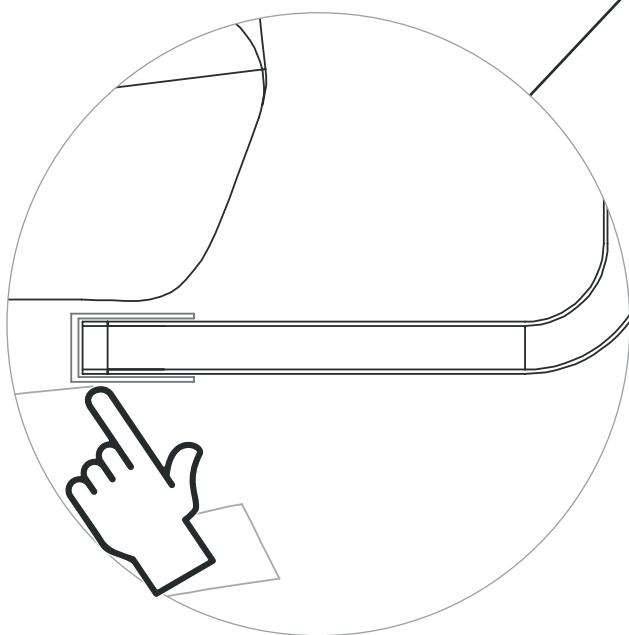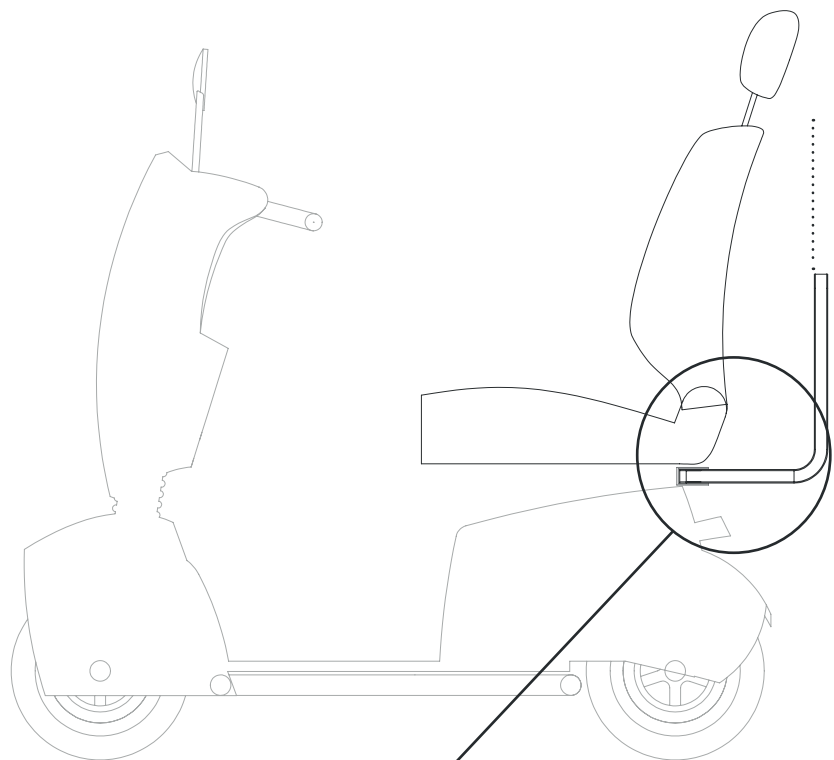
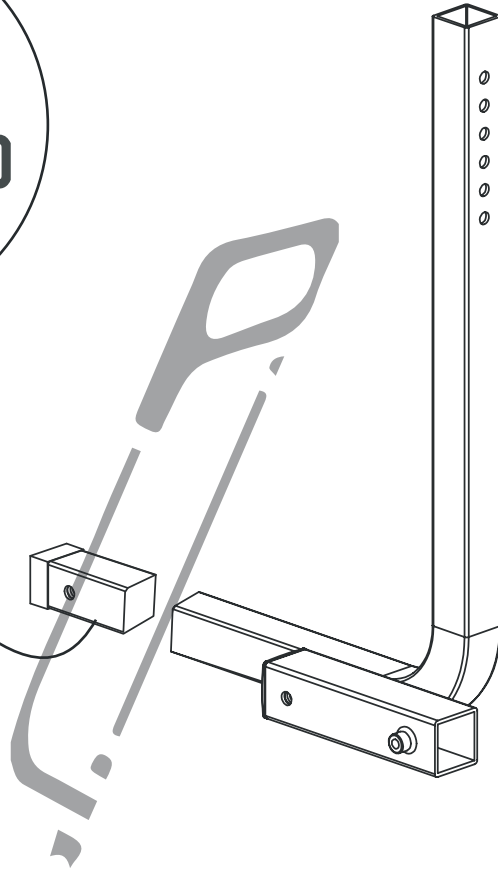
Work safely and always wear eye protection. Ask for help if you are not comfortable with the procedure.

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



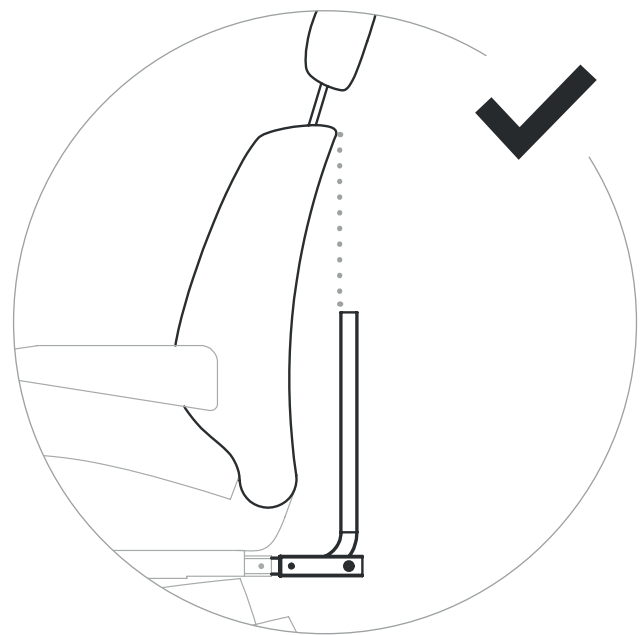
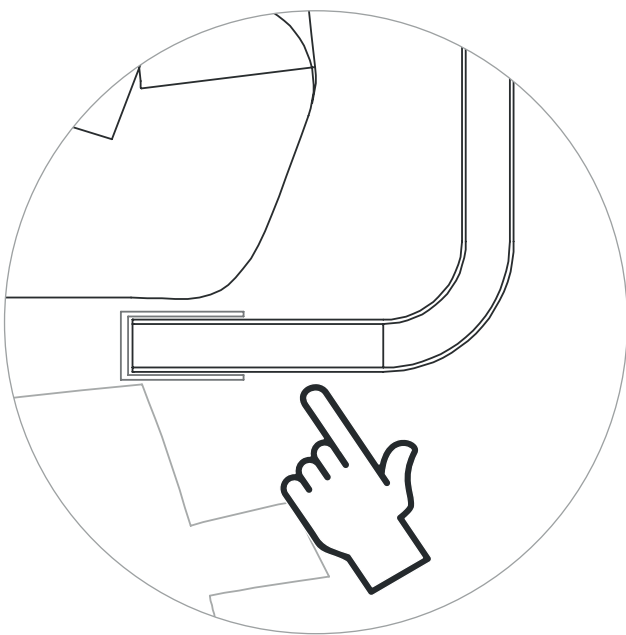
From its driving position, incline the backrest forward by one increment.

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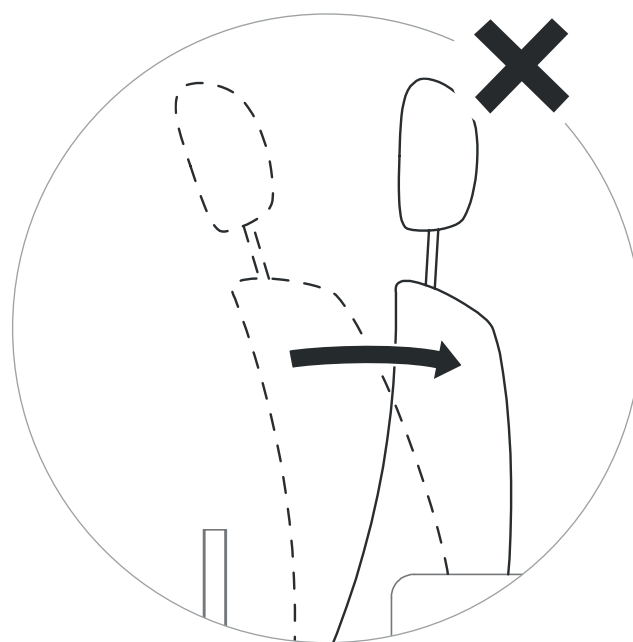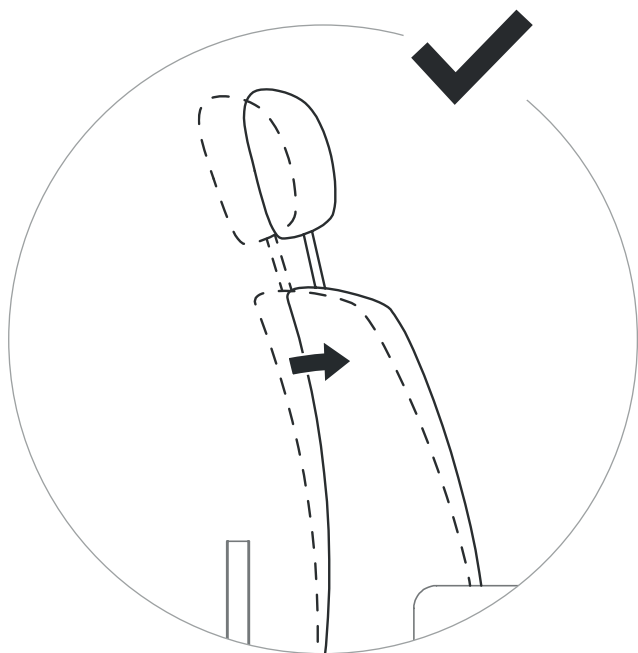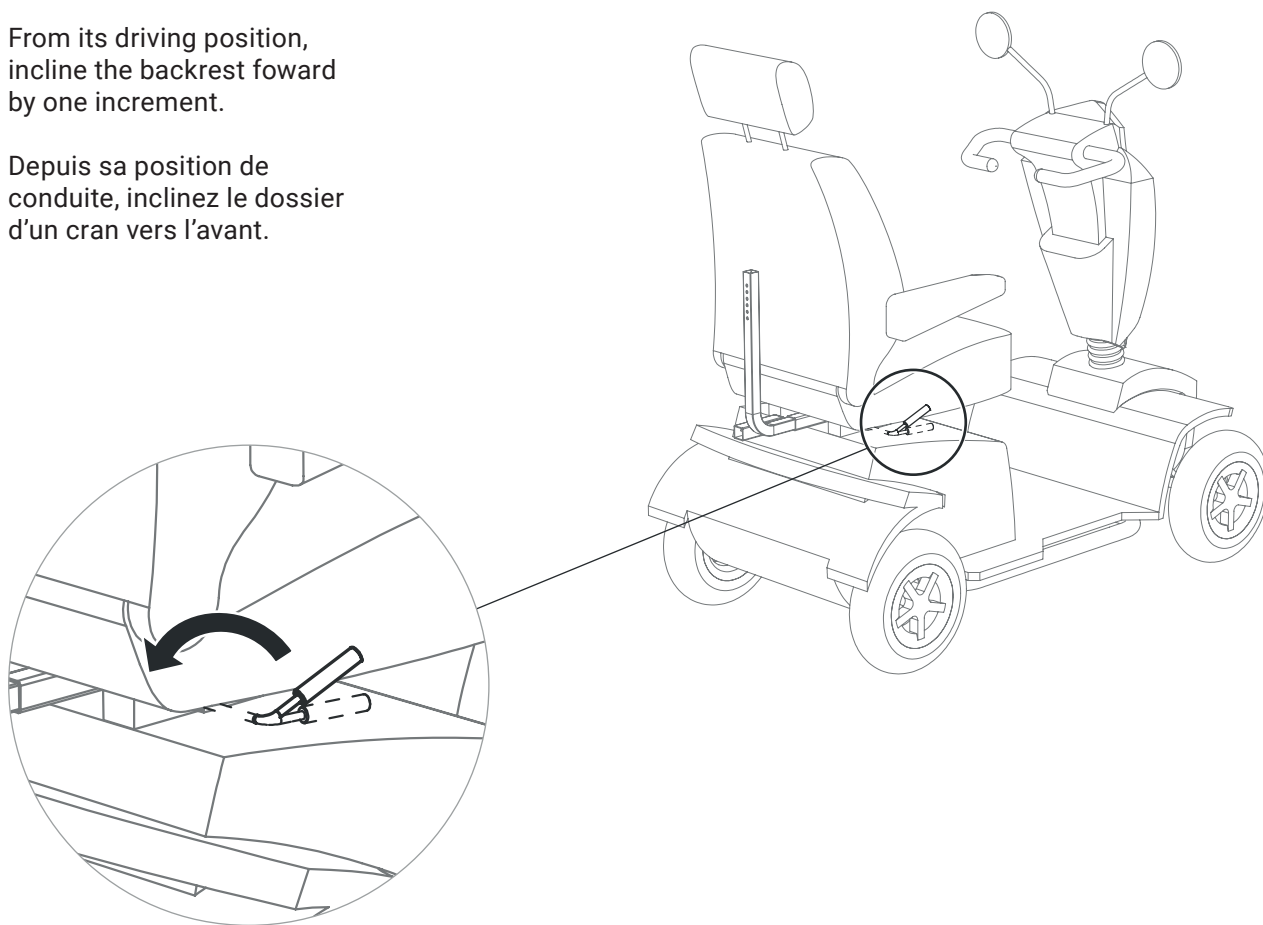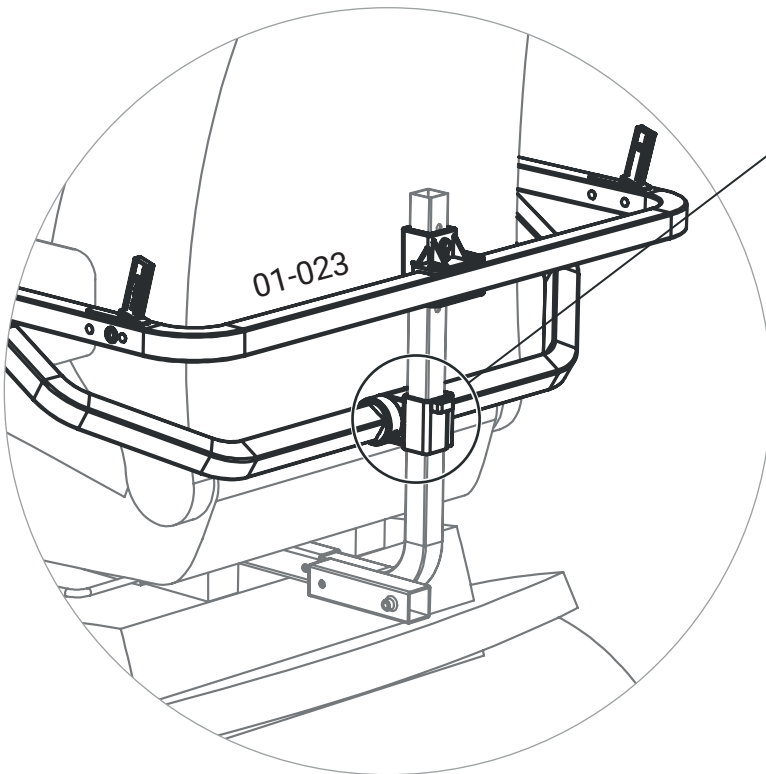
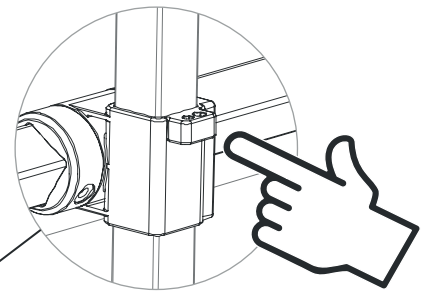
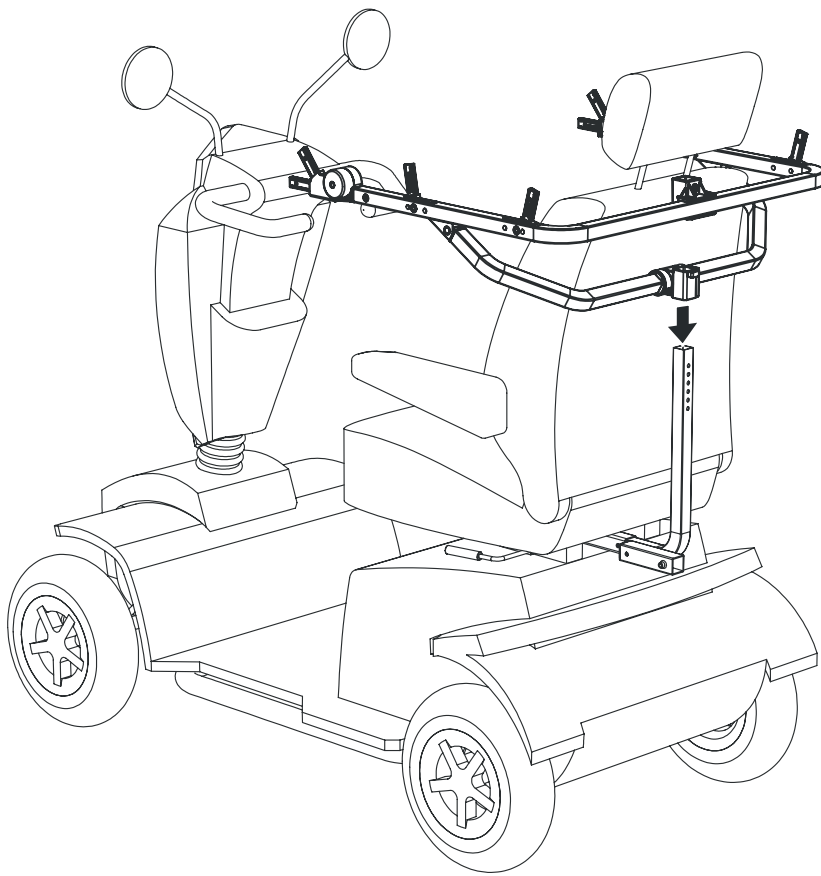
Make sure this part is well oriented. The allen key holder should face away from the seat.

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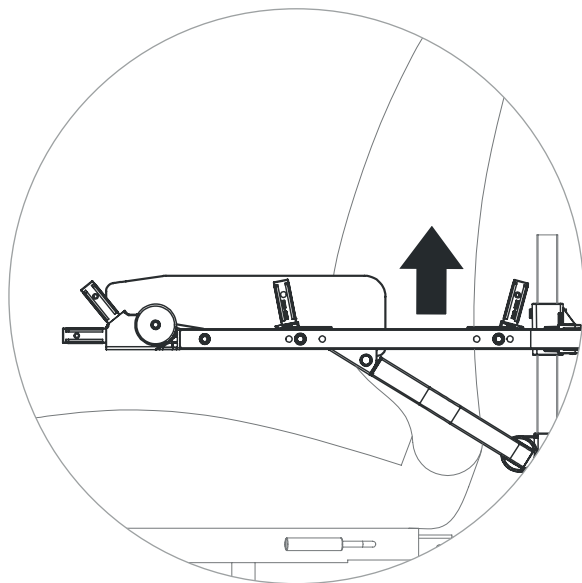
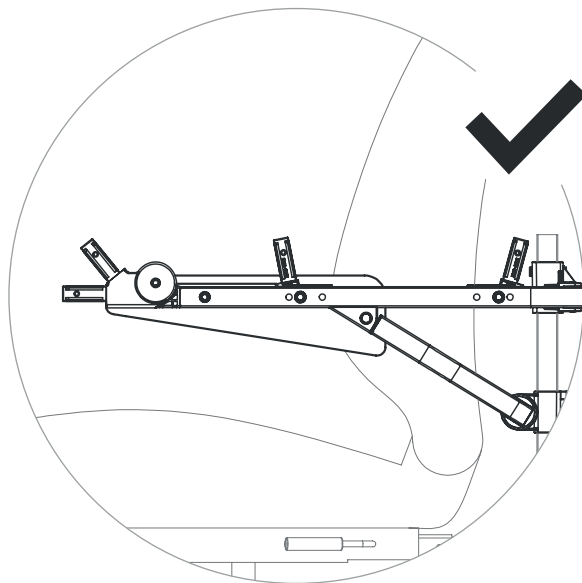
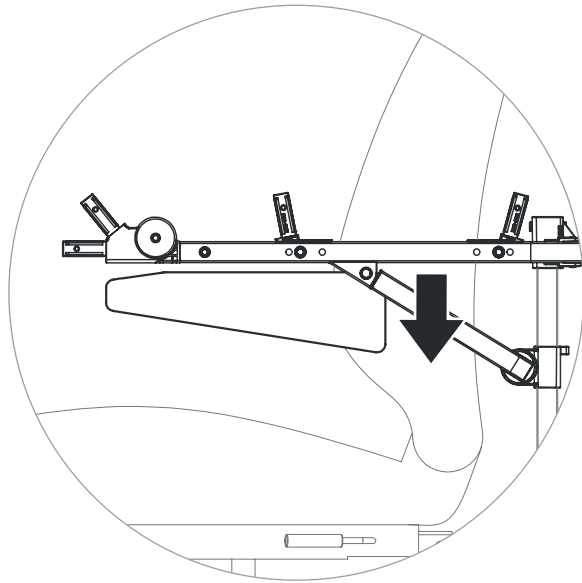
Insert 99-001 and tighten the two M5 x 1210 screws.

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




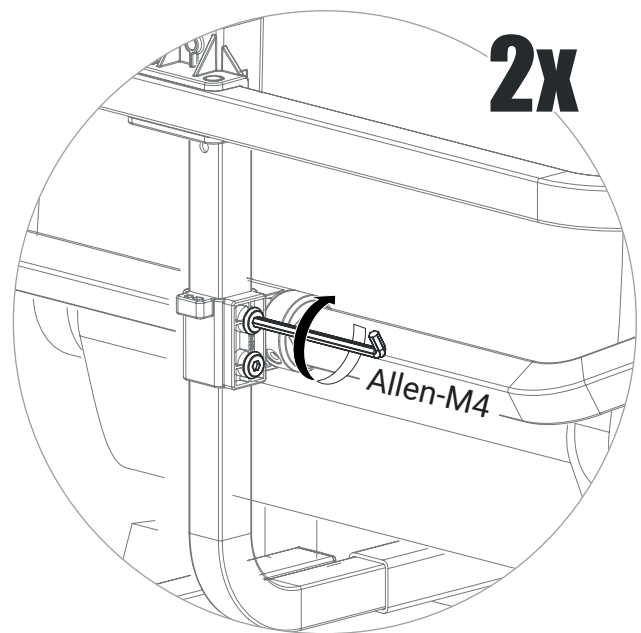
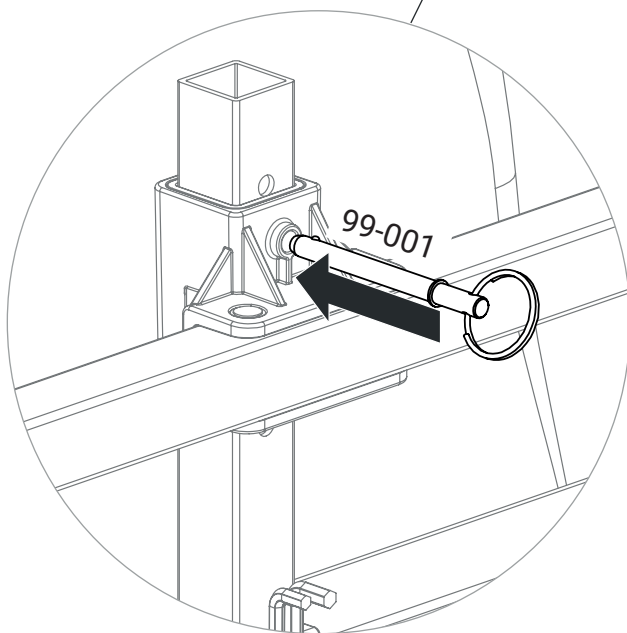
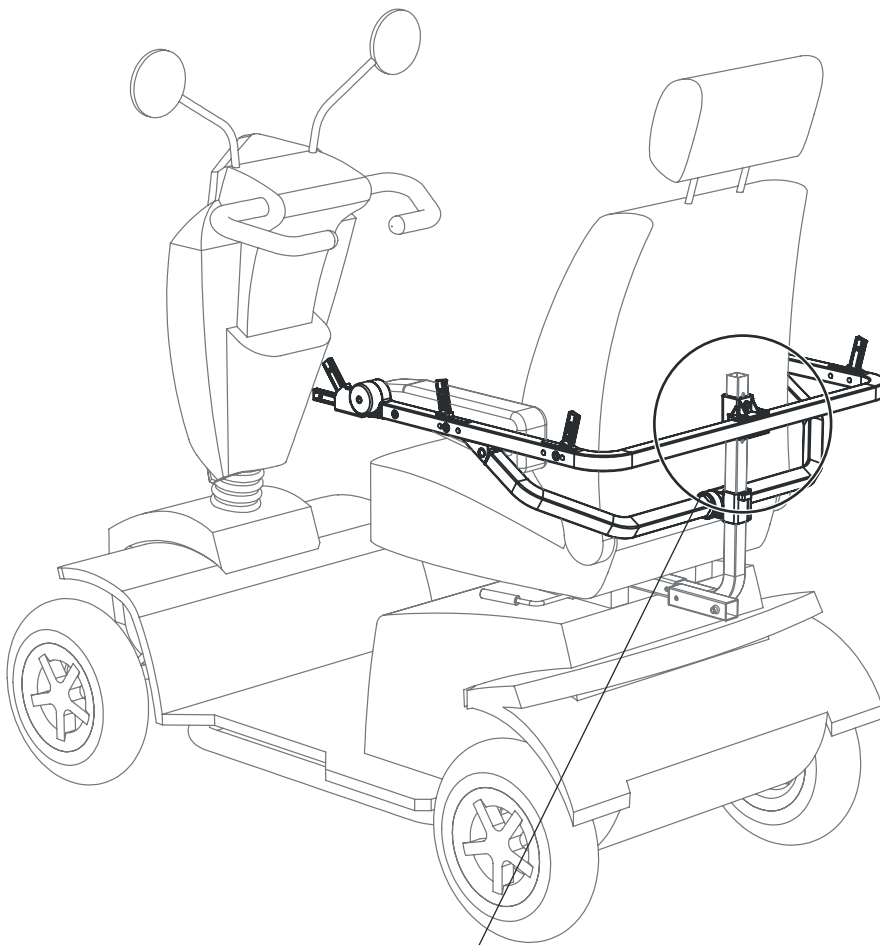
Remove 99-001 before proceeding to the next step.

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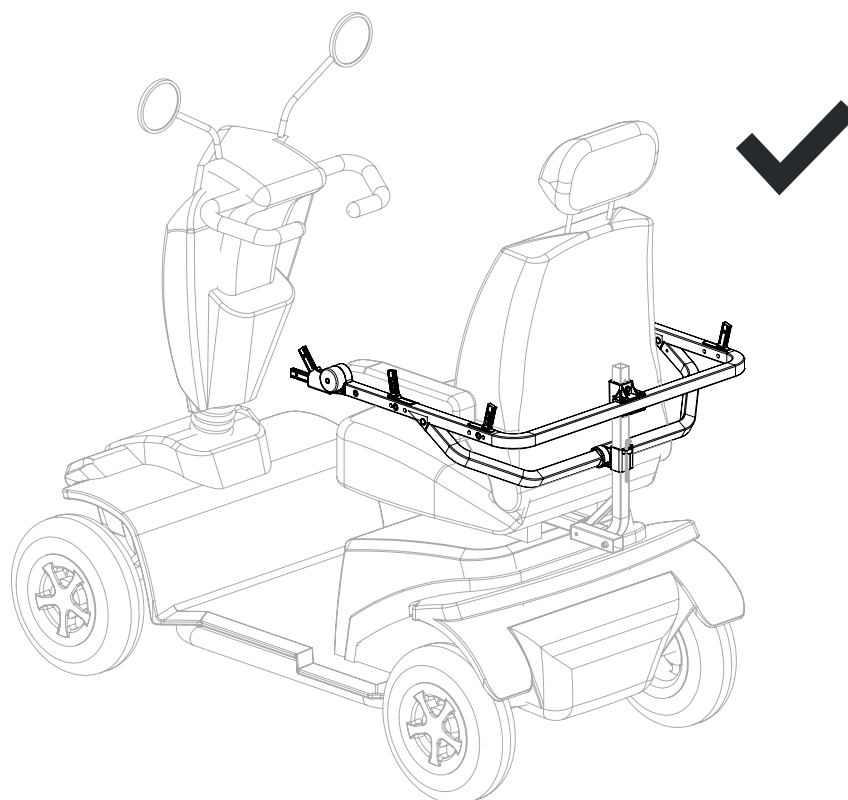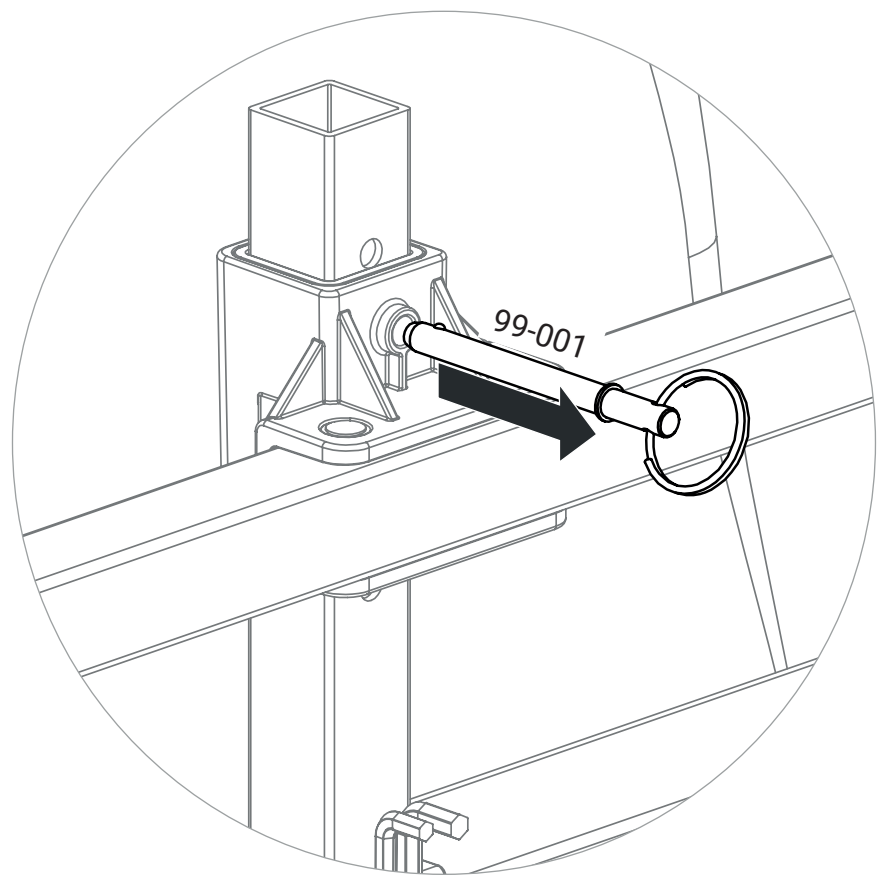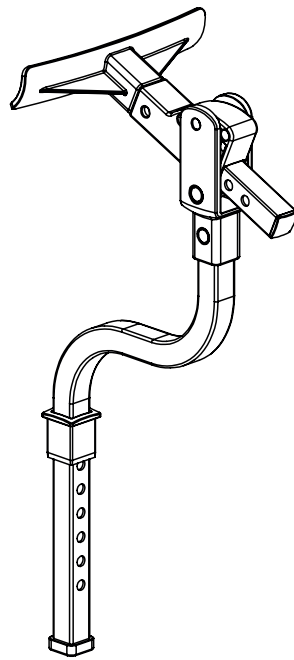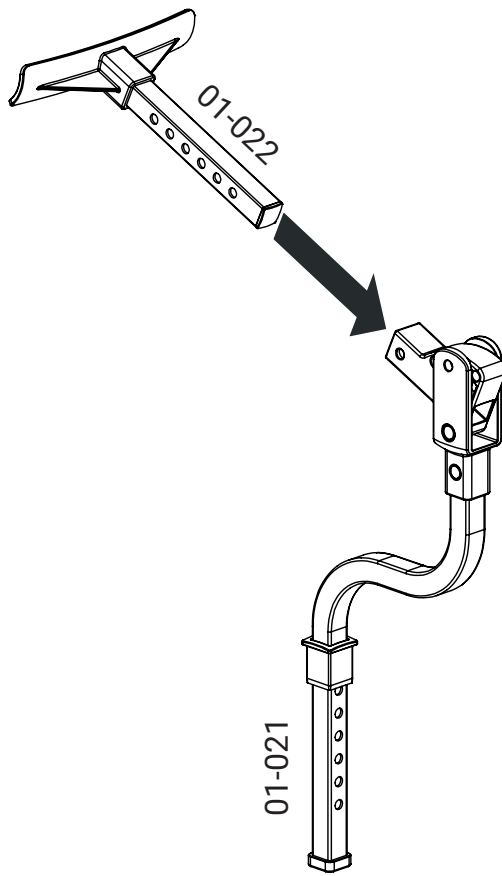
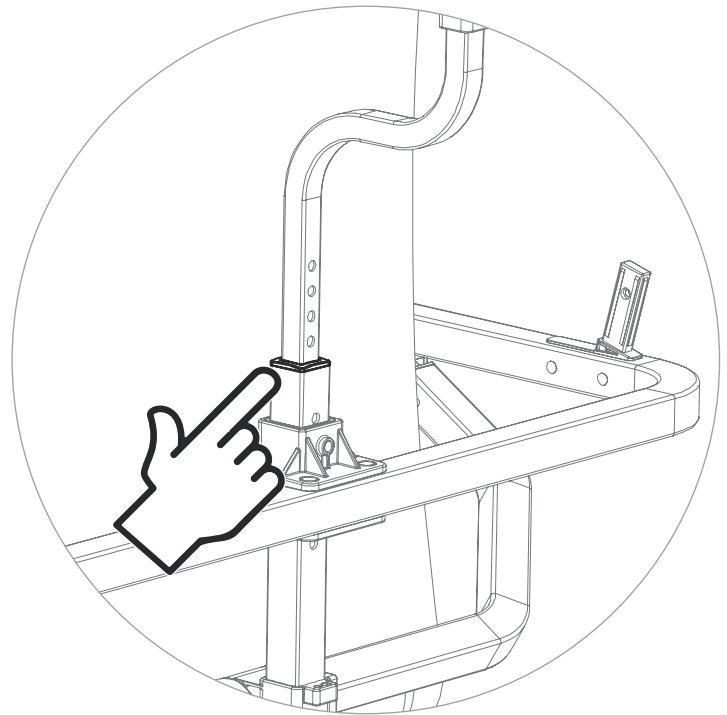
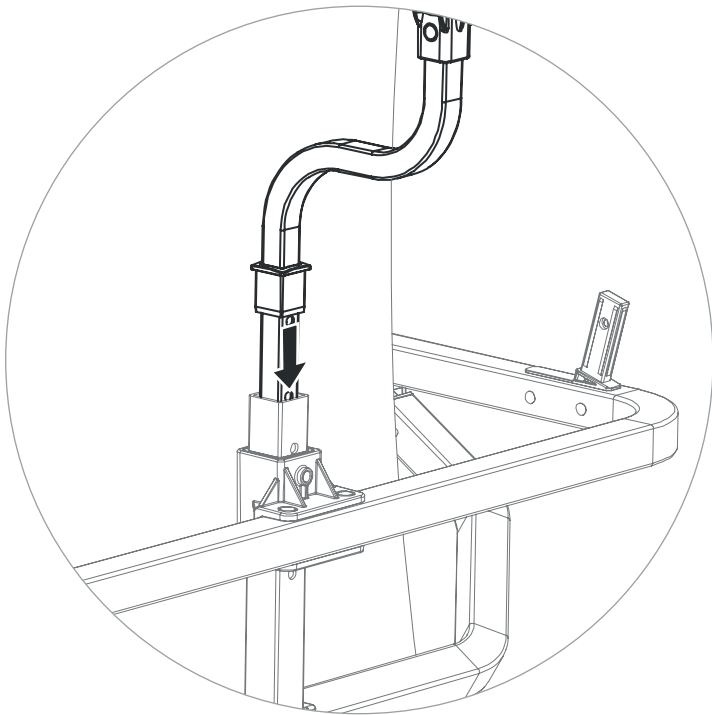
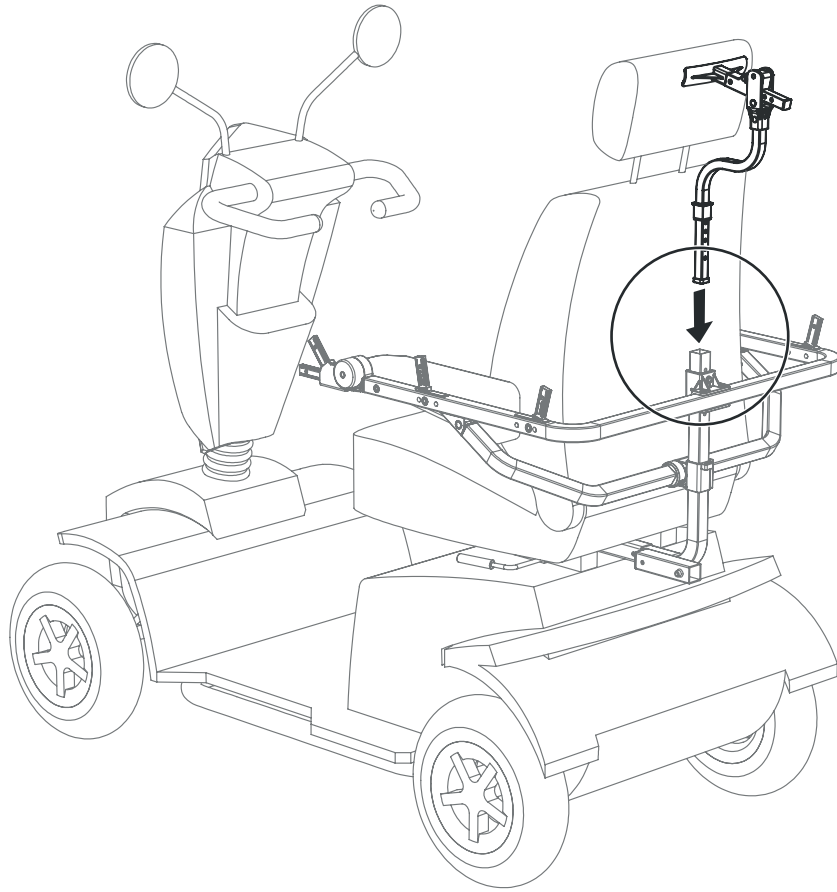
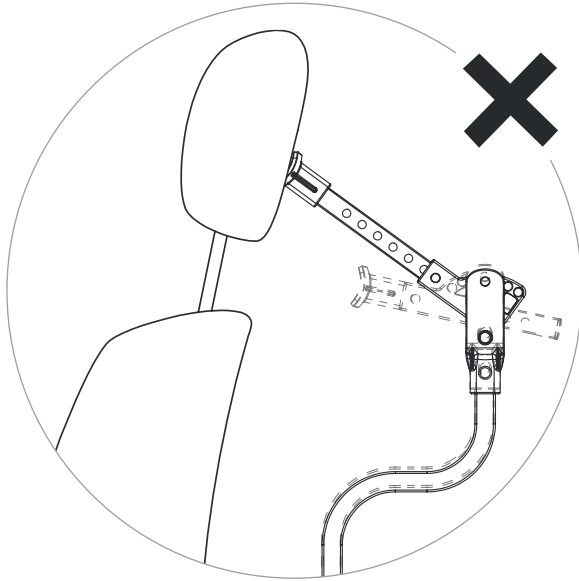
Adjust the height and angle of 01-021 so that 01-022 is pressing near the top of the backrest. Do not press 01-022 against the headrest.

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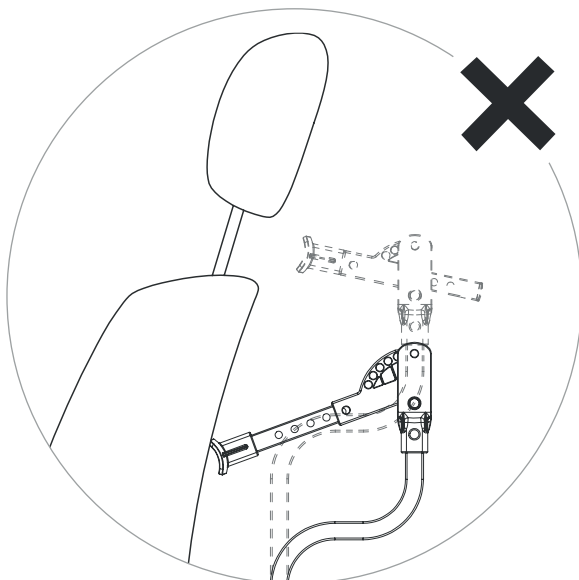
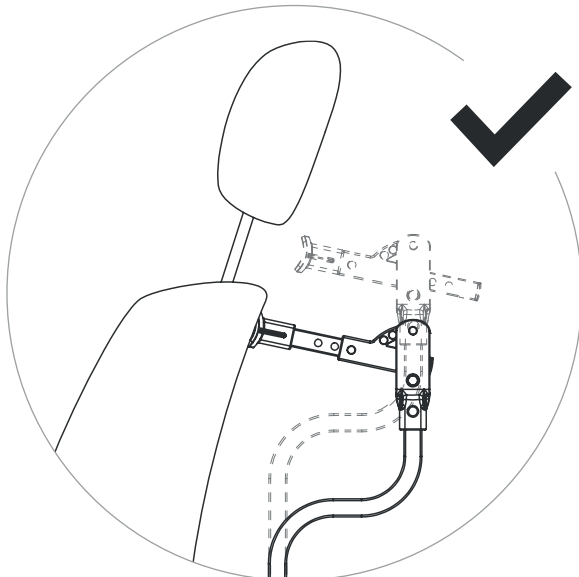
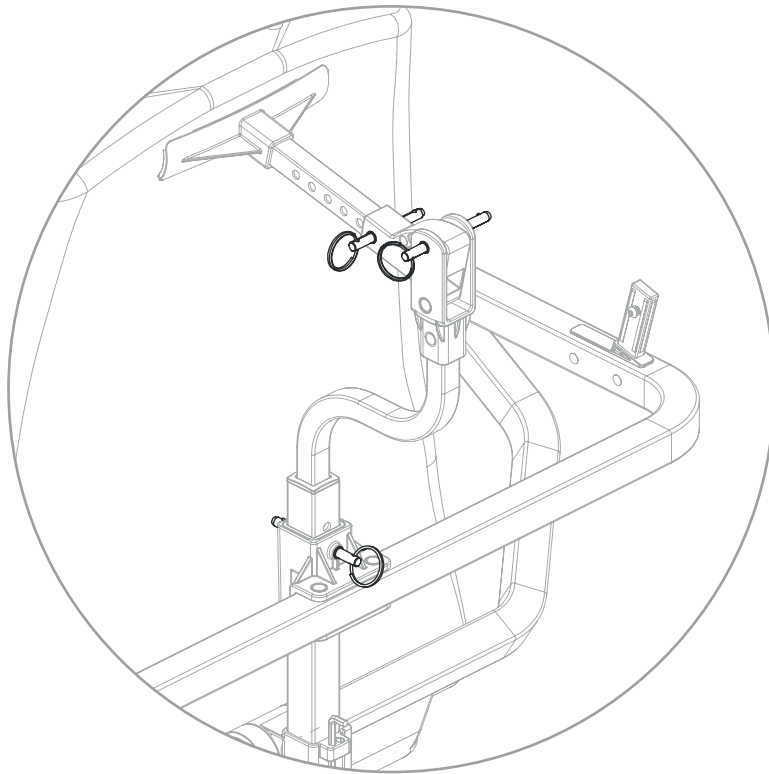
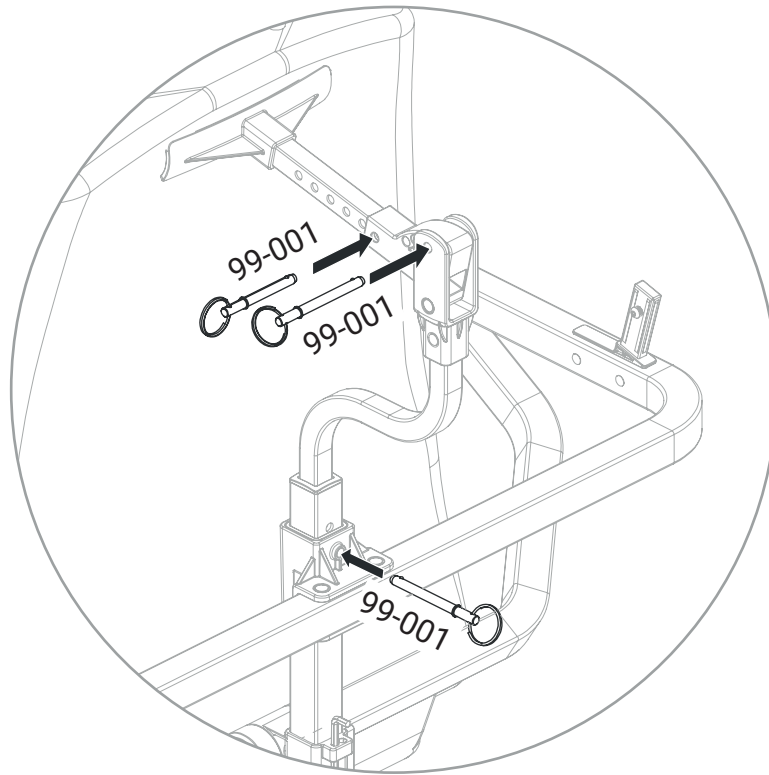
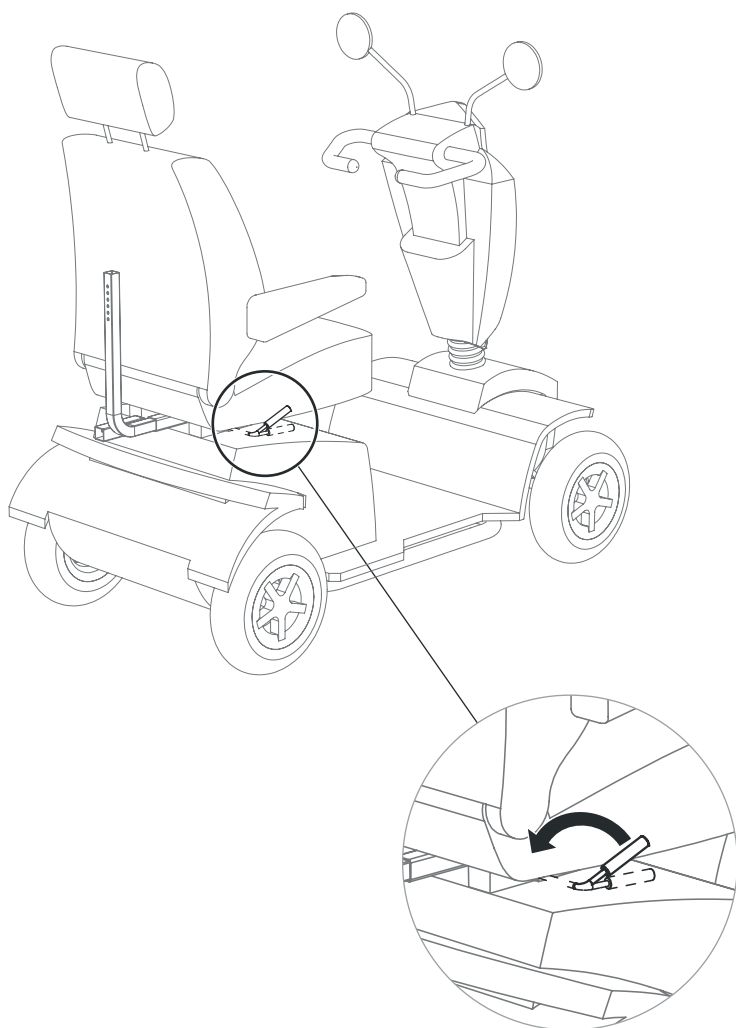
Recline the backrest back to its driving position. 01-021 should be pressed firmly against the backrest. Do not move the backrest past its driving position as it could damage 01-021.

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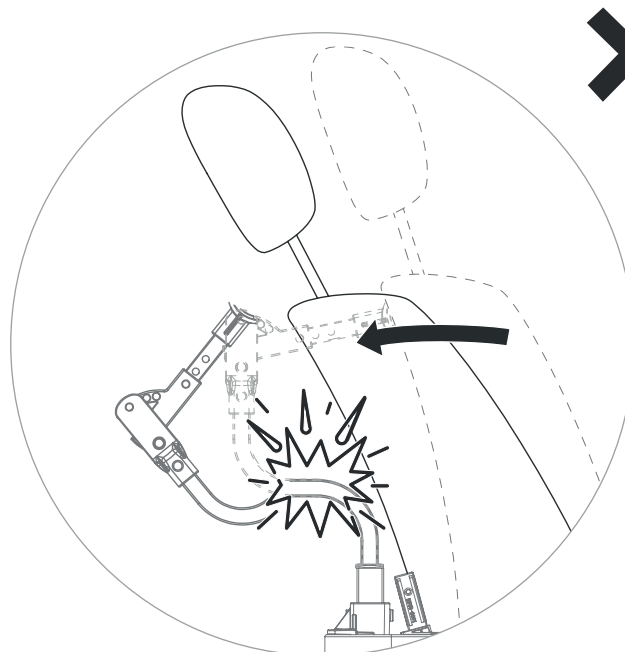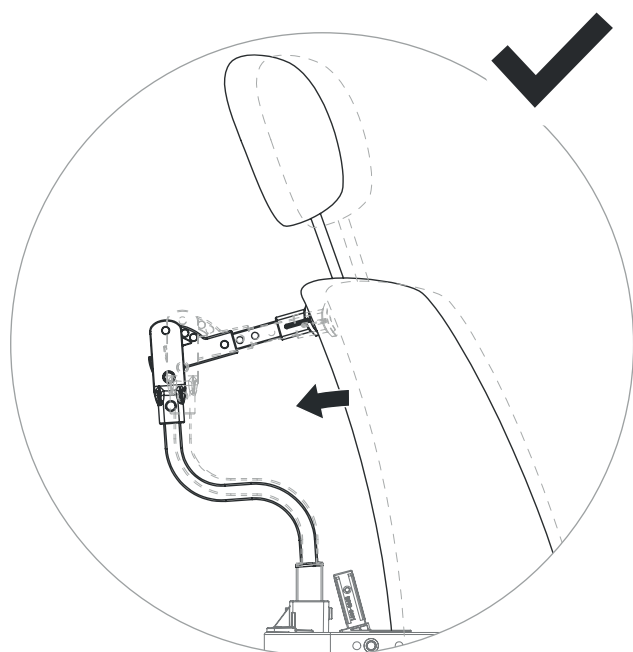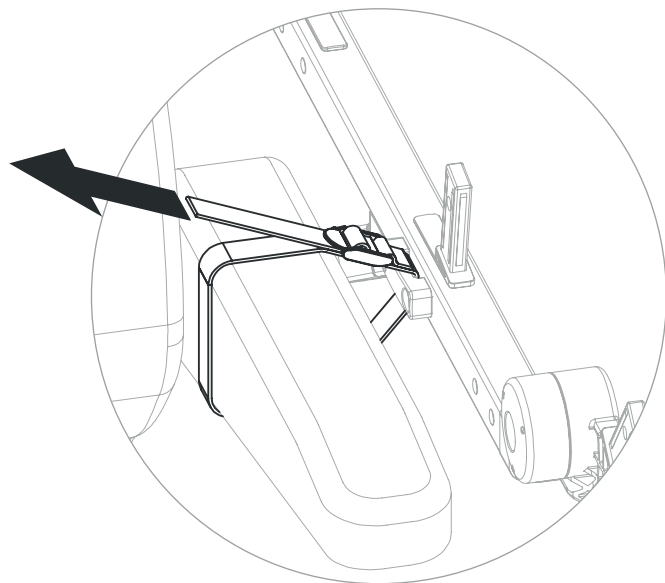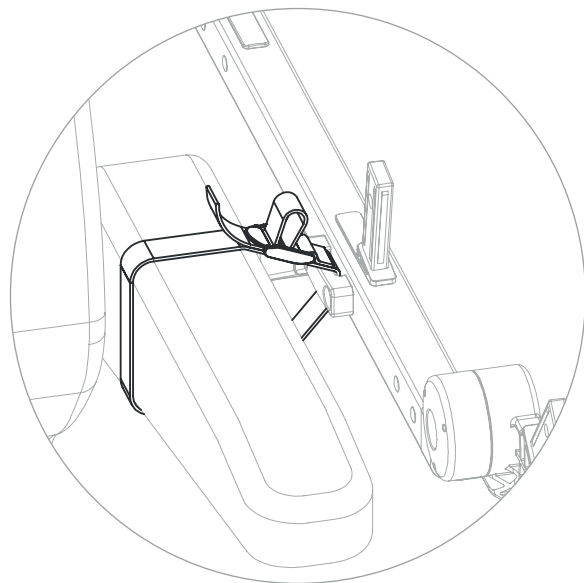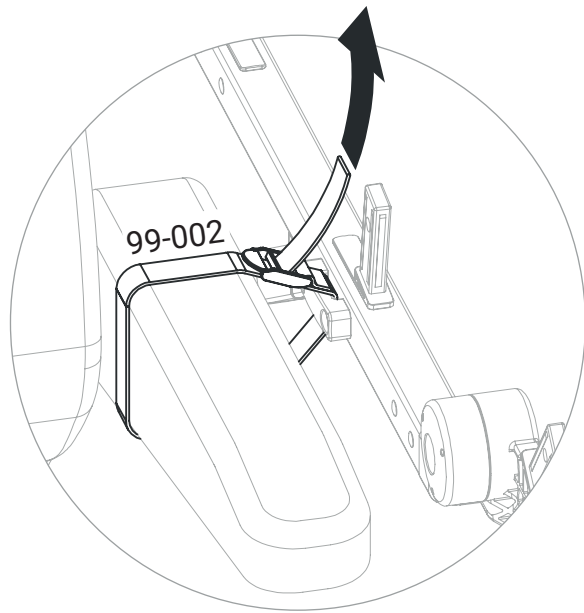
So far, 01-022 should be pressing against the backrest and 01-023 should be secured to the armrests using the 99-002 straps.

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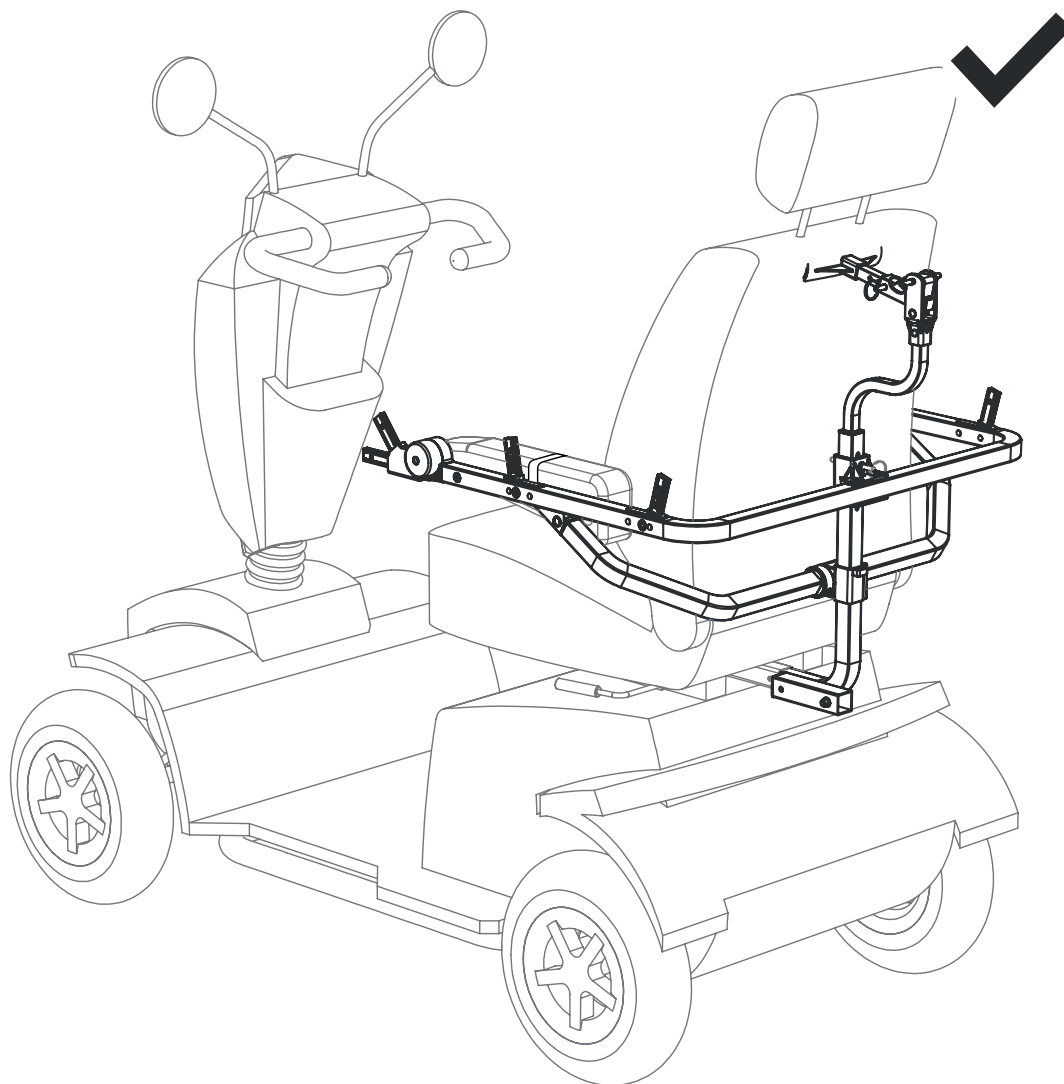
Make sure that all stickers are facing out. You should be able to see 2 and 4 from the same side and 1 and 5 from the other side.

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
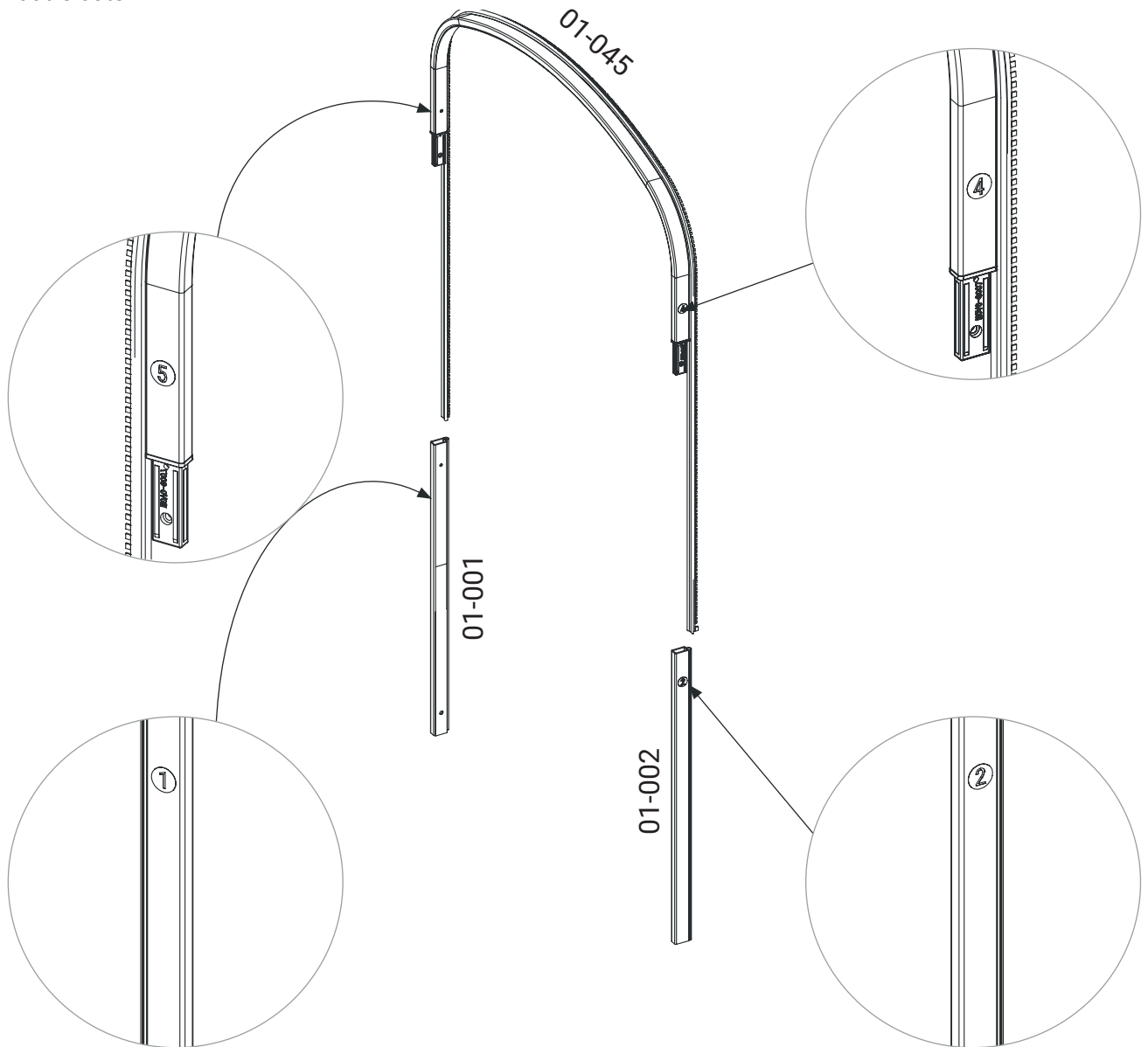
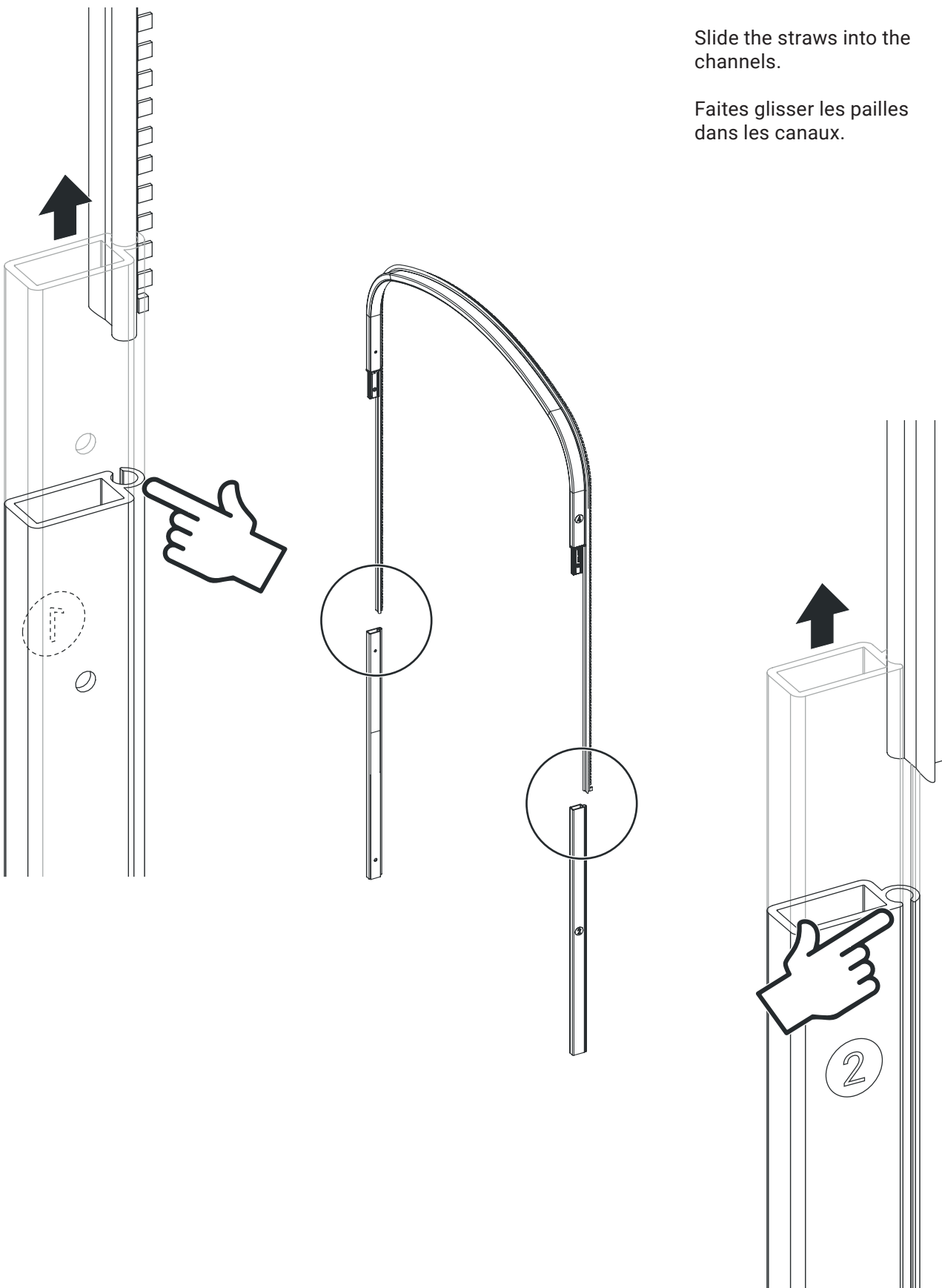
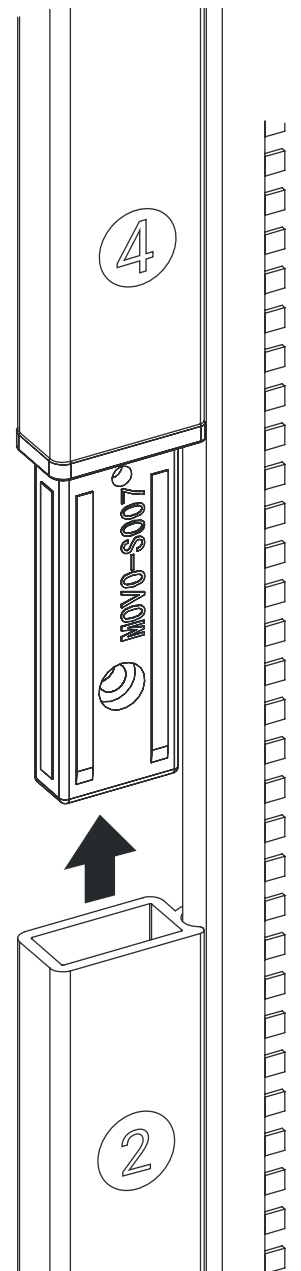
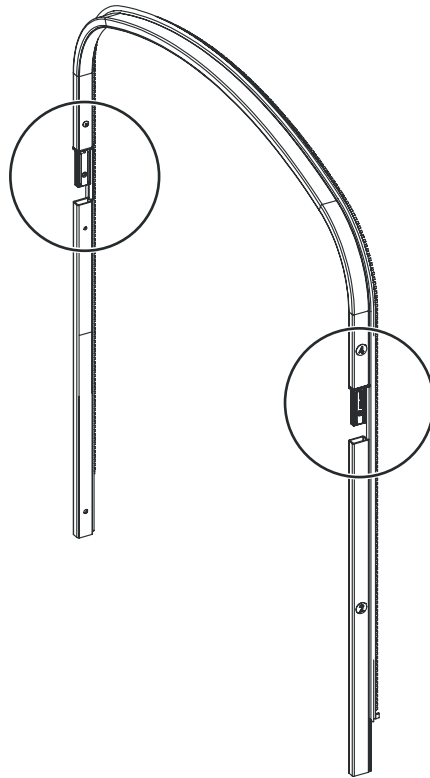
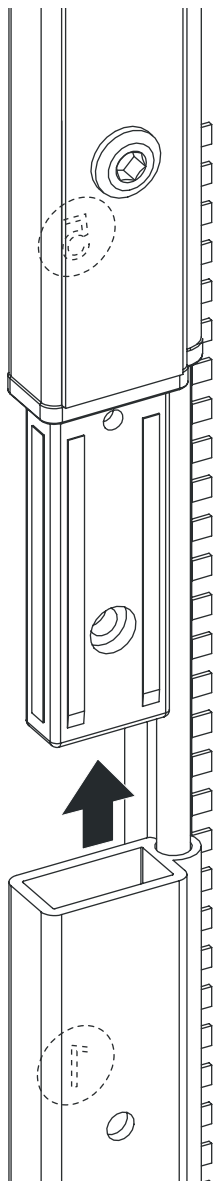
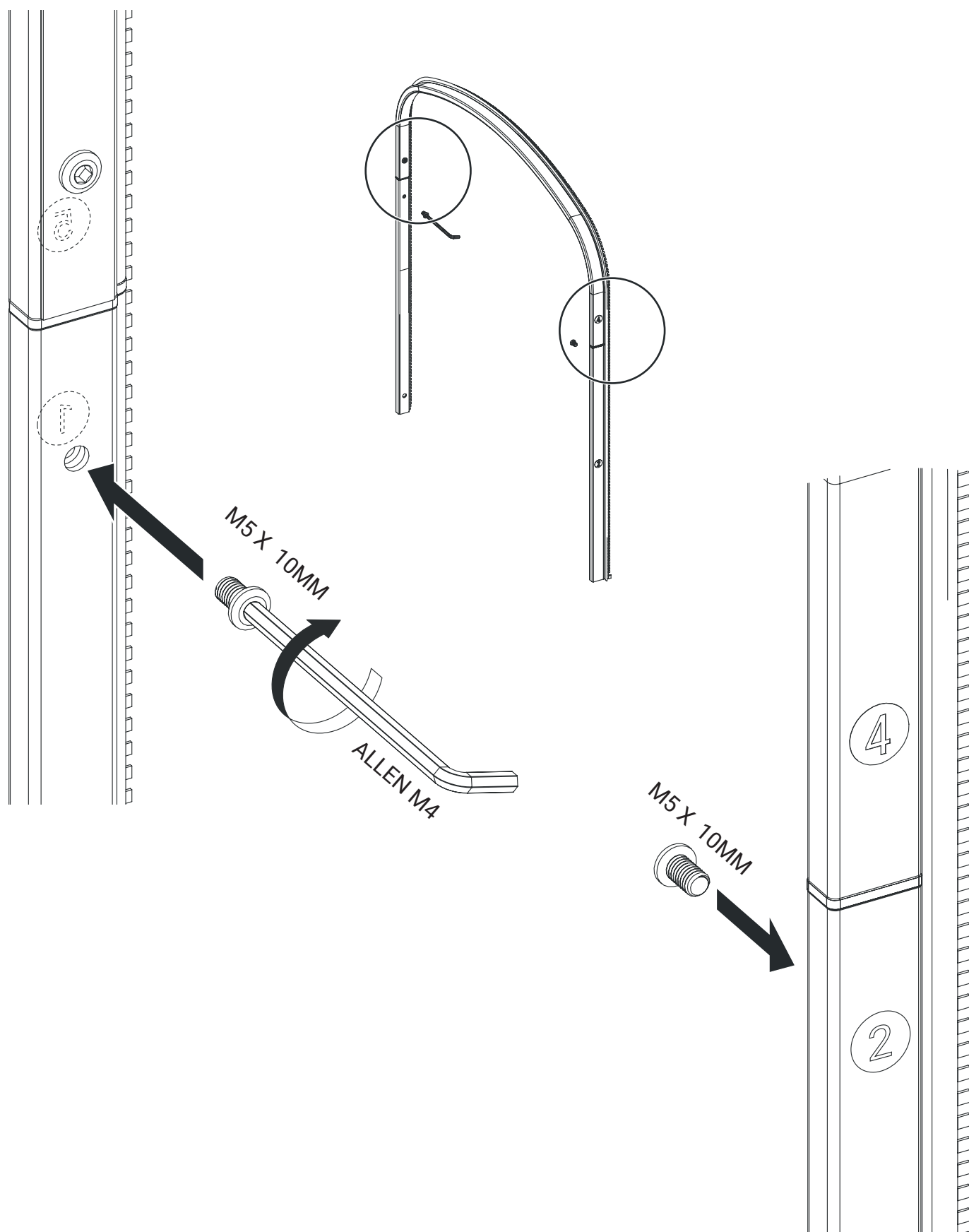
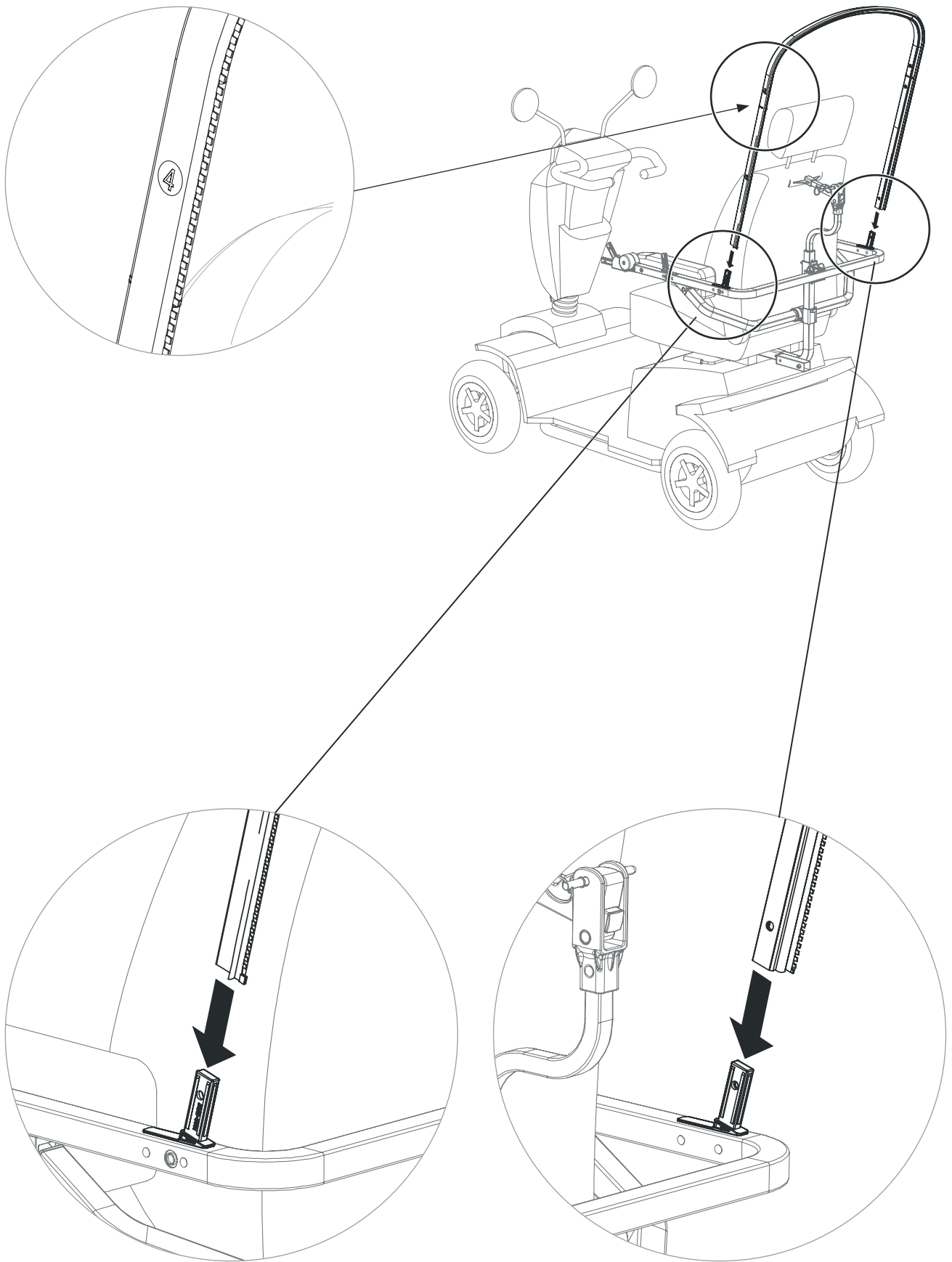
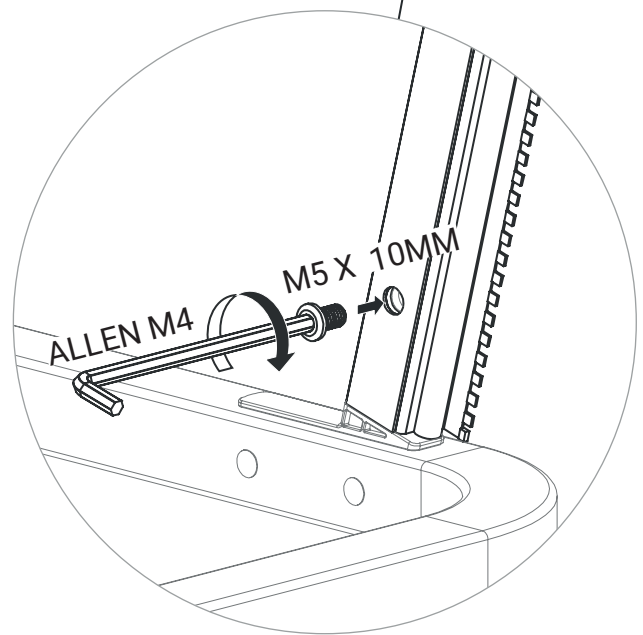
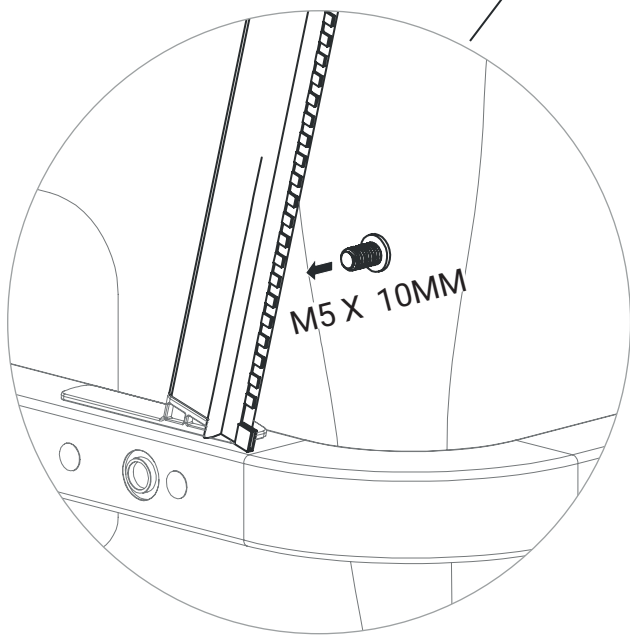
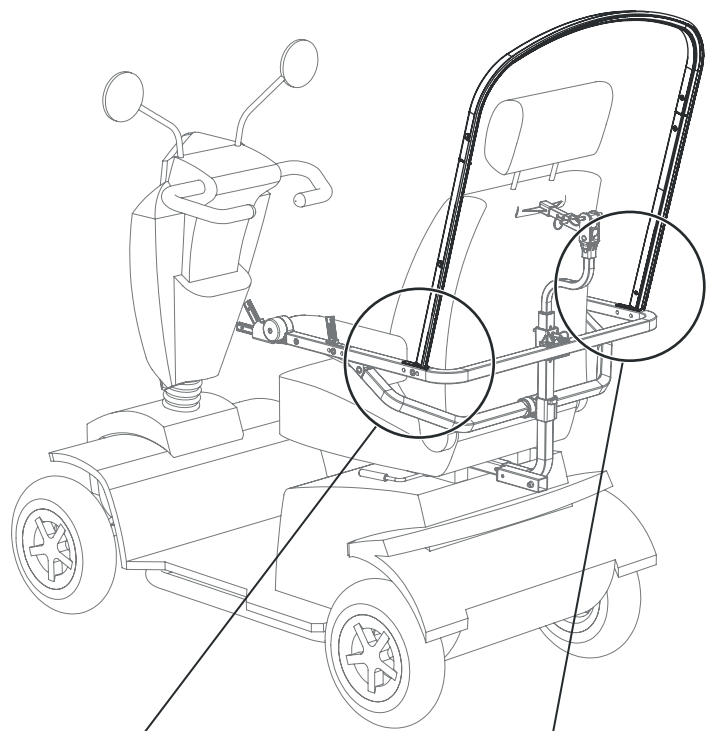
Make sure that all stickers are facing out. You should be able to see 7 and 3 from the same side and 6 and 13 from the other side.

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



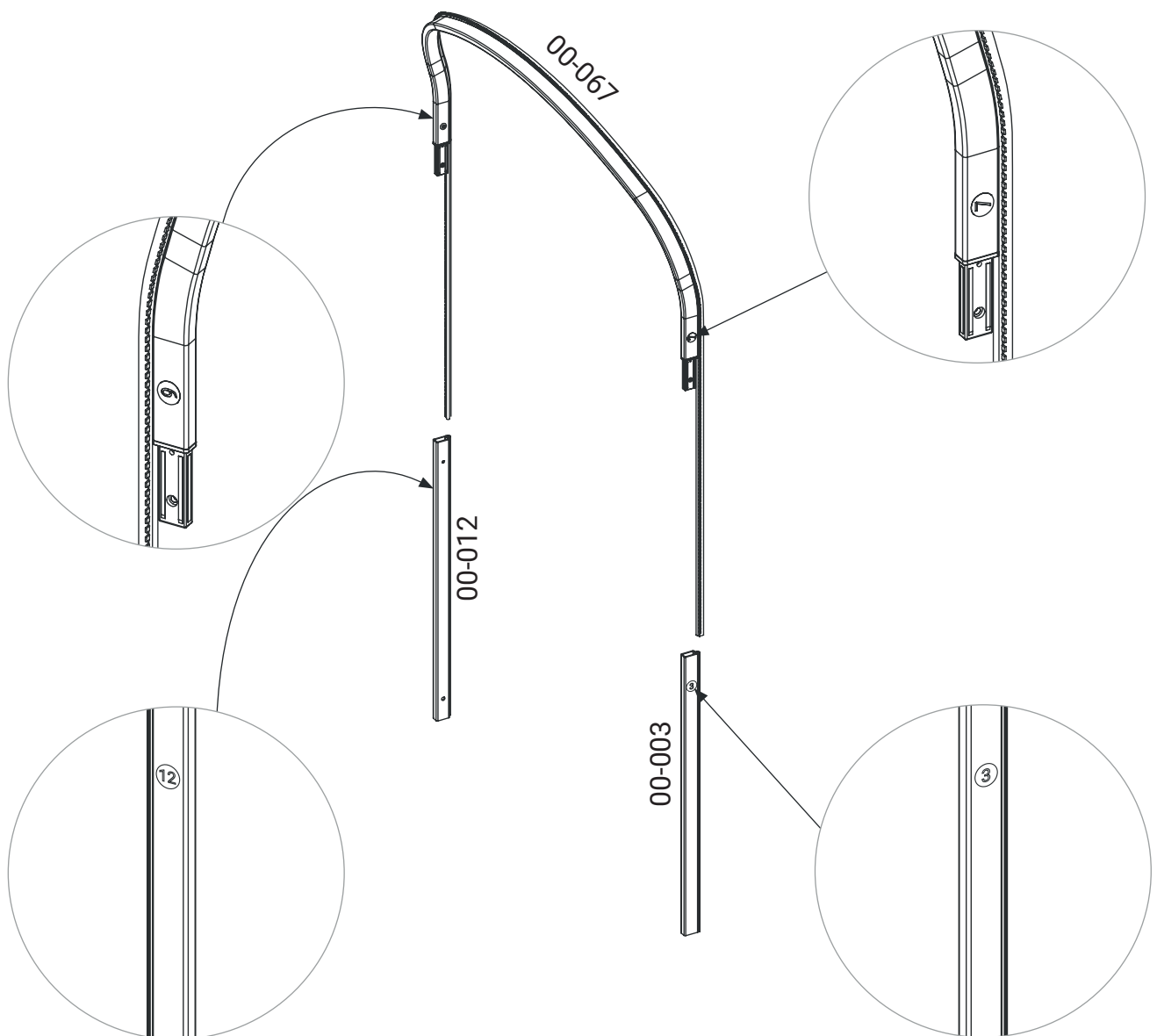
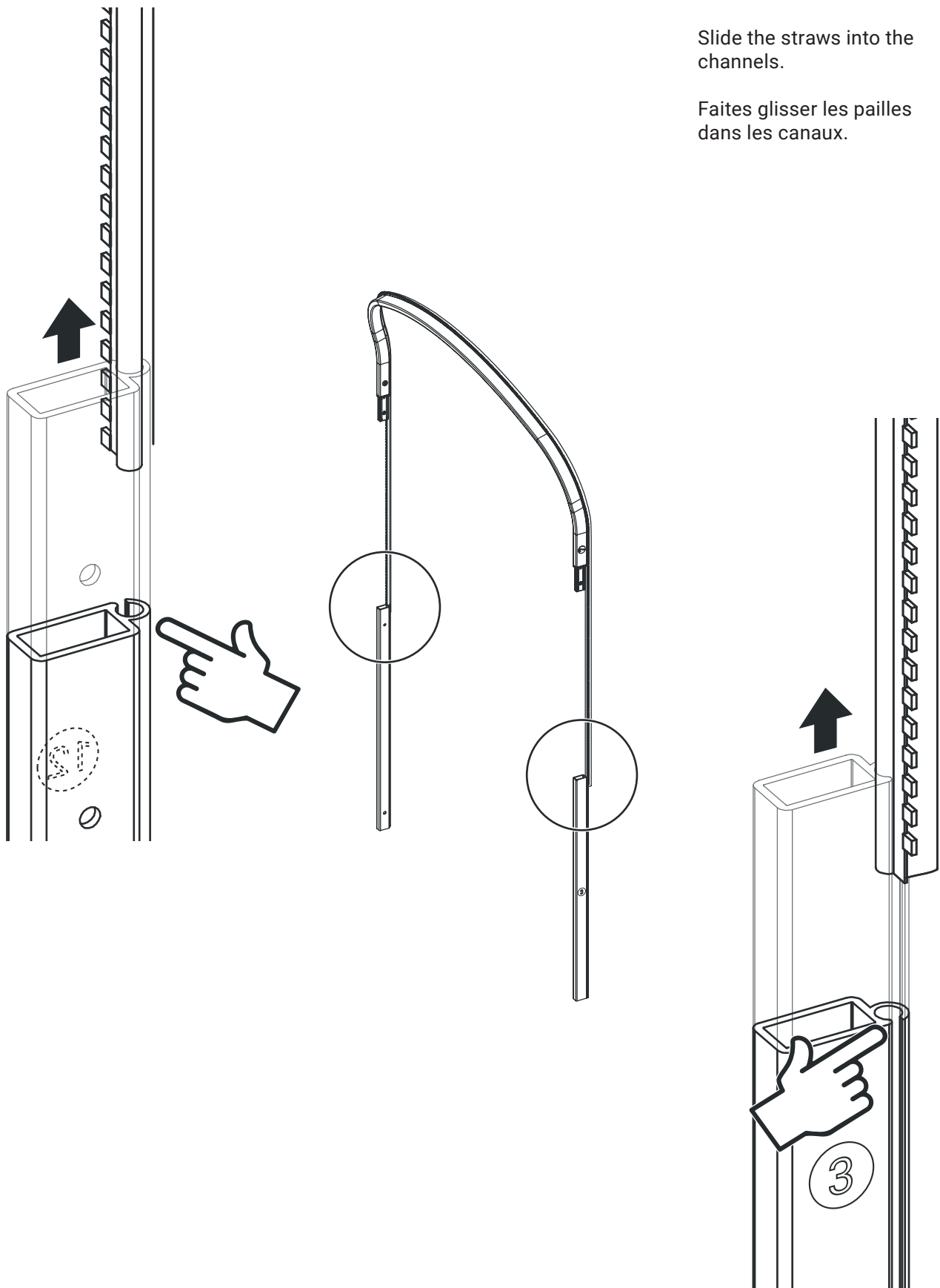
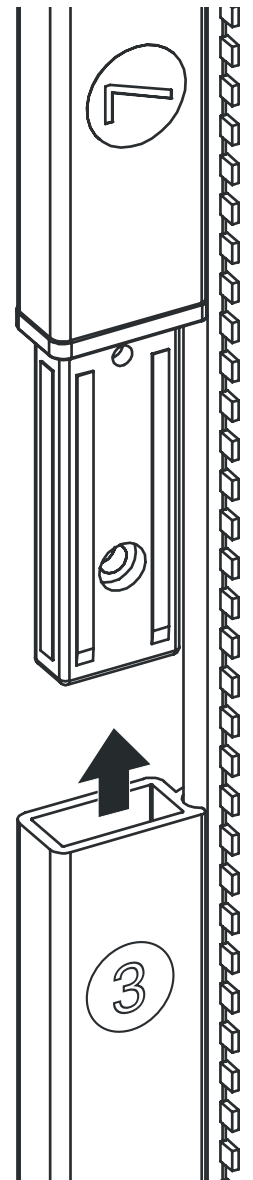
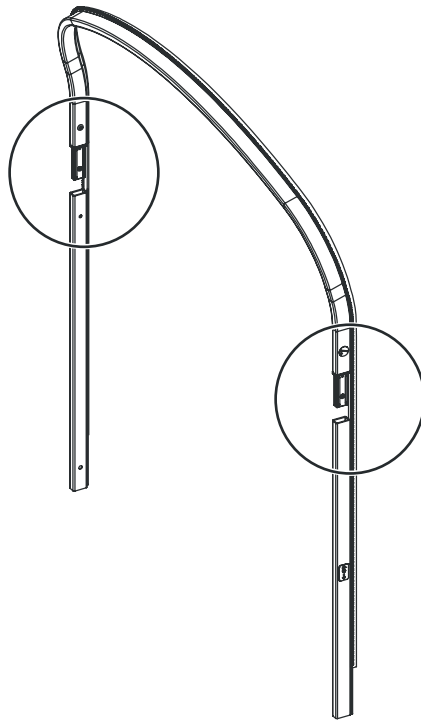
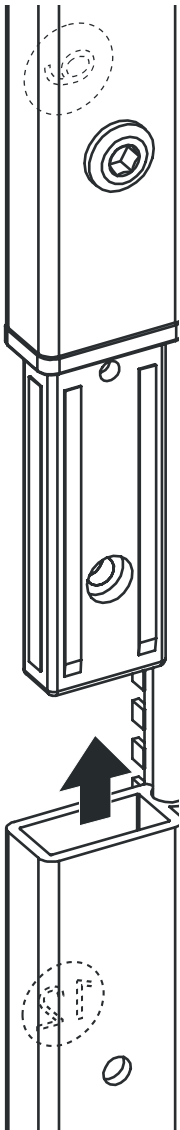
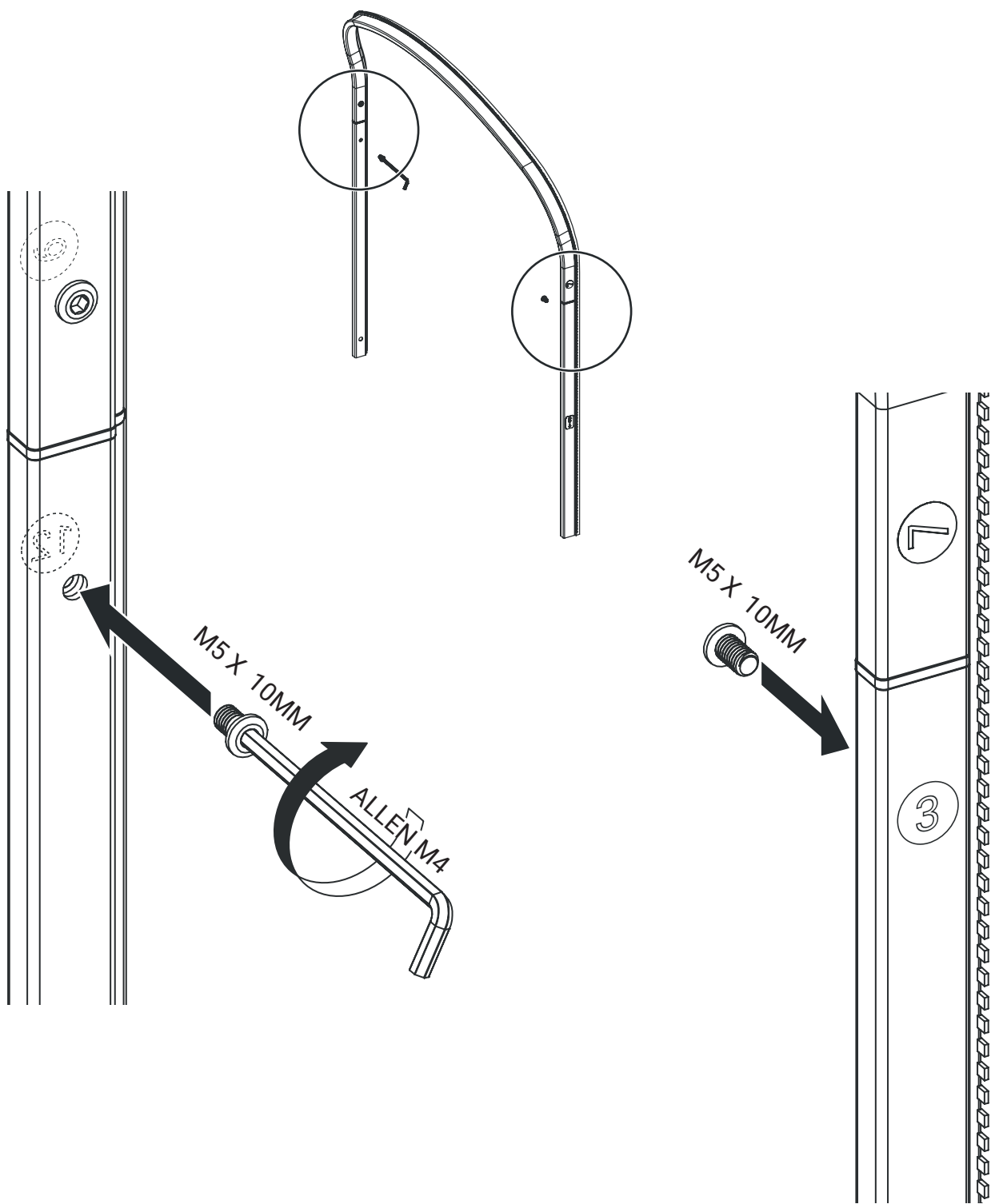
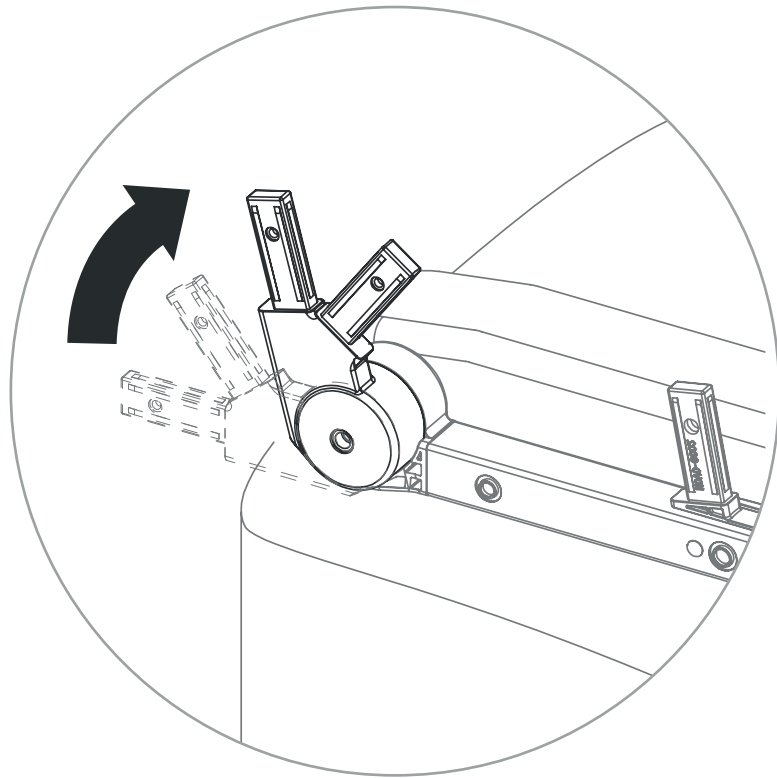
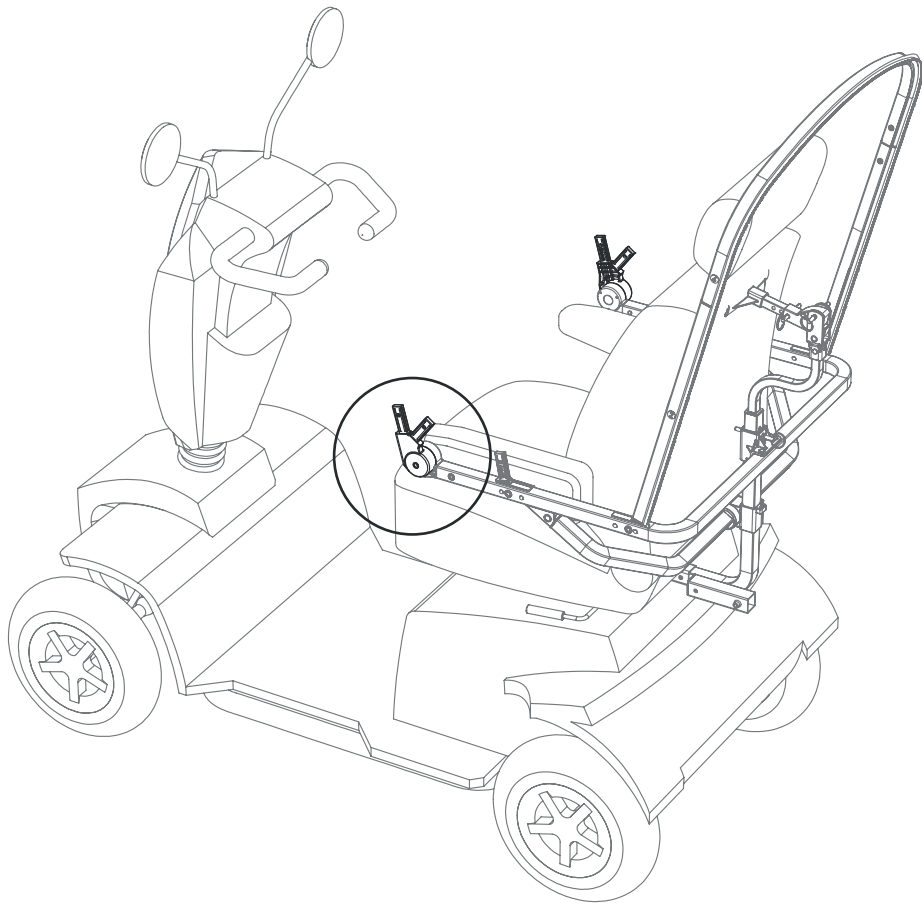
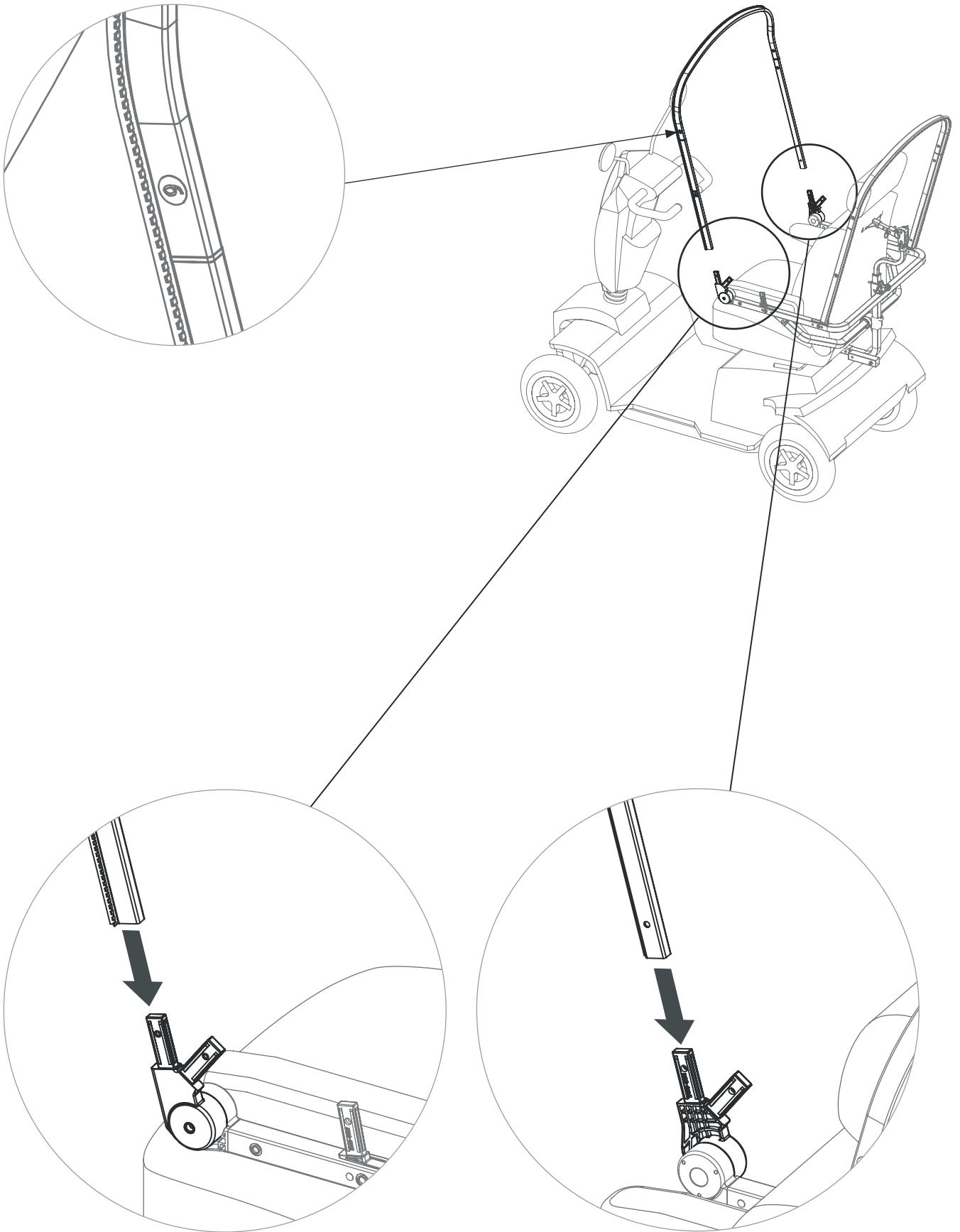
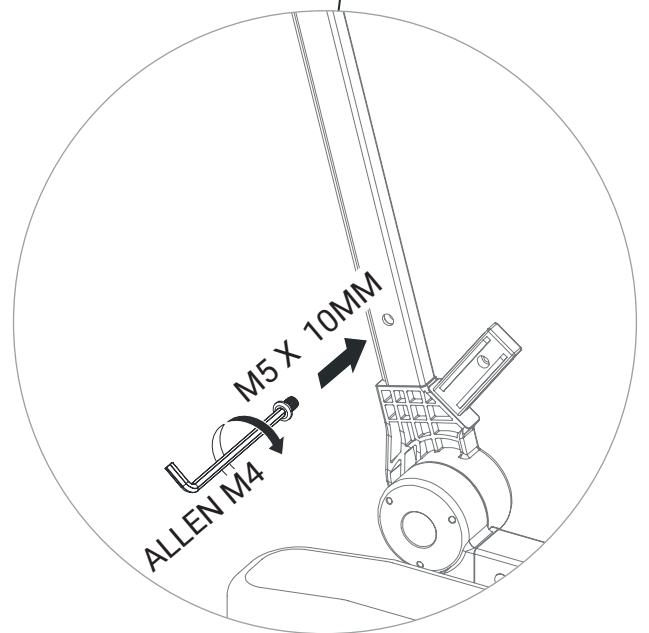
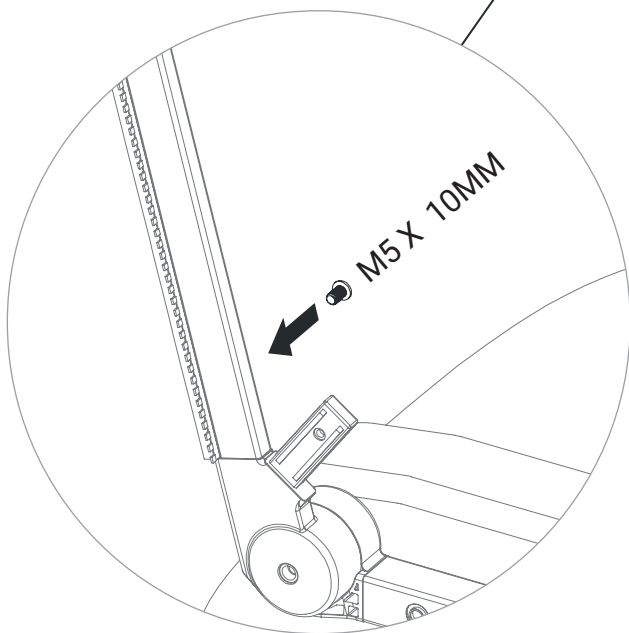
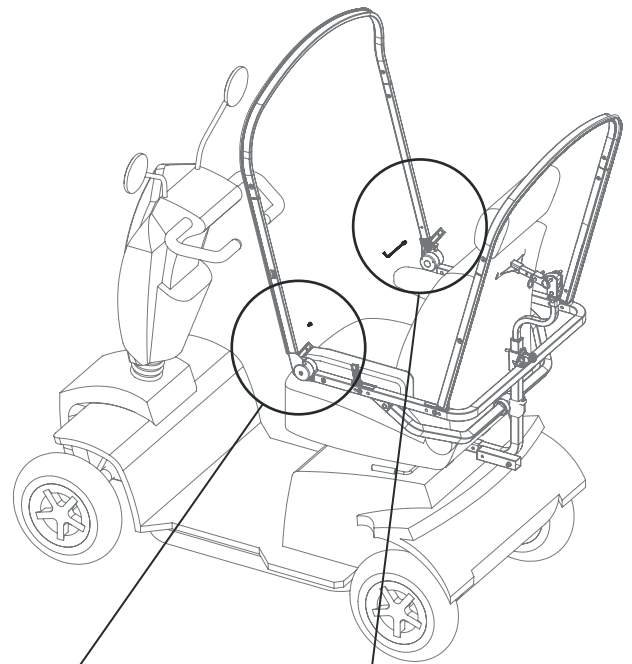
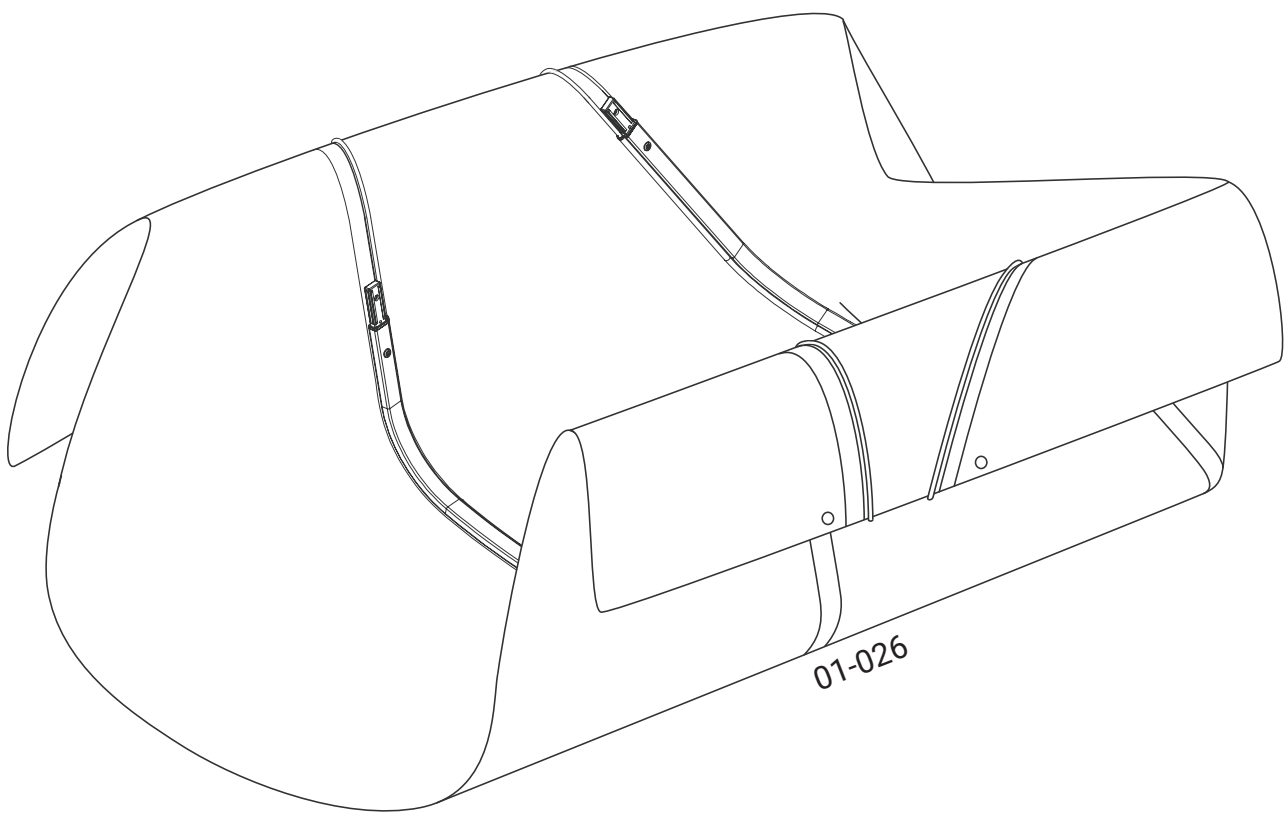
Orient the fabric so you can see sticker 10 and 11. Slide straw 11 in channel 13 and straw 10 in channel 14.

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


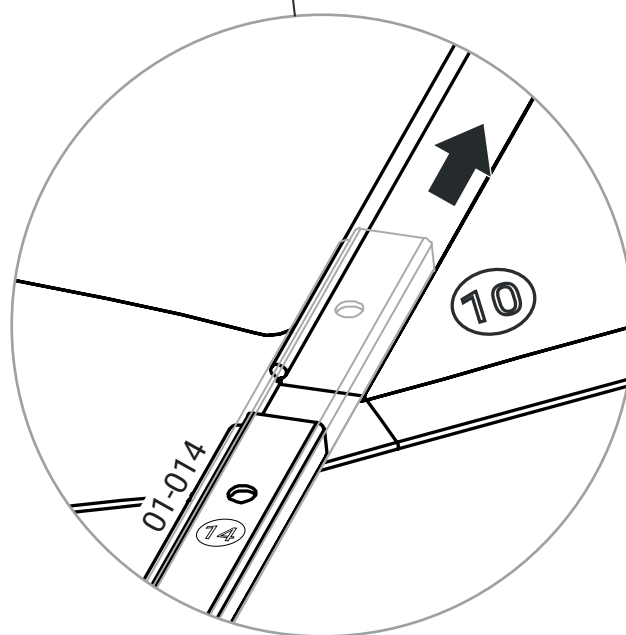
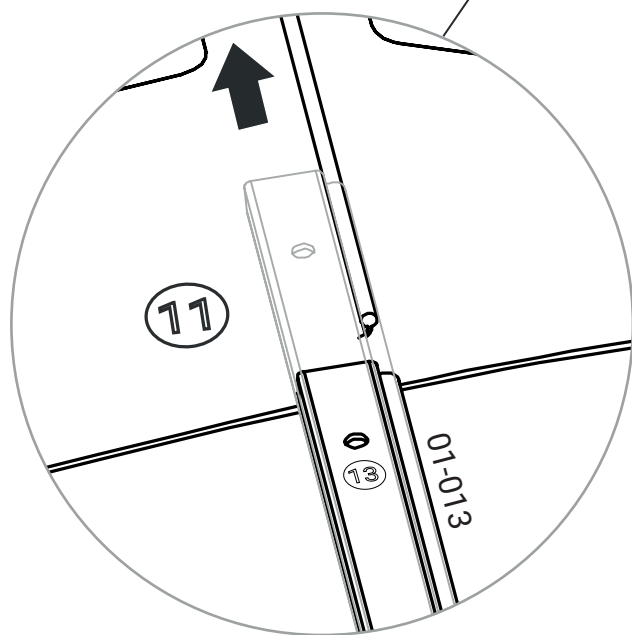
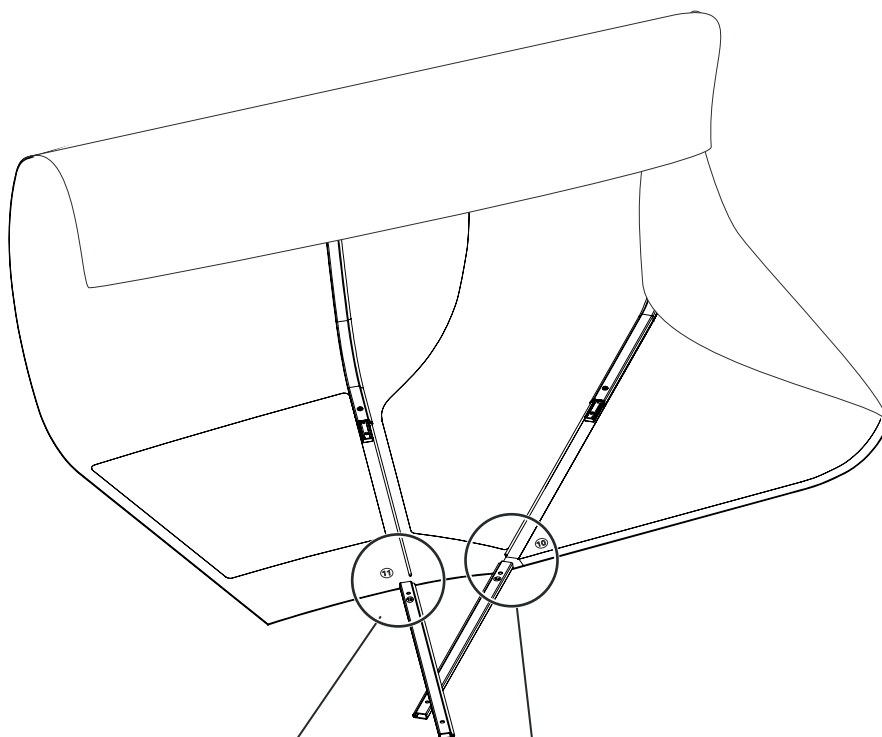
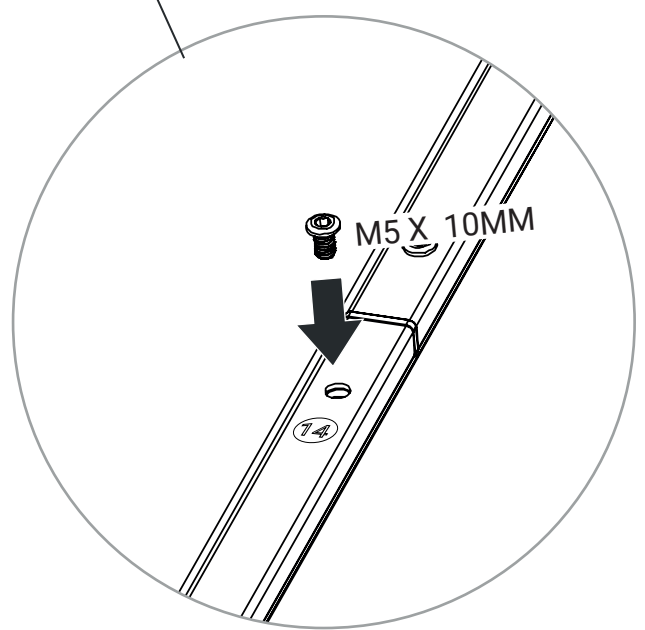
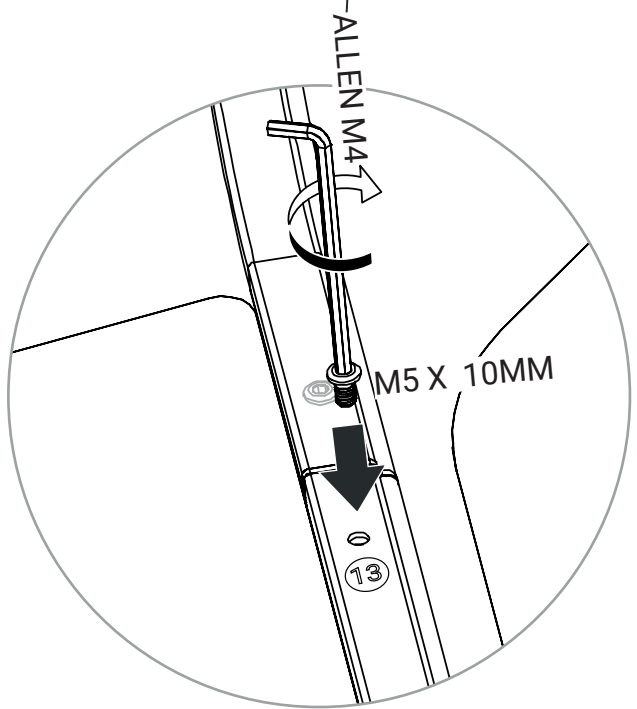
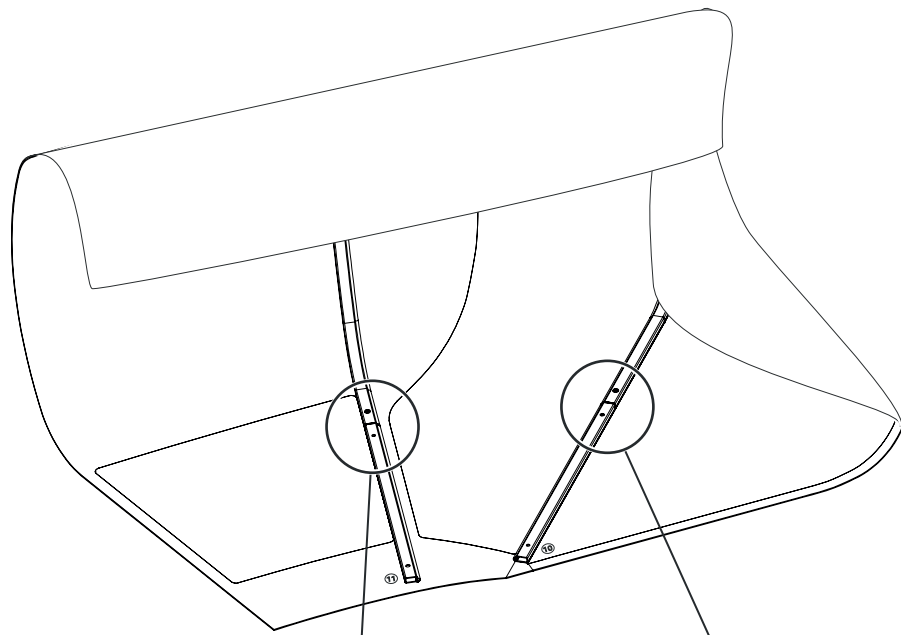
Orient the fabric so you can see sticker 8 and 9. Slide straw 8 in channel 13 and straw 9 in channel 14.

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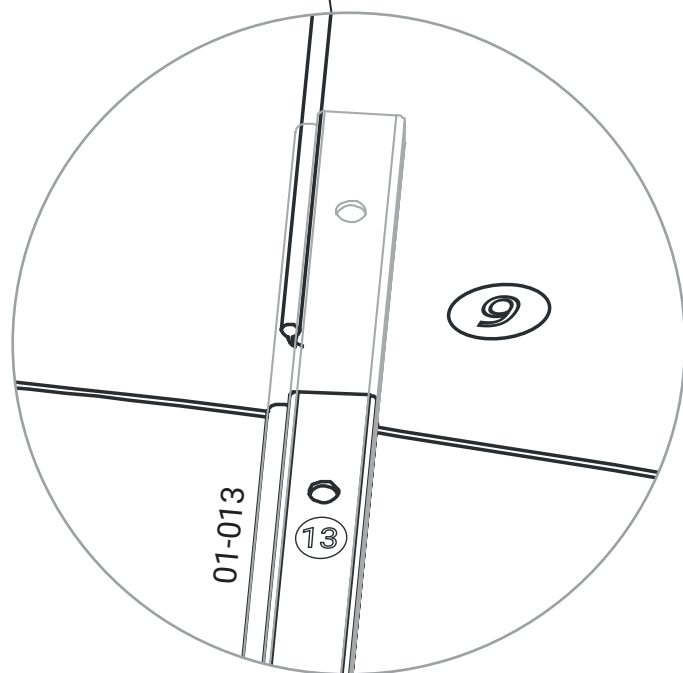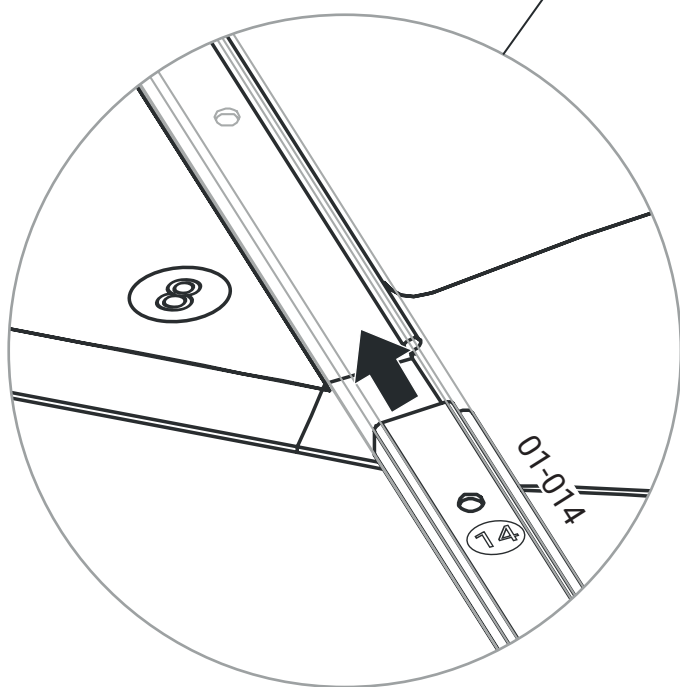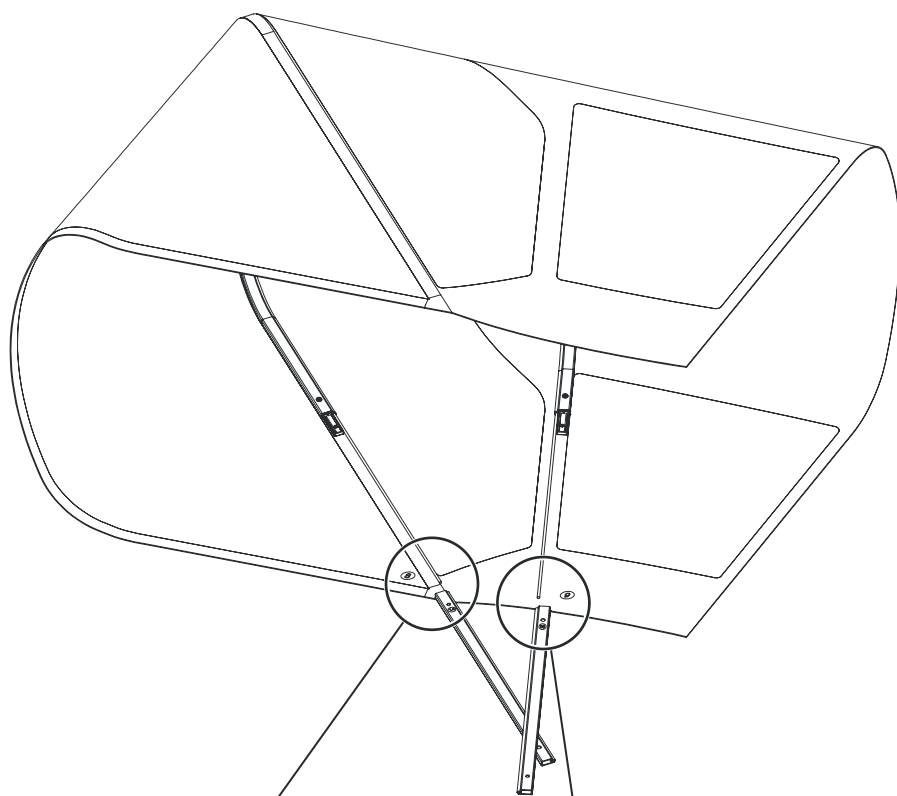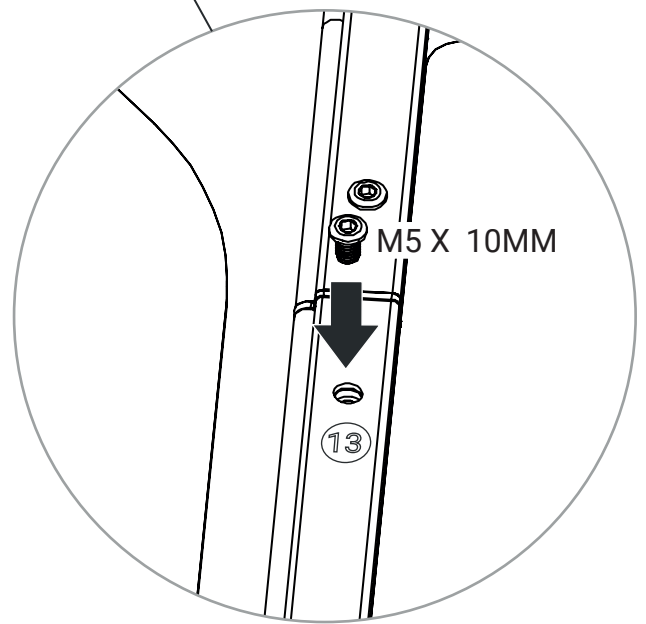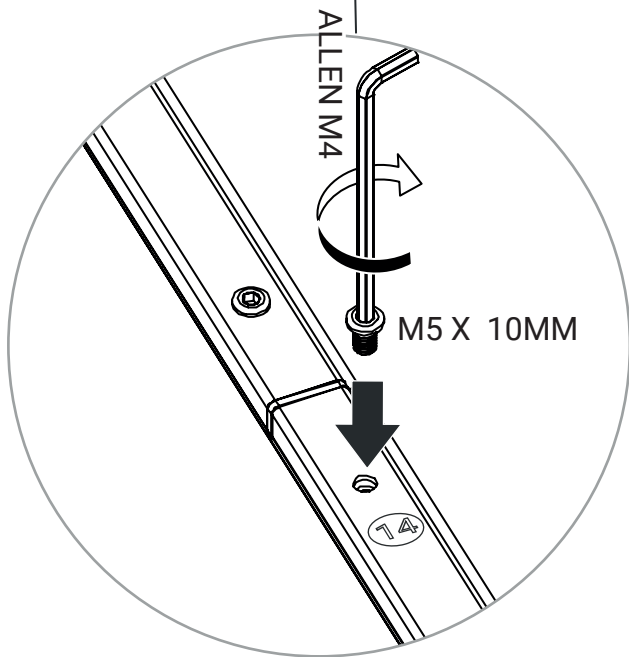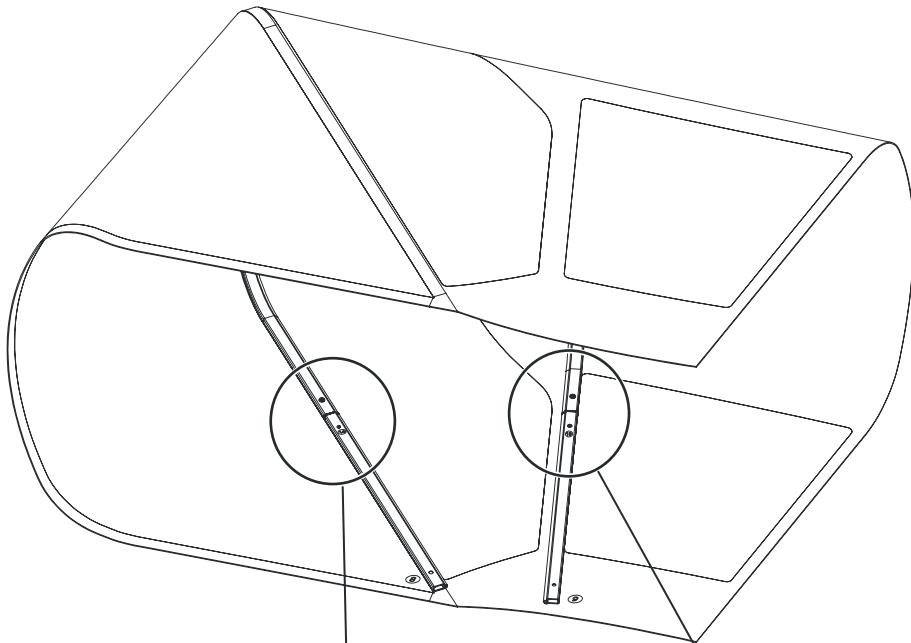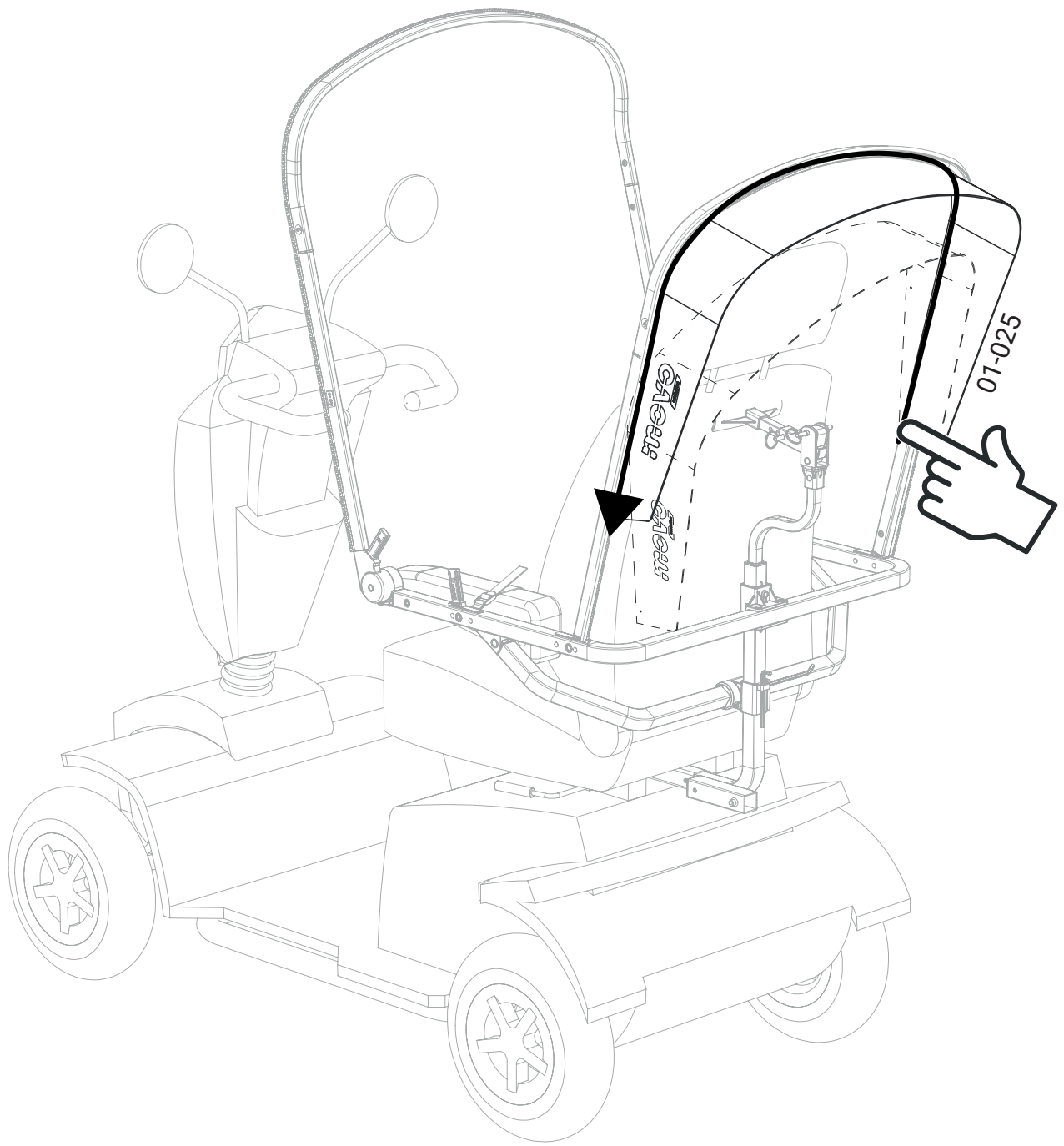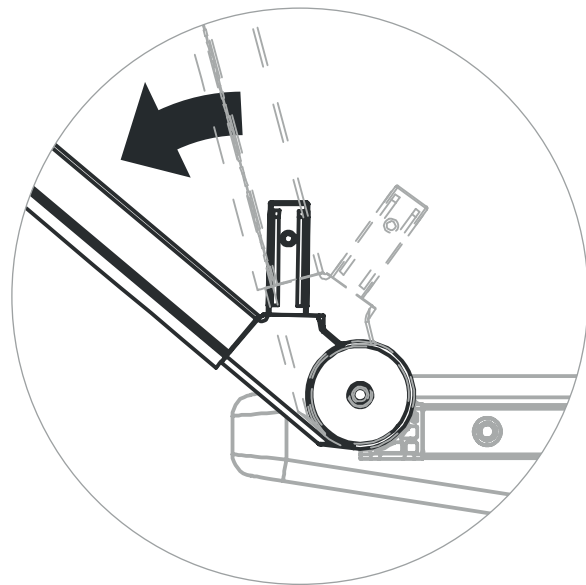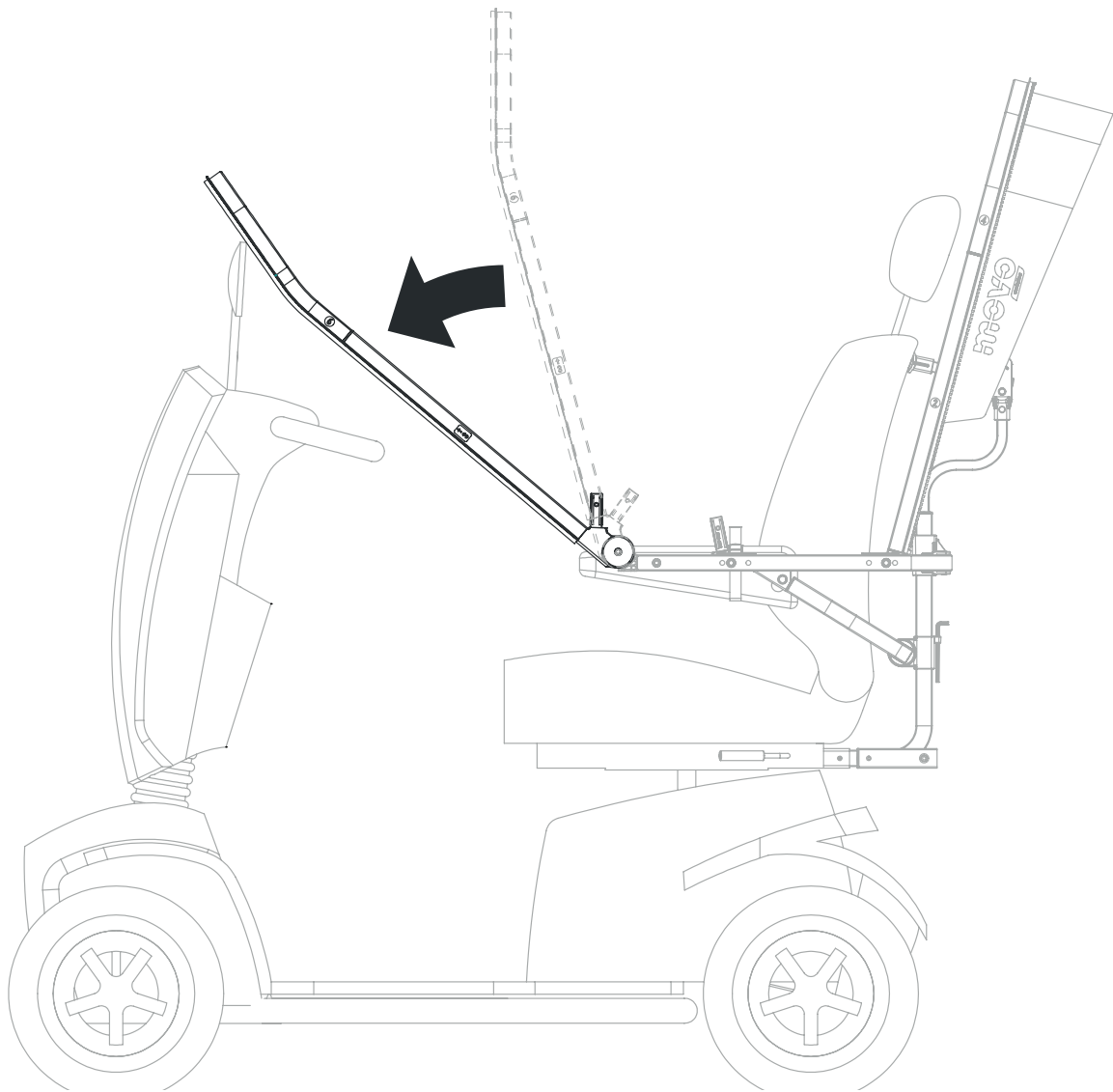It is recommended to work in pairs to align the fabric structure.

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