Moulded Valved Face Mask FFP2 NR

- FR Masque respiratoire moulé à valve
- DE Atemschutzmasken mit Ventil, geformt
- ES Mascarilla moldeada con válvula
- IT Scatola mascherina sagomata con valvola
- NL Gevormd stofmasker met ventiel
- PL Półmaska formowana z zaworem

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EN

Specification

Standard.....	EN149:2001+A1:2009
Shelf Life.....	Expiry date on product or individual packaging
Usage.....	Single Use
Protection Level.....	FFP2 NR
Protection duration.....	≤ 8hrs
Size.....	Standard
PPE Regulation 2016/425/EU	

Explanation of standards and protection levels

Mark	Meaning
EN149:2001+A1:2009	Respiratory standard number
FFP1 NR	Class 1, low filtering efficiency. Single use
FFP2 NR	Class 2, medium filtering efficiency. Single use
FPP3 NR	Class 3, high filtering efficiency. Single use

General Safety

- ⚠ WARNING:**
Always ensure the product is:
 - Suitable for the application
 - Always fitted correctly
 - Always worn during periods of exposure
 - Immediately replaced when necessary
- This product may be adversely affected by certain chemical substances. Contact Silverline for additional information
- To protect the wearer from hazard, correct training, selection, use and maintenance are required, in accordance with Silverline's instructions
- Always adhere to all WARNING statements on the product, on the information supplied with the product, or on the product's packaging
- Failure to follow all instructions on the use of these personal protection products and/or failure to properly wear the complete product during all periods of exposure may adversely affect the wearer's health, lead to severe or life-threatening illness or permanent disability
- Read all instructions and keep for future reference

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Specific Safety

IMPORTANT: The user should be trained in fitting the respirator and should be aware of the instructions for use prior to putting on the respirator. Use in adequately ventilated areas, which are not deficient of oxygen and do not contain explosive atmospheres. The respirator must be fitted correctly to ensure the level of protection expected is achieved. It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal.
Discard and replace the respirator if:
1. The respirator is removed whilst in the contaminated area
2. Clogging of the respirator causes breathing difficulties
3. The respirator becomes damaged
⚠ WARNING: Failure to follow the instructions and warnings on the use of this respirator during all times of exposure can reduce the effectiveness of the respirator and could result in illness or disability.

Intended Use

Single use face mask respirators for protection against non-toxic dusts, fibres & aqueous mists.

Fitting Instructions

IMPORTANT: To provide the required level of protection, it is essential that the respirator is fitted correctly. Please follow the guidance below and refer to step-by-step diagram provided:
1. Hold the respirator in your hand with the nosepiece at your fingertips, allowing the elasticated headbands to hang freely below your hand
2. Press the respirator firmly against your face with the nosepiece positioned on the bridge of your nose
3. Draw the headbands over your head one at a time. Position the top band high on the back of your head. Position the bottom band just below your ears
4. Using both hands, mould the metal nosepiece to the shape of your nose

Fitting check

- ⚠ WARNING:** Always perform this check prior to entering a respiratory hazard area.
- 1. Place both hands completely over the respirator and inhale vigorously.
- 2. If air leaks around the edges of the respirator, readjust the respirator, manipulating the nose bridge and headband to achieve an acceptable seal
- 3. Perform rechecks, and if a proper seal cannot be achieved, DO NOT enter the contaminated area

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Maintenance

Storage and shelf life

This respirator should be stored in a container, protected from dust, chemicals, moisture, away from direct sunlight and in a clean, cool and dry atmosphere with humidity factor less than 75% and in a temperature between +2°C / +35.6°F and +55°C / 131°F. Failure to observe these requirements may affect shelf life and performance.

Cleaning and maintenance

Do not carry out any cleaning and maintenance procedures on this product. Do not use this product if it has not been stored correctly and/or is not in a fit state for use. Dispose of the product after use.

Disposal

After use, carefully remove the mask in a clean, still environment and place in a sealable bag prior to disposal.

Contact

For Declaration of Conformity please visit www.silverlinetools.com
For technical or repair service advice, please contact the helpline on (+44) 1935 382 222
Web: silverlinetools.com/en-GB/Support

UK Address: Toolstream Ltd., Boundary Way, Lufton Trading Estate, Yeovil, Somerset, BA22 8HZ, United Kingdom
Approved body: BSI 389 Chiswick High Road, London, W4 4AL, United Kingdom
Approved body number: 0086

EU Address: Toolstream B.V., Hultum-Noordweg 11, Unit 4, 6121 RE Born, The Netherlands
Notified body: BSI Group The Netherlands B.V. Say Building, John M. Keynesplein 9, 1066 EP Amsterdam, Netherlands
Notified body number: 2797

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Indirizzo (RU): Toolstream Ltd., Boundary Way, Lufton Trading Estate, Yeovil, Somerset, BA22 8HZ, Regno Unito
Indirizzo (UE): Toolstream B.V., Holtum-Noordweg 11, Unit 4, 6121 RE Born, Paesi Bassi
VK-Adres: Toolstream Ltd., Boundary Way, Lufton Trading Estate, Yeovil, Somerset, BA22 8HZ, Verenigd Koninkrijk
EU-Adres: Toolstream B.V., Holtum-Noordweg 11, Unit 4, 6121 RE Born, Nederland
Keuringsinstantie: BSI Group The Netherlands B.V. Say Building, John M. Keynesplein 9, 1066 EP. Amsterdam, Nederland