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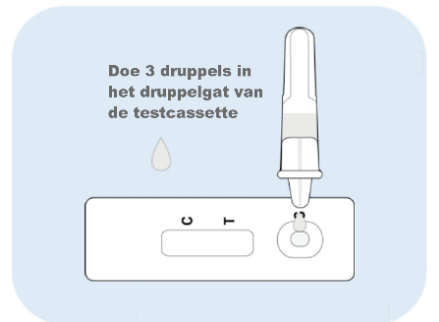
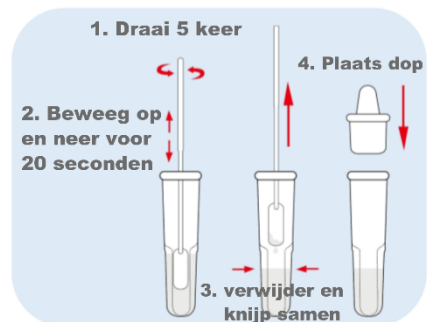
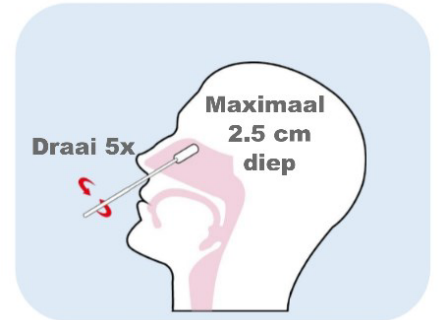
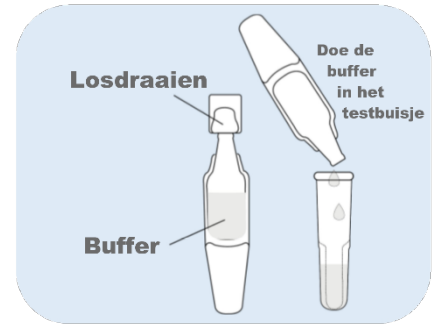
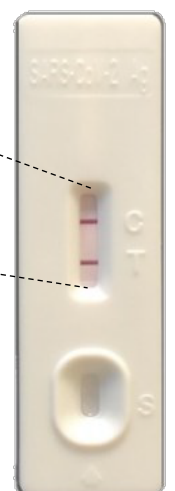
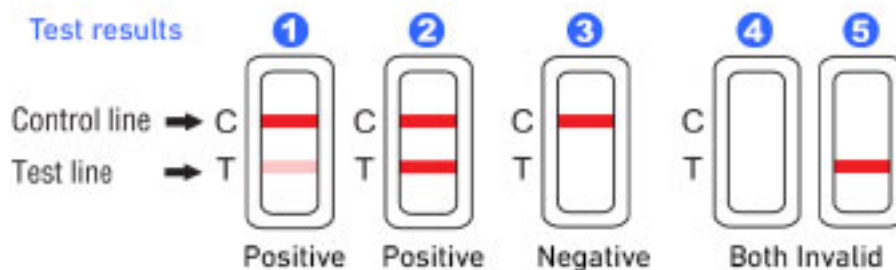
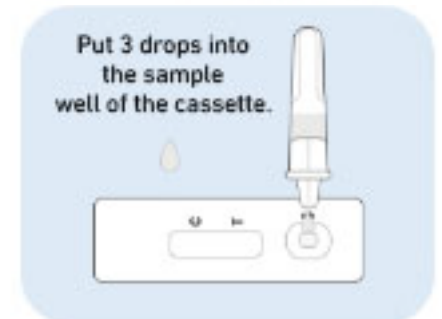
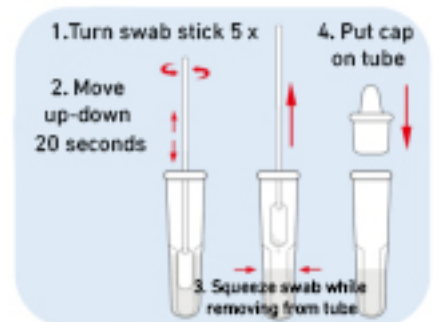
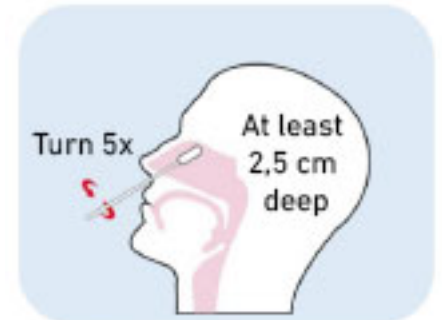
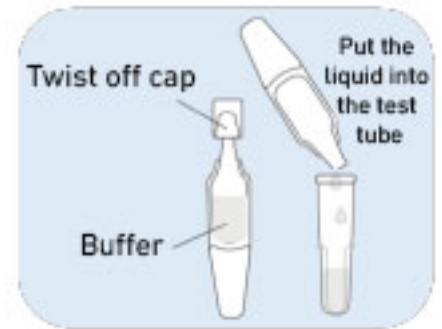
Contents: Romed<sup>®</sup> antigen test cassette, cotton swabs, test tube, drip cap, liquid buffer, instructions for use (technical manual). Please be aware that this quick guide does not replace the technical manual.

1. Blow the nose, wash hands with soap and dry your hands.
2. Place the test tube in the tube holder (the tube holder can be found in the inside flap of the box). Empty the liquid cap into the tube.
3. Take the Romed test cassette out of the foil pack. Put the cassette on a flat/horizontal surface.
4. Take the swap from the foil. Carefully not to contaminate the testing.
5. Take a nasal swab by putting the swab stick in **both** nostrils, **about 2,5 cm** into the nostril. Roll the swab stick 5 times firmly around to sample both mucous and cells from the mucous membrane.
6. Place the swab stick into the tube with liquid and turn the swab firmly **around 5 times** and up-and-down for 20 seconds. Squeeze the tube while removing the swab stick from the tube and be sure to wring out the swab stick. Put the cap on the tube.
7. Put 3 drops from your tube into the drip section, section S (round).

8. The test result is ready after 15 to 20 minutes. Beware that the test result is invalid after 20 minutes. Two lines (C and T) means that you are tested Covid-19 positive. Further instructions are mentioned in the section below.

**Instructions for a COVID-19 positive test result:**

Stay at home in quarantine. Avoid direct contact and do not invite any people. Contact your local health service for further instructions and a PCR test. Contact your doctor when you require care.

**Instructions for a COVID-19 negative test:**

The general hygiene instructions with regard to COVID-19 as advised by your government.