**ADHESIVE
REMOVER
SPRAY**

Elo.

INSTRUCTIONS FOR USE

optimummedical®

GBR | INSTRUCTIONS FOR USE

PRODUCT REF

4001 - Elo Adhesive Remover 50ml

This is a product information leaflet for the user. Read all of the product information leaflet prior to use and retain for your information.

If you are unsure about how to use this product, or whether it is suitable for you, please seek advice from a Healthcare Professional.

PRODUCT DESCRIPTION

Silicone based medical adhesive remover spray. Quick drying formula, leaving no sticky residue behind.

INGREDIENTS/COMPOSITION

100% Hexamethyldisiloxane

INTENDED PURPOSE

Elo Adhesive Remover is a silicone based, medical adhesive remover spray indicated for the gentle removal of adhesive appliances and adhesive residue from intact skin.

WARNINGS & PRECAUTIONS

Do not use the **Elo Adhesive Remover** if you are sensitive or allergic to any of the ingredients. For external use on intact skin only. Use only as directed. Do not use on broken skin or delicate/sensitive areas including eyes, mouth, ears, nose and vagina. If in doubt, patch test on a smaller area prior to use. Discontinue use if irritation occurs. Do not ingest. If sprayed in eye or ingested, seek medical attention. Do not breathe fumes/aerosol. Use only in well-ventilated areas. Do not use the **Elo Adhesive Remover** if the container is damaged. Keep out of reach of children.

DANGER: extremely flammable aerosol. Pressurised container; may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F.

UNDESIRABLE SIDE EFFECTS

In rare cases, local irritation/hypersensitivity reactions

may occur such as redness, itching or blistering. If this occurs, immediately stop using **Elo Adhesive Remover** and consult your Healthcare Professional.

Any serious incident or malfunction that has occurred in relation to the **Elo Adhesive Remover** should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

HOW TO USE

Do not use if packaging is damaged or unintentionally opened prior to use.

1. Spray remover around the edge of the adhesive.
2. After a few seconds, gently peel the adhesive away from the skin using additional medical adhesive remover if necessary.

Dries in seconds. Once dried, **Elo Adhesive Remover** will not affect the adhesion of products applied later.

STORAGE AND HANDLING

Unopened packs have a 2-year shelf life. Do not use after expiry date.

DISPOSAL

Elo Adhesive Remover must be disposed of according to local policies and waste disposal procedures, including any consumables used with the device.

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	Extremely flammable		Non sterile
	Legal manufacturer		Catalogue Number
	Medical Device		UDI - Indicates Unique Device Information
	Use-by Date YYYY-MM-DD		Batch Code
	Country of manufacture		Caution
	Keep Away from Sunlight		Approximate volume
	Consult instructions for Use		The can is recyclable
	The inner box is recyclable		The outer case has been made from 75% recycled material
	Indicates that the product is in compliance with European legislation for medical devices		
	Compliance with Aerosol Dispenser Directive 75/324/EEC		
	Brimful capacity - the maximum volume of a bottle if the contents were filled to the brim		
	Indicates the upper limit of temperature to which the medical device can be safely exposed		
	Authorised Representative in the European Community		
	Indicates the entity importing the medical device into the European market		

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Optimum Medical Solutions Ltd,
MT Promedt Consulting GmbH,
MedEnvoy Global B.V.,