EN - USER'S MANUAL


MEDICAL ANTI-DECUBITUS SYSTEM

(Mattress with Adjustable Pump)

MODEL: APP-P08A

Pump: P08A

- Pressure adjustable knob easy operation (max patient weight 135 kg)
- Auto pressure control through air sensor
- 12 min cycle time options
- Static mode ,low pressure normal pressure indication
- Sound alarm

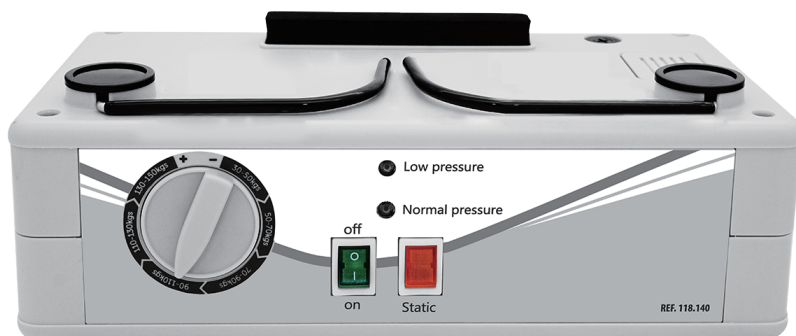Operating Principles

- Alternating pressure at a sufficient rhythm avoids prolonged vascular compression which is likely to cause tissue hypoxia.
- Alternating pressure adapts the mattress to the patient's position.

Guidelines For Use

The traditionally accepted method of pressure sore control is to turn the patient at regular intervals and to observe the areas of risk regularly.

Alternating pressure mattress is intended to compliment this treatment, observation of the patient and turning when necessary should still be considered as part of the treatment.

General

- For patients who are likely to spend more than 15 hours per day in bed.
- For patients who are classed as medium to high risk of developing pressure ulcers.

Installation / Use T05

1. Unroll the mattress.
2. Position it on the bed.
3. Secure it to the bed with the fixing straps.
4. Hang the compressor (pump) at the end of the bed (foot end) using the hooks.
5. Connect the mattress feed pipes to the compressor (pump) connector by matching the appropriate outlets.
6. Connect the electrical power cable to the compressor (pump).
7. Guide it along the bed to the mattress plug socket and plug in.
8. Press switch to turn on the compressor (pump).
9. Before positioning the patient onto the mattress, wait until the mattress has been fully inflated.
10. Select the pressure corresponding to the patient's morphology.

Safety Measures

- DO TAKE CARE when operating this electrical equipment.
- DO NOT leave it connected to electrical supply when not in use.
- DO NOT expose the equipment to excessive humidity.
- DO NOT use corrosive cleaning chemicals such as industrial degreasers or acetic solvents.
- DO NOT attempt to relocate this equipment whilst connected to the electrical supply.

Cleaning / Disinfection

CLEARING:

Perform the following cleaning procedures before use and as necessary thereafter.

Pump:

Wipe the pump with a clean, damp cloth and mild detergent, and keep it away from dust. Air dry.

- Caution: Do not use corrosive or powdered cleansers to clean the pump.
- Caution: Do not immerse or soak the pump.
- Caution: Do not clean with phenolic products.

Mattress

Wipe the mattress with a clean, damp cloth and mild detergent.

Air dry all parts thoroughly before use.

- Caution: Do not clean with phenolic products.
- Caution: After cleaning, air dry the bubble pad with out direct exposure to sunlight.

Use a cloth impregnated with a detergent solution or a surface detergent / disinfectant with CE mark following the usage concentrations recommended by the manufacturer and follow the endurance times.

Maintenance

Compressor (pump) maintenance operations must be strictly carried out by a technician authorized and trained by the importer.

Problem	Possible cause	Solution
The pump are on,mattress do not work	Tube connect between mattress and pump are lose	Ensure air tube are firmly secured to mattress and pump
The pump are on,tube connector are well connected,mattress do not work	CPR may not closed or not closed firmly.	Ensure CPR are closed before start working
CPR well closed,Connector are firmly connected,pump working well,but mattress not do not alternate at beginning	Mattress need about 15-30 min inflated and before start alternating.	This is normal status,please wait after 30 min when you start working the system.
CPR well closed,Connector are firmly connected,pump working well,but mattress seems not work always	Kinks or air tubes may breaks	Please contact your equipment provider for replacement

Disposal:

A damaged system should not be disposed of as unsorted municipal waste.
Contact your local distributor for disposal

Mattress Characteristics

B01 Specification:
Max. User Weight : 135kg
Mattress Dimension : 200*90CM
Mattress Materials :PVC

Compressor (pump) Characteristics
Pump
Item: APP-P08A
Power Supply: 220V/50Hz
Air Output: 6-8 liter/min.
Pressure Range: 30 mmHg -80 mmHg
Cycle Time: 12 min.
Auto Air Control;Sound Alarm;Static
Size: 27cm(L) x 13cm(W) x 9cm(H)
Weight:1.6kg

Importer:
Gohy S.A - Route zenobe gramme 33B, 4821 Andrimont,Belgium

European Technology

Sungo Europe B.V Olympisch Stadion 24 1076 DE Amsterdam Netherlands
--

Manufactured by XIAMEN SENYANG CO., LTD. No. 189-1, Tongji South Road, Jimei District, Xiamen, 361000, Fujian, P.R. China

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