VetPen®

Automatic Insulin Delivery Pen for Animal Use only

VetPen® is only for use with
2.7 ml Caninsulin® cartridges

VetPen® is available in two versions: VetPen 8, which delivers 0.5 to 8 units per injection, and VetPen 16, which delivers 1 to 16 units per injection.

COMPONENTS

1. Dose selector
2. Pen body
3. Release button
4. Internal plunger
5. Cartridge holder
6. Pen cap
7. Dose selector adapter
8. Release button extension
9. Pen needle and protective caps (inner and outer)
10. Cartridge plunger
11. Caninsulin cartridge (not included)
12. Needle remover

PEN NEEDLES

The insulin pen comes with specially lubricated 29G/12 mm VetPen needles. Sterility is guaranteed for 5 years from the date of sterilization if the package seal is undamaged. The insulin pen should be used solely with 29G/12 mm pen needles.

How to Use VetPen®

**PLEASE CONSULT YOUR VETERINARY SURGEON
AND FOLLOW THESE STEP-BY-STEP INSTRUCTIONS
CAREFULLY BEFORE USING VETPEN.**

- Failure to follow these instructions may result in an inaccurate dose.
- VetPen should only be used with Caninsulin 2.7 ml cartridges.

Loading a cartridge

- A. Prior to loading, turn the insulin cartridge up and down at least 10 times until the insulin appears uniformly milky. Do not use if there are agglomerates in the insulin after mixing thoroughly.

- B. Pull off the pen cap and unscrew the pen body from the cartridge holder.
- C. Check to ensure that the internal plunger is fully retracted. If the internal plunger is extended, please refer to step M to see how to wind it back in.
- D. Load the insulin cartridge into the cartridge holder by inserting the metal cap in first.
- E. Screw the cartridge holder and pen body gently but firmly together.

Priming the insulin pen

IMPORTANT: Steps F to H must be followed before each injection.

- F. Peel the protective film from a new pen needle. Screw the pen needle straight onto the cartridge holder. Remove the outer protective needle cap from the pen needle.

Replace the pen cap onto the insulin pen. Turn the pen up and down at least 10 times.

- G. It is important to remove any air from the cartridge (prime the pen) before use to avoid injection of air and to ensure proper dosing. Dial 1 unit on the dose selector. Remove the pen cap and the inner protective needle cap.**

- H. Position the pen carefully with the needle pointing vertically upwards. Tap the cartridge gently with your finger a few times to push any air bubbles to the top of the cartridge. Push the release button and keep it pushed towards the needle until the start line (—) on the dose selector returns to the arrow (►) on the pen body (►—).

A small amount of insulin may appear at the needle tip before the pen is fully primed i.e. before all air has been fully removed. Repeat steps G and H (may need to be repeated several times) until insulin actively drips or squirts out of the needle tip. Confirm that the pen is fully primed by ensuring that no air is visible in the cartridge window during inversion. The insulin pen is now primed and ready for use.

If the start line (—) on the dose selector does not return to the arrow (►) on the pen body, please refer to the 'Priming Advice'.

Priming Advice

If the dose selector does not return to the start line, and no insulin has been expelled, this may indicate one of two possibilities.

- 1) The pen needle that you are using may be blocked. Remove the pen needle currently attached to the pen and replace it with a new one. Then return to steps G and H.
- 2) If the line still does not return to the arrow, the cartridge plunger may require releasing. If so:
 - a. Replace the outer protective cap onto the needle.
 - b. Unscrew the cartridge holder.
 - c. Slide the release button towards the internal plunger and hold in place until the start line on the dose selector returns to the arrow on the body.

- d. Dial up 2 units on the dose selector without screwing the insulin pen back together and simply push and hold down the release button towards the internal plunger until the start line on the dose selector returns to the arrow on the body.
- e. Without rewinding the internal plunger, screw the cartridge holder and body firmly together. This will release the cartridge plunger and expel some of the insulin. However, the insulin pen is **not** yet fully primed.
- f. Holding the device upright, repeat steps G – H until the device is fully primed.

Injecting the dose into your pet

- I. Make sure the arrow (▶) on the pen body points to the start line (—) on the dose selector.

Dial up the number of units required per your veterinary surgeon's direction. (The dose shown in photographs 'I' on the cover pages of this leaflet is only an example. Your veterinary surgeon will tell you the recommended dose for your dog or cat.)

Never attempt to forcefully dial back the dose selector, as doing so may damage the insulin pen and result in inaccurate dosing.

If at any time you select too high a dose, the incorrect dose should not be injected into your pet. The incorrect dose should be completely expelled and discarded appropriately (for example, into the sink). Select the correct dose prescribed by your veterinary surgeon. Before injecting, make sure that the arrow (▶) on the pen body points towards the correct number of units on the dose selector.

- J. The site and method of injection should be performed as directed by your veterinary surgeon. The injection should be performed subcutaneously, 2 to 5 cm (¾ to 2 in) from the dorsal midline, varying from behind the scapulae (shoulder) to the mid-lumbar region (mid-back) and alternating sides. After inserting the needle, push the release button towards the pen needle and keep it pushed down until the start line on the dose selector returns to the arrow on the pen body (▶—).

Releasing the button prematurely will result in incomplete delivery of the dose.

After the start line on the dose selector returns to the arrow on the pen body (▶—), **wait for a minimum of 5 seconds** before removing the needle from the skin.

If your pet dislodges the needle, or it is removed before at least 5 seconds has elapsed, do not attempt to re-dose. Wait and give your pet its usual dose at the next injection.

If the dose selector stops before the start line on the dose selector returns to the arrow on the pen body (▶—), this indicates that your pet has not received a full dose. If a partial dose has been administered, do not attempt to re-dose. Wait and give your pet its usual dose at the next injection.

If, on withdrawal of the VetPen needle from the skin, insulin actively drips or squirts out of the needle, it is possible that a partial dose has been administered. Do not attempt to re-dose. Wait and give your pet its usual dose at the next injection. Check that the VetPen has been primed correctly. If insulin actively drips or squirts out of the needle after two consecutive injections, contact your veterinary surgeon for further advice on how to prime and use the VetPen.

Removing the needle

- K. Remove the needle by inserting it in the removal device and unscrewing it. Press the blue tip on the device to release the needle. Always check that your pen needle has been removed.

Replace the pen cap onto the insulin pen. Dispose of used pen needles carefully in accordance with the advice given by your veterinary surgeon or local regulations.

Changing the cartridge

- L. Before changing the cartridge, always make sure that the pen needle has been removed to prevent accidental injury. Then unscrew the cartridge holder from the pen body and remove the cartridge.
- M. Rewind the internal plunger by holding the white lower part of the pen body between the thumb and finger. Turn the pen body until the internal plunger is fully retracted. Then return to step A to load the next cartridge.

Note: If the internal plunger is rewound incorrectly it can damage the VetPen.

SAFETY PRECAUTIONS

- **VETPEN SHOULD BE USED ONLY AFTER CONSULTATION WITH YOUR VETERINARY SURGEON.**
- Please refer to the Caninsulin cartridge package insert for relevant storage conditions for the insulin prior to and during use.
- Always ensure that VetPen has been fully primed in accordance with the instructions detailed in steps G and H. Failure to follow the exact priming procedures could result in an inaccurate dose being delivered.
- To avoid potential damage to the VetPen, never dial a dose and push the release button without a cartridge in the cartridge holder.
- A new needle should be used for each injection.
- The needle should be removed with the needle remover and safely disposed of immediately after each injection. If the needle is left on it may become blocked and affect the dose delivered.
- Consult your veterinary surgeon should you encounter difficulty in injecting your pet or should your pet display any changes in behaviour or clinical condition.

Storage and Cleaning

VetPen should always be stored or carried with the needle removed, and the pen cap on. To clean VetPen, wipe with a damp cloth. Do not immerse in water.

PEN NEEDLES

- VetPen comes with specially lubricated 29G/12 mm VetPen needles. Sterility is guaranteed for 5 years from the date of sterilization if the package seal is undamaged
- VetPen should always be used with 29G/12 mm VetPen needles.

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