Stechbecken RFM Edelstahl inkl. Deckel

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Stainless steel RFM bedpan including lid

Instruction manual

Specific function

To support patients who cannot leave their beds; for temporary or permanent confinement to bed. Serves to securely collect dejections and urinary excretions. Also suitable for people who cannot get up unaided during the night and when there is a risk of falls.

Indication

Use of the bedpan is advised during strict bed rest, following surgical interventions or for leg thromboses, as well as during restricted mobility, as a result of stroke, for example.

Use (this is supported by nursing staff)

Before use, the bedpan is stabilised using a paper towel if necessary so that it does not slide away. Due to varying clinical pictures, attention should be paid to the bedpan being positioned underneath the pelvis from the correct side.

If the patient had a stroke, the bedpan should be positioned underneath the pelvis from the “affected” side. For femoral fractures or with artificial hip joints, the bedpan is positioned underneath the pelvis from the undamaged/healthy side of the body.

1. The patient turns themselves onto their side keeping their legs bent, with the help of nursing staff if necessary. The bedpan is placed on the buttocks in a position whereby it is at the correct point between the genital area and the bed when the patient turns back. The bent legs serve as a lever for turning the patient back. The nursing staff can check the correct positioning; the handle should be visible on the outside.

OR

2. The patient positions their legs like a bridge and pulls themselves upwards a little using the raising aid so that the buttocks are raised. The bedpan is now inserted from the front. Attention should be

paid here to the patient's sacrum being on the edge of the bedpan. The edge can be padded slightly using a small paper towel if necessary or the bedpan's position can be adjusted using a small paper towel.

Tip: a slightly raised torso position facilitates expulsion. For male patients, a urine bottle should be used simultaneously. Female patients should stretch their legs to avoid urine flowing off into the bed.

Once the bedpan has been removed by the same member of nursing staff who positioned it, it must be stored outwith range of the bed / the patient's reach with its lid closed in order to avoid injury and contamination. The bedpan must never be stored underneath the buttocks. This can lead to injury. Fully empty the bedpan after use.

Contraindication

Using the bedpan presents a very high risk of bedsores and not permitted for use with paraplegic patients. Use of the bedpan is contraindicated in case of pronounced weight loss, palsies and severe pain in the back and buttocks area.

Cleaning and disinfection in a clinical setting

The bedpan must be completely emptied prior to this and the lid must be cleaned and disinfected. A bedpan cleaning and disinfection device should be used for this in a clinical setting.

Cleaning and disinfection in a residential setting

The bedpan must be completely emptied. The bedpan and the lid must then be cleaned with a soap solution and subsequently rinsed. During cleaning, care should be taken to ensure that no residue remains visible. Then dry both parts with a soft cloth. Both parts are then disinfected through wiping using a commercially available disinfectant, e.g. Descosept AF. Do not rinse the disinfectant off with water and leave the parts to air dry. Tip: Follow the disinfectant manufacturer's instructions and safety advice.

Material/properties/scope of delivery

Bedpan and lid with button, Material: 100% stainless steel Maximum load: 150 kg

Dimensions: Diameter around 30 cm, Opening diameter around 21 cm, Handle length around 12 cm

Height around 9 cm, Weight: around 1 kg, Capacity around 2 l

Autoclavable for sterilisation up to 130°C

Tip: Protect the bedpan from direct sunlight and store it in a dry area away from light.

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