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Drive Medical Ltd. * Ainley's Industrial Estate; Elland * GB West Yorkshire HX5 9JP
Phone +44 1422-314488 * Fax +44 1422-314489
www.drivemedical.co.uk

www.drivedevilbiss.de
www.drivedevilbiss.co.uk

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Safety- and assembling instructions

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With the attachment of the product, the cultivation height of 850 mm (floor – upper edge pipe) is to be considered. Due to different wall- soil – and ceiling conditions, are to be selected individually certified fixings, depending upon wall construction.

The provided standard attachment contains plastic pegs as well as stainless-steel screws Ø 6 x 60. These are suitable for the attachment at the following wall types:

Cement, natural stone, solid brick and lime sandstone.

With other wall types, a specialized company of the attachment technology must be consulted. The assembly has to come for safety reasons via a specialist of the building services engineering. For the assembly no guarantee is taken over. For the security of the user, examined max. load not to be exceeded.

The operating instructions as well as the safety and assembling instructions are to be considered absolutely!

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Product Contents

Grab Bar, Mounting material (6 screws, 6 pegs)

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Cleaning / Maintenance

The employment depends on the spatial conditions. The high-grade steel has to be cleaned and disinfected in regular intervals for saving the longevity. For the cleaning and disinfection, commercial cleaning products without scrubbing effect are to be used.

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Warranty

We issue a warranty of 24 months from date of purchase.

Subject to technical alterations.