




Bathtub Grab Bar

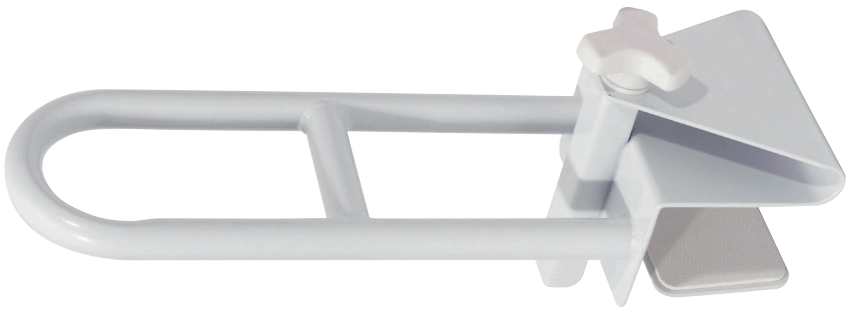

VY421

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for purchasing the Bathtub Grab Bar from Aidapt.

These instructions should be read carefully and left with the user of the product for future reference.

Please ensure this equipment is installed by a competent person.

VY421 – Bathtub Grab Bar **136kg (21St.) Weight Load**

Do not exceed the weight limit stated – doing so could put you at risk.

INTENDED USE

The removable Bathtub Grab Bar is designed to give you extra support when getting in and out of your bath. This product securely clamps to the side of your bath easily and with no tools required. The grab rail is fully removable for travel use and storing away when not in use.

FEATURES

- Provides you with support when getting in and out of the bath
- Easily installed without the need for tools
- Removable for easy storage
- Ideal for travel use

ASSEMBLY INSTRUCTIONS

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

1. Carefully remove all packaging and check the product thoroughly for damage or obvious faults.
2. Clean bathtub thoroughly and wipe dry before applying the bath rail. Contact surfaces must be dry and free from soapy residue to assure secure installation.
3. Firstly, loosen the star shaped hand wheel (A) by turning it anti-clockwise until it is wide enough to slip on side of bathtub.
4. Choose a desired location and safely place the grab bar on your bathtub (*as shown in fig. 1*).
5. Ensure star-shaped hand wheel is facing outside the bathtub (*fig.2*).
6. Tighten the Bathtub Grab Bar by turning the star shaped hand wheel clockwise (*fig. 2*).
7. Before use, test to ensure the grab bar is securely in place by pulling forward and back, and side to side. If the bar moves, or slips, please check to ensure surfaces are dry and free from soapy residue. Remove your grab bar and repeat these steps.



fig. 1



fig. 2

WARNING: The Bathtub Grab Bar requires a risk assessment before.

WARNING: The Bathtub Grab Bar must be checked periodically to ensure it is securely in place, retighten as required.

WARNING: Use extreme care when installing this grab bar on fibreglass bathtubs. DO NOT overtighten as the clamp may crack the fibreglass.

CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your Grab Bar using a non-abrasive cleaner or mild detergent with a soft cloth avoid using lime scale remover. Abrasive cleaners or abrasive cleaning pads will seriously damage your product beyond repair.

If to be disinfected by heat, one of the following three temperatures and duration of exposition can be used:

- a) a temperature of 90°C for 1 minute
- b) a temperature of 85°C for 3 minutes
- c) a temperature of 80°C for 10 minutes

CARE, MAINTENANCE AND YOUR OBLIGATION

Please check the Bathtub Grab Bar periodically for any signs of damage and if in doubt contact your supplier.

TECHNICAL INFORMATION

Width:	155mm
Depth:	140mm
Height:	495mm
Net Weight:	3.02kgs

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

SERVICE WARRANTY

Aidapt Bathrooms Ltd guarantees the product free from defects in material and workmanship for a period of one year.

Should this product be operated under conditions other than those recommended, or any attempts made to service or modify the product, then the warranty is rendered void.

The product you buy may differ slightly from illustrations. This warranty is in addition to, and does not affect your statutory rights.

Our guarantee is administered by our retailers.

If your product arrives damaged, you must contact the retailer from whom you bought it. The retailers contact details will be on the invoice that arrived with the product, or on the email you received when you placed the order. Do not contact Aidapt Bathrooms Ltd, only your retailer can arrange a replacement or refund.

If your product fails within the guarantee period, please contact the retailer from whom you bought it.

If you have received your product and require technical help, please call our help desk on 01744 745 000

A copy of the instructions leaflet can be downloaded from our website.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 Fax: +44 (0) 1744 745 001

Web: www.aidapt.com

Email: sales@aidapt.co.uk