DSF 130

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Table of contents

Preface.....	7
Declaration of conformity.....	7
Service life.....	7
Disposal.....	7
Identification plate.....	7
Reuse.....	7
Warranty.....	7
Indication.....	8
Scope of delivery.....	8
Safety instructions.....	8
Assembly.....	9
Height adjustment of the feet.....	9
Cleaning Maintenance.....	9
Technical data.....	9

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co.KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

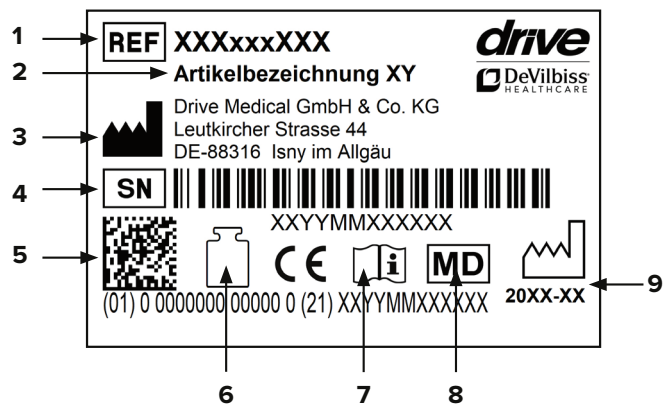1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year/month)

Reuse

This product can be reused. Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available at www.drivedevilbiss.de or from your specialist dealer.

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

Exclusion of warranty

Parts subject to wear include (among others): rubber feet

Parts of the product

DSF 130

1. **Backrest**
2. **Armrest**
3. **Lock for seat section**
4. **Adjustable foot**
5. **Seat section**

Indication

INDICATION:

Impossibility to securely stand in the shower due to impaired function of the lower extremity with limited or absent mobility and muscle strength and/or impaired vestibular function or tendency to fall due to other etiology with simultaneously adequate stable torso position for sitting and sufficient residual functions of the upper extremities.

Scope of delivery

1 x	Shower chair
1 x	Instructions for use

Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use. Do not use product if there are any defects!
- Clothing or body parts can be pinched when sitting down or getting up and when locking the seat section into place.
- Caution – risk of slipping/falling!
- The product is significantly less stable than regular chairs, therefore avoid leaning to the side – risk of tipping!
- Always sit in the middle of the seat and all the way back against the backrest and not on the edge – risk of tipping and slipping!
- The product should not be exposed to temperatures above 60 °C for long periods of time (for example, use in a sauna, etc.).
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.

Assembly


The shower chair is preassembled; only the 4 adjustable feet need to be assembled (see next section).

Unfold it as shown and lift the seat section upwards. Swing both side parts forwards and then bring the seat section down until it rests on the retaining clips (**pic. 1+2**).

Now push the seat section down firmly until it audibly clicks into place in the left and right retaining clips (**pic. 3**).


Check the proper position of the seat section in the retaining clips (**pic. 4**).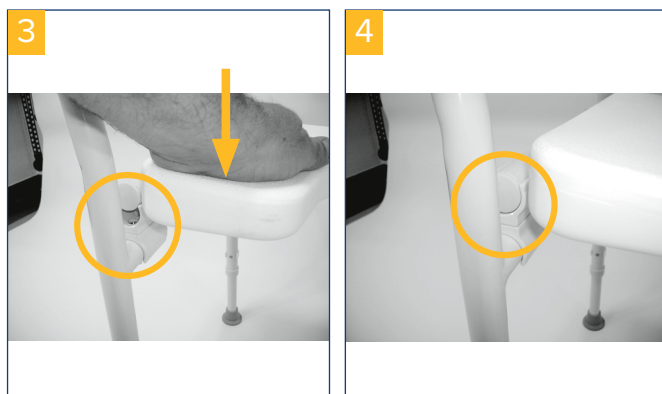

Height adjustment of the feet

Before using the product, adjust it to the desired height (**pic. 5**). To adjust the height of the legs:

Press button (1) and at the same time, pull the leg out or push it in until the button clicks into place in the desired position (2). Ensure that the pushbutton has correctly clicked into place.

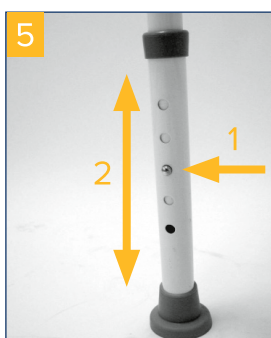Ensure that all four legs are adjusted to the same height.

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60 °C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.

Technical data

Product	DSF 130
Article number	550200100
HMV (medical aids register) no	04.40.03.2021
Overall depth	56 cm - 58 cm
Overall width	53 cm
Overall height	90 - 100 cm
Height of backrest	46 cm - 48 cm
Seat (W/D/H)	41 cm / 41 cm / 52 cm
Seat adjustment settings	5
Weight	3.8 kg
Max. load	130 kg
Frame material	Welded steel tubing, powder-coated
Seat / armrest cover material	PP (polypropylene) / PU (polyurethane)

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Drive Medical GmbH & Co. KG