

# wiroPINNY S





## PE BIB APRON

Tie straps at the waist | smooth



PREMIUM

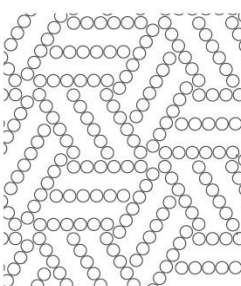

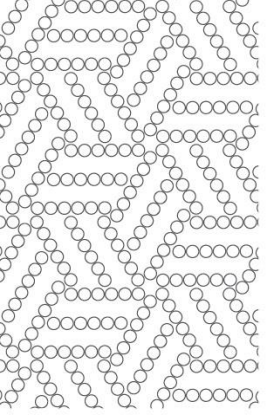

# wiroPINNY S



## Product description

Brand	care & serve®
Category	PREMIUM
Item description	- care & serve® - PREMIUM   wiroPINNY S   PE bib apron Tie straps at the waist   smooth
Properties	Tie straps at the waist smooth

## MDR/PSA use

Intended use	Medical protective clothing as defined in the Technical Rules for Biological Agents (TRBA 250) to prevent contamination of work or normal clothes during interventions or care measures resulting from direct contact with blood, secretions, excretions or other contaminated material.
Scope of application	There is no direct medical application on the patient. Medical protective clothing is intended to cover clothing and/or hair.
Application group	Employees and visitors from laboratories, the pharmaceutical industry, hospitals or similar institutions.
Patient group	Medical protective clothing is suitable for all patient groups.
Indication	Medical clothing as a barrier to cover clothing and/or hair.
Contraindication	No known contraindications.
Notes on use	Disposable product Not sterile
Note	All serious incidents directly related to this product must be reported to the manufacturer and the competent authorities.




## Material


Material	Polyethylene
Thickness	≈ 0.018 mm (± 5%)
Material information	Latex-free
Health advice	This product contains no known toxic, carcinogenic, mutagenic, reprotoxic or otherwise harmful substances or allergens. Contact with the skin can cause allergic reactions in sensitive individuals. In such cases, remove the suit in accordance with the applicable safety guidelines and consult a doctor.



## Standards

CE marking	Class I medical device in accordance with EU 2017/745		
SRN	DE-MF-000004956		
Basis UDI DI	4051642-wiroPINNYS-QN		
EUV 10/2011	Regulation on plastic materials and articles intended to come into contact with food		
EG 1935/2004	Regulation regarding plastic materials and articles intended to come into contact with food.		
	Product use	Result	
	Contact with food produce	Direct contact with food produce	
	Specifications for use	Result	
	Intended food types	For all foods	
	Intended conditions	2 hours at 40 °C	
	Tested surface/volume ratio	2 dm <sup>2</sup> to 100 ml test food	
Information	The general material is suitable for foodstuffs. Only the material has been tested, not each individual product.		

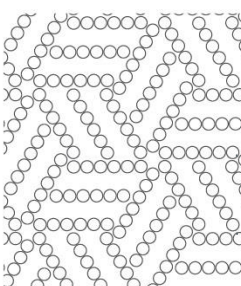

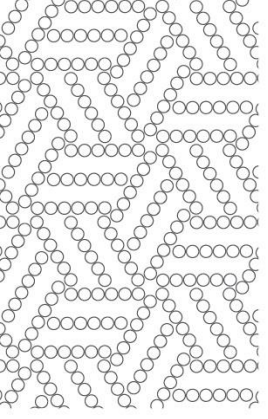

# wiroPINNY S



## Use

Traceability	Batch-related traceability from the determination of requirements to issuing. Product data sheet Version 1 Valid from Batch 2121 +
Instructions for use	<p>Before use</p> <p>Please make sure that you have selected the right product for your needs and areas of use. Please do not wear medical protective clothing if it is faulty or has already been used. Medical protective clothing is intended for short-term use and use is limited to one patient in the medical field.</p> <p>Use</p> <p>Unfold the folded apron. Pull the neck strap over your head and place it around your neck. Smooth the apron down slightly and knot the ties on your back or side. To remove the apron, tear the neck strap on your nape and fold the bib down, then tear the tie straps on your back and fold the sides forward so that the clean, uncontaminated area of the apron is on the outside and the contaminated area is on the inside. In order to avoid inadvertent contact with the contaminated area, the apron should not be crumpled up, but rather folded again for disposal.</p>
Disposal	Unused and uncontaminated products can be incinerated in an environmentally friendly manner or disposed of at a waste disposal site. Contaminated products must be disposed of in accordance with the national laws and regulations valid for the respective type of contamination.
Shelf life	5 years from the production date, if unused and stored correctly.
Storage	<p>Keep out of direct sunlight.</p> <p>Store in a dry place.</p> <p>Incorrect storage can lead to a reduction in the shelf life.</p>



## Packaging

Recycling	Licensed in accordance with VerpackG (German Packaging Act), Licence number DE1618662887322. Please recycle the packaging.
-----------	---




## WWW


PDF IFU	<a href="https://www.wiros.de/IFU/IFU-wiroPINNY-S.pdf">https://www.wiros.de/IFU/IFU-wiroPINNY-S.pdf</a>
---------	---

## Variant 1

Variant feature	Punched
Sales unit	1000 Unit/Cardboard box
Sub-packaging	50 Unit/Polybag
Minimum order quantity	1 Cardboard box

Item numbers	Item number	Colour	Size (W x L)	Tolerance
	20105303	white	≈ 75 x 125 cm	± 5 %
	20105323	blue	≈ 75 x 125 cm	± 5 %
	20105313	green	≈ 75 x 125 cm	± 5 %
	20105333	red	≈ 75 x 125 cm	± 5 %
	20105343	yellow	≈ 75 x 125 cm	± 5 %
	20105304	white	≈ 75 x 140 cm	± 5 %
	20105324	blue	≈ 75 x 140 cm	± 5 %

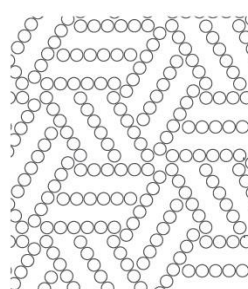

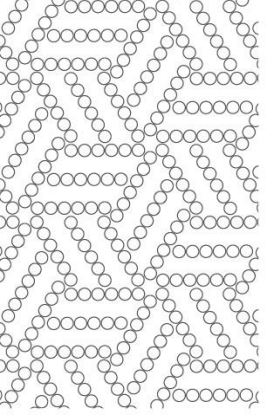

# wiroPINNY S



GTINs	Item number	Unit of Use UDI	Packaging level 1 UDI	Packaging level 2 UDI	Cardboard box UDI
	20105303	4051642004471	4051642050133	-----	4051642550138
	20105323	4051642004488	4051642050157	-----	4051642550152
	20105313	4051642004495	4051642001364	-----	4051642501369
	20105333	4051642004501	4051642001340	-----	4051642501345
	20105343	4051642004518	4051642001357	-----	4051642501352
	20105304	4051642004525	4051642050140	-----	4051642550145
	20105324	4051642004532	4051642050164	-----	4051642550169

The numerical data for the measurements, weights, thicknesses and tolerances are target values, and there may be some deviation.

The information represents the current state of our knowledge. It offers no guarantee of the product characteristics and does not form the basis of a contractual legal understanding. Please reconfirm any information that is older than 2 years.

Typographical and other errors excepted.

The product data sheets provided are the property of WIROS Wilfried Rosbach GmbH. WIROS Wilfried Rosbach GmbH is the owner of all copyrights/rights of use and rights of exploitation of the product data sheets. It is not permitted to change the product data sheets or to use the pictograms. The product data sheets provided may only be used in connection with original WIROS products.

