


How to Use

INDICATOR:
SHOWS WHICH WAY THE
SHARP EDGE MUST FACE.

THIS PART OF THE
SCALPEL HANDLE IS
CALLED THE TANG.

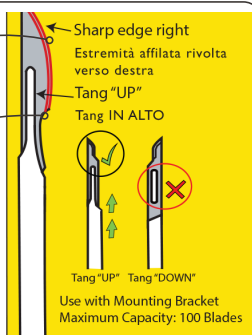

COUNTER


LANDING PAD



MOUNT TO WALL OR TROLLEY:

- If you haven't mounted your BladeFlask EVO before, prepare the universal mounting bracket*.

A BRACKET HAS DOUBLE SIDED TAPE ON BOTTOM

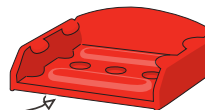

B OR USE SCREWS.



* See fitting template and instructions inside the bracket packaging.

- BladeFlask EVO can be mounted on a wall or on a trolley or bench.

SLIDE DOWN INTO BRACKET




SLIDE BACK INTO BRACKET




NOTE: BladeFlask EVO is to be mounted on the Mounting Bracket during use.

REMOVE BLADE SINGLE-HANDED ACTION:


- Place blade on the landing pad with the sharp edge facing to the right (or 'tang' up).

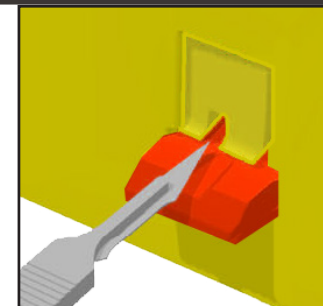

- Push up the scalpel handle into BladeFlask EVO along the ramp until you hear a click.

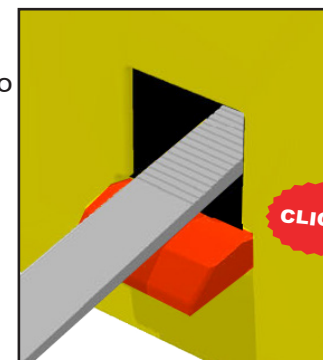

- Remove the handle.

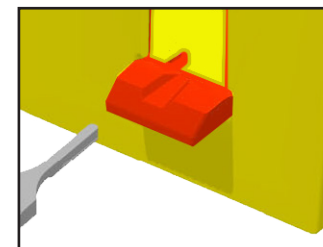

NOTE: BladeFlask EVO is disposed of as medical waste after the counter automatically shuts off when it reaches full capacity.