

Sieve Modules Replacement Instructions:

1. Turn off the device and disconnect AC or DC power source (if connected).
2. Remove the device from the carry bag.
3. Remove the battery from the device.
4. Press the four release buttons located at the top of the device to remove the top cover of the device as shown in Fig. 9.
5. Use the tool included with the replacement sieve modules package to rotate the sieve counter clockwise 3 - 4 turns until it can be removed by pulling up towards the top of the device.
6. Repeat for the second sieve module.
7. Remove the replacement sieves from the packaging, remove the air plug on the sieve module and immediately install the replacement sieve module. Caution: Do not leave the replacement sieve exposed.
8. Install the replacement sieve, with thread at the top of the device as shown in Fig. 10.
9. Hand tighten the sieve.
10. Use the tool to finish installing.



Fig. 9













Fig. 10

CAUTION: Do not over-tighten!

11. Repeat steps 8 - 10 for the second replacement sieve.
12. Re-install the top cover to the device by pushing down over the top of the device until the four buttons snap into place.
13. Re-insert the battery into the device.
14. Connect the device to an AC or DC power source.

DO NOT POWER ON!

15. Press the alarm mute  and battery  buttons simultaneously until the display shows "Leak Test".
 - a. The device will indicate progress by decimals right of the "Leak test" text.
 - b. When completed, the device will indicate "Test Passed" or "Test Failed".
 - i Test Passed - Press battery button  to exit, continue to step 16.
 - ii Test Failed - Press battery button  to exit, then repeat steps 4-15.
16. If the leak test has passed, then clear the device sieve hour counter.
 - a. Turn on your Zen-O lite portable oxygen concentrator.
 - b. Simultaneously press the  and  buttons until "Diagnostics" is displayed.
 - c. Press the  or  button to navigate the diagnostic menu until "XXXH Sieve" is displayed, where XXXX = number of hours for the old sieve was utilised.
 - d. Press the alarm mute button  until the sieve hour counter shows "0H Sieve".
 - e. Press the battery button  to exit the diagnostic mode.
 - f. The device is now ready to be used.