



## Inogen One® G5 Column Change Procedure

1. Turn off the Inogen One® G5 concentrator by pressing the power button to shut down the device.
2. Remove the Inogen One® G5 concentrator from the carry case.
3. Remove the battery from the Inogen One® G5 concentrator.
4. Place the Inogen One® G5 concentrator on its side so that the underside is visible. The metal column assembly can be seen on one side of the device.



(Actual appearance may vary, depending on model with or without pull handle.)

5. A. Unlock the column assembly by pushing the latch button away from the columns, or  
B. By inserting the column tool (as shown). See step 8 and remove top dust cap to obtain tool.



6. A. While holding the button open, slide the column assembly out of the device by pulling on the column pull handle or

B. Insert tool and press down between latch and columns.



7. A. Remove the columns completely from the Inogen One® G5. Both columns are removed as one piece or

B. Rotate tool up to push columns out.



8. A. Column (metal tube) Installation: Remove dust caps of new column assembly. Make sure there is no dust or debris where the dust caps were located or



B. Remove dust caps of new column assembly. Make sure to keep the top cap as it is also a tool for column removal.



9. A/B. Insert column assembly into the Inogen One® G5 concentrator. Do not leave the column ends exposed; column assembly should be inserted into the Inogen One® G5 as soon as the dust caps have been removed.
10. Push the column assembly into the device such that the columns are fully seated into the Inogen One® G5 concentrator. The spring loaded latch button should fully return to the closed position.
11. Connect the AC power supply cord to the Inogen One® G5 and plug the power supply AC cord into an electrical outlet. Do Not Power on the Inogen One® G5 concentrator.

The following steps can be done by pressing specific buttons on the device's screen or within Inogen Connect App.

Steps from your device's display:

- Press and hold the plus (+) and (-) minus button for 5 seconds. The screen will display the following informational icon. Release button once icon is displayed on screen.
- Press the alert  button once and screen will display the following informational icons.
- Press the power  button to turn on the Inogen One® G5, and use normally.

Steps using Inogen Connect App.

- If you are using Inogen Connect App, navigate to the Advanced screen, then to Additional Information screen and click on the Column Reset button.



Closed and locked



(Actual appearance may vary, depending on model with or without pull handle.)

