
# NasoFree

## Instructions for use

### RECOMMENDED BY EXPERTS

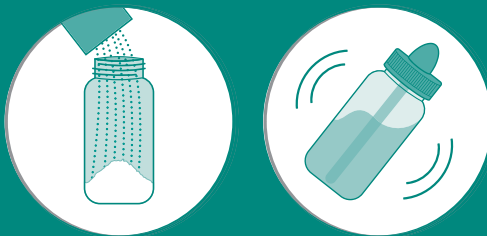1.

2.

3.

4.

## NL

### Benaming

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### Product description

- NasoFree nasal irrigator and NasoFree nasal saline are suitable for cleaning the nostrils and nasal sinuses. In order to flush, for example, excessive mucus, viruses, bacteria, dust or allergens from the nose.
- Nasal saline dilutes mucus and softens scabs. These are then flushed away by the volume of the saline solution. As a result, it's easier to breathe and/or allergic symptoms are reduced. The effect is normally immediate.
- Nasal rinsing is suitable for all ages, from the age a child can sit up. Children should preferably rinse their nose under adult supervision.

### Composition

- NasoFree nasal saline: Sodium chloride, sodium bicarbonate.
- NasoFree nasal saline with xylitol: Sodium chloride, sodium bicarbonate, xylitol and peppermint oil.

### Features

- NasoFree nasal irrigator is effective due to the volume and slight positive pressure involved when rinsing. The saline solution is squeezed from the irrigator container, from bottom to top. This cleans both the nose and nasal sinuses. The nasal irrigator is easy to use and simple to clean.
- NasoFree nasal saline is a combination of sodium chloride and sodium bicarbonate. One sachet is sufficient for one 250ml rinse. The saline solution cleans effectively and is gentle to sensitive nasal mucosa.
- NasoFree nasal saline with added xylitol and a hint of mint is especially for people with chronic nasal sinusitis. Bacteria often play a role in (chronic) nasal sinusitis. The bacteria produce a layer of self-protective mucus (biofilm). Scientific research has shown that xylitol breaks down this mucus layer around the bacteria. This can help control (chronic) nasal sinusitis.

### Indications/applications

- Sinusitis (rhinosinusitis).
- Airway allergy to grass and tree pollen, house dust mites and/or animal dander.
- Colds.
- Recovery after nasal surgery (in consultation with your ENT specialist).
- During or after a stay in a dusty environment.
- Snoring, as a result of colds or airway allergies.

### How to use

- Empty one sachet of nasal rinsing salt into the nasal douche. Then fill the irrigator with distilled or previously boiled, lukewarm water (at 37°C or below), up to the screw cap/neck of the bottle (i.e. 250 ml).
- Shake the nasal irrigator briefly before use so that the nasal saline and water are well mixed. While doing so, keep your finger on the cap of the nasal irrigator. Test the temperature of the water on your palm or forearm.
- Bend your head forward over a basin and insert the nasal irrigator at an angle of about 45° into one nostril. Gently squeeze the nasal irrigator. The fluid may flow back through the same nostril (in case of severe blockage), or through the other nostril. Therefore, always leave the other nostril free and do not block it.
- Blow your nose lightly between rinses and after rinsing. You may experience pressure differences in the sinus cavities and ears. This is not serious.

Repeat steps 3 and 4 for the other nostril. You can continue to alternate nostrils until the nasal irrigator is empty.

Saline solution may get into your throat during rinsing. This is not harmful. You may spit out the solution. You can prevent saline entering your throat by applying only gentle pressure when squeezing the nasal irrigator.

### Frequency and duration of use

The nose may be rinsed as needed, usually 1-4 times per day. There is no maximum for frequency or duration of use.

### Cleaning the nasal irrigator

- Rinse the NasoFree nasal irrigator with warm water before and after use, as required.
- Clean the nasal irrigator by hand, using water and, if required, soap.
- After cleaning, rinse the nasal irrigator thoroughly and let it air dry.
- Remove the screw cap from the nasal irrigator while it is drying.

### Use during pregnancy and breast feeding

There is no evidence that nasal rinses cannot be used during pregnancy and breast feeding.

### Contraindications - when not to use this product

- Do not use if there is a known allergy to any of the ingredients.
- Do not use if there is a direct connection between the nose and the fluid spaces of the brain.
- Do not use for injuries to the nasal roof and nasal sinuses.
- Do not use in cases of severe nosebleeds.
- Do not use if the packaging is damaged on receipt of the product.

### Special attention and precautions

- Consult your doctor if your symptoms do not improve.
- Never block the nostril through which the saline should flow out.
- Never use the NasoFree nasal irrigator while lying down.
- Always use your own nasal irrigator, to avoid cross-contamination.
- Never use water that is warmer than body temperature, to avoid irritation and burning.
- Always rinse with nasal saline. Rinsing with water can be painful and lead to irritation. Do not use other solutions to rinse your nose.
- Make a fresh saline solution before each rinse.
- If using medication that is applied to the nasal mucosa, observe the following sequence: use the nasal rinse, wait a few minutes, then administer the medication. The medication will then be applied to clean nasal mucosa.
- Never expose the NasoFree nasal irrigator or nasal saline directly to a heat source. Boiled water should first cool to 37°C or below before using it to fill the nasal irrigator.
- Keep out of reach of children.

### Side effects

- Rinsing with NasoFree nasal saline (with xylitol) may possibly cause a stinging sensation.
- If there are serious adverse effects related to use of this product, report them to the manufacturer and to the competent authorities (of the Member State) where the user is located.

### Storage

- The expiry date of NasoFree nasal saline (with xylitol) is indicated on the packaging.
- Due to wear and tear, and for hygienic reasons, replace the NasoFree nasal irrigator with a new one after 1 year.

Date of last revision of instructions for use February 2023

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