DOS Medical Smell Training Reuktraining Riechtraining Training olfactif

Name

StimuScent Smell Training by DOS Medical

Product description

- DOS Medical StimuScent Smell Training is suitable for training the olfactory sense and the brain to once again (correctly) recognise smells. Studies show that smell training supports and accelerates the recovery of an impaired sense of smell.
- StimuScent Smell Training sets each contain a detailed training schedule and four, different, internationally recognised fragrances.
- Smell training is suitable for all ages. Children should do smell training under adult supervision.
- Many people benefit from smell training, though the end-result cannot be predicted in advance.
- There is insufficient evidence available on the use of Smell Training during pregnancy or while nursing. Utilising it during pregnancy or nursing is therefore not advised.

Composition

Smell Training Set 1

- Essential oils (3 ml per bottle).
- Rose, lemon, eucalyptus, clove.

Smell Training Set 2

- Essential oils (3 ml per bottle).
- Jasmine, mandarin, mint, thyme.

Smell Training Set 3

- Essential oils (3 ml per bottle).
- Gardenia, bergamot, green tea, rosemary.

Characteristics

A DOS Medical StimuScent Smell Training set consists of four, different, internationally recognised fragrances and a detailed training schedule. As such, a StimuScent Smell Training set contains everything needed to easily carry out smell training. Smell training can be seen as therapy for the nose. The sense of smell is stimulated and the brain is trained to once again (correctly) recognise smells.

Indications/applications

Impaired sense of smell.

Instructions for use (see illustrations)

- 1 Open one of the fragrance bottles. The order in which you use them does not matter.
- 2 Hold the bottle approximately 3 cm under your nose. Do not touch the bottle with your nose.
- 3 Relax. Take one, slow, deep breath through your nose. This will enable good fragrance inhalation. It is important that you focus on the fragrance you are trying to smell. Read the scent on the bottle before using it. Do not smell blindly.
- 4 Repeat step 3 four times. You will then have inhaled this fragrance a total of five times.

- 5 Close the bottle. Rest your nose for 5 minutes and continue with the next fragrance.
- 6 Repeat this process with all the fragrances from the set. The entire session should take about 30 minutes.
- 7 Repeat the training session twice a day (morning and evening). Carry on for 12 weeks per set.

Frequency and duration of use:

- Smell training should preferably be done twice a day, for example in the morning and in the evening.
- To achieve the best possible results, smell training should be continued for at least 24 weeks.
- After 12 weeks, switch to another set of fragrances. There are three StimuScent Smell Training sets by DOS Medical.

Special attention and precautions:

- **Keep out of reach of children and pets.**
- **Never ingest the essential oil. Swallowing the essential oil causes physical injury.**
- **In case of swallowing, do not induce vomiting. Alert a doctor immediately and take the packaging with you.**

After treatment, report the incident to the manufacturer.

- If the essential oil gets on your skin, wash it off with soap and water as quickly as possible.
- Do not inhale too quickly, otherwise the fragrance will not penetrate properly.
- Focus on smelling and take your time. Make sure you know which fragrance you are smelling and try to be aware of it while smelling. Do not blindly select and smell a bottle.
- After use, close each bottle tightly, otherwise the essential oil will evaporate.
- Never expose the Smell Training set directly to a heat source.
- Bottles and any remaining oil should be disposed of at your local authority's environmental service/waste station.
- An odourless diluent (isopropyl palmitate) has been added to the oil in the rose and jasmine bottles. This diluent can congeal/crystallise in the cold. Once the oil has reached room temperature, it becomes liquid again. This has no effect on the quality of the fragrance.

Side effects

No side effects have been identified to date.

Storage

- The expiry date is stated on the packaging.
- Store at room temperature.
- Can be stored for 3 months after opening.

Date of last revision of the instructions for use June 2022.

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