airmax®

EE

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GB

TOP AID TO BREATHE FREELY THROUGH YOUR NOSE AND TO STOP SNORING

Using the Airmax relieves nasal breathing and improves airflow resulting in:

- Less "blocked" noses
- Better breathing during sleep
- Less trouble with snoring
- Improves the functions of the nose (warming, cleaning, and moisturising the air)
- More airflow through the nose while exercising

Airmax has been developed for every person from the age of 12 to improve breathing through the nose.

WHY THE AIRMAX?

The Airmax is a development in the field of nasal sprays. The shape of the Airmax has been developed by airflow experts and its effect has been tested by ENT doctors. The Airmax widens the nasal passage in the narrowest part of the nose and is made of a soft, odourless, and medically certified plastic (S.E.B.S.) which makes wearing the Airmax extremely comfortable. The Airmax has an extremely effective operation and achieves a scientifically proven average of 76.1% increase in air flow.

INSTRUCTIONS OF USE

Rinse the Airmax before and after use with water. Do not use any abrasives or disinfectants for this. We recommend that you use a mirror the first time you insert the Airmax in the nose to guarantee the correct placement of the Airmax. The Airmax will fit properly when the product is placed in both nostrils, the legs widen the nasal entrance above the nostrils and the "connecting bridge" fits snugly against the nasal septum. See image.

Use the Airmax for at least 2 weeks consecutively to get used to it.

Note that the Sport Airmax has small "hooks" so that the Airmax cannot easily fall out during exercise.

THE AIRMAX CAN BE USED FOR 3 MONTHS.

The product should be stored dry and kept out of direct UV light. Improper storage can lead to degradation, resulting in odour development, colour changes and can have negative effects on the physical properties of this product. The negative physical properties have no effect on the direct action of the device. However, this will make the product less comfortable during use.

WHEN SHOULD YOU NOT USE THE AIRMAX?

If you have difficulties placing the Airmax in the nose because your nose is not symmetrical or if you still cannot breathe better after placing the Airmax in the nose, we recommend that you see an ENT specialist. To date no allergic reactions or other contra-indications have been reported.

ATTENTION!

The Airmax is only suitable for placement in the nose. Not suitable for children under 12 years. Keep the Airmax out of reach of small children. The Airmax may only be used by 1 person to prevent contamination. Any serious incident related to the device must be reported to the manufacturer and the competent authority of the member State where the user is established. When disposing of this product, the general statutory regulations for waste separation apply.

We wish you a lot of fun, more air and a good night's sleep!

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