## GB User Instructions

Before using the Rhino Horn™ Junior for the first time, you should rinse it thoroughly with running water.

## The Water

Prepare a saline solution by putting one measuring spoon (included) of fine grained kitchen salt (NaCl) into the Rhino Horn Junior, followed by filling it with lukewarm water (37° C / 98.6° F). Close the openings of the Rhino Horn™ Junior with your palm / finger and shake it a few times to dissolve the salt. If there are concerns about water quality, boil the water for one minute and then let it cool to body temperature. If the tap water is safe, you may use it directly from the warm tap.

## Rinsing the Nose

To use the Rhino Horn™ Junior, tilt your head sideways over the sink and from the side gently place the spout into the upper nostril. Start with your less congested nostril. Breathing normally through your open mouth, pour the saline solution into your upper nostril, and if the angle of the head is correct, the water will drain directly through the lower nostril. Do not stick your chin out; keep it tucked in towards your chest, otherwise the water will run back from the nasal cavity into your throat. Repeat on the other side.

## Empty Your Nose

To clear any remaining water from your nose and sinuses after you use the Rhino Horn™ Junior, bend forward with the top of your head down, look to the side, close the lower nostril and gently blow into a tissue through the open nostril. Be careful not to blow so hard that your ears pop. Repeat on the other side. Do this a few times.

## Cleaning the Rhino Horn

After each use, the Rhino Horn™ Junior should be rinsed out and left to air-dry. The Rhino Horn™ Junior may be washed in a dishwasher and may even be cleaned in boiling water for a few minutes (do not boil the spoon). Washing or boiling is necessary only if two persons are using the same Rhino Horn™ Junior right after one another. Normally air-drying is enough.

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[www.rhinohorn.nl/neus-spoelen-in-4-stappen](http://www.rhinohorn.nl/neus-spoelen-in-4-stappen)

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<https://rhinohorn.be/fr/comment-se-laver-le-nez>

*GB: Please report any benefit or serious incident from using Rhino Horn to [post@yogaprocess.com](mailto:post@yogaprocess.com)*

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Designed in Norway

Manufactured under ISO 13485:2017 by:

No Phthalates – No Bisphenol A  
Toxic free according to REACH 168 test

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