
**BREATHE BETTER SLEEP BETTER FEEL BETTER**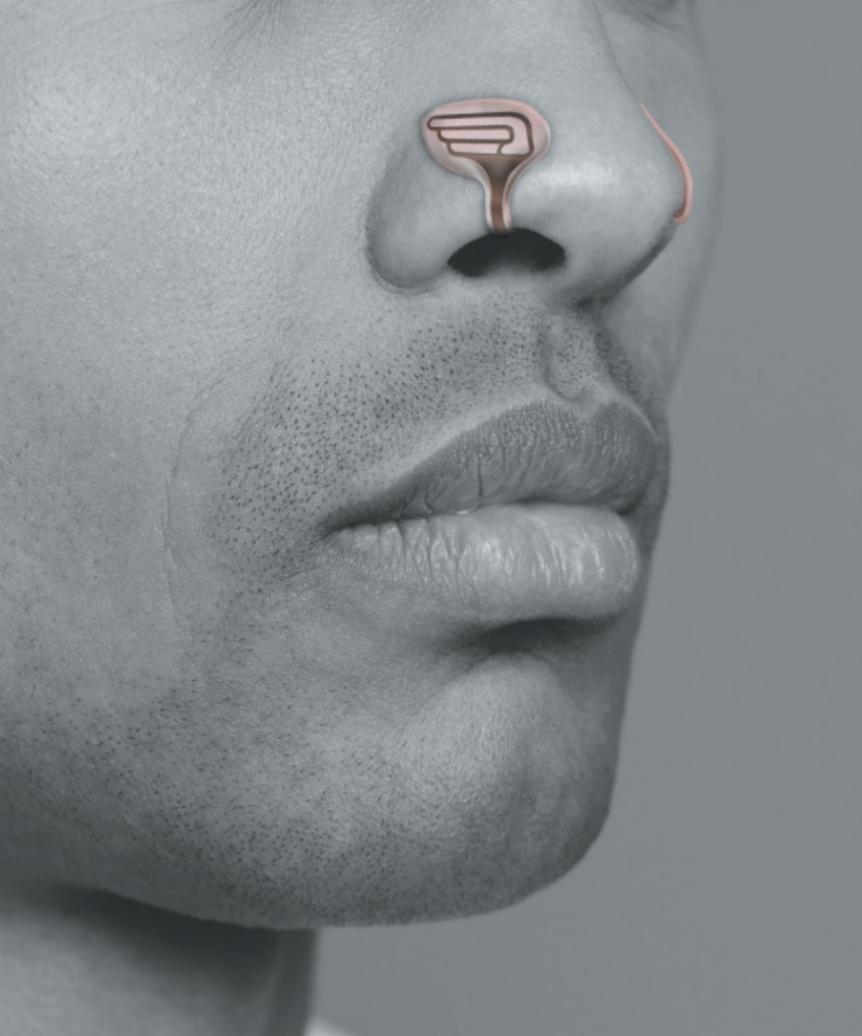

- EN** 4. What is NoseWings
- 5. Instructions for use
- NL** 7. Wat is NoseWings
- 8. Gebruiksaanwijzing
- FR** 10. Qu'est-ce que c'est que NoseWings
- 11. Notice d'utilisation
- DE** 13. Was ist NoseWings
- 14. Gebrauchsanweisung
- IT** 16. Cos'è NoseWings
- 17. Istruzioni per l'uso
- ES** 19. ¿Que es NoseWings?
- 20. Instrucciones de uso

Last revised: May 2021

## magnetic nasal dilator

### EN What is NoseWings:

NoseWings is a medical device intended to facilitate nasal airflow by widening the nasal airways by using the repellent force of strong permanent magnets. NoseWings are used for treatment of symptoms of obstruction in the nasal airways that may cause difficulties with nasal breathing.

NoseWings reduce the discomfort of a blocked nose and its concomitant effects such as sleeping difficulties, snoring and discomfort during yoga or sporting activities.

**Warning:** NoseWings are not intended for children and should always be kept out of reach of small children to avoid the risk of swallowing or choking.


**Contra-indications:** Although there are no definite contra-indications, caution is advised for users with cardiac devices such as pacemakers or other implants which may be sensitive to magnetic influences.

It is advised not to use the device more than 12 consecutive hours.

NoseWings are intended for personal use only and should not be shared with others. **Product care and hygiene:** With proper care NoseWings should remain functional for many months. For hygienic reasons we advise regular rinsing with mild detergent before and after use. Stop using the product if the product is damaged.

Any serious incident that may have occurred in relation to the product should be reported to the manufacturer at [info@nosewings.com](mailto:info@nosewings.com) and the competent authority of the country in which the user is established.

### EN Instructions for use 1/2

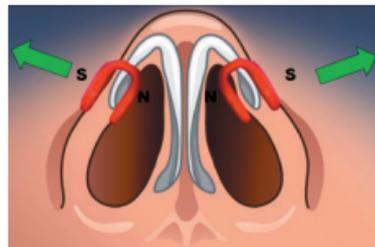NoseWings work by widening the nasal airways by using the repellent force of strong permanent magnets. To be effective NoseWings need to be properly positioned on the left and right wings of the nose with the magnets on the inside and the wings on the outside as shown on the illustration and demonstrated in the instructional video.

There is a left and a right NoseWing as indicated by the markings L and R on the inside of the container. NoseWings are shaped to fit most people with little to no need for adjustment but if necessary, they can easily be widened or narrowed for maximum comfort and effectiveness.

For optimal placement and positioning it's best to use a mirror. Take the NoseWing you wish to place first between your thumb and index finger, slightly widen the frame and slide it into position. When you let go the NoseWing should stay in place but not feel tight. If it does, take it out, widen it a little bit more a place it back again until it feels comfortable. If it's too loose, slightly narrow the frame. Once the first NoseWing is in

## EN Instructions for use 2/2

properly positioned, place the second one in the same way.

When properly placed you should notice a widening of your nasal passages and feel less resistance while breathing. Depending on the size and shape of your nose small adjustments of the position or angle may help to find the sweet spot in which NoseWings feel most comfortable and are most effective. Once in place, allow a little time to get used to NoseWings and after a short while you should hardly notice you are wearing them, except for the increased ease of breathing they will give you.

For any further inquiries or remarks  
please contact us at [info@nosewings.com](mailto:info@nosewings.com).

Manufacturer: Rhinolandia B.V.  
Jan Willem Brouwersstraat 21  
1026 CM Amsterdam  
The Netherlands

Medical Device Class I  
Revision date: May 2021

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Fabricante: Rhinolandia B.V.