

WARNINGS

- Never wash for anyone else.
- Never force a child to wash and never wash for a child!
- Never microwave plastic bottles.
- Always use sterilized, distilled or previously boiled water.
- Never use hot water for washing.
- Wash on congested side first and do not force.
- Never block the opposite nostril during washing or blowing.
- Never wash when you are completely congested (completely blocked on both sides) or have an ear infection without first consulting your physician.

RECOMMENDATIONS

- Replace bottle every 3-6 months.
- If ear popping occurs, blow more gently.
- Do not use immediately before bed or activity, as water may drip from nose for several minutes after washing.
- Always begin with isotonic and advance as tolerated.
- Do not share bottles.
- Nasopure bottle and cap are dishwasher safe.
- Wash with Nasopure prior to using prescribed nasal sprays.
- Consult a doctor if symptoms worsen or if you have a significant medical condition.
- If Nasopure causes discomfort, discontinue use & contact BeWell Health or your medical professional.

NASOPURE PRODUCTS

- Nasopure System Kit**
8 oz Nasopure bottle, 20 buffered salt packets
- Nasopure Refill Kit**
40 buffered salt packets
- Nasopure Little Squirt® Kit**
4 oz Nasopure bottle, 20 buffered salt packets
- Nasopure Sampler Kit**
8 oz Nasopure bottle, 4 buffered salt packets
- Nasopure Little Sampler Kit**
4 oz Nasopure bottle, 4 buffered salt packets
- Nasopure Personal Refill Kit**
20 buffered salt packets

Clearing The Air One Nose At A Time:

Caring for Your Personal Filter

By Hana R. Solomon, M.D.

Includes FREE 4 oz. sample



The Most COMFORTABLE Nose Wash in the World!®



EC REP

ISO 13485
CERTIFIED

CE

European Authorized Representative

Opelis s.a.

Bd Général Wabis 53

1030 Brussels

Belgium

Tel: +32 2 732-59-54

Fax: +32 2 732-60-03

mail@obelis.net

© BeWell Health, LLC.

Nasal Wash Bottle U.S. Patent Nos.

9,533,091 and D495,954

F-PD04-A (11/2018)



The Most COMFORTABLE Nose Wash in the World!®

Welcome to Dr. Hana's Nasopure®, *The Most Comfortable Nose Wash in the World!* Washing your nose with Nasopure helps clean the body's air filter of irritants such as allergens, infectious particles and pollutants that lead to a host of nasal woes.

Nasopure manages and relieves symptoms associated with allergies, cold, flu, sinusitis, and rhinitis. It all begins with Nasopure's patented, ergonomic bottle design, allowing for perfect control of a steady flow of solution along the nasal floor. This design eliminates the need to bend or twist the neck and avoids the common complaint of the 'drowning' sensation often felt with other nasal washes.

Our magic is in the mix! Nasopure's ideally buffered saline solution cleans the nasal passages of excessive mucus and debris, while soothing irritated membranes.



100%
Recyclable



Satisfaction
Guaranteed



Assembled
by Adults
with Disabilities



BPA-Free
Plastic



Made
in USA



MommyMD
Recommended

Nasal Congestion ~ Sinus Pressure ~ Nasal Dryness
Post Nasal Drip ~ Runny Nose ~ Sneezing ~ Coughing

Wash your nose clean the natural way with Dr. Hana's Nasopure!
Simple, Soothing, Effective and Safe - Try it, you'll love it!

Developed by a pediatrician using a mother's wisdom

Dr. Hana **BeWell**

Hana R. Solomon, M.D. HEALTH

Questions, comments or requests?

877-853-PURE (7873) • orders@nasopure.com

BeWell Health, LLC

Admin Office: 1204 Sutton Drive, Columbia, MO 65203

Marketing & Sales Office: 2950 S. Jamaica Ct, Unit 301, Aurora, CO 80014

www.nasopure.com

fb.com/nasopure.nicer.neti