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(EN) Instructions for use:

NitrAdine® disinfecting tablets disinfect removable dentures, removable orthodontic appliances, anti-snoring (mandibular advancement) and anti-bruxism devices. The tablets are highly effective against *Candida albicans*, *Pseudomonas aeruginosa*, *Staphylococcus aureus*, *Enterococcus hirae*, *Adenovirus*, *Poliiovirus* and *Murine Norovirus*.

Intended users: Wearers of full and partial removable metal or acrylic dentures, removable orthodontic appliances or anti-snoring (mandibular advancement) and anti-bruxism devices.

Packaging: Product available in the following tablet box sizes: 12, 20, 32 and 64. The number of tablets contained in this package can be found on the front of the product box.

Precautions for use: For external use only. Discard solution after use. Re-using the solution can result in its contamination by microorganisms or external particles and loss of the product's efficacy. Do not use if allergic to any of the ingredients or to persulfates. Do not put tablet in mouth. Tablet powder may cause eye damage. Do not drink the solution or use as a mouthwash. Rinse appliance well before replacing it in your mouth. Discontinue use and seek medical attention if any irritation, swelling or pain develops following the first or any later use. Consult instructions for use. Store in a dry place. For single use only. Do not use after expiry date. Do not use if blister foil is defect. Please report any serious incident to the manufacturer and the competent authority of your country. Disposal must be performed through an authorized waste management firm, in compliance with national and local regulations.

Danger

Causes serious eye damage. Causes skin irritation. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Wash hands thoroughly after handling. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER / doctor. IF ON SKIN: Wash with plenty of water. Contains: Sodium dodecyl sulphate • Potassium monopersulfate.

Instructions for use

1. Rinse your appliance under running water. If necessary, use a toothbrush or denture brush to remove any food debris.

2. Place the appliance in a glass of 150 ml of warm water (33°C - 47°C) ensuring that it is fully immersed and immediately place 1 tablet in the water. Soak for a minimum of 15 mins to ensure reduction of the biofilm. Acrylic appliances can be soaked overnight.

NB: If your tablet is broken the efficacy and usage of the product will not be affected (provided the whole tablet is used).

3. Remove the appliance from the solution and rinse well under water before replacing it in the mouth. If a taste should remain on your appliance after replacing it in your mouth, place it in warm water for 10 mins.

Recommended usage: Based on clinical testing, we recommend to begin the treatment with a daily use of the tablet for 21 days. This will ensure the reduction of the biofilm to a lower level. You can then adjust the usage based on your needs with a minimum usage of 2 times per week. Proven efficacy against *Candida albicans*, *Pseudomonas aeruginosa*, *Staphylococcus aureus*, *Enterococcus hirae*, *Adenovirus*, *Poliiovirus* and *Murine Norovirus*.

Conditions of the disinfecting activity testing: The bactericidal and yeasticidal tests were performed using 1 tablet in 150ml of lukewarm hard water (at 37°C) soaking for 15 minutes under clean conditions: in 80% final concentration/low load level. The virucidal test was performed using 1 tablet in 150ml of hard water (at 20°C) soaking for 15 minutes under clean conditions in 97% final concentration/low load level.

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