
USER INSTRUCTIONS

Instruction video at www.medcoat.com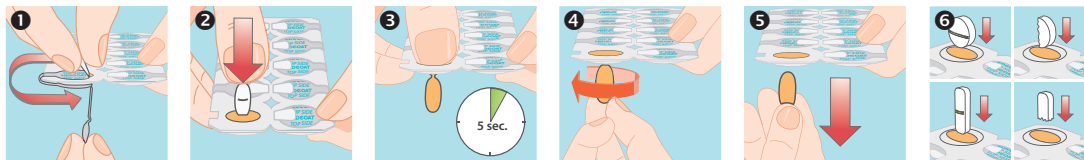

(EN) User instructions

1. Peel off the foil from the top and bottom side.
2. Push the tablet through the opening on the top side.
3. Push until the whole tablet is coated. Wait for 5 sec.
4. Twist the tablet twice without pulling.
5. Remove it with a quick pull. Swallow the tablet with the recommended amount of water.
6. How to push through different types of tablets.

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(EN) Additional information

MEDCOAT® is a slippery and saliva stimulating coating that makes tablets easier to swallow and taste good. MEDCOAT® can be used for tablets and capsules of different sizes and shapes, as well as for split tablets (not all tablets can be split). Each applicator can coat 1 big or 2 small tablets. MEDCOAT® can be used for adults and for children from the time they start to swallow whole or split tablets. Children can often learn to swallow small tablets from the age of 3-4 years. Consult your doctor or your pharmacist if you are uncertain. The coating quickly dissolves after the tablet has been swallowed and it only contains food classified ingredients. Do not use MEDCOAT® with chewable tablets, tablets that should melt in the mouth, tablets that should be dissolved in water and tablets/capsules that must be taken alone with plain water only. Use opened package within 3 months. Store and use in room temperature.

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