CE

User manual
Bedienungsanleitung
Mode d'emploi
Gebruikershandleiding

1.01
9/2018

ULTRASOFT WHEELZ

wheelzahead®

LET'S GET MOVING

USER MANUAL

Contents

1	Technical specifications	3
2	How do I replace the wheels?	4
3	How do I adjust the brakes?	5
4	Warranty	6

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Diameter \varnothing 8 '' / 20.32 cm
Weight 0.296 Kg
Width 3.9 cm

Operating conditions

Temperature between 0 and +40° Celcius
Humidity between 20 and 80%

Storage conditions

Temperature between -20 and +50° Celcius
Humidity between 45 and 60%

Materials

Rim PP
Tire PU

CE CE Feature according to EU Directive 93/42 / EWG for Medical articles

Warning!

To avoid danger of suffocation, keep this plastic bag away from babies and children.
Do not use this bag in cribs, beds, carriages or play pens. This bag is not a toy.

2 How do I replace the wheels?

Front wheels

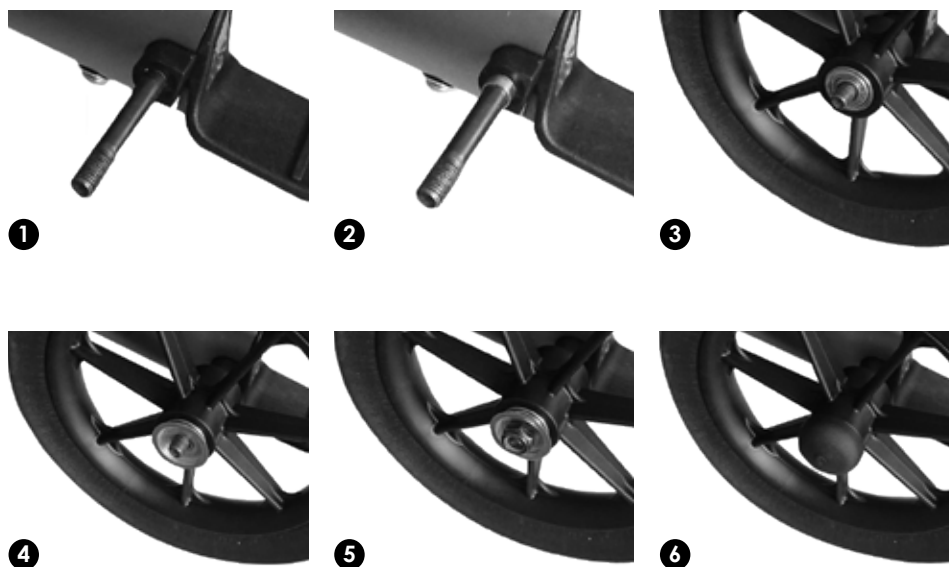
1. Loosen the bolt that forms the wheel axle with a hex key and remove the bolt and nut.
2. Remove the wheel and position the new wheel in the front fork.
3. Reinsert the bolt and tighten well.

Please note!

We advise treating the bolt with Loctite 435 before insertion.

Rear wheels

1. Remove the plastic cover from the cover fastening ring on the outside of the wheel axle.
2. Loosen the bolt that forms the wheel axle with a hex key and remove the nut, the cover fastening ring, the wheel and the small ring from the bolt.
3. Position the new ring, the new wheel, the cover fastening ring and the nut on the bolt and tighten well.
4. Re-attach the plastic cover to the cover fastening ring on the outside of the wheel axle.

Please note!

We advise treating the bolt with Loctite 435 before insertion.

Don't forget to readjust your brakes and check they are working properly!

3 How do I adjust the brakes?

This is essentially the same process as adjusting the brakes of a bicycle. On the bottom of the frame, close to the rear wheel, you will find the end of the braking cable. By turning the screw at the end, you can adjust the braking cable, tighter or looser. Should you have any doubts about the tension of the braking cable or about general brake functioning, please cease using the rollator until you have consulted a professional.

4 Warranty

1. WheelzAhead® warrants that, during the Warranty Period, the Product will, with normal use and service, be free from faulty parts, manufacture defects or workmanship errors.
2. The Ultrasoft Wheelz comes with a 12 month warranty on the frame and the cross bars, starting from the date of purchase by the original buyer. Any manufacturer defects (production errors or material failure) also carry a warranty of 12 months.
3. This warranty is valid only for Products that are purchased new and unused directly from WheelzAhead® or from authorized third party resellers or distributors.
4. The Customer must provide the original proof of purchase and the serial number of the Product which demonstrates compliance with the above conditions to receive any services under this warranty.
5. The warranty is non-transferable.
6. This warranty only applies where a defect has arisen, wholly or substantially, as a result of faulty manufacture, parts or workmanship during the Warranty Period.
7. The warranty does not apply where damage is caused by other factors, including without limitation:
 - A normal wear and tear;
 - B abuse, mishandling, accident or failure to follow operating instructions;
 - C servicing or modification of the Product other than by WheelzAhead, or their authorized service agents;
 - D use of the Product with other accessories, attachments, product supplies, parts or devices that do not conform to WheelzAhead specifications; or
 - E shipment or other transit.
8. This warranty does not cover:
 - A repair or replacement of any Product damaged during transit to or from WheelzAhead®; or
 - B any loss or consequential damage incurred while the Product is being repaired.
9. Instructions for making a warranty claim are included in the enclosed materials.

If the warranty claim is not accepted:

 - A WheelzAhead® will inform the Reseller, Distributor or the Customer in case of a purchase directly from WheelzAhead;
 - B if requested to do so by the Customer, the Reseller, Service Agent, Distributor or WheelzAhead in case of a purchase directly from WheelzAhead®, will repair the Product provided you pay the standard charges for such repair; and
 - C if applicable the Customer will be responsible for all costs associated with collection of the Product from the Reseller, Service Agent, Distributor or WheelzAhead® in case of a purchase directly from WheelzAhead®.

Making a warranty claim

Please contact your Reseller, Distributor or WheelzAhead® in case of a purchase directly from WheelzAhead®, by phone or email to make a warranty claim.

- A You must inform your Reseller, Distributor or WheelzAhead in case of a purchase directly from WheelzAhead® as soon as the warranty claim arises.
- B You must send to your Reseller, Distributor or WheelzAhead® in case of a purchase directly from WheelzAhead®, your original proof of purchase, your serial number and your written description of the fault (including image samples and any other relevant material); and
- C Subject to the applicable consumer laws in your jurisdiction, you must pay for all packing, freight and insurance costs for transit of the Product to your Reseller, Distributor or WheelzAhead® in case of a purchase directly from WheelzAhead®.
- D If the warranty claim is accepted your Reseller, Distributor or WheelzAhead® in case of a purchase directly from WheelzAhead® will, subject to the applicable consumer laws in your jurisdiction, at its cost: repair or replace any faulty parts or rectify any faulty workmanship; and return the Product to you.

WheelzAhead® BV

Polehof 38
6681 JB Bommel, Nederland

+31 (0)481 - 45 01 10
office@wheelzahead.com
www.wheelzahead.com

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