WheelzAhead INDOOR 2.0

User Manual 2

1	General information on your rollator	5
2	Technical specifications	7
3	Guide to safe use of your rollator	8
4	How to operate your rollator	10
5	Warranty	14

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Congratulations on your WheelzAhead® INDOOR 2.0 rollator purchase!

Before you start using the rollator we ask that you,
and your accompaniment, carefully read this manual.

If you have questions about the rollator or the content of this manual,
please get in touch with 2MOBILITY B.V., owner of the brand WheelzAhead®.

*The production process of your rollator has been TÜV certified
and adheres to the ISO 13485 regulations.*

*2MOBILITY B.V. hereby declares that your WheelzAhead® rollator adheres to the
European guidelines for medical support products 93/42/EEG and 2007/47/EEG.*

© Copyright 2025 2MOBILITY B.V., owner of the brand WheelzAhead®
*The information in this document can be changed or modified at any time and without
prior notice. 2MOBILITY B.V. cannot be held responsible for inaccuracies, typographical
errors or photographic errors in this document.*

1 General information on your rollator

What have I purchased?

You have purchased a foldable, light weight WheelzAhead INDOOR 2.0 rollator. This rollator is meant for indoor use for those that are mobile to a lesser degree.

Where can I find the rollator serial number?

On the back of the frame under the storage bag you will find a sticker with the serial number of your rollator.

What is the maximum allowed weight my rollator can carry?

The maximum allowed weight is 100 Kg. This includes any luggage in the designated compartments that are being carried with the rollator.

When do I use the rollator?

The WheelzAhead INDOOR 2.0 rollator supports people who are limited in their mobility indoor use. The main goal of this rollator is to make people who are limited in their mobility more mobile for indoor use.

What does the rollator consists of?

- | | | | |
|---|-----------------------------------|----|------------------------|
| 1 | Handle | 6 | Front wheels |
| 2 | Hand brake / Parking brake | 7 | Rear wheels |
| 3 | Frame | 8 | Wheel brake |
| 4 | Height adjustment | 9 | Extendable storage bag |
| 5 | Foam + front bar to press to fold | 10 | Removable tray |

Delivery

• 1 Carton box incl. product protection • 1 Rollator incl Net bag • 1 Tray • 1 User manual

Warning!

To avoid danger of suffocation, keep this plastic bag away from babies and children. Do not use this bag in cribs, beds, carriages or play pens. This bag is not a toy.

2 Technical specifications

INDOOR> 2.0

Length	680 mm
Total width	608 mm
Height	880 - 1010 mm
Measurement folded	608 x 265 x 965 mm
Total weight without accessories	6,5 Kg
Maximum load bearing capacity Bag	5,0 Kg
Maximum load bearing capacity Tray	5,0 Kg
Maximum allowed weight	100,0 Kg

User conditions

Temperature between 0 en +40 degrees Celcius
Humidity between 20 en 80%

Materials

Frame parts	Aluminum
Basket frame	Steel + aluminum
Wheels	EVA, PP (Rim)
Handle	Aluminum
Handle cover	EVA
Brake bar	Steel
Tray	ABS

3 Using your rollator safely

Should I check anything before use?

Always make sure that the following question can be answered affirmatively before using the rollator:

- Are the brakes working properly?

After receiving your rollator, check the delivery for completeness. If you notice any damage, the rollator may not be used. Contact your dealer please.

Do adjustments to the rollator affect safety?

Adjustments can negatively affect the safe use of the rollator. 2MOBILITY can no longer will no longer vouch for the rollator's quality if adjustments have been made. The warranty also expires when adjustments are made.

When is my rollator at risk of tipping over?

This is possible in the following situations:

- You are on a slanting surface
- When riding onto and off sidewalks
- On irregular ground, around drains for instance
- You are over-reaching to grab something in front of, behind or next to the rollator

Practice walking with the rollator on a slanting surface as well as getting on and off sidewalks. By shifting the weight of your upper body slightly forwards or backwards, you change the rollator's center of gravity. This way you can safely mount a small obstacle. Should you be in doubt, never attempt to do this without assistance.

When do I put on the parking brakes?

You should always secure the parking brakes in the following cases:

- When you lean on the rollator and are getting out of your seat/sofa
- In a lift or on a wheelchair lift
- When stopping on an uneven surface

How do I adjust the brakes?

This is essentially the same process as adjusting the brakes of a bicycle. On the bottom of the frame, close to the rear wheel, you will find the end of the braking cable. By turning the screw at the end, you can adjust the braking cable, tighter or looser. Should you have any doubts about the tension of the braking cable or about general brake functioning, please stop using the rollator until you have consulted a professional.

What should I consider when reaching for item in front of, next to or behind the rollator?

The main thing to take into account is never to shift the center of balance too much when reaching for items; this may tip over the rollator. Always pay attention to the following:

- Place the rollator as closely as possible to what it is you want to pick up
- Put the rollator on its parking brakes

How do I get up and down stairs?

Do not attempt to go up or down stairs while leaning on the rollator. Fold up the rollator and carry it up or down. Neither attempt to use the rollator on an escalator.

For whom is the INDOOR> 2.0 not suited?

The INDOOR 2.0 is not meant to be used by people who:

- Have severe balance disorders
- Have severe perception disorders

Any other tips?

- If you leave the rollator out in the sun for some time, the metal parts may heat up considerably
- Maneuver carefully if the bag is loaded
- Be careful with fire and burning cigarettes
- Please note: There is an increased risk of pinching when unfolding and folding the rollator and when extending the net bag in and out
- If you are not using the rollator, it can be stored in a dry place protected from the sun.

4 How to operate your rollator

How do I unfold the rollator?

Your rollator is easy to use.

- 1 See the rollator in image 1 in folded position. With one hand, push the handle slightly backwards and, as you can see below, with your other hand, push the metal tray ring / storage bag slightly upwards. The front and rear wheels will now move apart and you will hear a confirming 'click' when the frame lock closes. Place the tray on the rollator as desired.

How I fold the rollator back up?

It is just as easy to fold as it is to unfold:

- 2 Remove the tray if necessary and stand behind the rollator. Place your hand over the front bar of the rollator frame and the front bar of the storage bag. Squeeze your hand and the front bar of the construction storage bag moves you forward with one hand and with your other hand you push the handle of the rollator forward and the rollator folds together.

When you place the front wheels after folding against the frame near the rear wheels, the INDOOR 2.0 is self-standing in the folded position and you can put it simply aside.

How do I adjust the handles in height?

On the outside of your rollator's frame you will find 5 different positioning options for your handles. To come to the correct height please mind the following:

- Stand between the rear wheels
- Try to stand as upright as possible
- The arms should hang loosely from relaxed shoulders
- The handles are to be set at wrist height
- The elbows should be lightly bent when you hold the handles
- Please make sure to always have the handlebars at the same height for improved stability.

NOTE: Always check that the height adjustment is locked before using the rollator.

How do I attach the tray?

The storage bag is already installed and ready for use. Place the loose tray in the metal tray ring at the top of the storage bag. The tray edge falls slightly over the tray ring and the soft foam provides comfort and prevents the tray from shifting. Always check that the tray is placed correctly on the tray ring before using the rollator with tray.

What is the proper posture when walking behind the rollator?

In order to come to the right posture behind the rollator, please mind the following:

- Try to walk as upright as possible
- Stay close to the rollator when using it
- Do not lean on the handles too much
- Make sure you can reach and apply the brakes at all times
- Prevent a crooked posture due to fatigue

You need enough balance and strength in both arms and legs to be able to walk around with a rollator, you also need a pair of sturdy (laced up) shoes and proper eye sight. Wear glasses if necessary.

What should I pay attention to when walking with the rollator inside?

Walking with the rollator inside requires the following:

- Make sure you have enough space to move around; have furniture moved if necessary
- Remove any unsecured rugs and thresholds
- Sufficiently light all rooms that you enter (including the hallway)

How do I use the parking brakes?

By moving the brake handle down until it clicks into place, you secure the rollator's parking brakes. The rear wheels of the rollator are now secured. When you move these handles back up, you deactivate the parking brakes.

You can always brake (temporarily) by hand when walking by pushing the brake lever up. This is called the handbrake. With the parking brake for structural braking, use as described above.

How do I use the extendable storage bag?

If you hold the rollator with one hand, you can slide out the storage bag with the other. This extra functionality is for instance practical if you have items on your tray that you can still reach your storage bag. This way you can always use your tray and storage bag efficiently. Handy, right?!

How am I to take care of my rollator?

Weekly:

- Check if the brakes are still functioning properly

Every 3 months:

- Check if the cross frame still allows for easy unfolding
- Check if all bolts and screws are secure
- Loose screws have to be fixed and secured with Loctite 435.
Please adhere the produces user manual
- Check the tire surfaces for wear, tear and any uneven areas
- Check the front and rear wheels for wheel clearance

All hinged parts are easily kept functioning smoothly by lubricating them regularly.

How do I keep my rollator clean?

With a mild soap and water you can keep textiles, frame and polymer parts clean. We also recommend you to wax the frame from time to time.

Never use:

- Abrasive products
- Pressure cleaner
- Steam
- Abrasive or corrosive wax
- Aggressive chemicals
- Silicone spray

Always dry the rollator carefully in case it got dowsed in water or rain.

Re-use

The WheelzAhead INDOOR 2.0 rollator can be re-used by others. Before re-use the rollator must be thoroughly cleaned en disinfected. The rollator must be checked for user safety and approved by an authorised dealer.

The expected lifespan is, when safety and maintenance regulations are observed, up to 5 years. If the rollator is in good condition, the rollator can also be used after this period.

5 Warranty

1. 2MOBILITY B.V. warrants that, during the Warranty Period, the Product will, with normal use and service, be free from faulty parts, manufacture defects or workmanship errors.
2. The WheelzAhead INDOOR 2.0 Rollator comes with a 24 month warranty on the frame and the cross bars, starting from the date of purchase by the original buyer. Any manufacturer defects (production errors or material failure) also carry a warranty of 24 months.
3. This warranty is valid only for Products that are purchased new and unused directly from 2MOBILITY or from authorized third party resellers or distributors.
4. The Customer must provide the original proof of purchase and the serial number of the Product which demonstrates compliance with the above conditions to receive any services under this warranty.
5. The warranty is non-transferable.
6. This warranty only applies where a defect has arisen, wholly or substantially, as a result of faulty manufacture, parts or workmanship during the Warranty Period.
7. The warranty does not apply where damage is caused by other factors, including without limitation:
 - A normal wear and tear;
 - B abuse, mishandling, accident or failure to follow operating instructions;
 - C servicing or modification of the Product other than by 2MOBILITY, or their authorized service agents;
 - D use of the Product with other accessories, attachments, product supplies, parts or devices that do not conform to WheelzAhead specifications; or
 - E shipment or other transit.
8. This warranty does not cover:
 - A the replacement of any consumables such as tires, brake pads, brake cables, textiles, etc.
 - B repair or replacement of any Product damaged during transit to or from WheelzAhead® ; or
 - C any loss or consequential damage incurred while the Product is being repaired.
9. Instructions for making a warranty claim are included in the enclosed materials. If the warranty claim is not accepted:
 - A 2MOBILITY will inform the Reseller, Distributor or the Customer in case of a purchase directly from 2MOBILITY;
 - B if requested to do so by the Customer, the Reseller, Service Agent, Distributor or 2MOBILITY in case of a purchase directly from 2MOBILITY, will repair the Product provided you pay the standard charges for such repair; and
 - C if applicable the Customer will be responsible for all costs associated with collection of the Product from the Reseller, Service Agent, Distributor or 2MOBILITY in case of a purchase directly from 2MOBILITY .

Making a warranty claim

Please inform your Reseller, Distributor or 2MOBILITY in case of a purchase directly from 2MOBILITY, by phone or email as soon as the warranty claim arises.

- A You must send to your Reseller, Distributor or 2MOBILITY in case of a purchase directly from 2MOBILITY, your original proof of purchase, your serial number and your written description of the fault (including image samples and any other relevant material); and
- B Subject to the applicable consumer laws in your jurisdiction, you must pay for all packing, freight and insurance costs for transit of the Product to Reseller, Distributor or 2MOBILITY in case of a purchase directly from 2MOBILITY.
- C If the warranty claim is accepted your Reseller, Distributor or 2MOBILITY in case of a purchase directly from 2MOBILITY will, subject to the applicable consumer laws in your jurisdiction, at its cost: repair or replace any faulty parts or rectify any faulty workmanship; and return the Product to you.

Dispose

If you wish to dispose of your rollator, please ask your dealer or local waste disposal company about the regulations applicable in your area.

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© Copyright 2025 - 2MOBILITY B.V.
© Copyright 2025 - 2MOBILITY B.V. eigenaar merk Wheelzahead®.